

Hable One function manual Android

Introduction	4
Special Hable One Shortcuts	4
Play and pause TalkBack	4
TalkBack menu	4
Turn on TalkBack	4
Android Menu App opener	4
Navigation	4
Move to next item	4
Move to previous item	4
Move to item below	4
Move to item above	4
Select an item	5
Go back	5
Home screen	5
Jump to first item on screen	5
Jump to last item on screen	5
Show actions	5
Quick access Shortcuts	5
Notifications	5
Play or pause media	5
Status bar	5
Hold 2, 3 and 4 (s)	5
Google Now	5
Long press item	5
Recent Apps / app chooser	5
Screenshot	5
Reading shortcuts	6
Read from next	6
Read from top	6
Keyboard shortcuts	6
Search on the screen	6
Hold 1, 2 and 4 (f)	6
Switch keyboard	6
	1

Lock screen	6
Sound	6
Mute media	6
Volume down (repeatable)	6
Volume up (repeatable)	6
Tabs	6
Close tab	6
New tab	6
Forward tab (in browser)	6
Backward tab (in browser)	6
Android Menu App opener	7
Open Calculator	7
Open Browser	7
Open Calendar	7
Open Contacts	7
Open Mail	7
Open Maps	7
Open Music	7
Open texting	7
Typing	7
Typing letters	7
Typing contractions	7
Space	7
Backspace	7
Backspace a word	8
Enter (or new line)	8
Shift enter	8
Numbers	8
Capitals	8
Text Editing 1 - navigating in a text box	8
Turn Contractions on or off (for typing)	8
Next character	8
Previous character	8
Select single character to the right	8
Select single character to the left	8
Select next line	8
	2

Select previous line	8
Jump to the beginning of the line	8
Jump to the end of the line	9
Jump to beginning of paragraph	9
Jump to end of paragraph	9
Jump to the beginning of the page (in Browser)	9
Jump to the end of the page (in Browser)	9
Go to next table	9
Go to previous table	9
Select all	9
Text Editing 2 - Functions with selected text	9
Copy	9
Paste	9
Cut	9
Bold text	9
Cursive text	9
Underline text	9
Undo	10
Redo	10
Make new	10
Search in document	10
Print	10
Hable Menu	10
Open the Hable Menu	10
Set operating system as Android	10
Set operating system as iOS	10
Remove all Bluetooth connections	10
Set the duration of the "hold down time"	10
Exchange button 1 with button 3 and button 4 with button 6	10
Put the language to Dutch	10
Put the language to Flemish (Belgium)	10
Put the language to English (UEB)	11
Put the language to French	11
Put the language to German	11
Battery status of the Hable One	11
Product Specifications	12
Weight	12

Dimensions (L.W.H.)	12
OS Support	12
Braille Input Keys	12
Languages Supported	12
Interfaces & Ports	12
Battery	12
Battery Life	12
Charging Method	12
Charging Time	12
Environmental Conditions	12
Safety & Compliance	12
Regulatory Compliance Statements	13

Introduction

This is a quick overview of all the functions the Hable One can perform. To get some more detailed explanations of the functions refer to the training guides. You can navigate through the headings to find exactly what you're looking for or make Ctrl+f your best friend.

Special Hable One Shortcuts

Play and pause TalkBack

Hold 1

TalkBack menu

Hold 2 and 4

Turn on TalkBack

Hold 1, 2, 3 and 8

Hold again to turn TalkBack off again.

Android Menu App opener

Hold 3

Navigation

Move to next item

Hold 7 and press 8

Move to previous item

Hold 8 and press 7

Move to item below

Hold 7, then press 6

Move to item above

Hold 6, then press 7

Select an item

Press 7 and 8

Go back

Hold 1 and 2 (b)

Home screen

Hold 1, 2 and 5 (h)

Jump to first item on screen

Hold 1, 2 and 3

Jump to last item on screen

Hold 4, 5 and 6

Show actions

Hold 1 and 6

Quick access Shortcuts

Notifications

Hold 1, 3, 4 and 5 (n)

Play or pause media

Hold 1, 2, 3 and 4 (p)

Status bar

Hold 2, 3 and 4 (s)

Google Now

Hold 1, 4 and 5

Long press item

Hold 3 and 8

Recent Apps / app chooser

Hold 2 and 3

To close recent apps, you can swipe up with two fingers.

Screenshot

Hold 3 and 6

Reading shortcuts

Read from next

Hold 1, 2, 3 and 5 (r)

Read from top

Hold 2, 4, 5 and 6

Keyboard shortcuts

Search on the screen

Hold 1, 2 and 4 (f)

Switch keyboard

Hold 1, 5 and 6

Lock screen

Hold 2 and 5

Sound

Mute media

Hold 1, 3 and 4

Volume down (repeatable)

Hold 1, 2 and 6

Volume up (repeatable)

Hold 3, 4 and 5

Tabs

Close tab

Hold 2, 3 and 5

New tab

Hold 2, 5 and 6

Forward tab (in browser)

Hold 2, 3, 4 and 5

Backward tab (in browser)

Hold 1, 2, 5 and 6

Android Menu App opener

For all these shortcuts, you must first start the app category opener by pressing and holding 3.

Open Calculator

Hold 1

Open Browser

Hold 1 and 2

Open Calendar

Hold 1, 2 and 3

Open Contacts

Hold 1 and 4

Open Mail

Hold 1 and 5

Open Maps

Hold 1, 3 and 4

Open Music

Hold 1, 2, 3, 4

Open texting

Hold 2, 3 and 4

Typing

Typing letters

Letters are the same as the braille alphabet

Typing contractions

Contractions work in the same way as UEB grade 1 braille

Space

Press 8

Backspace

Press 7

Backspace a word

Press 5 and 7

Enter (or new line)

Press 1 and 8

Shift enter

Press 2 and 8

Numbers

Press 3,4,5,6 first to enable numbers. The numbers are the same as braille. To go back to typing alphabets, press button 8 for space and continue.

Capitals

Press 6 first, followed by the letter you want to type. The letter after that will automatically be lower case again.

Pressing 6 twice activates capitals for the next word. Adding a space deactivates the capitals. Pressing 6 three times will activate CapsLock for the coming sentence. An Enter will deactivate the capitals.

Text Editing 1 - navigating in a text box

Turn Contractions on or off (for typing)

Hold 1, 2, 4 and 5 (g)

Next character

Hold 7 and press 4

Previous character

Hold 4 and press 7

Select single character to the right

Hold 7 and press 4 and 5

Select single character to the left

Hold 4 and 5 and press 7

Select next line

Hold 7 and press 5 and 6

Select previous line

Hold 5 and 6 and press 7

Jump to the beginning of the line

Hold 1, 4, 5 and 6

Jump to the end of the line

Hold 3, 4, 5 and 6

Jump to beginning of paragraph

Hold 1, 2 and 3

Jump to end of paragraph

Hold 4, 5 and 6

Jump to the beginning of the page (in Browser)

Hold 2, 3 and 6

Jump to the end of the page (in Browser)

Hold 3, 5 and 6

Go to next table

Press 2, 3, 4, 5 and 8

Go to previous table

Press 2, 3, 4, 5 and 7

Select all

Hold 2, 3, 5 and 6

Text Editing 2 - Functions with selected text

Copy

Hold 1 and 4 (c)

Paste

Hold 1, 2, 3 and 6 (v)

Cut

Hold 1, 3, 4 and 6 (x)

Bold text

Hold 1, 2 and 8

Cursive text

Hold 1, 3 and 8

Underline text

Hold 2, 3 and 8

Undo

Hold 1, 3, 5 and 6 (z)

Redo

Hold 1, 3, 4, 5 and 6 (y)

Make new

Hold 2, 3, 4, 5 and 6

Search in document

Hold 1, 2, 4, 5 and 6

Print

Hold 1, 2, 3, 4 and 8

Hable Menu

Open the Hable Menu

Hold 1, 2, 3, 4, 5 and 6

All next functions only work once you enter the Hable Menu. After a function is executed, you automatically leave the Hable Menu.

Set operating system as Android

Hold 1 (you feel 3 vibrations)

Set operating system as iOS

Hold 2 (you feel 2 vibrations)

Remove all Bluetooth connections

Hold 1, 2, 3 and 5 (r)

Set the duration of the "hold down time"

Hold 1, 2 and 3 (l)

4 vibrations means you need to hold the buttons the longest, one vibration means the shortest.

Exchange button 1 with button 3 and button 4 with button 6

Hold 1, 3, 4 and 6 (x)

Put the language to Dutch

Hold 1, 4 and 5 (d)

Put the language to Flemish (Belgium)

Hold 1 and 2 (b)

Put the language to English (UEB)

Hold 1 and 5 (e)

Put the language to French

Hold 1, 2 and 4 (f)

Put the language to German

Hold 1, 2, 4 and 5 (g)

Put the language to Spanish

Hold 2, 3 and 4 (s)

Put the language to Danish

Hold 1, 3, 4 and 5 (n)

Battery status of the Hable One

Hold 3 and 6

- 3 vibrations: High battery level.
- 2 vibrations: Medium battery level.
- 1 vibration: Low battery level.

Product Specifications

Weight

100 grams

Dimensions (L.W.H.)

120 × 65 × 32 mm

OS Support

iOS (iPhone & iPad): Voiceover

Android: TalkBack

Mac OS: Writing support

Windows: Writing support

Braille Input Keys

Six-dot braille input

Languages Supported

UEB (contracted; uncontracted), Dutch, German, French

Interfaces & Ports

One Micro USB-C, Firmware upgrades and battery storage, Bluetooth V5, Slider Switch

Battery

Non-replaceable, Lithium-ion battery

Battery Life

14 days of typical use

Charging Method

PC charger: Charge over USB Port – 5V, 500 mA Max Wall charger: Charge over USB Port – 5V, 1000 mA Max;

Charging Time

The Hable one is fully charged in about 3 hours.

Environmental Conditions

Operational

Temperature: five °C to 45 °C

Humidity: 10% - 70% relative humidity Storage

Temperature: -15 °C to 70 °C Humidity: 5% - 90% relative humidity

Safety & Compliance

Thank you for purchasing the Hable One™ product (the Product). With this document we aim to inform you about the safety, compliance, and warranty aspects of the Product.

Important Safety Instructions

General warnings



Read, keep, and follow these instructions.

Heed all warnings.

Do not use the Product near water. This can result in a risk of fire or electric shock.

Substances in the Product and its battery may harm the environment or cause injury if handled and disposed of improperly.

Clean only with a dry cloth. Do not use abrasive cleaners.

Do not block any ventilation openings.

Do not use or dispose of the Product at any heat sources, such as radiators, stoves and other apparatus that produce heat. Turn off the product when it feels warm or hot.

Only use attachments/accessories specified by Hable or sold with the Product (such as a USB-C cable).

Refer all servicing to qualified service personnel. Do not open the Product yourself. Servicing is required when the Product has been damaged in any way, e.g. when the charging port is defective or when liquid is spilled over the Product.

The Product may not be used by persons under the age of 8.

Long periods of repetitive motion using the Product may be associated with nerve, tendon or muscle injury in your hands, wrists, arms, shoulders, neck or back. See a qualified health professional for pain, numbness, weakness, swelling, burning, cramping, stiffness or any other medical discomfort that may occur when using the Product.

Specific battery warning

Do not attempt to replace the battery or open the enclosure or disassemble the Product yourself. Doing so will void the warranty and can result in a safety hazard. If the Product is no longer charging, please contact Hable Customer Support for repair or replacement information.

Disposal and Recycling information

Due to the battery incorporated in the Product, the Product may not be disposed with ordinary household waste.



Regulatory Compliance Statements



Hable is authorized to apply the UKCA Mark on the Product. The following Directives and Standards have been applied:

2011/65/EU

2012/19/EU

2006/66/EC

1907/2006

2014/53/EU

2014/35/EU

Warranty

The Limited Warranty Policy (LWP) for Hable One Products applies to the delivery of the Product. Provided that for customers (any natural person acting for purposes which are outside his trade, business or profession) under UK law the Warranty Period (as defined in the LWP) is six years in England, Wales and Northern Ireland and five years in Scotland. The customer may choose which Warranty Services (as defined in the LWP) are desirable. During the repair or replacement of the Product, the Warranty Period shall be suspended.