RAPTOR

SERVER ANALYSIS SOFTWARE

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INDHOLDSFORTEGNELSE

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Welcome to RAPTOR SERVER Analysis Software, the analysis software for the [ALEX multiplex test](#). Please log in below to continue.

Log in with Azure AD / Office 365	Log in with Username & password
If your organization integrates the software into its Microsoft Azure/Office 365 user directory (single-sign-on) click the button below. You will be redirected to the Microsoft login screen.  Log in	Please enter your username and password. Your username Your password   Log in Forgot your password? No problem, you can request a new one.

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Log in with Username & password

Please enter your username and password.

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Contract approval required

Please review the following documents. You are required to confirm that you have read and agree with each document in order to use RAPTOR SERVER:

☐ **Bitte lesen** (last updated 23.11.2023 18:00)

☐ **RAPTOR SERVER IFU** (last updated 04.12.2023 11:05)

☐ **Terms of Service** (last updated 22.11.2023 01:00)

Confirm

By clicking the button, I confirm that I have reviewed these documents and agree.

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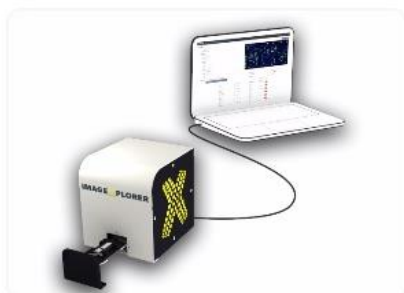
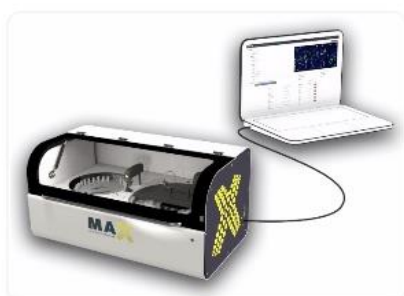
9K-Test-Instrument

Not initialized

[Initialize](#) [Shutdown](#) [Monthly Maintenance](#)

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E-Mail address:

Language: English (en)

The language is used to determine the display language.

Culture: English (United States) (en)

The culture is used to format numbers and dates.

Timezone: (UTC+01:00) Amsterdam, Berlin, Bern, Rome, Stockholm, Vienna

Configure your timezone to see date and times in your local time instead of UTC.

Multi-Factor Authentication

[Manage MFA](#)

Use multi-factor authentication (MFA) to significantly increase security. It is highly recommended to enable MFA; this adds a second layer of protection against phishing attacks or weak passwords.


Configuration

Research spots: Show research spots

If enabled, "research use only" arrays. This does not affect the results.

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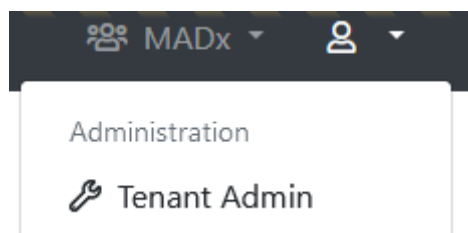Configure "MADx"

General configuration	
* Name:	MADx
Default timezone:	(UTC+01:00) Amsterdam, Berlin, Bern, R _e
Configure a timezone to see date and times in (instead of UTC). Please note that users can choose to use a different timezone.	
Default report template (ALEX2):	ALEX2
Default report template (ALEX3, ALEX Air, ALEX Food):	check 1.21
Default report template (FOX):	FOX (2025)
Default report template (PAX):	PAX
Default report language:	English (en)
Default ALEX MOD:	< No MOD >
Default FOX MOD:	< No MOD >
Default PAX MOD:	< No MOD >
Force MOD:	False
Requires a measurement to always have an MOD assigned. If this setting is enabled, a measurement cannot be approved without an MOD assigned.	
Hide Global MOD:	False
If this setting is enabled, only the tenant-specific MOD will be available to select, and the global system-wide MOD will be hidden.	
Display MOD Excluded Spots As Research Use Only:	False
If this setting is enabled, spots excluded by an MOD will not be hidden from the measurement image, but displayed as a "RUO spot".	
MFA:	☐ Force MFA
If this setting is enabled, users are forced to set up an MFA on sign-in, in case they haven't yet.	

MAX device default settings	
* Default Minimum RunId:	10
When a new MAX device is created, this value will be used for its Minimum Run Id	

Tenant-specific batch releases	
Restrict this tenant to only allow the following batches to be scanned or to restrict certain batches. Enter one per line.	
Use:	
☒ Whitelist	
☐ Blacklist	
Whitelisted Batches:	Enter one batch code including product identifier per line, for example "02AAA"
Blacklisted Batches:	Enter one batch code including product identifier per line, for example "02AAA"

Company Details

* Legal Entity Name:	MacroArray Diagnostics
* Street and Number:	Street 4
* Zip Code:	1230
* City:	New York
* Country:	United States of America
* Phone:	+4318652573
* E-Mail:	office@macroarraydx.com
* Legal Representative:	Dr. Christian Harwanegg
Website:	
Vat Number:	
Company Register Number:	

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Tenant-specific batch releases

Restrict this tenant to only allow the following batches to be scanned or to restrict certain batches. Enter one per line.

Use:

☒ **Whitelist**

☐ **Blacklist**

Whitelisted Batches:

02AAA
02ABC
02BAA

Blacklisted Batches:

Enter one batch code including product identifier per line, for example "02AAA"

Enter one batch code including product identifier per line, for example "02AAA"

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You requested a new measurement from ImageExplorer "IX Trainingsraum". Please wait until the measurement is completed ...

Invalid batch detected! The measurement can't be processed because its batch is blocked.

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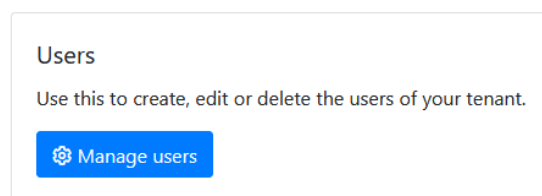
Login Credentials

Type: Username and Password

Please select if the user should be validated against Azure AD (Office 365), or if you want to specify username and password yourself. It is recommended to use Azure AD whenever possible. This allows for increased security by supporting two-factor authentication. Also, it is more convenient for the user, because he or she does not have to remember a new username and password (single-sign-on).

Username:

It is a good idea to use the users e-mail-address as his username, because the username needs to be unique across the entire system.

Password:

Automatically send invitation e-mail

Access Rights

Preconfigured groups

Use these pre-configured groups to automatically assign a set of access rights to this user. You can still add or remove specific access rights.

Reset Lab User Super Lab User Support/Distributor User System Administrator

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Tenants

This user is connected to the following tenants and has the following access rights:

☐ Tenants	Can share measurements	Can edit measurements	Can edit personal data	Can approve measurements	Can manage basic tenant configuration	Can manage API keys	Can manage ImageXplorers	Can view agent connections	Can manage users	Can manage users of Result Sharing Portal	Can manage report templates	Can manage measurement data completion	Can unapprove tenant specific batch releases	System Administration	Can unapprove measurements	Can use a custom calibration string	Can manage Run Configuration	Can skip
☐ Another MADx Tenant	☒	☒	☒	☒	☒	☒	☒	☒	☒	☒	☒	☒	☒	☒	☒	☒	☒	☒
☐ MADx Tenant	☒	☒	☒	☒	☒	☒	☒	☒	☒	☒	☒	☒	☒	☒	☒	☒	☒	☒

Additionally this user can be added to the following tenants by checking the tenant name and hitting save afterwards. User rights w applied from current open tenant user.

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Additionally this user can be added to the following tenants by checking the tenant name and hitting save afterwards. User right applied from current open tenant user.

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Tenants

This user is connected to the following tenants and has the following access rights:

Edit selected tenants rights								
☐	Tenants	Can share measurements	Can edit measurements	Can edit personal data	Can approve measurements	Can manage basic tenant configuration	Can manage API keys	Can manage Imagey
☒	Another MADx Tenant	✓	✓	✓	✓	✓	✓	✓

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ImageXplorers

Use this to configure ImageXplorers, or add new ones. Every ImageXplorer needs to be known to Raptor-Server, otherwise it won't be able to connect.

[Manage ImageXplorers](#)

Add new ImageXplorer

General information

Name:

Key:
This key is required when installing an Agent.

ImageXplorer Serialnumber:
Serialnumber of connected ImageXplorer.

Notes:

Internal Notes:
For internal use only.

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RAPTOR

New measurements Approved measurements Start measurement Automata

Details

Barcode: @2AE3488
Sample Code:
Requisition Code:
Sampling Date: TT.MM.JJJJ
Assay Date: 04.10.2021
Patient Code:
Patient Name:
Date of Birth: TT.MM.JJJJ
Info:
Doctor Info:
Choose a doctor... Insert doctor info
Tags:
MOD: < No MOD >
Report Template: Version 2 (ALEX2 only) - Deutsch (de)
Report Save Save and approve

B

Received: 04.10.2021 16:23
ImageXplorer: 22124137
Analysis: 04.10.2021 16:23
Automatic QC: OK

Name	Code	Biochemical	Tissue & Exposure	E/M	I2 Feature	KUA/L
Kivi <i>Actinidia deliciosa</i>	f501	Kiwifelin	Plant Food Food	M	Act d 5	4.94
Alternaria alternata <i>Alternaria alternata</i>	m229	Alt a 1-Family	Microorganisms & Spores Inhalation	M	Alt a 1	2.65
Ragweed <i>Ambrosia artemisiifolia</i>	w1		Pollen Inhalation	E	Amb a	4.04
Ragweed <i>Ambrosia artemisiifolia</i>	w230	Pectate lyase	Pollen Inhalation	M	Amb a 1	11.66
Cashew <i>Anacardium occidentale</i>	f443	2S Albumin	Plant Food Food	M	Ana o 3	6.78
Herring worm <i>Anisakis simplex</i>	p10	Kunitz Serin Protease Inhibitor	Animal Food Food	M	Ani s 1	8.22
Herring worm <i>Anisakis simplex</i>	p11	Tropomyosin	Animal Food Food	M	Ani s 3	22.58

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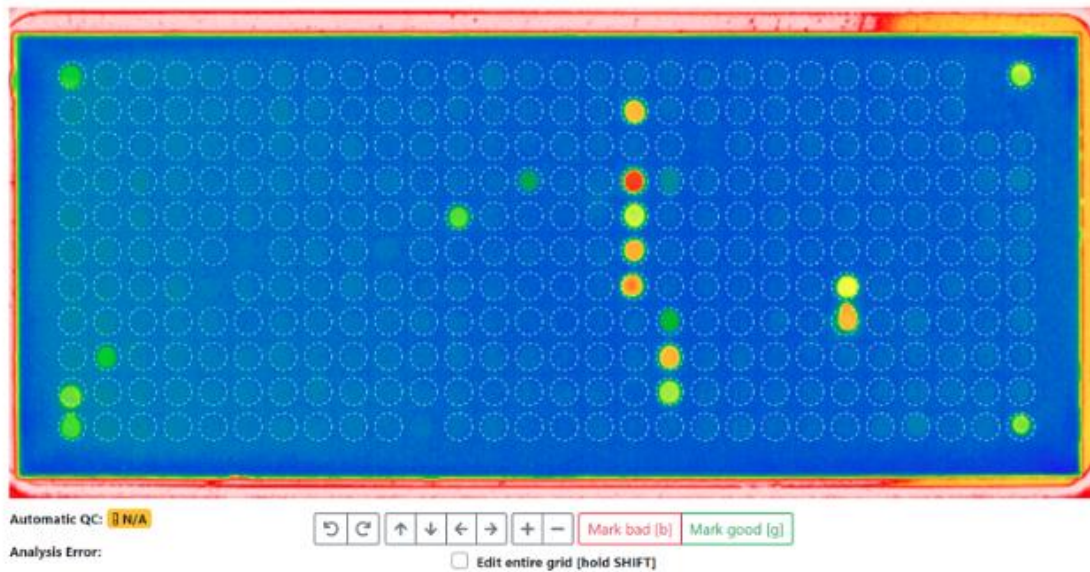
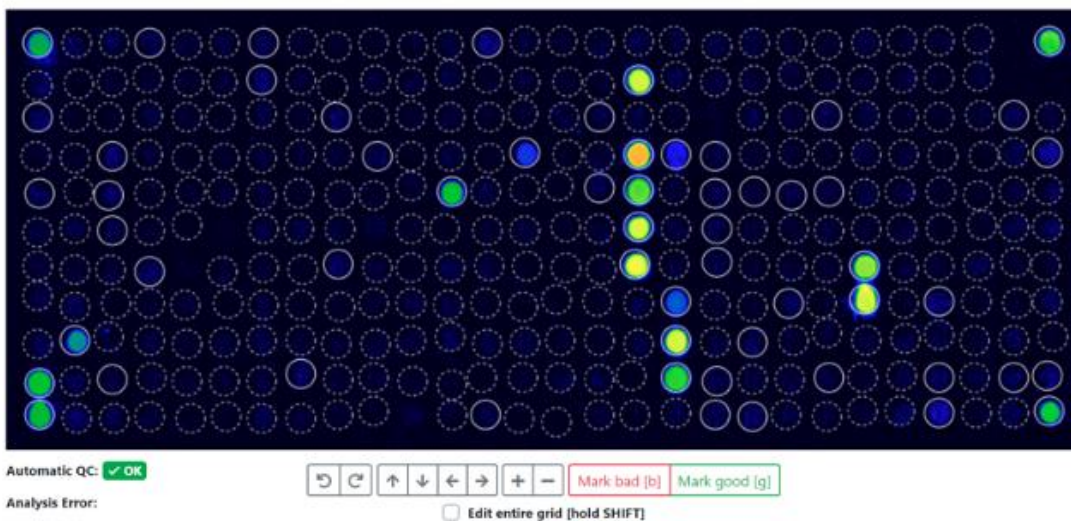
Manual Analysis
Image
Grid
More
Show Analysis & QC Logs

Reset everything and reprocess

This discards all modifications to the array, and re-runs the complete analysis process from the original image data. Please note that any manual changes will be irreversible lost.

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Analysis Log

```
'PreProcessedPixels' took 128ms.
Using definition '02_ALEXII_IVD_v1.6.0'.
'ApplyArrayDefinition' took 74ms.
Parsing calibration string ...
'ApplyDefinitions' took 75ms.
'FindGrid' took 1ms.
'AlignFeatures' took 862ms.
'CropImage' took 1ms.
'QuantifyFeatures' took 231ms.
'SubtractBackground' took 601ms.
Parsing calibration string ...
Using calibrators from QR-Code.
Total iqt (tqg) normalized by calculating 0,330999755859375E-07x²+0,05697999954223633x+-38,69269921875
(x=8515), which results in 515.
'ReQuantify' took 57ms.
```

QC Log

These are the values used for QC (ALEX v2): Curve Correction=0,9997370867894242, tlgE=8515, GD1=28985 (SNR=105.38), GD2=29369 (SNR=185.28), GD3=28569 (SNR=147.53), GD4=28385 (SNR=128.1). Reference values are as follows: Guide Dots must have a value above 5000; each Guide Dot spot must exist and be unique; Guide Dot spot must not be set to bad. Average background must be between 4000 and 16000. QC (ALEX v2) succeeded.

Background Log

Calculated background statistics for 164 pixels (from a total of 182, skipped smallest 9, and highest 9 pixels). The average is 9,419, the minimum 7973 and the maximum 11792 (minimum and maximum is calculated including skipped pixels). This results in an average difference of 628, which is an SD of 49,17 and a CV of 0,52%. For comparison, there is an overall total of 282944 pixels. The overall average is 14,871, the minimum 6,012 and the maximum 64,965.

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
Tissue & Source		Feature	kUA/L
Pollen Inhalation	E	Aca m	≤ 0.10
	E	Aca s	7.11

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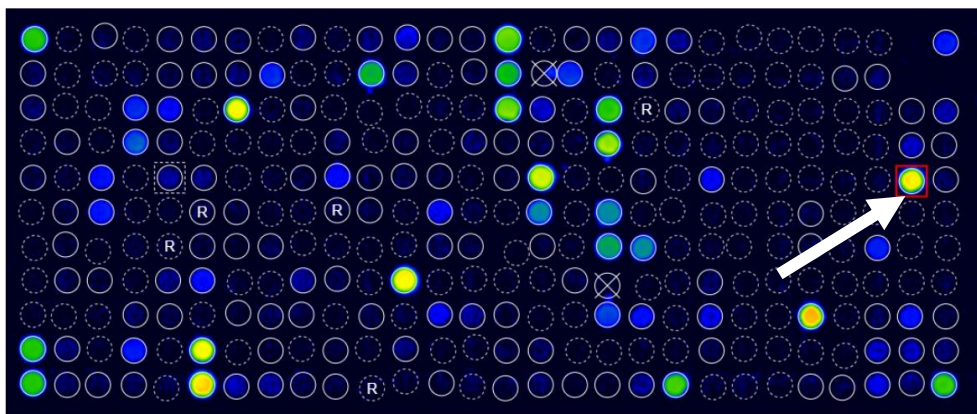New Measurements

Search by QR-Code, patient, sample, tag, analysis date, approval date ...

☒ Selected measurements

☐	Source	QC	Tags	Product type
☐	IX	✓ OK		ALEX2
☒	IX	✓ OK		ALEX2
☒	IX	✓ OK		ALEX2
☒	IX	✓ OK		ALEX2

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Re-process measurements

You are about to re-process the following measurements. undone, the existing analysis result will be irreversibly over

QR-Code	Sample Code
02APK3C8	Dummy8
02APK258	Dummy1
02APK07F	Dummy13

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New Measurements

	Source	QC	Tags	Product type
☐	Recaptured	✓ OK	recaptured	ALEX1
☐	⚠ Recaptured	✓ OK	recaptured	ALEX1
☐	QR-Code was scanned more than once.		recaptured	ALEX1

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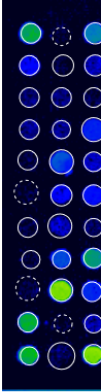
Approve measurements

You are about to approve the following measurements.

There are measurements with one or more warnings. Please make sure to double-check these measurements before approval.

QR-Code	Sample Code	Analysis Date
02AKV035		28.04.2022 11:14

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RAPTOR
New measurements
Approved measurements
Start m

Approved Measurements

Search by QR-Code, patient, sample, tag, analysis date ...

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RAPTOR
New measurements
Approved measurements
Start measurement
Automats
MADx Testtenant


Exports

Certain types of data exports (for example, the export of many PDF reports at once) take a few minutes to prepare for download. Here download your requested exports. Please note that you can keep working, all exports are done in the background and you can download when they are completed.

Export requested	# of measurements included	Export completed
8/16/2021 11:07 AM	4	0 / 4
8/16/2021 11:06 AM	4	8/16/2021 11:06 AM
8/16/2021 11:04 AM	3	8/16/2021 11:04 AM

Administration
Tenant Admin
Exports
Personal Settings
Log out
Support
Imprint
Download
Download

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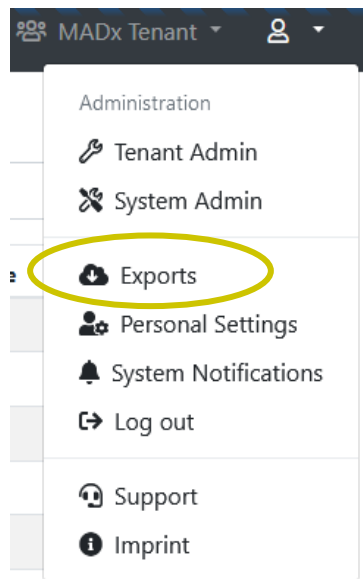Exports

Certain types of data exports (for example, the export of many PDF reports at once) may take some time. Please note that you can keep working, all exports are done in the background.

[Export multiple measurements](#)

Export requested	# of measurements included
06.07.2022 08:52	3

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Export multiple measurements

Please note that **Report PDF** and **ASTM** export is only available for certain data types.

QR-Code/Product/Batch

02ABC

Sa

Analysis Date (from)

TT . MM . JJJJ

Analysis Date (to)

TT . MM . JJJJ

Tag

Status

All

☐ Export CSV
 ☐ Export Excel

Search Measurements

✕ Cancel

Export multiple measurements

Please note that **Report PDF** and **ASTM** export is only

QR-Code/Product/Batch

02aaa

Analysis Date (from)

TT . MM . JJJJ

Analysis Date (to)

TT . MM . JJJJ

Tag

Status

All

☐ Export CSV

☐ Export Excel

 Start export ...

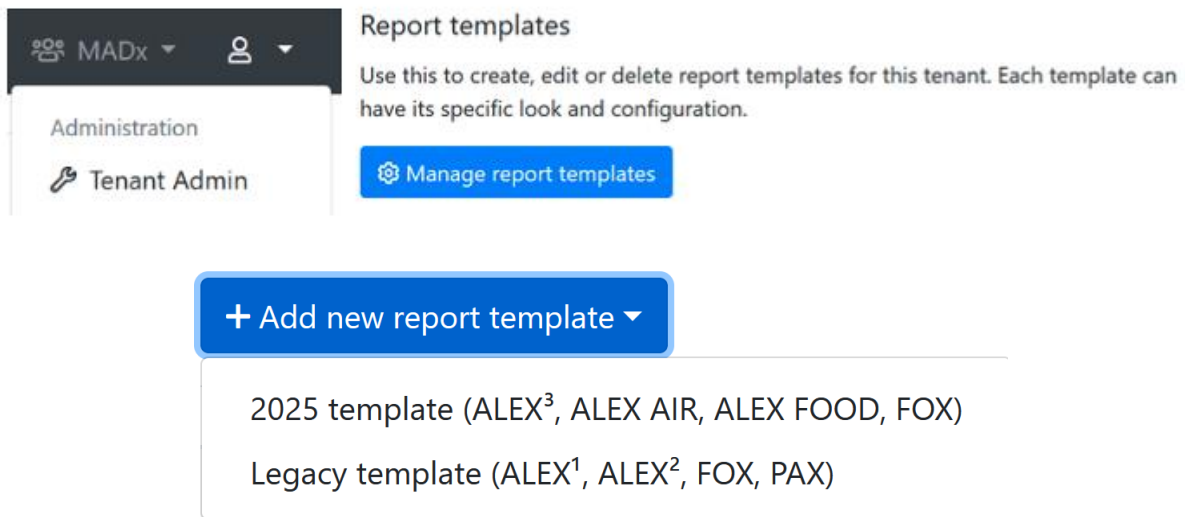
← Back

66 Measurements found that match your criteria

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Edit report template

Your changes have been saved successfully.

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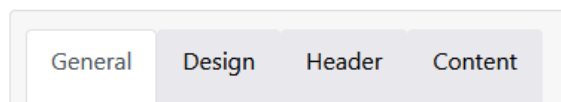
Configure report template "New report template"

General	Design	Header	Content
Name: New report template This is the name of the template which shows up in the dropdown list while generating a report.			
Base Template: Version 2 (ALEX2 only) A base template which specifies basic functionality must be chosen.			
Preferred language: English (United States) (en-us) A default language can be specified which will be used if there isn't yet a language stored to a measurement's report.			

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The filename of the report PDF can be customized. Either built automatically based on your selection below or manually adjusted in the order of your needs.

PDF naming

Setting Type: Automatic

Include Patient ID: ☒

Include Patient Name: ☒

Include Sample ID: ☐

Include Measurement QR-Code: ☐

Include Date & Time: ☒

The filename of the report PDF can be customized. Either built automatically based on your selection below or manually adjusted in the order of your needs.

Footer

Footer text:

Will be printed in the footer if provided and rendered as HTML.

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Configure report template "New report template"

The screenshot shows the 'Design' tab of a configuration interface. It has four sub-tabs: 'General', 'Design', 'Header', and 'Content'. The 'Design' tab is active. It contains three main settings:

- PDF format:** Two radio buttons, 'A4' (selected) and 'Letter'. A note to the right says: 'Specifies the paper size of the output PDF.'
- Font:** A dropdown menu showing 'Arial'. A note to the right says: 'The font family of the whole report texts can be adjusted.'
- Background image:** A text input field followed by a 'Browse' button. A note to the right says: 'If set, this image is used as background on the first page of the PDF.' Below the input field, it says 'Maximum file size 1 MB'.

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Third level headline color:

Logos

Logo (left): Browse
Maximum file size 1 MB
This logo will appear on the **top left corner** of the first page of the report.

Text (center):
This text will appear either on the left, right, or center of the top of the first page (depending on your configured logos). Supports the placeholders `{{ PATIENT_PORTAL_QRCODE }}` and `{{ PATIENT_PORTAL_URL }}` to dynamically insert measurement information.

Logo (right): Browse
Maximum file size 1 MB
This logo will appear on the **top right corner** of the first page of the report.

Small logo: Browse
Maximum file size 1 MB
This logo is used on all detail pages. If not set, the default logo is used.

Signatures

Left: Browse
Maximum file size 1 MB
This signature or image will appear on the last page, left aligned.

Center: Browse
Maximum file size 1 MB
This signature or image will appear on the last page, center aligned.

Right: Browse
Maximum file size 1 MB
This signature or image will appear on the last page, right aligned.

Indicators

Custom intensity indicator:
A custom SVG for displaying allergen intensity can be pasted here. Will be styled according to specified colors.

⚠️ The SVG element must contain the attributes **width** and **height** with a minimum dimension of 16 for correct display in PDF.

🌿 SVGs paths which should be colored must require the **colorize** class as well as a fill defined in the **defs** segment of the SVG.

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Configure report template "New report template"

Header content:	
Left half	Right half
Custom headline ☐ Patient Name Patient Name Custom headline	Custom headline ☐ Doctor Information None
Date of Birth	None
Sample ID	None
QR-Code	Additional Info text
Analysis Date	None
Amount of tested allergens	None
Test method	Quality Control text

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General	Design	Header	Content
Allergens to include:	All allergens	Specifies which kind of allergens should be included in the report.	
Allergens sort order:	Alphabetically by allergen	Specifies the sort order of the allergens.	
Include Overview:	☒	Whether the allergen overview should be printed or not.	
Include product type in overview:	☒	Whether the product type should be printed in the header of allergen overview.	
Include Total IgE:	On the front page	Whether the information of total IgE should be included in the report and where.	
Cross-reactive information:	Include text	This section will be displayed underneath the allergen list itself.	
Include Interpretation:	Include manually	It can be specified if the Raven report interpretation information should be included in the report or not. This section is printed underneath the allergens overview.	

Additional information (below the list of results)		Those informations are integrated into the report in an extra section underneath the allergen list.
Approval date:	☐	
Assay date:	☐	
Doctor info:	☐	The information about the doctor which accompanied the measurement/patient.
Print date:	☒	When the report was generated.
Report info:	☐	The „additional info“ field belonging to the measurement.
Sample date:	☒	

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Configure report template "New report template - Duplicate"

General	Design	Header	Content
Name:	New report template - Duplicate		
Support products:	ALEX ²		
Preferred language:	Deutsch (de)		

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General configuration

*** Name:** MADx

Default timezone: (UTC+01:00) Amsterdam, Berlin, Bern, Rome, Stockholm, Vienna
Configure a timezone to see date and times in (instead of UTC). Please note that users can choose to use a different timezone.

Default report template (ALEX 1): Version 1 (ALEX1 & ALEX2)

Default report template (ALEX 2): Version 2 (ALEX2 only)

Default report language: English (United Kingdom) (en-gb)

Default MOD: < No MOD >

Require ImageXplorer testarray scan: Warn after 60 days, enforce after 90 days.

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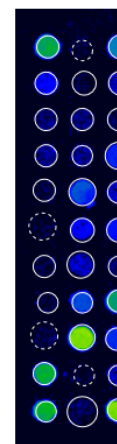
Raptor New measurements Approved measurements Start measurement Automate

Report

Template: Version 2 (ALEX2 only)

Language: English (en)

Usually this tenant's default template and language will be used when sharing or exporting measurements. You may change the template and language associated with this measurement by saving your current configuration.

You can download this report in PDF format once the measurement has been approved.

You have access to manage the report templates and settings via the tenant administration. Please note that any changes will affect all users.

MADX
MACROARRAY DIAGNOSTICS

PATIENT ID
PATIENT NAME
DATE OF BIRTH
SAMPLE ID
SWABCODE
02AD0042
ANALYSED ON
11/04/2022
TESTED ALLERGENS
306
TESTMETHODE
ALEX2

REFERRING PHYSICIAN

NOTE
The internal QC (Pauzability check for Q2) was within acceptance range.

Lab report: Summary on detectable sensitisations

POLLEN
Grass Pollen

MICROORGANISMS
Fungal Spores & Yeast

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The screenshot shows the RAPTOR web interface. At the top, there's a navigation bar with 'New measurements', 'Approved measurements', and 'Start m'. Below this is the 'Approved Measurements' section with a search bar and a 'Download selected measurements' dropdown. The table below has columns: Source, Tags, Product type, QR-Code, and Sample C. Three rows are visible, all with 'IX' as Source, 'ALEX2' as Product type, and 'Ø2AKV39D' as QR-Code. The 'Download selected measurements' dropdown is open, showing options: 'Report (PDF format)' (highlighted with a yellow circle), 'CSV format', 'Excel format', 'ASTM format', 'Update interpretation', 'Choose Reporttemplate', 'Share', 'Move', and 'Unapprove'.

	Source	Tags	Product type	QR-Code	Sample C
☒	IX		ALEX2	Ø2AKV39D	
☒	IX		ALEX2	Ø2AKV39D	
☒	IX		ALEX2	Ø2AKV39D	

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Tags:

MOD:

Report Template: Version 2 (ALEX2 only) - Deutsch (de)

Download Save >> Move to other tenant

[New measurements](#)
[Approved measurements](#)
[Start measurement](#)
[Automats](#)

Report

Template: Default
Language: English

[Save](#)
[Use defaults](#)

Usually this tenants default template and language will be used when sharing or exporting measurements. You may change the template and language associated with this measurement by saving your current configuration.

[Download PDF](#)

[Manage report template and settings](#)

You have access to manage the report templates and settings via the tenant administration. Please note that any changes will affect all users.

Patient ID:	
Patient Name:	
Date of Birth:	
Sample ID:	
Barcode:	02AAE
Sampled on:	
Tested on:	12/18/
Printed on:	12/19/
Note:	The internal QC (Plausibility)

Lab report: Summary on de


Pollen	Grass Pollen
	Tree Pollen
	Weed Pollen
Mites	House Dust M
Microorganisms	Fungal Spores
	Legumes
	Grain
	Spices
Plant-Based Food	Fruits
	Vegetables
	Nuts & Seeds
	Milk

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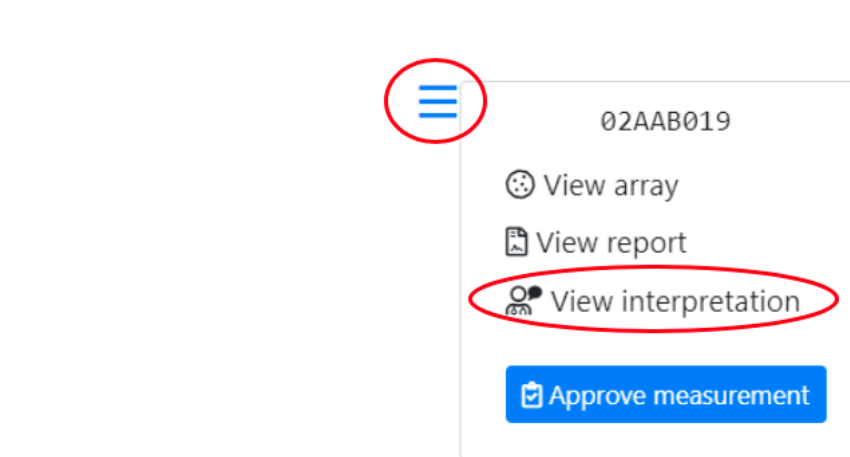Rule set

RAVEN Ruleset A for ALEX

A rule set defines the RAVEN or RAVEN² engine that is used to interpret results.

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AIT

Medication

MADx data declaration for smartRaven

☒ I accept to enter clinical data for using the full interpretation function of smartRAVEN

Patient has reported any symptoms

☐ Yes ☐ No

Additional information about the patient

☐ Atopic Dermatitis

☐ Allergen-specific immunotherapy (AIT) - ongoing, completed or discontinued

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[Request new interpretation](#)

This measurement already has an interpretation stored. Usually, it should not be necessary to request those again, but it could be necessary when the patients information has changed.

Interpretation

Raven Interpretation Summary

Sample Information

The sample was tested on ALEX² Barcode 02AAB019, interpretation date 12/19/2019.

Of the tested 295 allergens, 25 were above the detection limit of 0.3 kU_L/L. A sensitization can be an indicator of an allergy. For selected allergens, comments for interpretation are listed below.

Total IgE: 512 kU/L

The measured total IgE was 512 kU/L. Above a total IgE of 100 kU/L, allergy is likely.

Cross-Reactive Allergen Sensitization detected

Sensitization against allergen molecules which are markers of (broad) cross-reactivity between different allergen sources.

Detected cross-reactive allergen sensitizations:

- PR-10s: Aln g 1, Bet v 1, Cor a 1.0103, Fag s 1
- Polcalcins: Aln g 4, Phi p 7

PR-10 Proteins (PR10)

PR-10 inhalative: The major birch pollen allergen, Bet v 1, represents the prototype of all PR-10 allergens and is the primary sensitizer in regions with birch pollen exposure. The presence of PR-10 allergens in Fagales tree pollen explains IgE cross-reactivity between pollen from hazel, alder, beech, oak and hornbeam. PR-10 nutritive: PR-10 allergens in raw fruits, nuts, vegetable and legumes can induce oral allergy syndrome and sometimes severe allergic reactions in sensitized individuals. PR-10 allergens are not stable to heat and digestion.

Polcalcins (Ca++ binding proteins, PC)

Members of the PC family can cause inhalative symptoms. PC allergens can be found in pollen from grasses, trees and weeds. Inhalative symptoms manifest as allergic rhino-conjunctivitis and/or allergic asthma.

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The screenshot displays the 'Raptor' software interface. At the top, there's a navigation bar with 'New measurements', 'Approved measurements', 'Start measurement', and 'Automate'. Below this is the 'Details' section, which contains various input fields: 'Barcode' (024E1852), 'Sample Code', 'Sampling Date' (tt.mm.åååå), 'Assay Date' (07.10.2020), 'Patient Code', 'Patient Name', 'Date of Birth' (tt.mm.åååå), 'Info', 'Doctor Info', 'Tags', and 'MODs'. The 'MODs' field is highlighted with a red oval and contains the text 'Pollen allergens'. Below the 'MODs' field are buttons for 'Report', 'Save', and 'Save and approve'.

Below the 'Details' section is a large image of the multiplex assay results. The image shows a grid of circular spots, each containing a colored dot (blue or green). A red 'R' is visible in the center of the grid. Above the image are 'Previous' and 'Next' navigation buttons. Below the image, there's a summary section with the following information:

- Received: 07.10.2020 14:44
- ImageXplorer: 23156891
- Analysis: 07.10.2020 14:44
- Automatic QC:

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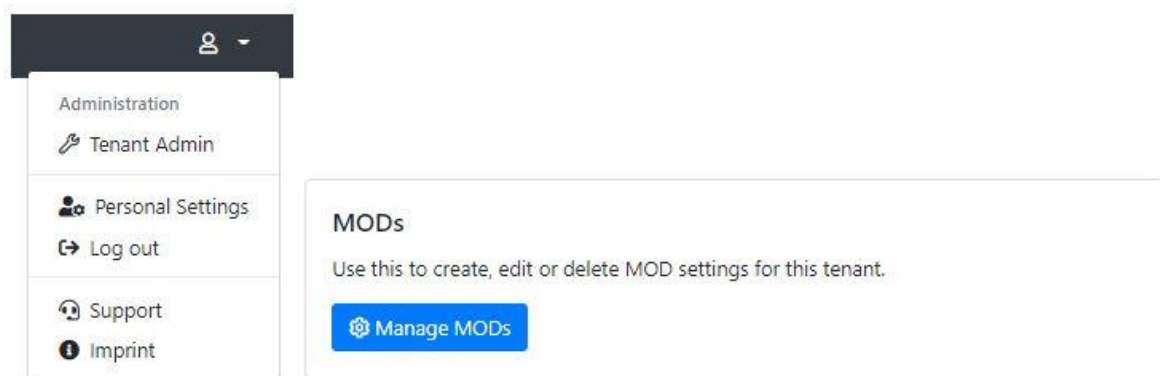
Raptor New measurements Approved measurements Start measurement Automats

Create new MOD

Name:

Type:

Deactivated:

You cannot delete an MOD once it is in use. But you can deactivate it, so that it's not available for new measurements anymore.

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* Name:

Default timezone:

Configure a timezone to see date and times in (instead of UTC). Please note that users can choose to use a different timezone.

Default report template (ALEX 1):

Default report template (ALEX 2):

Default report template (FOX):

Default report template (PAX):

Default report language:

Default ALEX2 MOD:

Default FOX MOD:

Default PAX MOD:

Force MOD:

Requires a measurement to always have an MOD assigned. If this setting is enabled, a measurement cannot be approved without an MOD assigned.

Hide Global MOD:

If this setting is enabled, only the tenant-specific MOD will be available to select, and the global system-wide MOD will be hidden.

Display MOD Excluded Spots As Research Use Only:

If this setting is enabled, spots excluded by an MOD will not be hidden from the measurement image, but displayed as a "RUO spot".

Require ImageXplorer ConfigXplorer scan: Warn after 60 days, enforce after 90 days.

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Force MOD:
False

Requires a measurement to always have an MOD assigned. If this setting is enabled, a measurement cannot be approved without an MOD assigned.

Hide Global MOD:
False

If this setting is enabled, only the tenant-specific MOD will be available to select, and the global system-wide MOD will be hidden.

Display MOD Excluded Spots As Research Use Only:
True

If this setting is enabled, spots excluded by an MOD will not be hidden from the measurement image, but displayed as a "RUO spot".

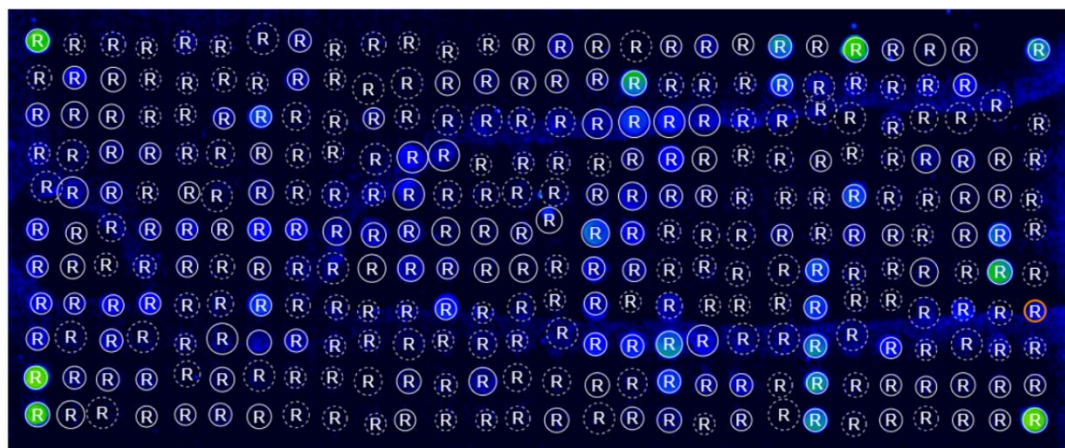
Configuration

Calibration string:

Allows to override the calibration information from the cartridge's QR-Code. Leave empty to use the calibration information from the QR-Code on the cartridge (the usual behaviour).

Research spots:
Show research spots

If enabled, "research use only" spots will be displayed on measurement details and arrays. This does not affect the export behaviour or the reports of measurements.

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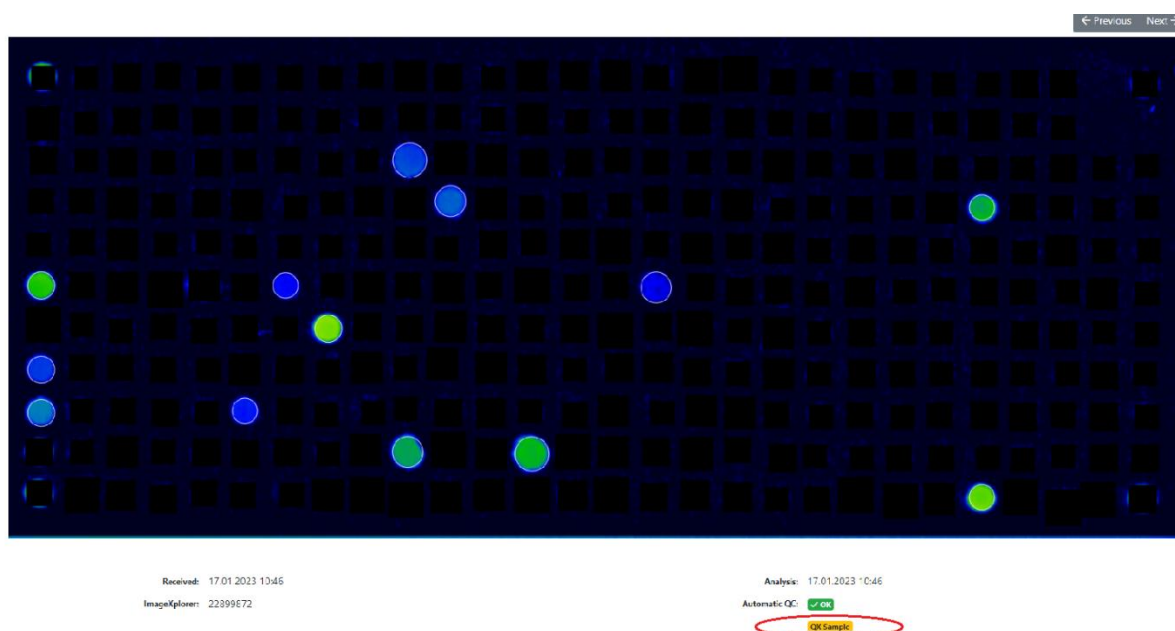
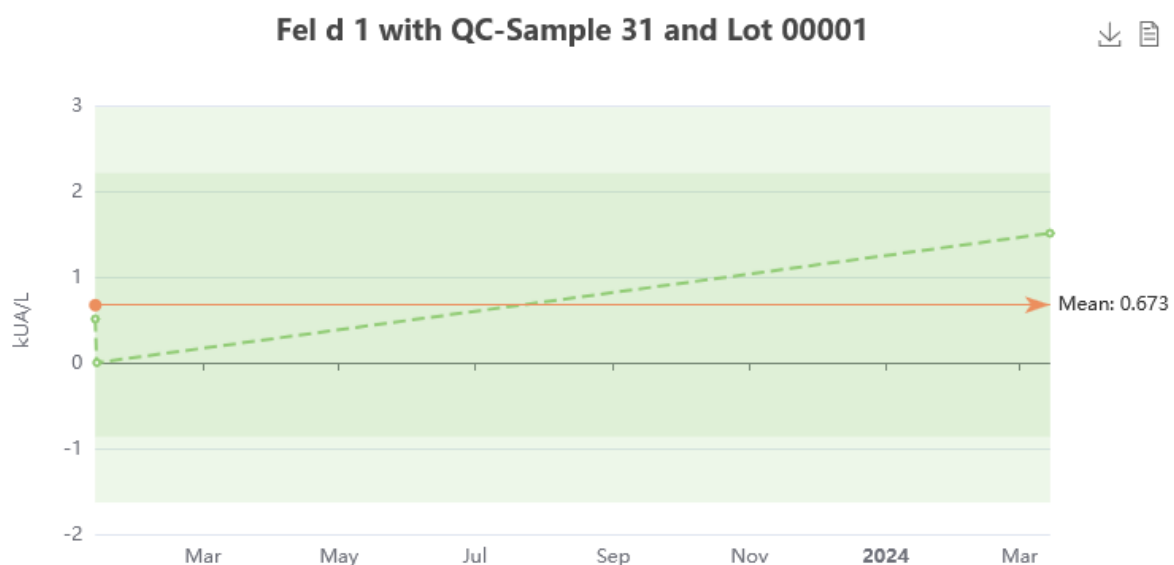
New Measurements

Search by barcode, patient, sample, tag, analysis date, a

Selected measurements ▾

- Download CSV format
- Download Excel format
- Approve
- Update interpretation
- Re-Process
- » Move
- Delete**

Tags	Product type
	FOX
	FOX
	ALEX2
	FOX
	ALEX2
	ALEX1

Delete measurements

You are about to delete the following measurements. Please note that this action is irreversible and cannot be undone.

Barcode	Analysis Date
02AAB160	16.12.2019 13:41

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Plant Food Food	M	Jug r 3	0.40
Plant Food Food	M	Jug r 1	0.38

Plant Food Food	M	Jug r 3	0.40
Plant Food Food	M	Jug r 1	0.38

Delete request scheduled now. [Cancel delete request](#)

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ign in. Can be

Background jobs

View details about running background jobs and manually schedule runs.

[View dashboard](#)

Measurement delete / unapproval log & requests

Show logging and pending requests for measurement delete and unapproval.

[Show log & requests](#)

Measurement unapproval / delete requests

Open requests Log & Finished requests				
Requested UTC	Requesting User (Email)	QR-Code	Request Type	Action
11.10.2023 13:05:28	Test Test (Test@test.com)	02AYQ145	Unapprove	Accept Reject
11.10.2023 12:59:27	Test Test (Test@test.com)	02AYQ14C	Delete	Accept Reject

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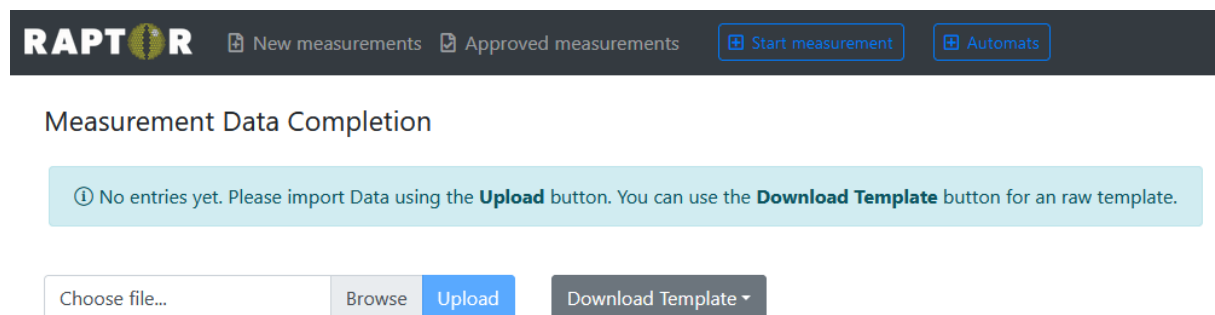
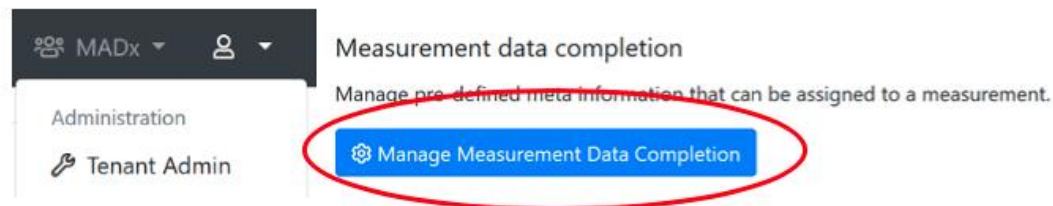
Add new user on Result Sharing Portal

General configuration

First name:

Last name:

E-Mail address:

Status:

Enabled (user can log in)

Username:

Password:

Send an invitation e-mail to user automatically

☒ **Automatically send an email summary in case new measurement(s) have been shared**
This email is sent once per day as a summary, just in case a new measurement was shared.

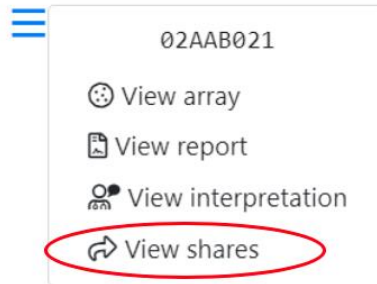
Doctor Info Text:

This information will be printed on a patients report.

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≡ Share

You are about to share a measurement with a user of the User Portal. Please note that the doctor will have access to the results of the measurement. Once you share a measurement you'll be provided with a link to access the shared information via the encrypted User F

Barcode: 02AAB021

Share with: X Dr. (testuser2@macroarraydx.com)

If you can't find a specific user of the User Portal, go to the [tenant administration](#) to configure the list of available users of the User Portal.

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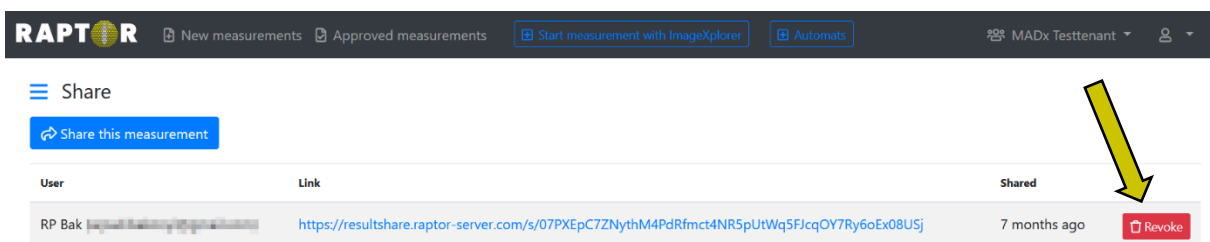
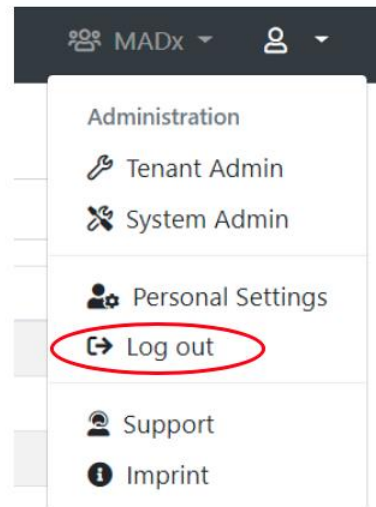
Resultatdelingsportalen kan tilgås via <https://resultshare.raptor-server.com>.

 Result Sharing Portal English (en) ▼

Welcome to the Result Sharing Portal

The Raptor-Server Result Sharing Portal allows users to view shared measurements. You will need a username and password to log in to view all measurements that have been shared with you. Also, a sharing link will allow to view specific shared measurements.

Please enter your username and your password:

Did you forget your password? No problem, you can [request a new one](#).

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Result Sharing Portal
English (en) ▼
Change password
Log out

Measurements

These are the measurements that are available for you. You may view the details of each measurement, or download the report PDF file.

Measurement	Product	Patient	Sample	Date
01AAK239	ALEX1		11852162	7/12/2019
01AAK281	ALEX1		04824821	12/18/2019
01AAK891	ALEX1		66560260	1/28/2020
02AAA015	ALEX2			2/21/2020

Listen over delte målinger kan sorteres efter QR-kode, produkttype, produktnavn, patientnavn, prøve-ID eller dato.

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Measurement 02AYQ143

Product: ALEX2
Patient:
Date of Birth:
Sample:
Info:
Doctor Info:
Sampling Date:
Assay Date: 10/12/2023
Processed Date: 10/12/2023
Approval Date: 10/12/2023

Report Language:

English (United Kingdom) ▼

You can request a new RAVEN interpretation that will be included in the report.

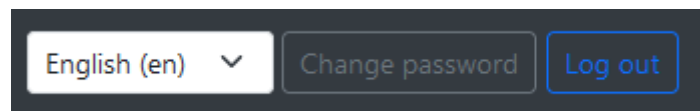
[Request new RAVEN interpretation](#)

You can download a PDF report that includes all information and results of this measurement.

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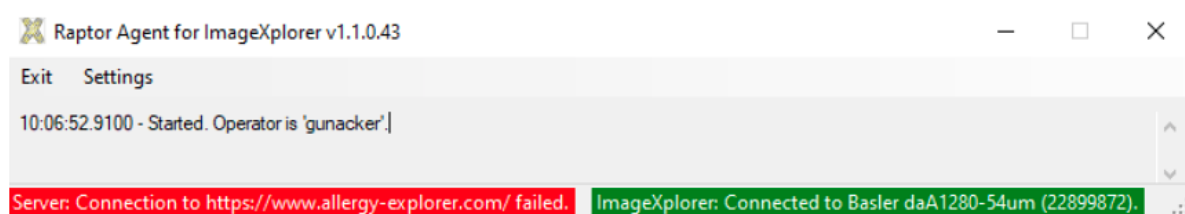
Settings

Please enter the settings for your ImageXplorer. You can find your ImageXplorer key when you log in to Raptor. Please contact support if you're not sure what the proper settings are for your ImageXplorer.

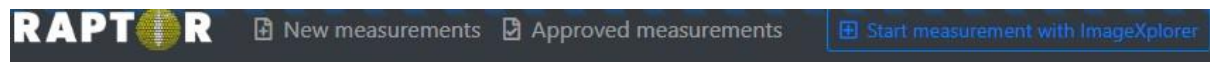
Raptor URL:

ImageXplorer Key:

Continue

Startmåling er ikke mulig:

The ImageXplorer IX training cannot be used, because it requires a ConfigXplorer scan.

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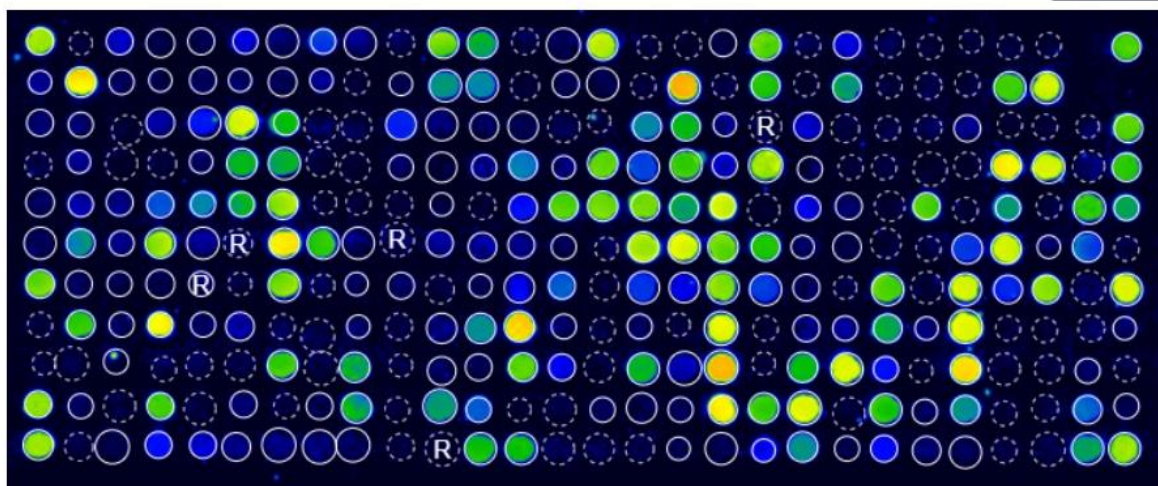
ImageXplorer: 40287249

Automatic QC: X Failed

Analysis Error: Insufficient data transfer - check connection.

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RAPTOR SERVER Analysis Software i kombination med andre MADx-testsæt:

© Copyright by MacroArray Diagnostics

MacroArray Diagnostics (MADx)

Lemböckgasse 59, Top 4

1230 Vienna, Austria

+43 (0)1 865 2573

www.madx.com

Versionsnummer: 22-IFU-01-DA-09 (EN-26)

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