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Welcome to RAPTOR SERVER Analysis Software, the analysis software for the [ALEX multiplex test](#).
Please log in below to continue.

Log in with Azure AD / Office 365

If your organization integrates the software into its Microsoft Azure/Office 365 user directory (single-sign-on) click the button below. You will be redirected to the Microsoft login screen.

 Log in

Log in with Username & password

Please enter your username and password.

Your username

Your password

 Log in

Forgot your password? No problem, you can [request a new one](#).

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Contract approval required

Please review the following documents. You are required to confirm that you have read and agree with each document in order to use RAPTOR SERVER:

☐ **Bitte lesen** (last updated 23.11.2023 18:00)

☐ **RAPTOR SERVER IFU** (last updated 04.12.2023 11:05)

☐ **Terms of Service** (last updated 22.11.2023 01:00)

Confirm

By clicking the button, I confirm that I have reviewed these documents and agree.

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New Measurements					Last 7 days
QC	Product type / Source	QR-Code	Sample Code	Analysis Date	
❌ Failed	Research MAX 45k	99AFG0F7	NK_FOX	30/03/2023 20:50	Details Report
❌ Failed	Research MAX 45k	99AFG0F8	N.G.	30/03/2023 20:50	Details Report
❌ Failed	Research MAX 45k	99AFG0F9	E18735	30/03/2023 20:50	Details Report
❌ Failed	Research MAX 45k	99AFG0FA	20220325w56	30/03/2023 20:50	Details Report
❌ Failed	Research MAX 45k	99AFG0EE	HM100038	30/03/2023 20:50	Details Report
❌ Failed	Research MAX 45k	99AFG04D	786118	30/03/2023 20:37	Details Report
❌ Failed	Research MAX 45k	99AFG016	HM100225	30/03/2023 20:37	Details Report
❌ Failed	Research MAX 45k	99AFG0CA	E17685_M	30/03/2023 20:37	Details Report
❌ Failed	Research MAX 45k	99AFG0CC	E17685_M	30/03/2023 20:37	Details Report
❌ Failed	Research MAX 45k	99AFG015	E17685_F	30/03/2023 20:37	Details Report

In total there are 270 new measurements in the selected time period.

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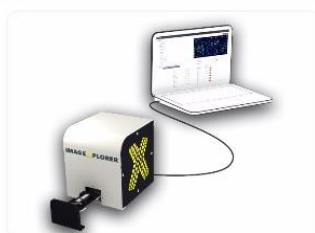
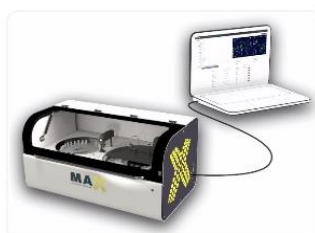
9K-Test-Instrument

Not initialized

[Initialize](#) [Shutdown](#) [Monthly Maintenance](#)

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
The screenshot shows the RAPTOR web application interface. At the top is a dark navigation bar with the RAPTOR logo and links for 'New measurements', 'Approved measurements', 'Start measurement', and 'Automats'. Below this, the 'Personal settings' section contains input fields for 'Name', 'E-Mail address', a dropdown for 'Language' (set to English (en)), a dropdown for 'Culture' (set to English (United States) (en)), and a dropdown for 'Timezone' (set to (UTC+01:00) Amsterdam, Berlin, Bern, Rome, Stockholm, Vienna). To the right, the 'Multi-Factor Authentication' section has a 'Manage MFA' button and explanatory text. Below that, the 'Configuration' section has a 'Research spots' dropdown set to 'Show research spots'.

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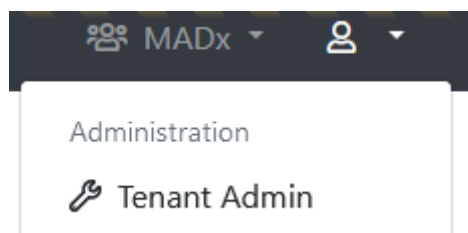Configure "MADx"

General configuration	
* Name:	MADx
Default timezone:	(UTC+01:00) Amsterdam, Berlin, Bern, R...
Configure a timezone to see date and times in (instead of UTC). Please note that users can choose to use a different timezone.	
Default report template (ALEX2):	ALEX2
Default report template (ALEX3, ALEX Air, ALEX Food):	check 1.21
Default report template (FOX):	FOX (2025)
Default report template (PAX):	PAX
Default report language:	English (en)
Default ALEX MOD:	< No MOD >
Default FOX MOD:	< No MOD >
Default PAX MOD:	< No MOD >
Force MOD:	False
Requires a measurement to always have an MOD assigned. If this setting is enabled, a measurement cannot be approved without an MOD assigned.	
Hide Global MOD:	False
If this setting is enabled, only the tenant-specific MOD will be available to select, and the global system-wide MOD will be hidden.	
Display MOD Excluded Spots As Research Use Only:	False
If this setting is enabled, spots excluded by an MOD will not be hidden from the measurement image, but displayed as a "RUO spot".	
MFA:	☐ Force MFA
If this setting is enabled, users are forced to set up an MFA on sign-in, in case they haven't yet.	

MAX device default settings	
* Default Minimum RunId:	10
When a new MAX device is created, this value will be used for its Minimum Run Id	

Tenant-specific batch releases	
Restrict this tenant to only allow the following batches to be scanned or to restrict certain batches. Enter one per line.	
Use:	
☒ Whitelist	
☐ Blacklist	
Whitelisted Batches:	
Enter one batch code including product identifier per line, for example "02AAA"	
Blacklisted Batches:	
Enter one batch code including product identifier per line, for example "02AAA"	

Company Details

* Legal Entity Name:	MacroArray Diagnostics
* Street and Number:	Street 4
* Zip Code:	1230
* City:	New York
* Country:	United States of America
* Phone:	+4318652573
* E-Mail:	office@macroarraydx.com
* Legal Representative:	Dr. Christian Harwanegg
Website:	
Vat Number:	
Company Register Number:	

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Tenant-specific batch releases
Restrict this tenant to only allow the following batches to be scanned or to restrict certain batches. Enter one per line.
Use:
☒ **Whitelist**
☐ **Blacklist**

Whitelisted Batches:

02AAA
02ABC
02BAA

Enter one batch code including product identifier per line, for example "02AAA"

Blacklisted Batches:

Enter one batch code including product identifier per line, for example "02AAA"

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You requested a new measurement from ImageExplorer "IX Trainingsraum". Please wait until the measurement is completed ...

Invalid batch detected! The measurement can't be processed because its batch is blocked.

Hvis en batch er hvitelistet, vil RAPTOR SERVER kun tillate skanning av batcher med denne koden.

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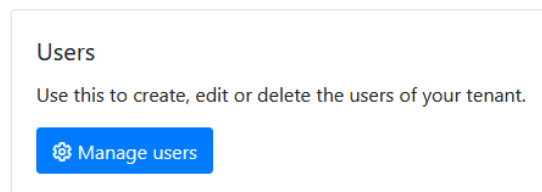
Login Credentials

Type:

Please select if the user should be validated against Azure AD (Office 365), or if you want to specify username and password yourself. It is recommended to use Azure AD whenever possible. This allows for increased security by supporting two-factor authentication. Also, it is more convenient for the user, because he or she does not have to remember a new username and password (single-sign-on).

Username:

Password:

It is a good idea to use the users e-mail-address as his username, because the username needs to be unique across the entire system.

Access Rights

Preconfigured groups

Use these pre-configured groups to automatically assign a set of access rights to this user. You can still add or remove specific access rights.

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Tenants

This user is connected to the following tenants and has the following access rights:

☐ Tenants	Can share measurements	Can edit measurements	Can edit personal data	Can approve measurements	Can manage basic tenant configuration	Can manage API keys	Can manage ImageXplorers	Can view agent connections	Can manage users	Can manage users of Result Sharing Portal	Can manage report templates	Can manage measurement data completion	Can unapprove tenant specific batch releases	System Administration	Can use a custom calibration string	Can manage Run Configuration	Can skip
☐ Another MADx Tenant	☒	☒	☒	☒	☒	☒	☒	☒	☒	☒	☒	☒	☒	☒	☒	☒	☒
☐ MADx Tenant	☒	☒	☒	☒	☒	☒	☒	☒	☒	☒	☒	☒	☒	☒	☒	☒	☒

Additionally this user can be added to the following tenants by checking the tenant name and hitting save afterwards. User rights w applied from current open tenant user.

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Additionally this user can be added to the following tenants by checking the tenant name and hitting save afterwards. User right applied from current open tenant user.

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Tenants

This user is connected to the following tenants and has the following access rights:

 Edit selected tenants rights

☐ Tenants	Can share measurements	Can edit measurements	Can edit personal data	Can approve measurements	Can manage basic tenant configuration	Can manage API keys	Can manage Imagey
☒ Another MADx Tenant	☒	☒	☒	☒	☒	☒	☒

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The screenshot shows the MADx interface. On the left, a sidebar menu has 'Administration' and 'Tenant Admin' (with a key icon). The main area is titled 'ImageXplorers' and contains the text: 'Use this to configure ImageXplorers, or add new ones. Every ImageXplorer needs to be known to Raptor Server, otherwise it won't be able to connect.' Below this text, the 'Manage ImageXplorers' button is circled in red.

Add new ImageXplorer

General information

Name:

Key:

This key is required when installing an Agent.

ImageXplorer Serialnumber:

Serialnumber of connected ImageXplorer.

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Analysis Log

```
'PreProcessedPixels' took 128ms.
Using definition '02_ALEXII_IVD_v1.6.0'.
'ApplyArrayDefinition' took 74ms.
Parsing calibration string ...
'ApplyDefinitions' took 75ms.
'FindGrid' took 1ms.
'AlignFeatures' took 862ms.
'CropImage' took 1ms.
'QuantifyFeatures' took 231ms.
'SubtractBackground' took 601ms.
Parsing calibration string ...
Using calibrators from QR-Code.
Total Igt (tIgt) normalized by calculating 0,330999755859375E-07x²+0,05697999954223633x+-38,69269921875
(x=8515), which results in 515.
'ReQuantify' took 57ms.
```

QC Log

These are the values used for QC (ALEX v2): Curve Correction=0,9997370867894242, tIgt=8515, GD1=28985 (SNR=105.38), GD2=29369 (SNR=185.28), GD3=28569 (SNR=147.53), GD4=28385 (SNR=128.1). Reference values are as follows: Guide Dots must have a value above 5000; each Guide Dot spot must exist and be unique; Guide Dot spot must not be set to bad. Average background must be between 4000 and 16000. QC (ALEX v2) succeeded.


Background Log

Calculated background statistics for 164 pixels (from a total of 182, skipped smallest 9, and highest 9 pixels). The average is 9,419, the minimum 7973 and the maximum 11792 (minimum and maximum is calculated including skipped pixels). This results in an average difference of 628, which is an SD of 49,17 and a CV of 0,52%. For comparison, there is an overall total of 282944 pixels. The overall average is 14,871, the minimum 6,012 and the maximum 64,965.

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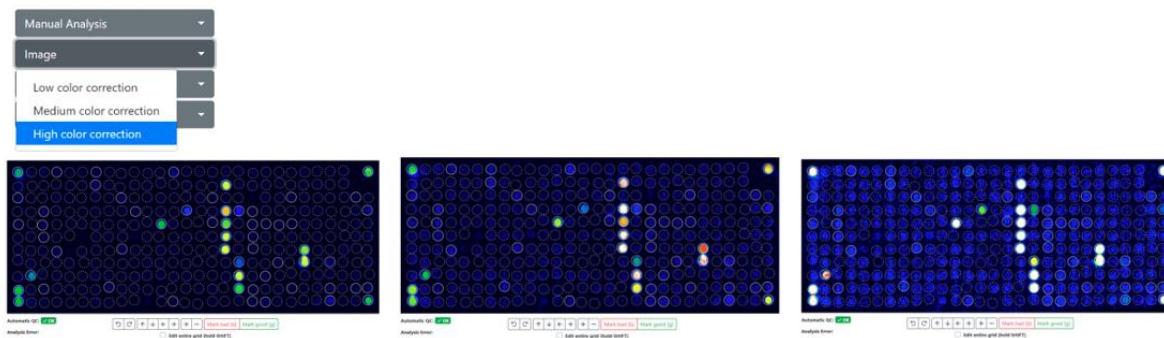
Tissue & Source		All allergens		Only positive allergens		kUA/L
Pollen	Inhalation	E	Aca m			≤ 0.10
Mites		E	Aca s			7.11

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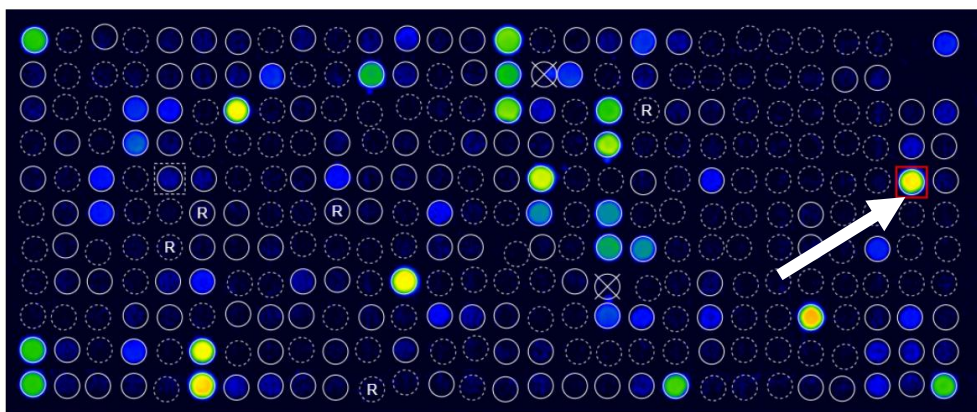New Measurements

Search by QR-Code, patient, sample, tag, analysis date, approval date ...

	Source	QC	Tags	Product type	QR Code	Sample Code	Patient Code	Patient Name	Analysis Date
☐	IX	OK		ALEX2	02AGT957				29.04.2021 10:51
☐	IX	OK		ALEX2	02AGT956				29.04.2021 10:55
☐	IX	OK		ALEX2	02AGT955				29.04.2021 10:50
☐	IX	OK		ALEX2	02AGT954				29.04.2021 10:51
☐	IX	OK		ALEX2	02AGT953				29.04.2021 10:51
☐	IX	OK		ALEX2	02AGT952				29.04.2021 10:50
☐	IX	OK		ALEX2	02AGT951				29.04.2021 10:53
☐	IX	OK		ALEX2	02AGT950				29.04.2021 10:56
☐	IX	OK		ALEX2	02AGT949				29.04.2021 10:56

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New Measurements

Search by QR-Code, patient, sample, tag, analysis date, approval date ...				
☐	Source	QC	Tags	Product type
☐	IX	✓ OK		ALEX2
☒	IX	✓ OK		ALEX2
☒	IX	✓ OK		ALEX2
☒	IX	✓ OK		ALEX2

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New Measurements

Search by QR-Code, patient	
Selected measurements ▾	
☐	IX
☒	IX
☒	IX
☒	IX

- Download CSV format
- Download Excel format
- Approve
- Update interpretation
- Choose Reporttemplate
- Modify Tags
- Re-Process**
- » Move
- Delete

Re-process measurements

You are about to re-process the following measurements. undone, the existing analysis result will be irreversibly over

QR-Code	Sample Code
02APK3C8	Dummy8
02APK258	Dummy1
02APK07F	Dummy13

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New Measurements

	Source	QC	Tags	Product type
☐	Recaptured	✓ OK	recaptured	ALEX1
☐	⚠ Recaptured	✓ OK	recaptured	ALEX1
☐	QR-Code was scanned more than once.		recaptured	ALEX1

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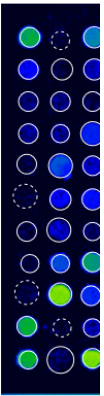
Approve measurements

You are about to approve the following measurements.

There are measurements with one or more warnings. Please make sure to double-check these measurements before approval.

QR-Code	Sample Code	Analysis Date
02AKV035		28.04.2022 11:14

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New measurements
Approved measurements
Start m

Approved Measurements

Search by QR-Code, patient, sample, tag, analysis date ...

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New measurements
Approved measurements
Start measurement
Automats
MADx Testtenant

Exports

Certain types of data exports (for example, the export of many PDF reports at once) take a few minutes to prepare for download. Here download your requested exports. Please note that you can keep working, all exports are done in the background and you can download when they are completed.

Export requested	# of measurements included	Export completed
8/16/2021 11:07 AM	4	0 / 4
8/16/2021 11:06 AM	4	8/16/2021 11:06 AM
8/16/2021 11:04 AM	3	8/16/2021 11:04 AM

Download
Download
Download

Administration
Tenant Admin
Exports
Personal Settings
Log out
Support
Imprint
Download
Download
Download

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The screenshot shows the MADx Tenant interface. On the left, a sidebar menu is open with the 'Exports' option highlighted by a yellow circle. The main content area is titled 'Exports' and contains a blue button labeled 'Export multiple measurements', also highlighted by a yellow circle. Below the button, a table shows the status of an export request.

Export requested	# of measurements included
06.07.2022 08:52	3

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The screenshot shows the 'Export multiple measurements' form. It includes a yellow warning box stating: 'Please note that **Report** and **ASTM** export is only available'. The form contains several input fields: 'Product Type' (set to ALEX2), 'Analysis Date From' (with a calendar icon), 'Tag' (with a placeholder 'use , to separate tags'), 'QR Code', 'Analysis Date', and 'Approved or n'. There are two checkboxes: 'Export CSV' and 'Export Exc'. At the bottom, there is a blue button labeled 'Search Measurements' (highlighted with a yellow circle) and a grey button labeled 'Cancel'.

Export multiple measurements

Please note that **Report** and **ASTM** export is only available for certain product types.

Product Type

ALEX2

QR Code

Analysis Date From

TT . MM . JJJJ

Analysis Date To

TT . MM . JJJJ

Tag

Approved by

Approved by

use , to separate tags

☐ Export CSV

☐ Export Report

 Start export ...

← Back

122 Measurements found that match your criteria

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Edit report template

Your changes have been saved successfully.

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Configure report template "New report template"

General	Design	Header	Content
Name: New report template This is the name of the template which shows up in the dropdown list while generating a report. Base Template: Version 2 (ALEX2 only) A base template which specifies basic functionality must be chosen. Preferred language: English (United States) (en-us) A default language can be specified which will be used if there isn't yet a language stored to a measurement's report.			

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PDF naming

Setting Type: Manual

The filename of the report PDF can be customized. Either built automatically based on your selection below or manually adjusted in the order of your needs.

Name Pattern: Report_{PatientId}_{MeasurementQrCode}

Available Elements: {PatientId} ; {PatientName} ; {SampleId} ; {MeasurementQrCode} ; {DateAndTime}

PDF naming

Setting Type: Automatic

The filename of the report PDF can be customized. Either built automatically based on your selection below or manually adjusted in the order of your needs.

Include Patient ID: ☒

Include Patient Name: ☒

Include Sample ID: ☐

Include Measurement QR-Code: ☐

Include Date & Time: ☒

Footer

Footer text:

Will be printed in the footer if provided and rendered as HTML.

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Configure report template "New report template"

The screenshot shows the 'Design' tab of a configuration window. It contains three main settings:

- PDF format:** Two radio buttons, 'A4' (selected) and 'Letter'. A note on the right states: 'Specifies the paper size of the output PDF.'
- Font:** A dropdown menu showing 'Arial'. A note on the right states: 'The font family of the whole report texts can be adjusted.'
- Background image:** A text input field followed by a 'Browse' button. A note on the right states: 'If set, this image is used as background on the first page of the PDF.' Below the input field, it says 'Maximum file size 1 MB'.

- Fargepalett: Grunnfargene i malen og resultatklassene kan tilpasses over hele fargepaletten
- Logoer: I tillegg til logoene på begge sider av forsidenes topptekst, kan det legges til en miniatyrlogo i toppteksten på alle sider. Som standard er dette en miniatyr av logoen på venstre side.
- Signatur: I tillegg kan et bilde av en signatur lastes opp og legges til på slutten av rapporten.
- Egendefinert intensitetsindikator (en egendefinert SVG for visning av allergenintensitet kan legges til)

Color settings

Color palette: Palette 1

Color 1:

Color 2:

Color 3:

Color 4:

First level headline color:

Second level headline color:

Third level headline color:

Logos

Logo (left): Browse

Maximum file size 1 MB

This logo will appear on the **top left corner** of the first page of the report.

Text (center):

This text will appear either on the left, right, or center of the top of the first page (depending on your configured logos). Supports the placeholders `{{ PATIENT_PORTAL_QRCODE }}` and `{{ PATIENT_PORTAL_URL }}` to dynamically insert measurement information.

Logo (right): Browse

Maximum file size 1 MB

This logo will appear on the **top right corner** of the first page of the report.

Small logo: Browse

This logo is used on all detail pages. If not set, the default logo is used.

Signatures

Left: Browse

Maximum file size 1 MB

This signature or image will appear on the last page, left aligned.

Center: Browse

Maximum file size 1 MB

This signature or image will appear on the last page, center aligned.

Right: Browse

Maximum file size 1 MB

This signature or image will appear on the last page, right aligned.

Indicators

Custom intensity indicator:

A custom SVG for displaying allergen intensity can be pasted here. Will be styled according to specified colors.

⚠ The SVG element must contain the attributes **width** and **height** with a minimum dimension of 16 for correct display in PDF.

💡 SVGs paths which should be colored must require the **colorize** class as well as a fill defined in the **defs** segment of the SVG.

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Configure report template "New report template"

Header content:	Left half	Right half
	Custom headline	Custom headline
Patient Name	☐	Doctor Information
Patient Name	☒	None
Custom headline		
Date of Birth	☐	None
Sample ID	☐	None
QR-Code	☐	Additional Info text
Analysis Date	☐	None
Amount of tested allergens	☐	None
Test method	☐	Quality Control text

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General	Design	Header	Content
Allergens to include:	All allergens	Specifies which kind of allergens should be included in the report.	
Allergens sort order:	Alphabetically by allergen	Specifies the sort order of the allergens.	
Include Overview:	☒	Whether the allergen overview should be printed or not.	
Include product type in overview:	☒	Whether the product type should be printed in the header of allergen overview.	
Include Total IgE:	On the front page	Whether the information of total IgE should be included in the report and where.	
Cross-reactive information:	Include text	This section will be displayed underneath the allergen list itself.	
Include Interpretation:	Include manually	It can be specified if the Raven report interpretation information should be included in the report or not. This section is printed underneath the allergens overview.	

Additional information (below the list of results)		Those informations are integrated into the report in an extra section underneath the allergen list.
Approval date:	☐	
Assay date:	☐	
Doctor info:	☐	The information about the doctor which accompanied the measurement/patient.
Print date:	☒	When the report was generated.
Report info:	☐	The „additional info“ field belonging to the measurement.
Sample date:	☒	

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Configure report template "New report template - Duplicate"

General	Design	Header	Content
Name:	New report template - Duplicate		
Support products:	ALEX ²		
Preferred language:	Deutsch (de)		

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General configuration

*** Name:** MADx

Default timezone: (UTC+01:00) Amsterdam, Berlin, Bern, Rome, Stockholm, Vienna
Configure a timezone to see date and times in (instead of UTC). Please note that users can choose to use a different timezone.

Default report template (ALEX 1): Version 1 (ALEX1 & ALEX2)

Default report template (ALEX 2): Version 2 (ALEX2 only)

Default report language: English (United Kingdom) (en-gb)

Default MOD: < No MOD >

Require ImageXplorer testarray scan: Warn after 60 days, enforce after 90 days.

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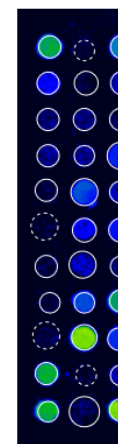
Tags:

MOD:

Report Template: Version 2 (ALEX2 only) - Deutsch (de)

Re
ImageX
Analysi

Deretter ser du en forhåndsvisning av rapporten, der du kan velge en rapportmal (standard eller tilpasset) og angi standardspråk. Det er imidlertid ikke mulig å laste ned denne PDF-rapporten. For dette formålet er det nødvendig å godkjenne målingen først.

Raptor

Report

Template: Version 2 (ALEX2 only)

Language: English (en)

Usually this tenant's default template and language will be used when sharing or exporting measurements. You may change the template and language associated with this measurement by saving your current configuration.

You can download this report in PDF format once the measurement has been approved.

You have access to manage the report templates and settings via the tenant administration. Please note that any changes will affect all users.

MAD X

PATIENT ID
PATIENT NAME
DATE OF BIRTH
SAMPLE ID
LAB CODE
LAB CODE
ANALYSED ON
TESTED ALLERGENS
205
TEST METHOD CODE
ALEX2

REFERRING PHYSICIAN

NOTE
The internal QC (Feasibility check for QC) was within acceptance range.

Lab report: Summary on detectable sensitisations

POLLEN
Grass Pollen

MICROORGANISMS
Fungal Spores & Yeast

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The screenshot shows the RAPTOR web interface. At the top, there's a navigation bar with 'New measurements' and 'Approved measurements' tabs. Below, the 'Approved Measurements' section has a search bar and a 'Download selected measurements' dropdown menu. The dropdown menu is open, showing options: 'Report (PDF format)' (highlighted with a yellow circle), 'CSV format', 'Excel format', 'ASTM format', 'Update interpretation', 'Choose Reporttemplate', 'Share', 'Move', and 'Unapprove'. Below the menu is a table of measurements.

	Source	Tags	Product type	QR-Code	Sample C
☒	IX		ALEX2	Ø2AKV39D	
☒	IX		ALEX2	Ø2AKV39D	
☒	IX		ALEX2	Ø2AKV39D	

Hvis du klikker på en måling, kommer du til detaljområdet for denne målingen. Der kan du også vise en forhåndsvisning av rapporten og endre innstillingene for rapportens overskrift og språk. Deretter kan du laste ned rapporten direkte derfra.

Details

QR-Code: Ø2AKV39D

Sample Code:

Requisition Code:

Sampling Date:

Assay Date: 27.09.2021

Patient Code:

Patient Name:

Date of Birth:

Info:

Doctor Info:

Analysis: 27.09.2021 15:39

Approved: 27.09.2021 15:39

ImageExplorer: 23167844

Tags:

MOD:

Report Template: Version 2 (ALEX2 only) - Deutsch (de)

[New measurements](#)
[Approved measurements](#)
[Start measurement](#)
[Automatic](#)

Report

Template: Default

Language: English

[Save](#)
[Use defaults](#)

Usually this tenants default template and language will be used when sharing or exporting measurements. You may change the template and language associated with this measurement by saving your current configuration.

[Download PDF](#)

[Manage report template and settings](#)

You have access to manage the report templates and settings via the tenant administration. Please note that any changes will affect all users.

Patient ID:	
Patient Name:	
Date of Birth:	
Sample ID:	
Barcode:	02AAE
Sampled on:	
Tested on:	12/18/
Printed on:	12/19/
Note:	The internal QC (Plausibility)

Lab report: Summary on de


Pollen	Grass Pollen
	Tree Pollen
	Weed Pollen
Mites	House Dust M
Microorganisms	Fungal Spores
	Legumes
	Grain
Plant-Based Food	Spices
	Fruits
	Vegetables
	Nuts & Seeds
	Milk

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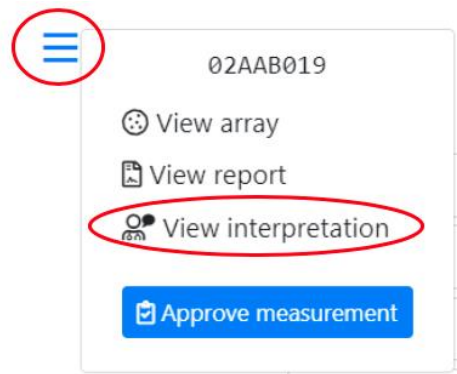Rule set

RAVEN Ruleset A for ALEX

A rule set defines the RAVEN or RAVEN² engine that is used to interpret results.

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AIT

Medication

MADx data declaration for smartRaven

☒ I accept to enter clinical data for using the full interpretation function of smartRAVEN

Patient has reported any symptoms

☐ Yes ☐ No

Additional information about the patient

☐ Atopic Dermatitis

☐ Allergen-specific immunotherapy (AIT) - ongoing, completed or discontinued

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[Request new interpretation](#)

This measurement already has an interpretation stored. Usually, it should not be necessary to request those again, but it could be necessary when the patients information has changed.

Interpretation

Raven Interpretation Summary

Sample Information

The sample was tested on ALEX² Barcode 02AAB019, interpretation date 12/19/2019.

Of the tested 295 allergens, 25 were above the detection limit of 0.3 kU_L/L. A sensitization can be an indicator of an allergy. For selected allergens, comments for interpretation are listed below.

Total IgE: 512 kU/L

The measured total IgE was 512 kU/L. Above a total IgE of 100 kU/L, allergy is likely.

Cross-Reactive Allergen Sensitization detected

Sensitization against allergen molecules which are markers of (broad) cross-reactivity between different allergen sources.

Detected cross-reactive allergen sensitizations:

- PR-10s: Aln g 1, Bet v 1, Cor a 1.0103, Fag s 1
- Polcalcins: Aln g 4, Phl p 7

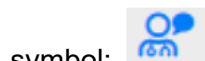
PR-10 Proteins (PR10)

PR-10 inhalative: The major birch pollen allergen, Bet v 1, represents the prototype of all PR-10 allergens and is the primary sensitizer in regions with birch pollen exposure. The presence of PR-10 allergens in Fagales tree pollen explains IgE cross-reactivity between pollen from hazel, alder, beech, oak and hornbeam. PR-10 nutritive: PR-10 allergens in raw fruits, nuts, vegetable and legumes can induce oral allergy syndrome and sometimes severe allergic reactions in sensitized individuals. PR-10 allergens are not stable to heat and digestion.

Polcalcins (Ca++ binding proteins, PC)


Members of the PC family can cause inhalative symptoms. PC allergens can be found in pollen from grasses, trees and weeds. Inhalative symptoms manifest as allergic rhino-conjunctivitis and/or allergic asthma.

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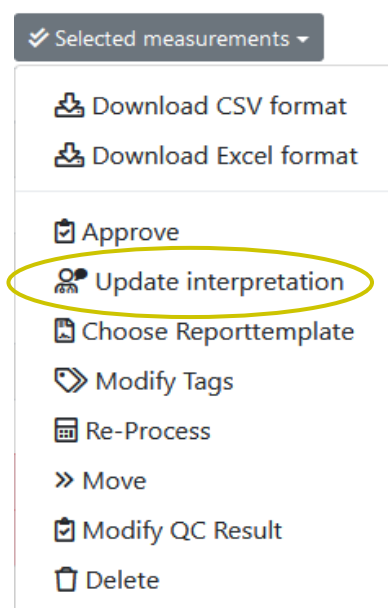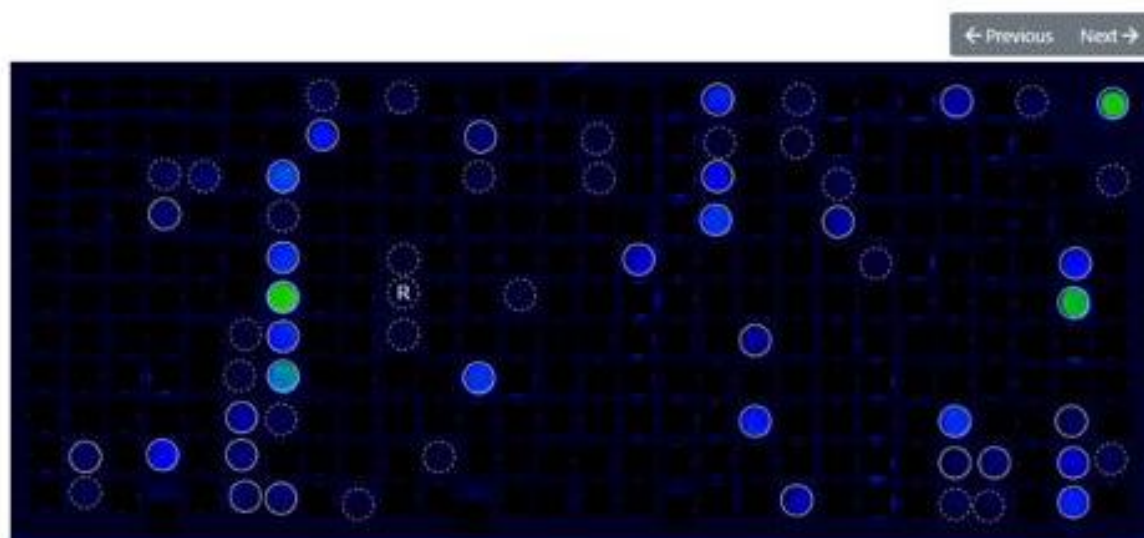
Tags:

MOD:

Report Template: Version 2 (ALEX2 only) - English (United States) (en-us)

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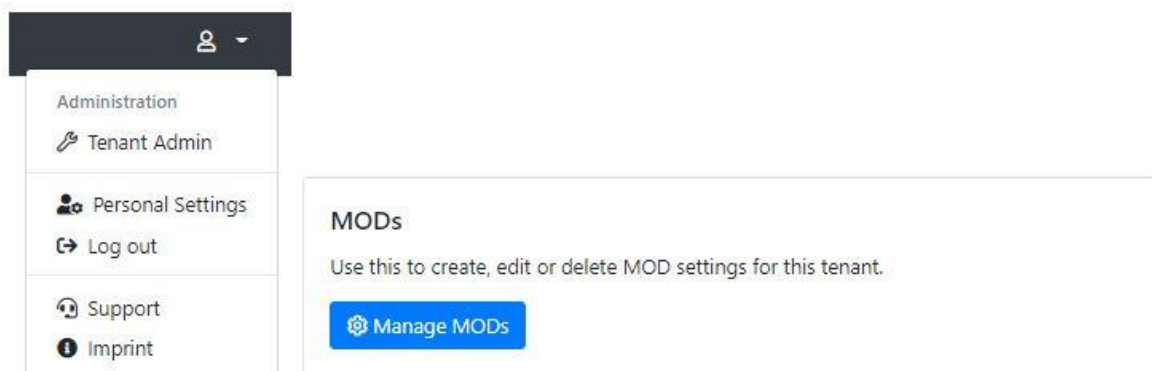
Raptor New measurements Approved measurements Start measurement Automats

Create new MOD

Name:

Type:

Deactivated:

You cannot delete an MOD once it is in use. But you can deactivate it, so that it's not available for new measurements anymore.

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* Name:

Default timezone:

Configure a timezone to see date and times in (instead of UTC). Please note that users can choose to use a different timezone.

Default report template (ALEX 1):

Default report template (ALEX 2):

Default report template (FOX):

Default report template (PAX):

Default report language:

Default ALEX2 MOD:

Default FOX MOD:

Default PAX MOD:

Force MOD:

Requires a measurement to always have an MOD assigned. If this setting is enabled, a measurement cannot be approved without an MOD assigned.

Hide Global MOD:

If this setting is enabled, only the tenant-specific MOD will be available to select, and the global system-wide MOD will be hidden.

Display MOD Excluded Spots As Research Use Only:

If this setting is enabled, spots excluded by an MOD will not be hidden from the measurement image, but displayed as a "RUO spot".

Require ImageXplorer ConfigXplorer scan: Warn after 60 days, enforce after 90 days.

- Force MOD: Hvis denne innstillingen er true, må en MOD velges når en måling lagres eller godkjennes.

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
Force MOD:	False	Requires a measurement to always have an MOD assigned. If this setting is enabled, a measurement cannot be approved without an MOD assigned.
Hide Global MOD:	False	If this setting is enabled, only the tenant-specific MOD will be available to select, and the global system-wide MOD will be hidden.
Display MOD Excluded Spots As Research Use Only:	True	If this setting is enabled, spots excluded by an MOD will not be hidden from the measurement image, but displayed as a "RUO spot".

Configuration	
Calibration string:	
	Allows to override the calibration information from the cartridge's QR-Code. Leave empty to use the calibration information from the QR-Code on the cartridge (the usual behaviour).
Research spots:	Show research spots
	If enabled, "research use only" spots will be displayed on measurement details and arrays. This does not affect the export behaviour or the reports of measurements.

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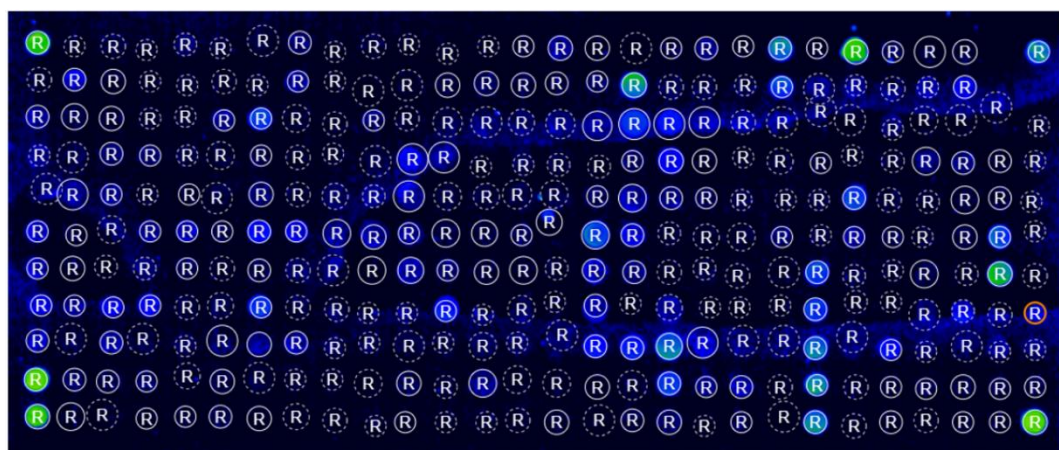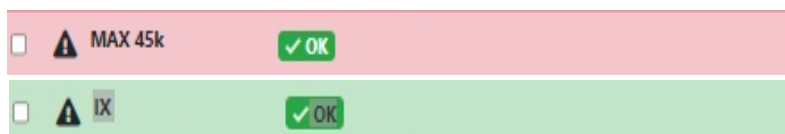
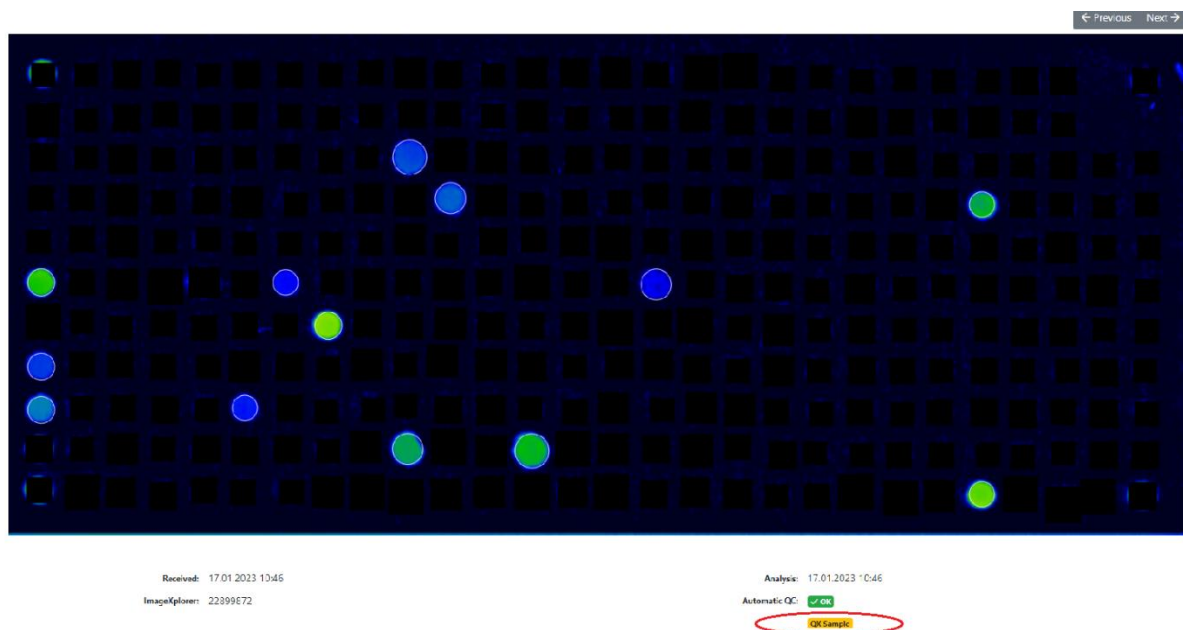
New Measurements

Search by barcode, patient, sample, tag, analysis date, a

Selected measurements ▾

- Download CSV format
- Download Excel format
- Approve
- Update interpretation
- Re-Process
- » Move
- Delete**

Tags	Product type
	FOX
	FOX
	ALEX2
	FOX
	ALEX2
	ALEX1

Delete measurements

You are about to delete the following measurements. Please note that this action is irreversible and cannot be undone.

Barcode	Analysis Date
02AAB160	16.12.2019 13:41

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Plant Food Food	M	Jug r 3	0.40
Plant Food Food	M	Jug r 1	0.38

Plant Food	M	Jug r 3	0.40
Food			
Plant Food	M	Jug r 1	0.38
Food			

Delete request scheduled now. [Cancel delete request](#)

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ign in. Can be

Background jobs

View details about running background jobs and manually schedule runs.

[View dashboard](#)

Measurement delete / unapproval log & requests

Show logging and pending requests for measurement delete and unapproval.

[Show log & requests](#)

Measurement unapproval / delete requests

Open requests Log & Finished requests				
Requested UTC	Requesting User (Email)	QR-Code	Request Type	Action
11.10.2023 13:05:28	Test Test (Test@test.com)	02AYQ145	Unapprove	Accept Reject
11.10.2023 12:59:27	Test Test (Test@test.com)	02AYQ14C	Delete	Accept Reject

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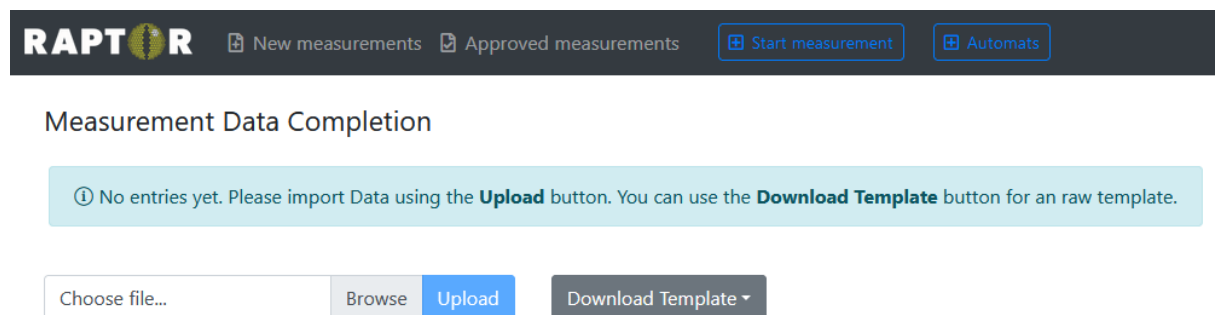
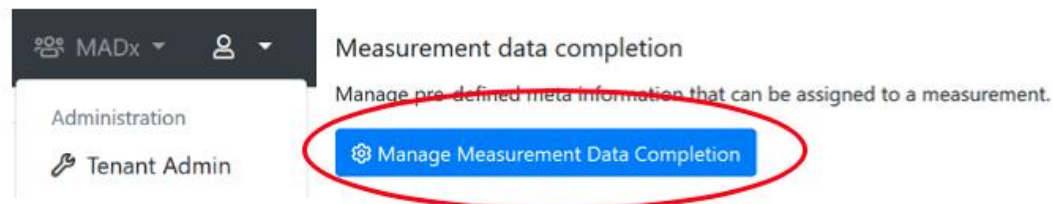
Add new user on Result Sharing Portal

General configuration

First name:

Last name:

E-Mail address:

Username:

Password:

Send an invitation e-mail to user automatically

Status:

Locked (user cannot log in)

Enabled (user can log in)

Locked (user cannot log in)

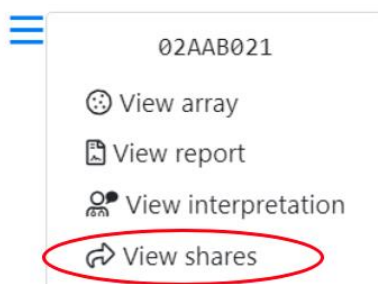
Doctor Info Text:

It is a good idea to use the users e-mail-address as username as well.

This information will be printed on a patients report.

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≡ Share

You are about to share a measurement with a user of the User Portal. Please note that the doctor will have access to the results of the measurement. Once you share a measurement you'll be provided with a link to access the shared information via the encrypted User F

Barcode: 02AAB021

Share with: X Dr. (testuser2@macroarraydx.com)

If you can't find a specific user of the User Portal, go to the [tenant administration](#) to configure the list of available users of the User Portal.

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User	Link	Shared
RP Bak	https://resultshare.raptor-server.com/s/07PXEpC7ZNytm4PdRfmc4NR5pUtWq5FJcqOY7Ry6oEx08USj	7 months ago Revoke

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- Administration
 - Tenant Admin
 - System Admin
- Personal Settings
- Log out**
- Support
- Imprint

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The screenshot shows the 'Result Sharing Portal' header with a language dropdown set to 'English (en)'. Below the header, a welcome message states: 'Welcome to the Result Sharing Portal. The Raptor-Server Result Sharing Portal allows users to view shared measurements. You will need a username and password to log in to view all measurements that have been shared with you. Also, a sharing link will allow to view specific shared measurements.' The login form includes a prompt 'Please enter your username and your password:', a 'Your username' input field, a 'Your password' input field with an eye icon, and a blue 'Log in' button. A link 'request a new one' is provided for users who forgot their password.

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This version of the login form has annotations. The prompt 'Please enter your username and your password:' is followed by a 'Your username' input field. The password field is labeled 'password' and has an eye icon with a diagonal line through it, indicating it is visible. A blue 'Log in' button is at the bottom.

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Log out

Measurements

These are the measurements that are available for you. You may view the details of each measurement, or download the report PDF file.

Measurement	Product	Patient	Sample	Date
01AAK239	ALEX1		11852162	7/12/2019
01AAK281	ALEX1		04824821	12/18/2019
01AAK891	ALEX1		66560260	1/28/2020
02AAA015	ALEX2			2/21/2020

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Measurement 02AYQ143

Product: ALEX2
Patient:
Date of Birth:
Sample:
Info:
Doctor Info:
Sampling Date:
Assay Date: 10/12/2023
Processed Date: 10/12/2023
Approval Date: 10/12/2023

Report Language:

English (United Kingdom) ▼

You can request a new RAVEN interpretation that will be included in the report.

[Request new RAVEN interpretation](#)

You can download a PDF report that includes all information and results of this measurement.

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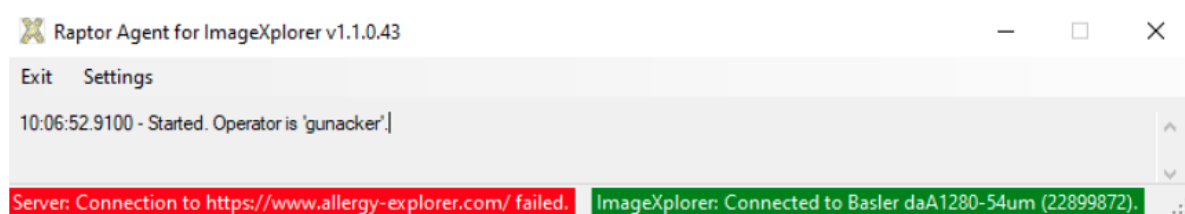
Settings

Please enter the settings for your ImageXplorer. You can find your ImageXplorer key when you log in to Raptor. Please contact support if you're not sure what the proper settings are for your ImageXplorer.

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Raptor Agent for ImageXplorer v1.1.0.43

Exit Settings Open Raptor

10:06:52.9100 - Started. Operator is 'gunacker'.
10:11:33.9399 - Disconnecting from server ...
10:11:35.0321 - Server confirms successful authentication to tenant MADx.]

Server: Connected to MADx. ImageXplorer: Connection failed.

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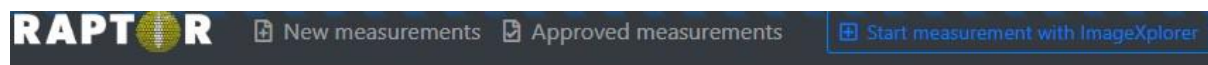
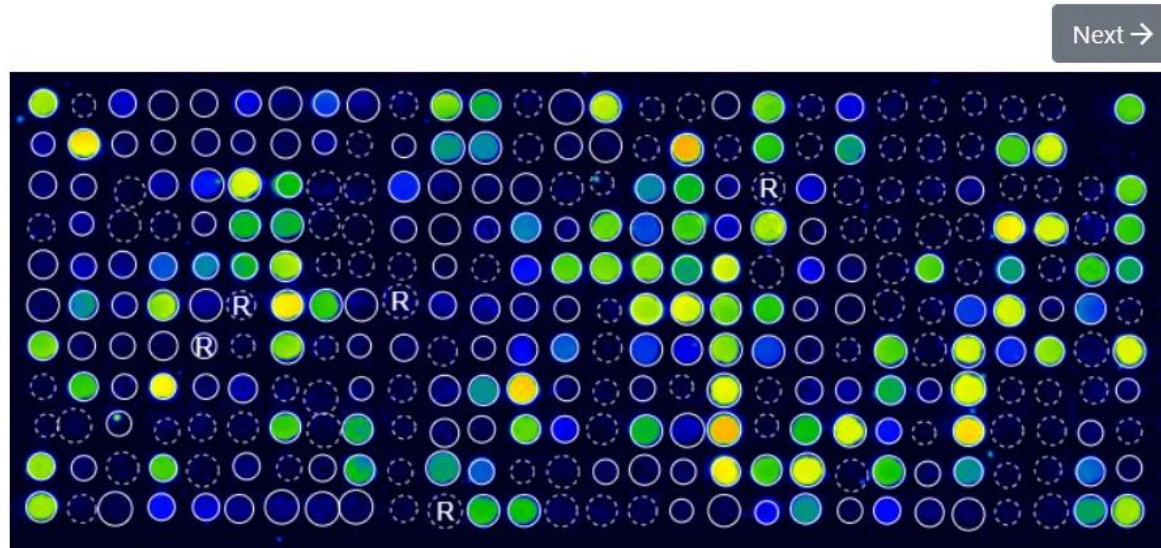
ImageXplorer: 40287249

Automatic QC: X Failed

Analysis Error: Insufficient data transfer - check connection.

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© Copyright by MacroArray Diagnostics

MacroArray Diagnostics (MADx)

Lemböckgasse 59, Top 4

1230 Vienna, Austria

+43 (0)1 865 2573

www.madx.com

Versjonsnummer: 22-IFU-01-NO-05 (EN-25)

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