RAPTOR

SERVER ANALYSIS SOFTWARE

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INDHOLDSFORTEGNELSE

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Welcome to RAPTOR SERVER Analysis Software, the analysis software for the [ALEX multiplex test](#). Please log in below to continue.

Log in with Azure AD / Office 365	Log in with Username & password
If your organization integrates the software into its Microsoft Azure/Office 365 user directory (single-sign-on) click the button below. You will be redirected to the Microsoft login screen.  Log in	Please enter your username and password. Your username Your password   Log in Forgot your password? No problem, you can request a new one.

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Log in with Username & password

Please enter your username and password.

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Contract approval required

Please review the following documents. You are required to confirm that you have read and agree with each document in order to use RAPTOR SERVER:

☐ **Bitte lesen** (last updated 23.11.2023 18:00)

☐ **RAPTOR SERVER IFU** (last updated 04.12.2023 11:05)

☐ **Terms of Service** (last updated 22.11.2023 01:00)

Confirm

By clicking the button, I confirm that I have reviewed these documents and agree.

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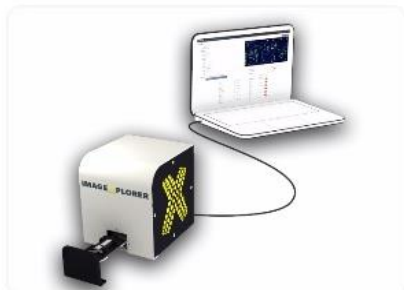
New Measurements					Last 7 days	
QC	Product type / Source	QR-Code	Sample Code	Analysis Date		
✖ Failed	Research MAX 45k	99AFG0F7	NK_F0X	30/03/2023 20:50	Details	Report
✖ Failed	Research MAX 45k	99AFG0F8	N.G.	30/03/2023 20:50	Details	Report
✖ Failed	Research MAX 45k	99AFG0F9	E18735	30/03/2023 20:50	Details	Report
✖ Failed	Research MAX 45k	99AFG0FA	20220325w56	30/03/2023 20:50	Details	Report
✖ Failed	Research MAX 45k	99AFG0EE	HM100038	30/03/2023 20:50	Details	Report
✖ Failed	Research MAX 45k	99AFG04D	786118	30/03/2023 20:37	Details	Report
✖ Failed	Research MAX 45k	99AFG016	HM100225	30/03/2023 20:37	Details	Report
✖ Failed	Research MAX 45k	99AFG0CA	E17685_M	30/03/2023 20:37	Details	Report
✖ Failed	Research MAX 45k	99AFG0CC	E17685_M	30/03/2023 20:37	Details	Report
✖ Failed	Research MAX 45k	99AFG015	E17685_F	30/03/2023 20:37	Details	Report
In total there are 278 new measurements in the selected time period.						
View all new measurements						

Approved Measurements						
QR-Code	Product type / Source	Sample Code	Analysis Date			
02AVJ227	ALEX2	Anna LMacley	28/03/2023 21:17	Details	Report	
02ARA379	ALEX2 IX	SM_28032023	28/03/2023 14:29	Details	Report	
02ARA393	ALEX2 IX	RG_28032023	28/03/2023 14:26	Details	Report	
02AYA31F	ALEX2 MAX 45k	PR_Mar2023_11	21/03/2023 15:05	Details	Report	
02AVI389	ALEX2 MAX 45k	KF	17/03/2023 14:46	Details	Report	
02AYT337	ALEX2 MAX 45k	2846846+ CCD	14/03/2023 19:06	Details	Report	
02ATL349	ALEX2 IX		22/02/2023 14:21	Details	Report	
02AVE383	ALEX2 MAX 45k	AO	03/02/2023 09:41	Details	Report	
02AXE29A	ALEX2 MAX 45k	OW	19/01/2023 19:12	Details	Report	
02ALY3AE	ALEX2 MAX 45k	BSZ_Okt2022_8	18/10/2022 18:12	Details	Report	
View all approved measurements						

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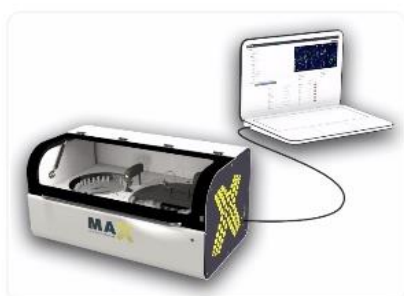
9K-Test-Instrument

Not initialized

[Initialize](#) [Shutdown](#) [Monthly Maintenance](#)

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E-Mail address:

Language: English (en)

The language is used to determine the display language.

Culture: English (United States) (en)

The culture is used to format numbers and dates.

Timezone: (UTC+01:00) Amsterdam, Berlin, Bern, Rome, Stockholm, Vienna

Configure your timezone to see date and times in your local time instead of UTC.

Multi-Factor Authentication

[Manage MFA](#)

Use multi-factor authentication (MFA) to significantly increase security. It is highly recommended to enable MFA; this adds a second layer of protection against phishing attacks or weak passwords.


Configuration

Research spots: Show research spots

If enabled, "research use only" arrays. This does not affect the results.

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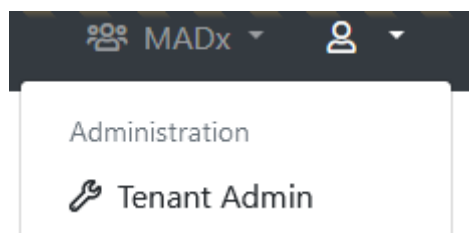General configuration

*** Name:**

Default timezone:
Configure a timezone to see date and times in (instead of UTC). Please note that users can choose to use a different timezone.

Default report template (ALEX2):

Default report template (ALEX3, ALEX Air, ALEX Food):

Default report template (FOX):

Default report template (PAX):

Default report language:

Default ALEX MOD:

Default FOX MOD:

Default PAX MOD:

Force MOD:
Requires a measurement to always have an MOD assigned. If this setting is enabled, a measurement cannot be approved without an MOD assigned.

Hide Global MOD:
If this setting is enabled, only the tenant-specific MOD will be available to select, and the global system-wide MOD will be hidden.

Display MOD Excluded Spots As Research Use Only:
If this setting is enabled, spots excluded by an MOD will not be hidden from the measurement image, but displayed as a "RUO spot".

MFA: ☐ **Force MFA**
If this setting is enabled, users are forced to set up an MFA on sign-in, in case they haven't yet.

MAX device default settings

*** Default Minimum RunId:**
When a new MAX device is created, this value will be used for its **Minimum Run Id**

Tenant-specific batch releases

Restrict this tenant to only allow the following batches to be scanned or to restrict certain batches. Enter one per line.

Use:

☒ **Whitelist**

☐ **Blacklist**

Whitelisted Batches:
Enter one batch code including product identifier per line, for example "02AAA"

Blacklisted Batches:
Enter one batch code including product identifier per line, for example "02AAA"

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Tenant-specific batch releases
Restrict this tenant to only allow the following batches to be scanned or to restrict certain batches. Enter one per line.
Use:
☒ **Whitelist**
☐ **Blacklist**

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You requested a new measurement from ImageExplorer "IX Trainingsraum". Please wait until the measurement is completed ...

Invalid batch detected! The measurement can't be processed because its batch is blocked.

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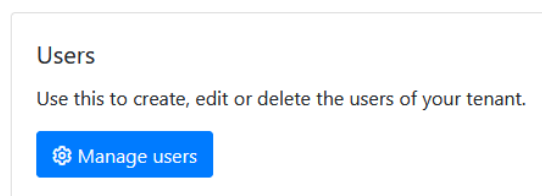
Login Credentials

Type:

Please select if the user should be validated against Azure AD (Office 365), or if you want to specify username and password yourself. It is recommended to use Azure AD whenever possible. This allows for increased security by supporting two-factor authentication. Also, it is more convenient for the user, because he or she does not have to remember a new username and password (single-sign-on).

Username:

Password:

It is a good idea to use the users e-mail-address as his username, because the username needs to be unique across the entire system.

Access Rights

Preconfigured groups

Use these pre-configured groups to automatically assign a set of access rights to this user. You can still add or remove specific access rights.

[Reset](#) [Lab User](#) [Super Lab User](#) [Support/Distributor User](#) [System Administrator](#)

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Tenants

This user is connected to the following tenants and has the following access rights:

☐ Tenants	Can share measurements	Can edit measurements	Can edit personal data	Can approve measurements	Can manage basic tenant configuration	Can manage API keys	Can manage ImageXplorers	Can view agent connections	Can manage users	Can manage users of Result Sharing Portal	Can manage report templates	Can manage measurement data completion	Can unapprove tenant specific batch releases	System Administration	Can unapprove measurements	Can use a custom calibration string	Can manage Run Configuration	Can skip
☐ Another MADx Tenant	☒	☒	☒	☒	☒	☒	☒	☒	☒	☒	☒	☒	☒	☒	☒	☒	☒	☒
☐ MADx Tenant	☒	☒	☒	☒	☒	☒	☒	☒	☒	☒	☒	☒	☒	☒	☒	☒	☒	☒

Additionally this user can be added to the following tenants by checking the tenant name and hitting save afterwards. User rights w applied from current open tenant user.

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Additionally this user can be added to the following tenants by checking the tenant name and hitting save afterwards. User right applied from current open tenant user.

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Tenants

This user is connected to the following tenants and has the following access rights:

Edit selected tenants rights								
☐	Tenants	Can share measurements	Can edit measurements	Can edit personal data	Can approve measurements	Can manage basic tenant configuration	Can manage API keys	Can manage ImageY
☒	Another MADx Tenant	✓	✓	✓	✓	✓	✓	✓

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ImageXplorers

Use this to configure ImageXplorers, or add new ones. Every ImageXplorer needs to be known to Raptor-Server, otherwise it won't be able to connect.

[Manage ImageXplorers](#)

Add new ImageXplorer

General information

Name:

Key:

This key is required when installing an Agent.

ImageXplorer Serialnumber:

Serialnumber of connected ImageXplorer.

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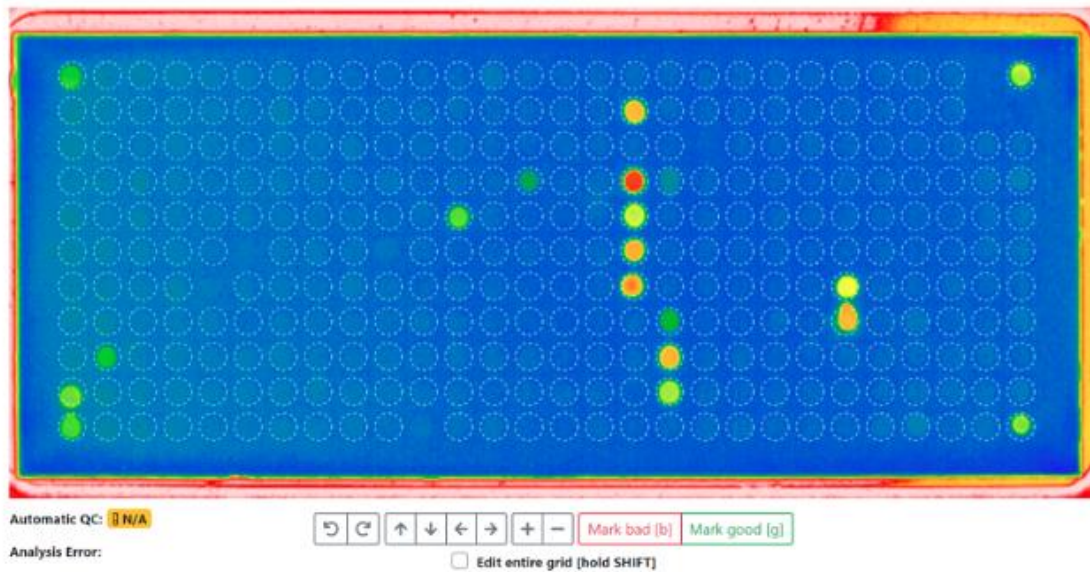
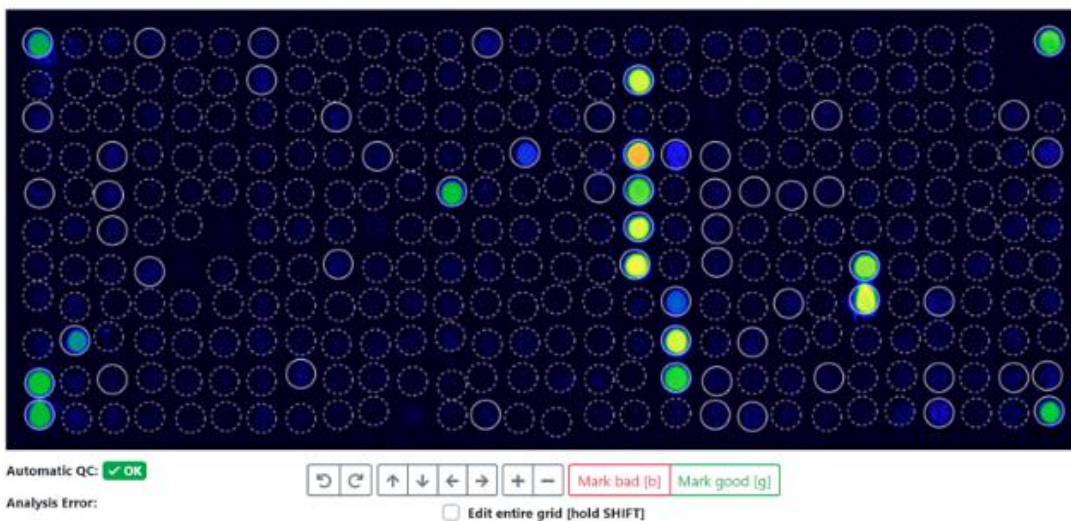
Show Analysis & QC Logs

Reset everything and reprocess

This discards all modifications to the array, and re-runs the complete analysis process from the original image data. Please note that any manual changes will be irreversible lost.

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Analysis Log

```
'PreProcessedPixels' took 128ms.
Using definition '02_ALEXII_IVD_v1.6.0'.
'ApplyArrayDefinition' took 74ms.
Parsing calibration string ...
'ApplyDefinitions' took 75ms.
'FindGrid' took 1ms.
'AlignFeatures' took 862ms.
'CropImage' took 1ms.
'QuantifyFeatures' took 231ms.
'SubtractBackground' took 681ms.
Parsing calibration string ...
Using calibrators from QR-Code.
Total Igt (tIgt) normalized by calculating 0,330999755859375E-07x²+0,05697999954223633x+-38,69269921875
(x=8515), which results in 515.
'ReQuantify' took 57ms.
```

QC Log

These are the values used for QC (ALEX v2): Curve Correction=0,9997370867894242, tIgt=8515, GD1=28985 (SNR=105.38), GD2=29369 (SNR=185.28), GD3=28569 (SNR=147.53), GD4=28385 (SNR=128.1). Reference values are as follows: Guide Dots must have a value above 5000; each Guide Dot spot must exist and be unique; Guide Dot spot must not be set to bad. Average background must be between 4000 and 16000. QC (ALEX v2) succeeded.

Background Log

Calculated background statistics for 164 pixels (from a total of 182, skipped smallest 9, and highest 9 pixels). The average is 9,419, the minimum 7973 and the maximum 11792 (minimum and maximum is calculated including skipped pixels). This results in an average difference of 628, which is an SD of 49,17 and a CV of 0,52%. For comparison, there is an overall total of 282944 pixels. The overall average is 14,871, the minimum 6,012 and the maximum 64,965.

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
Tissue & Source		All allergens		Only positive allergens		kUA/L
Pollen	Inhalation	E	Aca m			≤ 0.10
Mites		E	Aca s			7.11

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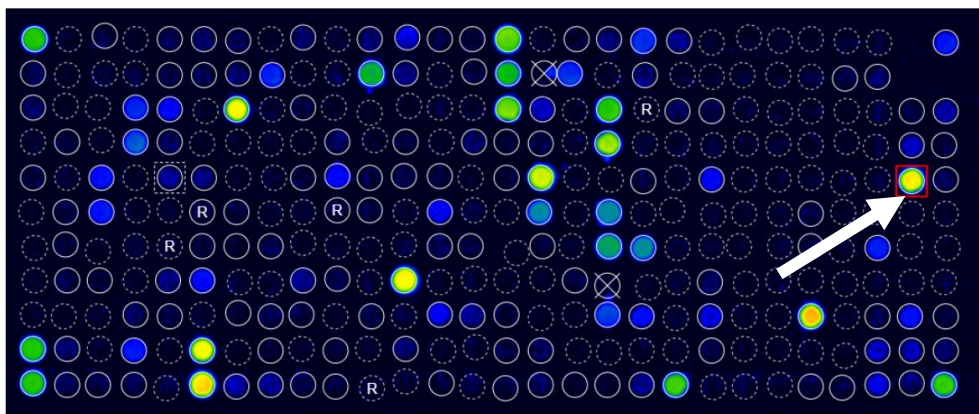New Measurements

Search by QR-Code, patient, sample, tag, analysis date, approval date ...

☒ Selected measurements ▼

☐	Source	QC	Tags	Product type
☐	IX	✓ OK		ALEX2
☒	IX	✓ OK		ALEX2
☒	IX	✓ OK		ALEX2
☒	IX	✓ OK		ALEX2

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Re-process measurements

You are about to re-process the following measurements. undone, the existing analysis result will be irreversibly over

QR-Code	Sample Code
02APK3C8	Dummy8
02APK258	Dummy1
02APK07F	Dummy13

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New Measurements

Q Search by QR-Code, patient, sample, tag, analysis date, approval date ...

	Source	QC	Tags	Product type
☐	Recaptured	✓ OK	recaptured	ALEX1
☐	⚠ Recaptured	✓ OK	recaptured	ALEX1
☐	QR-Code was scanned more than once. recaptured			ALEX1

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RAPTOR New measurements Approved measurements

New Measurements

Q Search by QR-Code, patient, sample, tag, analysis date, approval date ...

Selected measurements

	Source	QC	Tags	Product type	QR-Code
☒	IX	✓ OK		ALEX2	02AMC180
☒	IX	✓ OK		ALEX2	02AMC078
☒	IX	✓ OK		ALEX2	02AKV2D9

- Download CSV format
- Download Excel format
- Approve**
- Update interpretation
- Re-Process
- Move
- Delete

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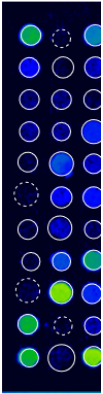
Approve measurements

You are about to approve the following measurements.

There are measurements with one or more warnings. Please make sure to double-check these measurements before approval.

QR-Code	Sample Code	Analysis Date
02AKV035		28.04.2022 11:14

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RAPTOR
New measurements
Approved measurements
Start m

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RAPTOR
New measurements
Approved measurements
Start measurement
Automats
MADx Testtenant


Exports

Certain types of data exports (for example, the export of many PDF reports at once) take a few minutes to prepare for download. Here download your requested exports. Please note that you can keep working, all exports are done in the background and you can download when they are completed.

Export requested	# of measurements included	Export completed
8/16/2021 11:07 AM	4	0 / 4
8/16/2021 11:06 AM	4	8/16/2021 11:06 AM
8/16/2021 11:04 AM	3	8/16/2021 11:04 AM

Administration
Tenant Admin
Exports
Personal Settings
Log out
Support
Imprint
Download
Download

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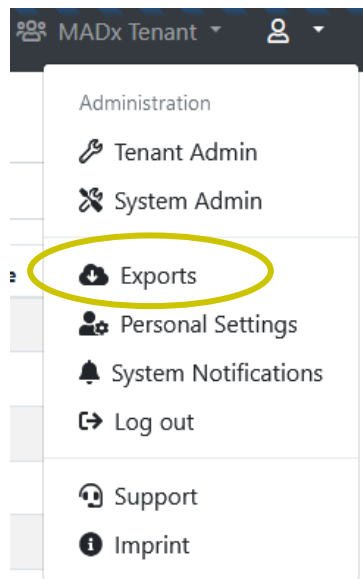Exports

Certain types of data exports (for example, the export of many PDF reports at once) may take some time. Please note that you can keep working, all exports are done in the background.

[Export multiple measurements](#)

Export requested	# of measurements included
06.07.2022 08:52	3

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Export multiple measurements

Please note that **Report PDF** and **ASTM** export is only available for certain data types.

QR-Code/Product/Batch

02ABC

Sa

Analysis Date (from)

TT . MM . JJJJ

Analysis Date (to)

TT . MM . JJJJ

Tag

Status

All

☐ Export CSV

☐ Export Excel

Search Measurements

✕ Cancel

Export multiple measurements

Please note that **Report PDF** and **ASTM** export is only

QR-Code/Product/Batch

02aaa

Analysis Date (from)

TT . MM . JJJJ

Analysis Date (to)

TT . MM . JJJJ

Tag

Status

All

☐ Export CSV

☐ Export Excel

 Start export ...

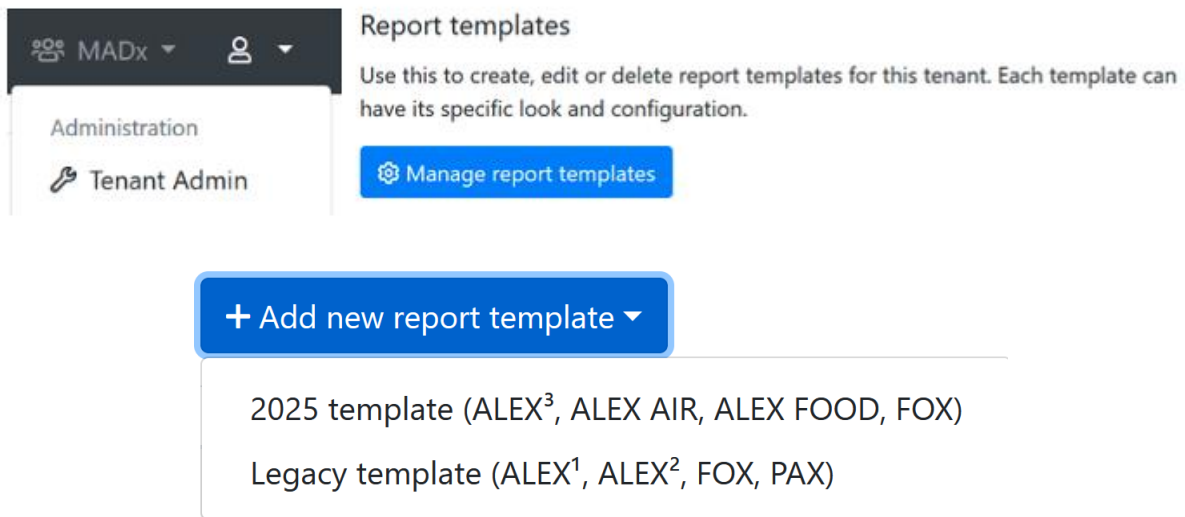
← Back

66 Measurements found that match your criteria

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Edit report template

Your changes have been saved successfully.

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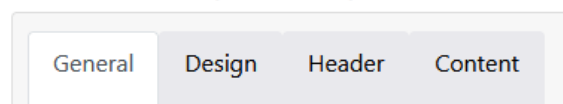
Configure report template "New report template"

General	Design	Header	Content
Name: New report template This is the name of the template which shows up in the dropdown list while generating a report.			
Base Template: Version 2 (ALEX2 only) A base template which specifies basic functionality must be chosen.			
Preferred language: English (United States) (en-us) A default language can be specified which will be used if there isn't yet a language stored to a measurement's report.			

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PDF naming

Setting Type: Manual

The filename of the report PDF can be customized. Either built automatically based on your selection below or manually adjusted in the order of your needs.

Name Pattern: Report_{PatientId}_{MeasurementQrCode}

Available Elements: {PatientId} ; {PatientName} ; {SampleId} ; {MeasurementQrCode} ; {DateAndTime}

PDF naming

Setting Type: Automatic

The filename of the report PDF can be customized. Either built automatically based on your selection below or manually adjusted in the order of your needs.

Include Patient ID: ☒

Include Patient Name: ☒

Include Sample ID: ☐

Include Measurement QR-Code: ☐

Include Date & Time: ☒

Footer

Footer text:

Will be printed in the footer if provided and rendered as HTML.

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Configure report template "New report template"

The screenshot shows the 'Design' tab of the 'Configure report template' interface. The 'General' tab is selected. The settings are as follows:

- PDF format:** A4 (selected), Letter (unselected). A note states: 'Specifies the paper size of the output PDF.'
- Font:** Arial (selected in a dropdown menu). A note states: 'The font family of the whole report texts can be adjusted.'
- Background image:** A text input field with a 'Browse' button. A note states: 'If set, this image is used as background on the first page of the PDF.'

Below the 'Background image' field, it says 'Maximum file size 1 MB'.

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Third level headline color:

Logos

Logo (left): Browse
Maximum file size 1 MB
This logo will appear on the **top left corner** of the first page of the report.

Text (center):
This text will appear either on the left, right, or center of the top of the first page (depending on your configured logos). Supports the placeholders `{{ PATIENT_PORTAL_QRCODE }}` and `{{ PATIENT_PORTAL_URL }}` to dynamically insert measurement information.

Logo (right): Browse
Maximum file size 1 MB
This logo will appear on the **top right corner** of the first page of the report.

Small logo: Browse
Maximum file size 1 MB
This logo is used on all detail pages. If not set, the default logo is used.

Signatures

Left: Browse
Maximum file size 1 MB
This signature or image will appear on the last page, left aligned.

Center: Browse
Maximum file size 1 MB
This signature or image will appear on the last page, center aligned.

Right: Browse
Maximum file size 1 MB
This signature or image will appear on the last page, right aligned.

Indicators

Custom intensity indicator:

A custom SVG for displaying allergen intensity can be pasted here. Will be styled according to specified colors.

⚠ The SVG element must contain the attributes **width** and **height** with a minimum dimension of 16 for correct display in PDF.

🌿 SVGs paths which should be colored must require the **colorize** class as well as a fill defined in the **defs** segment of the SVG.

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Configure report template "New report template"

Header content:	
Left half	Right half
Custom headline ☐ Patient Name Patient Name Custom headline	Custom headline ☐ Doctor Information None
Date of Birth	None
Sample ID	None
QR-Code	Additional Info text
Analysis Date	None
Amount of tested allergens	None
Test method	Quality Control text

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General	Design	Header	Content
Allergens to include:	All allergens	Specifies which kind of allergens should be included in the report.	
Allergens sort order:	Alphabetically by allergen	Specifies the sort order of the allergens.	
Include Overview:	☒	Whether the allergen overview should be printed or not.	
Include product type in overview:	☒	Whether the product type should be printed in the header of allergen overview.	
Include Total IgE:	On the front page	Whether the information of total IgE should be included in the report and where.	
Cross-reactive information:	Include text	This section will be displayed underneath the allergen list itself.	
Include Interpretation:	Include manually	It can be specified if the Raven report interpretation information should be included in the report or not. This section is printed underneath the allergens overview.	

Additional information (below the list of results)		Those informations are integrated into the report in an extra section underneath the allergen list.
Approval date:	☐	
Assay date:	☐	
Doctor info:	☐	The information about the doctor which accompanied the measurement/patient.
Print date:	☒	When the report was generated.
Report info:	☐	The „additional info“ field belonging to the measurement.
Sample date:	☒	

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Configure report template "New report template - Duplicate"

General	Design	Header	Content
Name:	New report template - Duplicate		
Support products:	ALEX ²		
Preferred language:	Deutsch (de)		

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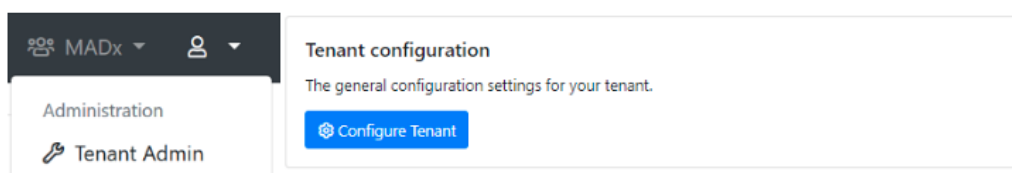
- Aserbajdsjansk
- Albansk
- Arabisk
- Arabisk (Saudi-Arabien)
- Bulgarsk
- Kinesisk
- Kinesisk (Hongkong)
- Kroatisk
- Tjekkisk
- Hollandsk
- Engelsk
- Engelsk (Indien)
- Engelsk (Det Forenede Kongerige)
- Engelsk (USA)
- Estisk
- Farsi
- Farsi (Iran)
- Fransk
- Georgisk
- Tysk
- Græsk
- Hindi
- Ungarsk
- Italiensk
- Kasakhisk
- Koreansk
- Lettisk
- Litauisk
- Makedonsk
- Norsk
- Persisk
- Polsk
- Portugisisk
- Portugisisk (Brasilien)
- Rumænsk
- Russisk
- Serbisk
- Slovakisk
- Slovensk
- Spansk
- Spansk (Latinamerikansk)

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* Name: MADx

Default timezone: (UTC+01:00) Amsterdam, Berlin, Bern, Rome, Stockholm, Vienna

Configure a timezone to see date and times in (instead of UTC). Please note that users can choose to use a different timezone.

Default report template (ALEX 1): Version 1 (ALEX1 & ALEX2)

Default report template (ALEX 2): Version 2 (ALEX2 only)

Default report language: English (United Kingdom) (en-gb)

Default MOD: < No MOD >

Require ImageXplorer testarray scan: Warn after 60 days, enforce after 90 days.

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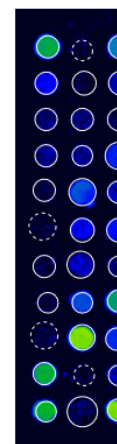
Raptor New measurements Approved measurements Start measurement Automate

Report

Template: Version 2 (ALEX2 only)

Language: English (en)

Usually this tenant's default template and language will be used when sharing or exporting measurements. You may change the template and language associated with this measurement by saving your current configuration.

You can download this report in PDF format once the measurement has been approved.

[Manage report template and settings](#)
You have access to manage the report templates and settings via the tenant administration. Please note that any changes will affect all users.

MADX
MACROARRAY DIAGNOSTICS

PATIENT ID
PATIENT NAME
DATE OF BIRTH
SAMPLE ID
SWABCODE
02AD0042
ANALYSED ON
11/04/2022
TESTED ALLERGENS
306
TESTMETHODE
ALEX2

REFERRING PHYSICIAN

NOTE
The internal QC (Pauzability check for QD) was within acceptance range.

Lab report: Summary on detectable sensitisations

POLLEN
Grass Pollen

MICROORGANISMS
Fungal Spores & Yeast

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RAPIOR New measurements Approved measurements Start m

Approved Measurements

Search by QR-Code, patient, sample, tag, analysis date ...

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Tags:

MOD:

Report Template: Version 2 (ALEX2 only) - Deutsch (de)

[New measurements](#)
[Approved measurements](#)
[Start measurement](#)
[Automats](#)

Report

Template: Default
Language: English

[Save](#)
[Use defaults](#)

Usually this tenants default template and language will be used when sharing or exporting measurements. You may change the template and language associated with this measurement by saving your current configuration.

[Download PDF](#)

[Manage report template and settings](#)

You have access to manage the report templates and settings via the tenant administration. Please note that any changes will affect all users.

Patient ID:	
Patient Name:	
Date of Birth:	
Sample ID:	
Barcode:	02AAE
Sampled on:	
Tested on:	12/18/
Printed on:	12/19/
Note:	The internal QC (Plausibility)

Lab report: Summary on de


Pollen	Grass Pollen
	Tree Pollen
	Weed Pollen
Mites	House Dust M
Microorganisms	Fungal Spores
	Legumes
Plant-Based Food	Grain
	Spices
	Fruits
	Vegetables
	Nuts & Seeds
	Milk

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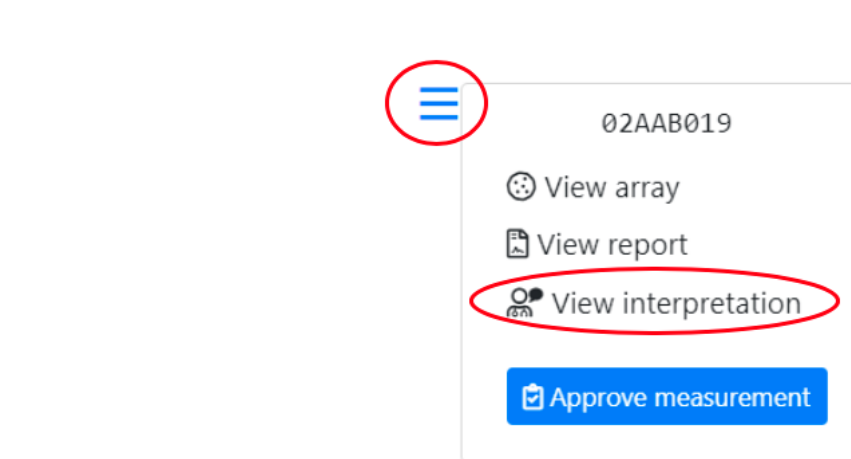Rule set

RAVEN Ruleset A for ALEX

A rule set defines the RAVEN or RAVEN² engine that is used to interpret results.

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AIT

Medication

MADx data declaration for smartRaven

☒ I accept to enter clinical data for using the full interpretation function of smartRAVEN

Patient has reported any symptoms

☐ Yes ☐ No

Additional information about the patient

☐ Atopic Dermatitis

☐ Allergen-specific immunotherapy (AIT) - ongoing, completed or discontinued

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[Request new interpretation](#)

This measurement already has an interpretation stored. Usually, it should not be necessary to request those again, but it could be necessary when the patients information has changed.

Interpretation

Raven Interpretation Summary

Sample Information

The sample was tested on ALEX² Barcode 02AAB019, interpretation date 12/19/2019.

Of the tested 295 allergens, 25 were above the detection limit of 0.3 kU_L/L. A sensitization can be an indicator of an allergy. For selected allergens, comments for interpretation are listed below.

Total IgE: 512 kU/L

The measured total IgE was 512 kU/L. Above a total IgE of 100 kU/L, allergy is likely.

Cross-Reactive Allergen Sensitization detected

Sensitization against allergen molecules which are markers of (broad) cross-reactivity between different allergen sources.

Detected cross-reactive allergen sensitizations:

- PR-10s: Aln g 1, Bet v 1, Cor a 1.0103, Fag s 1
- Polcalcins: Aln g 4, Phi p 7

PR-10 Proteins (PR10)

PR-10 inhalative: The major birch pollen allergen, Bet v 1, represents the prototype of all PR-10 allergens and is the primary sensitizer in regions with birch pollen exposure. The presence of PR-10 allergens in Fagales tree pollen explains IgE cross-reactivity between pollen from hazel, alder, beech, oak and hornbeam. PR-10 nutritive: PR-10 allergens in raw fruits, nuts, vegetable and legumes can induce oral allergy syndrome and sometimes severe allergic reactions in sensitized individuals. PR-10 allergens are not stable to heat and digestion.

Polcalcins (Ca++ binding proteins, PC)

Members of the PC family can cause inhalative symptoms. PC allergens can be found in pollen from grasses, trees and weeds. Inhalative symptoms manifest as allergic rhino-conjunctivitis and/or allergic asthma.

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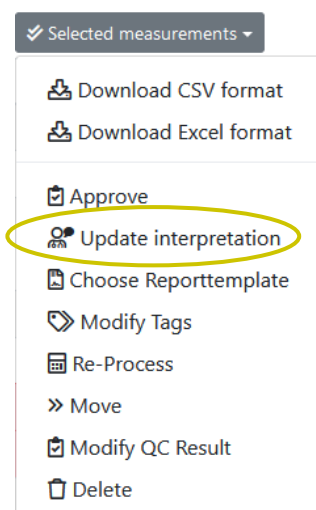
The screenshot displays the 'Raptor' software interface. At the top, there's a navigation bar with 'New measurements', 'Approved measurements', 'Start measurement', and 'Automate'. Below this is the 'Details' section, which contains various input fields: Barcode (024E1852), Sample Code, Sampling Date (tt.mm.åå), Assay Date (07.10.2020), Patient Code, Patient Name, Date of Birth (tt.mm.åå), Info, Doctor Info, and Tags. A red oval highlights the 'MOD: Pollen allergens' field. Below the form are buttons for 'Report', 'Save', and 'Save and approve'. The bottom section shows a large image of a multiplex assay plate with numerous blue and green spots. Navigation arrows 'Previous' and 'Next' are visible at the top right of the image. At the bottom, there's a status bar with 'Received: 07.10.2020 14:44', 'ImageXplorer: 23156891', 'Analysis: 07.10.2020 14:44', and 'Automatic QC: ✓ OK'.

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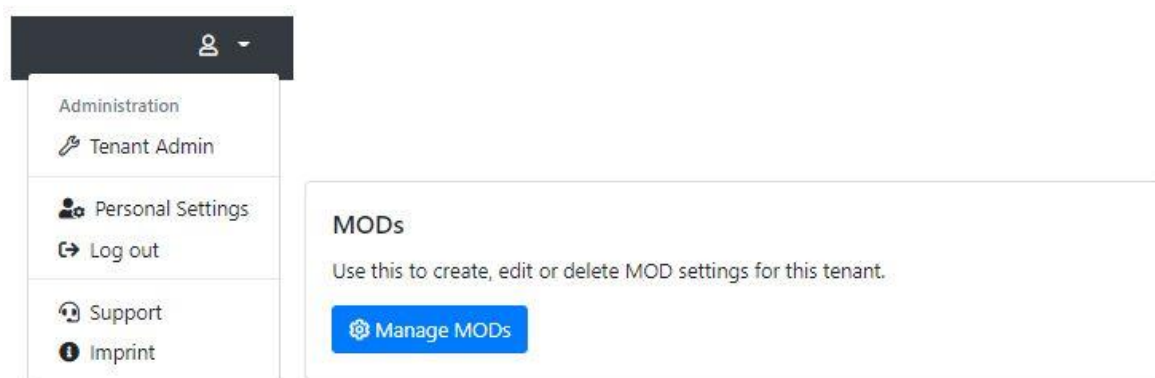
Type:

Deactivated:

You cannot delete an MOD once it is in use. But you can deactivate it, so that it's not available for new measurements anymore.

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*** Name:**

Default timezone:

Configure a timezone to see date and times in (instead of UTC). Please note that users can choose to use a different timezone.

Default report template (ALEX 1):

Default report template (ALEX 2):

Default report template (FOX):

Default report template (PAX):

Default report language:

Default ALEX2 MOD:

Default FOX MOD:

Default PAX MOD:

Force MOD:

Requires a measurement to always have an MOD assigned. If this setting is enabled, a measurement cannot be approved without an MOD assigned.

Hide Global MOD:

If this setting is enabled, only the tenant-specific MOD will be available to select, and the global system-wide MOD will be hidden.

Display MOD Excluded Spots As Research Use Only:

If this setting is enabled, spots excluded by an MOD will not be hidden from the measurement image, but displayed as a "RUO spot".

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Force MOD: False
Requires a measurement to always have an MOD assigned. If this setting is enabled, a measurement cannot be approved without an MOD assigned.

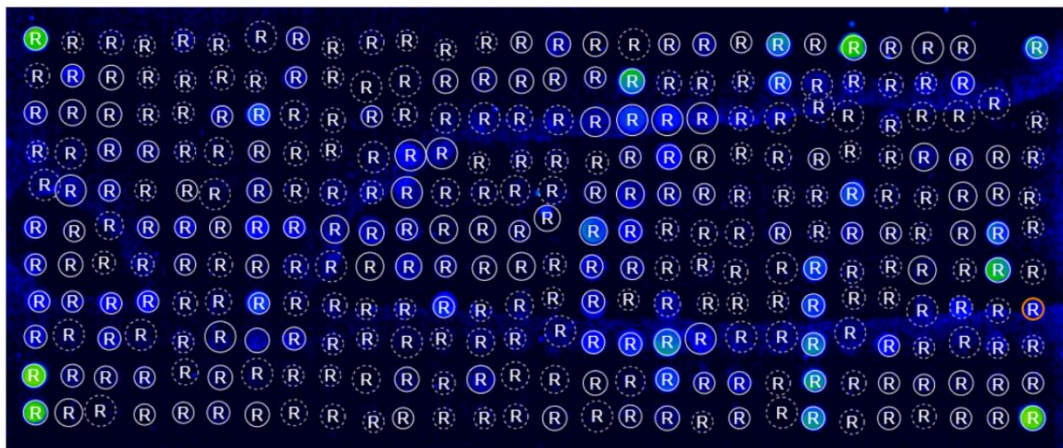
Hide Global MOD: False
If this setting is enabled, only the tenant-specific MOD will be available to select, and the global system-wide MOD will be hidden.

Display MOD Excluded Spots As Research Use Only: True
If this setting is enabled, spots excluded by an MOD will not be hidden from the measurement image, but displayed as a "RUO spot".

Configuration

Calibration string:
Allows to override the calibration information from the cartridge's QR-Code. Leave empty to use the calibration information from the QR-Code on the cartridge (the usual behaviour).

Research spots: Show research spots
If enabled, "research use only" spots will be displayed on measurement details and arrays. This does not affect the export behaviour or the reports of measurements.

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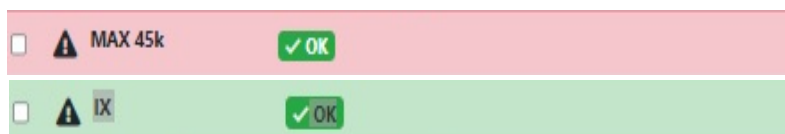
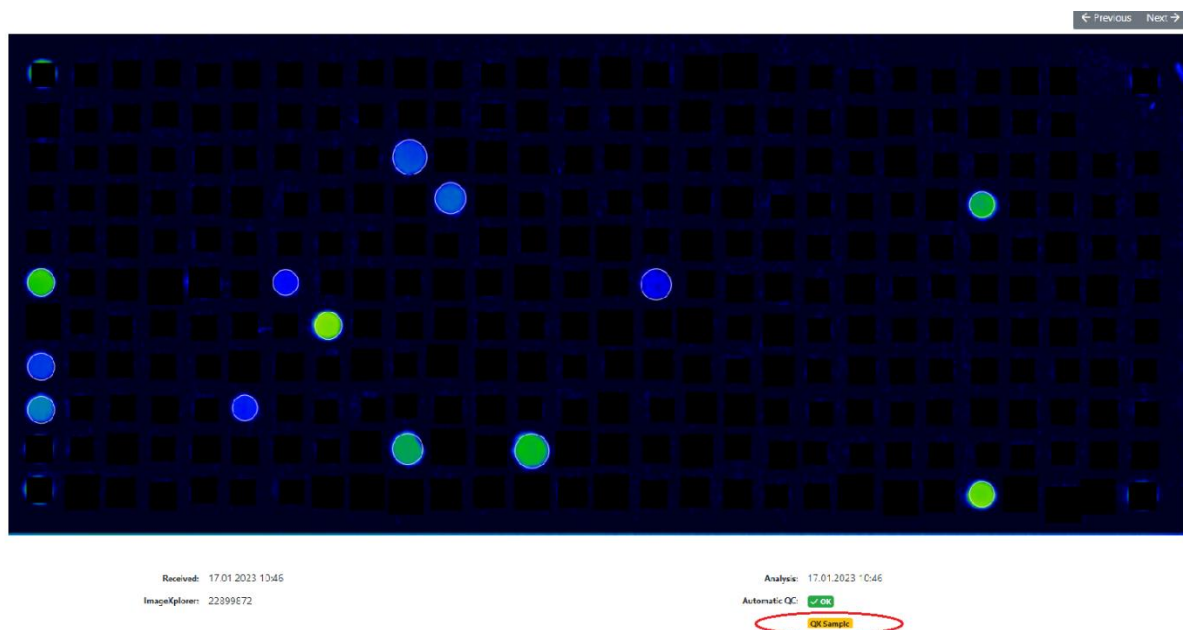
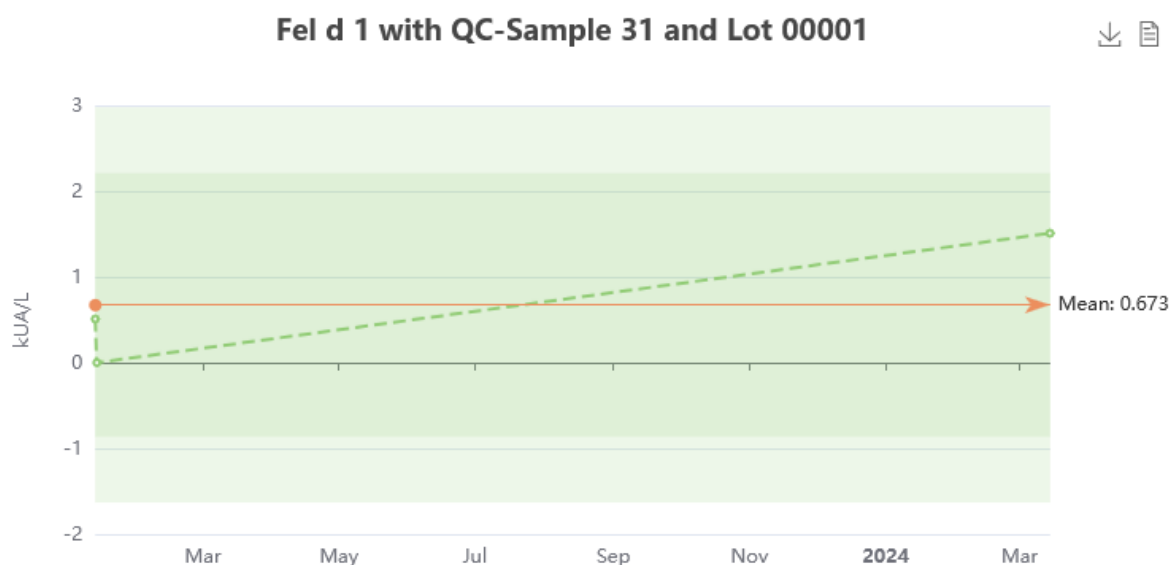
New Measurements

Search by barcode, patient, sample, tag, analysis date, a

✓ Selected measurements ▼

- Download CSV format
- Download Excel format
- Approve
- Update interpretation
- Re-Process
- » Move
- Delete**

Tags	Product type
	FOX
	FOX
	ALEX2
	FOX
	ALEX2
	ALEX1

Delete measurements

You are about to delete the following measurements. Please note that this action is irreversible and cannot be undone.

Barcode	Analysis Date
02AAB160	16.12.2019 13:41

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Plant Food Food	M	Jug r 3	0.40
Plant Food Food	M	Jug r 1	0.38

Plant Food Food	M	Jug r 3	0.40
Plant Food Food	M	Jug r 1	0.38

Delete request scheduled now. [Cancel delete request](#)

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ign in. Can be

Background jobs

View details about running background jobs and manually schedule runs.

[View dashboard](#)

Measurement delete / unapproval log & requests

Show logging and pending requests for measurement delete and unapproval.

[Show log & requests](#)

Measurement unapproval / delete requests

Open requests Log & Finished requests				
Requested UTC	Requesting User (Email)	QR-Code	Request Type	Action
11.10.2023 13:05:28	Test Test (Test@test.com)	02AYQ145	Unapprove	Accept Reject
11.10.2023 12:59:27	Test Test (Test@test.com)	02AYQ14C	Delete	Accept Reject

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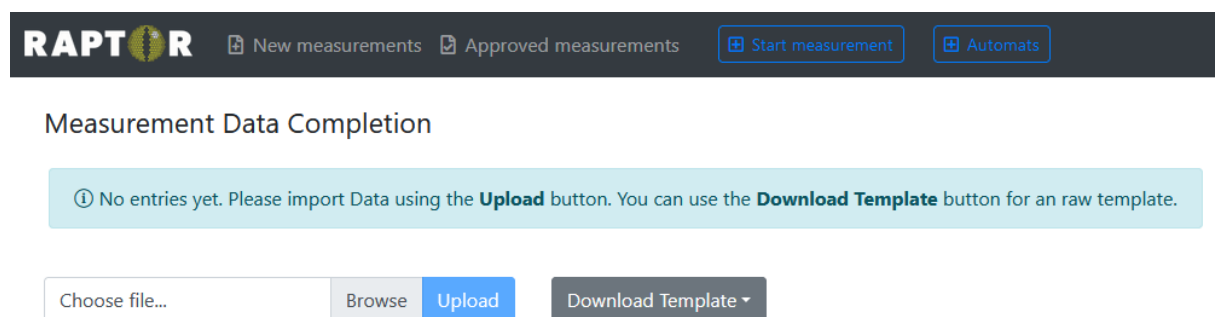
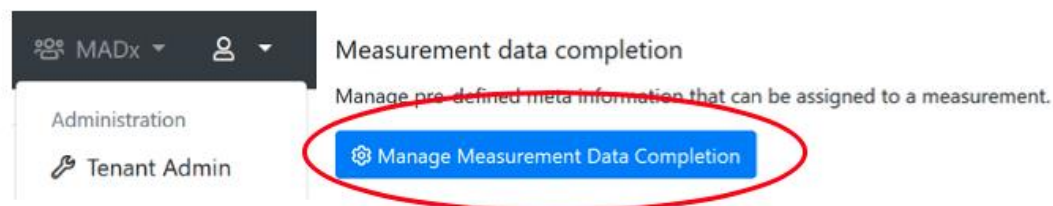
Add new user on Result Sharing Portal

General configuration

First name:

Last name:

E-Mail address:

Username:

It is a good idea to use the users e-mail-address as username as well.

Password:

 Send an invitation e-mail to user automatically

Status:

Locked (user cannot log in)

Enabled (user can log in)

Locked (user cannot log in)

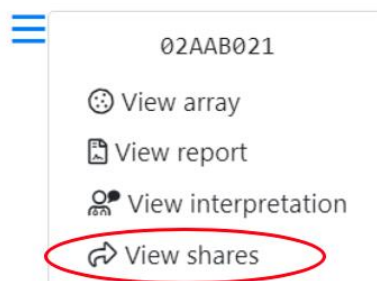
Doctor Info Text:

This information will be printed on a patients report.

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≡ Share

You are about to share a measurement with a user of the User Portal. Please note that the doctor will have access to the results of the measurement. Once you share a measurement you'll be provided with a link to access the shared information via the encrypted User F

Barcode: 02AAB021

Share with: X Dr. (testuser2@macroarraydx.com)

If you can't find a specific user of the User Portal, go to the [tenant administration](#) to configure the list of available users of the User Portal.

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
The screenshot shows the RAPTOR interface with a top navigation bar containing 'New measurements', 'Approved measurements', 'Start measurement with ImageXplorer', and 'Automats'. Below this is a 'Share' section with a 'Share this measurement' button. A table lists shared measurements with columns for 'User', 'Link', and 'Shared'. The 'Shared' column shows '7 months ago' and a 'Revoke' button. A yellow arrow points to the 'Revoke' button.

User	Link	Shared
RP Bak	https://resultshare.raptor-server.com/s/07PXEpC7ZNyhm4PdRfmc4NR5pUtWq5FJcqOY7Ry6oEx08USj	7 months ago Revoke

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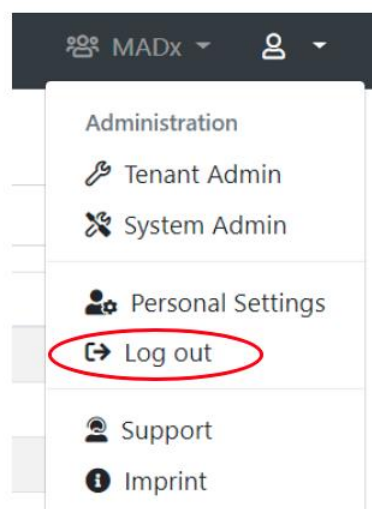VII.8 PORTAL TIL DELING AF RESULTATER

Resultatdelingsportalen kan tilgås via <https://resultshare.raptor-server.com>.

The screenshot shows the 'Result Sharing Portal' interface. At the top, there is a dark header bar with the MacroArray logo and the text 'Result Sharing Portal' on the left, and a language dropdown menu set to 'English (en)' on the right. Below the header, the main content area has a heading 'Welcome to the Result Sharing Portal' followed by a paragraph: 'The Raptor-Server Result Sharing Portal allows users to view shared measurements. You will need a username and password to log in to view all measurements that have been shared with you. Also, a sharing link will allow to view specific shared measurements.' Below this text is a login form with the prompt 'Please enter your username and your password:'. It contains two input fields: 'Your username' and 'Your password'. The password field has an eye icon to toggle visibility. A blue 'Log in' button is positioned below the fields. At the bottom of the form, there is a link: 'Did you forget your password? No problem, you can [request a new one](#).'

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This is a detailed view of the login form. It features the same prompt 'Please enter your username and your password:'. The 'Your username' field is a simple text input. The 'password' field is also a text input, but it includes an eye icon on the right side, which is used to show or hide the password. Below these fields is a prominent blue button labeled 'Log in'.

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Log out

Measurements

These are the measurements that are available for you. You may view the details of each measurement, or download the report PDF file.

Measurement	Product	Patient	Sample	Date
01AAK239	ALEX1		11852162	7/12/2019
01AAK281	ALEX1		04824821	12/18/2019
01AAK891	ALEX1		66560260	1/28/2020
02AAA015	ALEX2			2/21/2020

Listen over delte målinger kan sorteres efter QR-kode, produkttype, produktnavn, patientnavn, prøve-ID eller dato.

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Measurement 02AYQ143

Product: ALEX2
 Patient:
 Date of Birth:
 Sample:
 Info:
 Doctor Info:
 Sampling Date:
 Assay Date: 10/12/2023
 Processed Date: 10/12/2023
 Approval Date: 10/12/2023

Report Language:

English (United Kingdom) ▾

You can request a new RAVEN interpretation that will be included in the report.

[Request new RAVEN interpretation](#)

You can download a PDF report that includes all information and results of this measurement.

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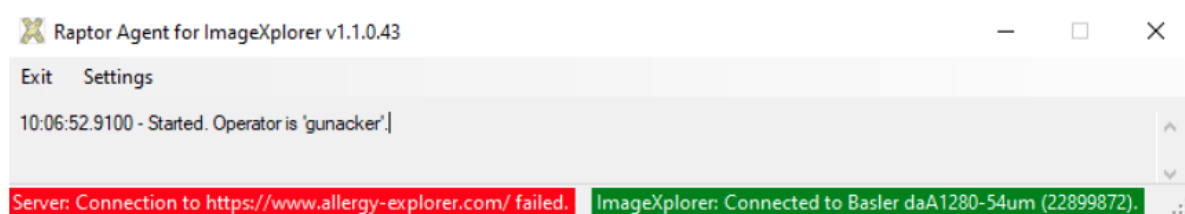
Settings

Please enter the settings for your ImageXplorer. You can find your ImageXplorer key when you log in to Raptor. Please contact support if you're not sure what the proper settings are for your ImageXplorer.

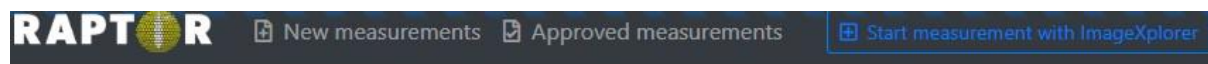
Raptor URL:

ImageXplorer Key:

Continue

Startmåling er ikke mulig:

The ImageXplorer IX training cannot be used, because it requires a ConfigXplorer scan.

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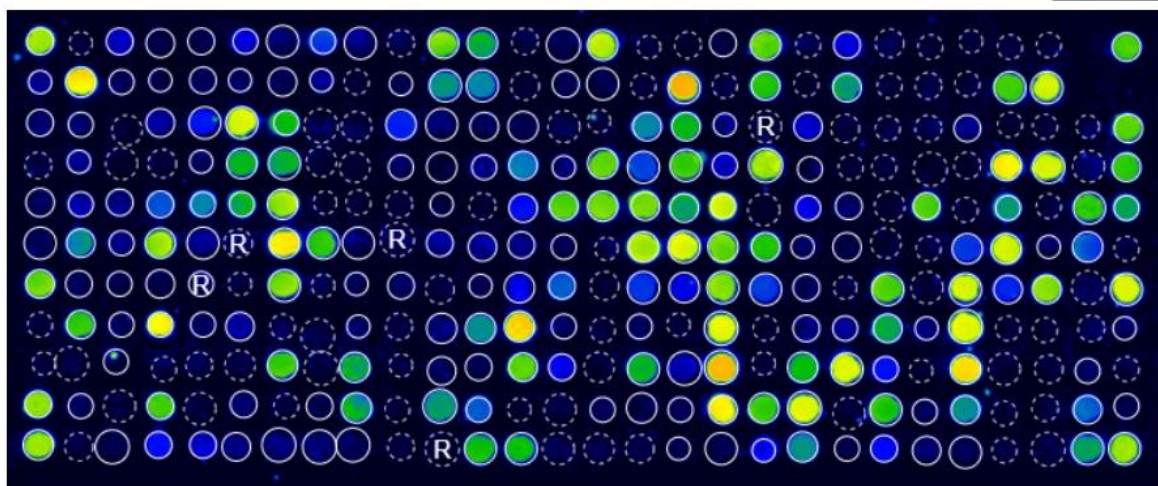
ImageXplorer: 40287249

Automatic QC: X Failed

Analysis Error: Insufficient data transfer - check connection.

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RAPTOR SERVER Analysis Software i kombination med andre MADx-testsæt:

© Copyright by MacroArray Diagnostics

MacroArray Diagnostics (MADx)

Lemböckgasse 59, Top 4

1230 Vienna, Austria

+43 (0)1 865 2573

www.madx.com

Versionsnummer: 22-IFU-01-DA-08 (EN-25)

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