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Welcome to RAPTOR SERVER Analysis Software, the analysis software for the [ALEX multiplex test](#).  
Please log in below to continue.

#### Log in with Azure AD / Office 365

If your organization integrates the software into its Microsoft Azure/Office 365 user directory (single-sign-on) click the button below. You will be redirected to the Microsoft login screen.

 Log in

#### Log in with Username & password

Please enter your username and password.

Your username

Your password

 Log in

Forgot your password? No problem, you can [request a new one](#).

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**Contract approval required**

Please review the following documents. You are required to confirm that you have read and agree with each document in order to use RAPTOR SERVER:

☐ **Bitte lesen** (last updated 23.11.2023 18:00)

☐ **RAPTOR SERVER IFU** (last updated 04.12.2023 11:05)

☐ **Terms of Service** (last updated 22.11.2023 01:00)

Confirm

By clicking the button, I confirm that I have reviewed these documents and agree.

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| New Measurements      |                       |          |             |                  | Last 7 days             |                        |
|-----------------------|-----------------------|----------|-------------|------------------|-------------------------|------------------------|
| QC                    | Product type / Source | QR-Code  | Sample Code | Analysis Date    |                         |                        |
| <span>❌ Failed</span> | Research MAX 45k      | 99AFG0F7 | NK_FOX      | 30/03/2023 20:50 | <a href="#">Details</a> | <a href="#">Report</a> |
| <span>❌ Failed</span> | Research MAX 45k      | 99AFG0F8 | N.G.        | 30/03/2023 20:50 | <a href="#">Details</a> | <a href="#">Report</a> |
| <span>❌ Failed</span> | Research MAX 45k      | 99AFG0F9 | E18735      | 30/03/2023 20:50 | <a href="#">Details</a> | <a href="#">Report</a> |
| <span>❌ Failed</span> | Research MAX 45k      | 99AFG0FA | 20220325w56 | 30/03/2023 20:50 | <a href="#">Details</a> | <a href="#">Report</a> |
| <span>❌ Failed</span> | Research MAX 45k      | 99AFG0EE | HM100038    | 30/03/2023 20:50 | <a href="#">Details</a> | <a href="#">Report</a> |
| <span>❌ Failed</span> | Research MAX 45k      | 99AFG04D | 786118      | 30/03/2023 20:37 | <a href="#">Details</a> | <a href="#">Report</a> |
| <span>❌ Failed</span> | Research MAX 45k      | 99AFG016 | HM100225    | 30/03/2023 20:37 | <a href="#">Details</a> | <a href="#">Report</a> |
| <span>❌ Failed</span> | Research MAX 45k      | 99AFG0CA | E17685_M    | 30/03/2023 20:37 | <a href="#">Details</a> | <a href="#">Report</a> |
| <span>❌ Failed</span> | Research MAX 45k      | 99AFG0CC | E17685_M    | 30/03/2023 20:37 | <a href="#">Details</a> | <a href="#">Report</a> |
| <span>❌ Failed</span> | Research MAX 45k      | 99AFG015 | E17685_F    | 30/03/2023 20:37 | <a href="#">Details</a> | <a href="#">Report</a> |

In total there are 270 new measurements in the selected time period.

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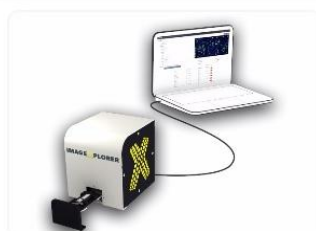
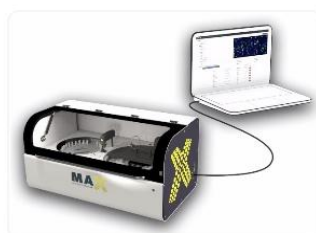
### 9K-Test-Instrument

Not initialized

[Initialize](#) [Shutdown](#) [Monthly Maintenance](#)

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
The screenshot shows the RAPTOR web application interface. At the top is a dark navigation bar with the RAPTOR logo and links for 'New measurements', 'Approved measurements', 'Start measurement', and 'Automats'. Below this, the 'Personal settings' section contains input fields for 'Name', 'E-Mail address', a dropdown for 'Language' (set to English (en)), a dropdown for 'Culture' (set to English (United States) (en)), and a dropdown for 'Timezone' (set to (UTC+01:00) Amsterdam, Berlin, Bern, Rome, Stockholm, Vienna). To the right, the 'Multi-Factor Authentication' section has a 'Manage MFA' button and explanatory text. Below that, the 'Configuration' section has a 'Research spots' dropdown set to 'Show research spots'.

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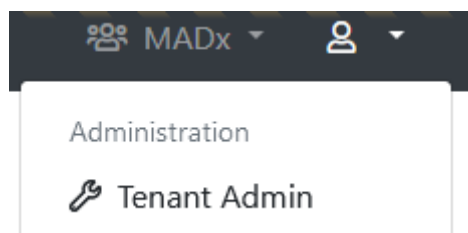## Configure "MADx"

| General configuration                                                                                                                        |                                           |
|----------------------------------------------------------------------------------------------------------------------------------------------|-------------------------------------------|
| * Name:                                                                                                                                      | MADx                                      |
| Default timezone:                                                                                                                            | (UTC+01:00) Amsterdam, Berlin, Bern, R... |
| Configure a timezone to see date and times in (instead of UTC). Please note that users can choose to use a different timezone.               |                                           |
| Default report template (ALEX2):                                                                                                             | ALEX2                                     |
| Default report template (ALEX3, ALEX Air, ALEX Food):                                                                                        | check 1.21                                |
| Default report template (FOX):                                                                                                               | FOX (2025)                                |
| Default report template (PAX):                                                                                                               | PAX                                       |
| Default report language:                                                                                                                     | English (en)                              |
| Default ALEX MOD:                                                                                                                            | < No MOD >                                |
| Default FOX MOD:                                                                                                                             | < No MOD >                                |
| Default PAX MOD:                                                                                                                             | < No MOD >                                |
| Force MOD:                                                                                                                                   | False                                     |
| Requires a measurement to always have an MOD assigned. If this setting is enabled, a measurement cannot be approved without an MOD assigned. |                                           |
| Hide Global MOD:                                                                                                                             | False                                     |
| If this setting is enabled, only the tenant-specific MOD will be available to select, and the global system-wide MOD will be hidden.         |                                           |
| Display MOD Excluded Spots As Research Use Only:                                                                                             | False                                     |
| If this setting is enabled, spots excluded by an MOD will not be hidden from the measurement image, but displayed as a "RUO spot".           |                                           |
| MFA:                                                                                                                                         | <input type="checkbox"/> Force MFA        |
| If this setting is enabled, users are forced to set up an MFA on sign-in, in case they haven't yet.                                          |                                           |

| MAX device default settings                                                             |    |
|-----------------------------------------------------------------------------------------|----|
| * Default Minimum RunId:                                                                | 10 |
| When a new MAX device is created, this value will be used for its <b>Minimum Run Id</b> |    |

| Tenant-specific batch releases                                                                                             |  |
|----------------------------------------------------------------------------------------------------------------------------|--|
| Restrict this tenant to only allow the following batches to be scanned or to restrict certain batches. Enter one per line. |  |
| Use:                                                                                                                       |  |
| <input checked="" type="radio"/> Whitelist                                                                                 |  |
| <input type="radio"/> Blacklist                                                                                            |  |
| Whitelisted Batches:                                                                                                       |  |
| Enter one batch code including product identifier per line, for example "02AAA"                                            |  |
| Blacklisted Batches:                                                                                                       |  |
| Enter one batch code including product identifier per line, for example "02AAA"                                            |  |

## Company Details

|                          |                          |
|--------------------------|--------------------------|
| * Legal Entity Name:     | MacroArray Diagnostics   |
| * Street and Number:     | Street 4                 |
| * Zip Code:              | 1230                     |
| * City:                  | New York                 |
| * Country:               | United States of America |
| * Phone:                 | +4318652573              |
| * E-Mail:                | office@macroarraydx.com  |
| * Legal Representative:  | Dr. Christian Harwanegg  |
| Website:                 |                          |
| Vat Number:              |                          |
| Company Register Number: |                          |

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**Tenant-specific batch releases**  
Restrict this tenant to only allow the following batches to be scanned or to restrict certain batches. Enter one per line.  
Use:  
☒ **Whitelist**  
☐ **Blacklist**

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You requested a new measurement from ImageExplorer "IX Trainingsraum". Please wait until the measurement is completed ...

Invalid batch detected! The measurement can't be processed because its batch is blocked.

Hvis en batch er hvitelistet, vil RAPTOR SERVER kun tillate skanning av batcher med denne koden.

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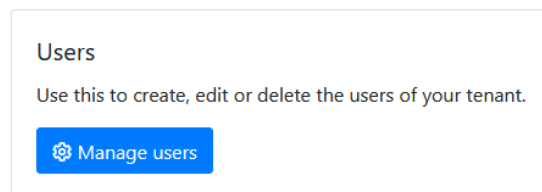
##### Login Credentials

Type:

Please select if the user should be validated against Azure AD (Office 365), or if you want to specify username and password yourself. It is recommended to use Azure AD whenever possible. This allows for increased security by supporting two-factor authentication. Also, it is more convenient for the user, because he or she does not have to remember a new username and password (single-sign-on).

Username:

It is a good idea to use the users e-mail-address as his username, because the username needs to be unique across the entire system.

Password:

##### Access Rights

##### Preconfigured groups

Use these pre-configured groups to automatically assign a set of access rights to this user. You can still add or remove specific access rights.

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## Tenants

This user is connected to the following tenants and has the following access rights:

| <input type="checkbox"/> Tenants             | Can share measurements              | Can edit measurements               | Can edit personal data              | Can approve measurements            | Can manage basic tenant configuration | Can manage API keys                 | Can manage ImageXplorers            | Can view agent connections          | Can manage users                    | Can manage users of Result Sharing Portal | Can manage report templates         | Can manage measurement data completion | Can unapprove tenant specific batch releases | System Administration               | Can unapprove measurements          | Can use a custom calibration string | Can manage Run Configuration        | Can skip                            |
|----------------------------------------------|-------------------------------------|-------------------------------------|-------------------------------------|-------------------------------------|---------------------------------------|-------------------------------------|-------------------------------------|-------------------------------------|-------------------------------------|-------------------------------------------|-------------------------------------|----------------------------------------|----------------------------------------------|-------------------------------------|-------------------------------------|-------------------------------------|-------------------------------------|-------------------------------------|
| <input type="checkbox"/> Another MADx Tenant | <input checked="" type="checkbox"/> | <input checked="" type="checkbox"/> | <input checked="" type="checkbox"/> | <input checked="" type="checkbox"/> | <input checked="" type="checkbox"/>   | <input checked="" type="checkbox"/> | <input checked="" type="checkbox"/> | <input checked="" type="checkbox"/> | <input checked="" type="checkbox"/> | <input checked="" type="checkbox"/>       | <input checked="" type="checkbox"/> | <input checked="" type="checkbox"/>    | <input checked="" type="checkbox"/>          | <input checked="" type="checkbox"/> | <input checked="" type="checkbox"/> | <input checked="" type="checkbox"/> | <input checked="" type="checkbox"/> | <input checked="" type="checkbox"/> |
| <input type="checkbox"/> MADx Tenant         | <input checked="" type="checkbox"/> | <input checked="" type="checkbox"/> | <input checked="" type="checkbox"/> | <input checked="" type="checkbox"/> | <input checked="" type="checkbox"/>   | <input checked="" type="checkbox"/> | <input checked="" type="checkbox"/> | <input checked="" type="checkbox"/> | <input checked="" type="checkbox"/> | <input checked="" type="checkbox"/>       | <input checked="" type="checkbox"/> | <input checked="" type="checkbox"/>    | <input checked="" type="checkbox"/>          | <input checked="" type="checkbox"/> | <input checked="" type="checkbox"/> | <input checked="" type="checkbox"/> | <input checked="" type="checkbox"/> | <input checked="" type="checkbox"/> |

Additionally this user can be added to the following tenants by checking the tenant name and hitting save afterwards. User rights w applied from current open tenant user.

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Additionally this user can be added to the following tenants by checking the tenant name and hitting save afterwards. User right applied from current open tenant user.

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## Tenants

This user is connected to the following tenants and has the following access rights:

 Edit selected tenants rights

| <input type="checkbox"/>            | Tenants             | Can share measurements              | Can edit measurements               | Can edit personal data              | Can approve measurements            | Can manage basic tenant configuration | Can manage API keys                 | Can manage Imagey                   |
|-------------------------------------|---------------------|-------------------------------------|-------------------------------------|-------------------------------------|-------------------------------------|---------------------------------------|-------------------------------------|-------------------------------------|
| <input checked="" type="checkbox"/> | Another MADx Tenant | <input checked="" type="checkbox"/> | <input checked="" type="checkbox"/> | <input checked="" type="checkbox"/> | <input checked="" type="checkbox"/> | <input checked="" type="checkbox"/>   | <input checked="" type="checkbox"/> | <input checked="" type="checkbox"/> |

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The screenshot shows the MADx Tenant Admin interface. On the left, a sidebar contains 'Administration' and 'Tenant Admin'. The main area is titled 'ImageXplorers' and includes a description: 'Use this to configure ImageXplorers, or add new ones. Every ImageXplorer needs to be known to Raptor Server, otherwise it won't be able to connect.' A blue button labeled 'Manage ImageXplorers' is circled in red.

Add new ImageXplorer

General information

Name:

Key:   
This key is required when installing an Agent.

ImageXplorer Serialnumber:   
Serialnumber of connected ImageXplorer.

Notes:

Internal Notes:   
For internal use only.

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Details

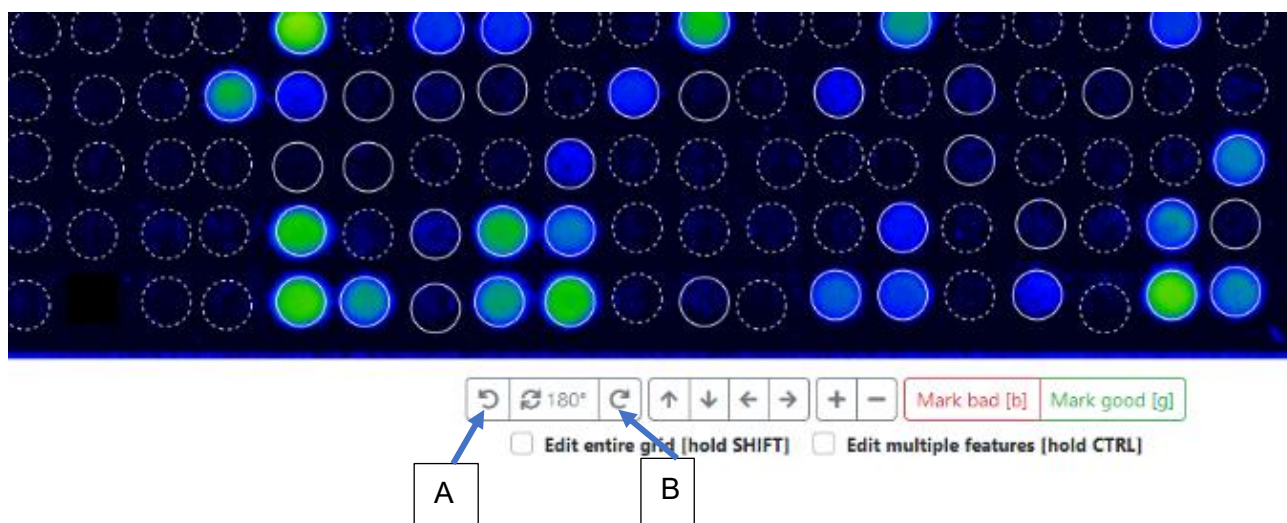
Barcode: 02AE3408  
Sample Code: 12  
Requisition Code:  
Sampling Date: 01.10.2021  
Assay Date: 04.10.2021  
Patient Code: TE  
Patient Name: Test  
Date of Birth: 17.05.2010  
Info:  
Doctor Info: Dr. A  
Choose a doctor... Insert doctor info  
Tag: Tag Info  
MOD: < No MOD >  
Report Template: Version 2 (ALEX2 only) - Deutsch (de)

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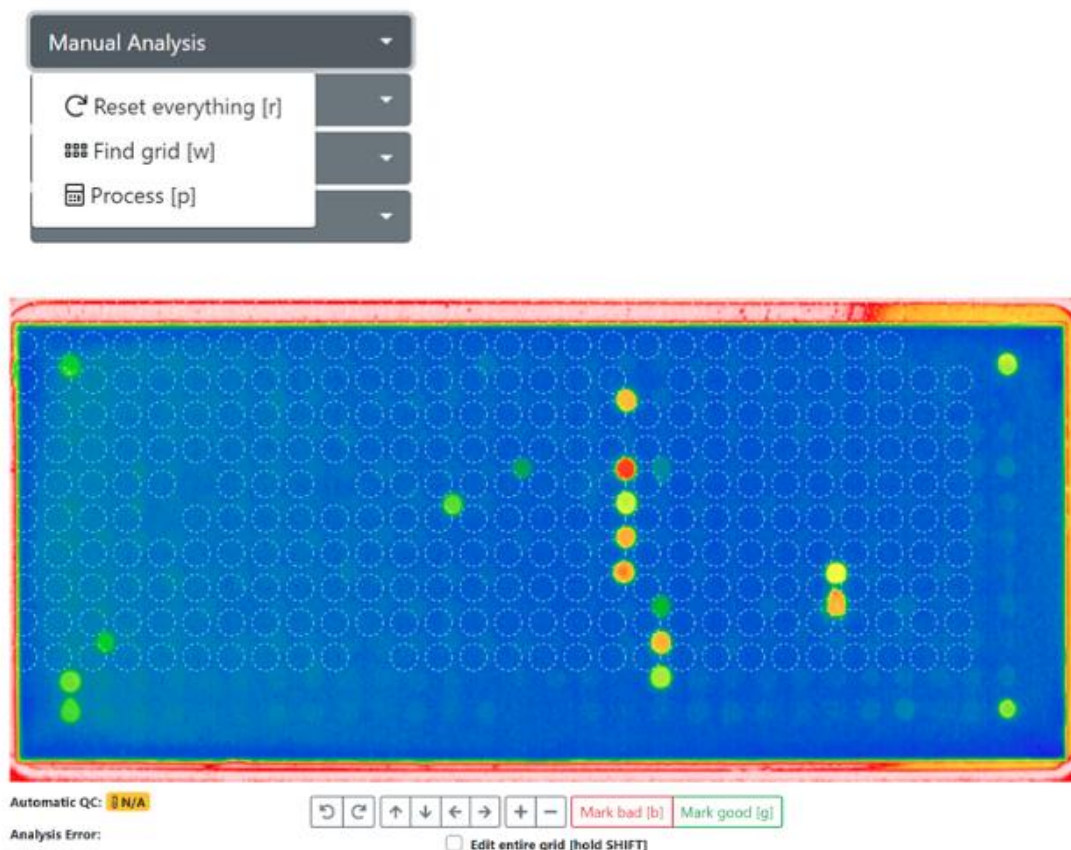
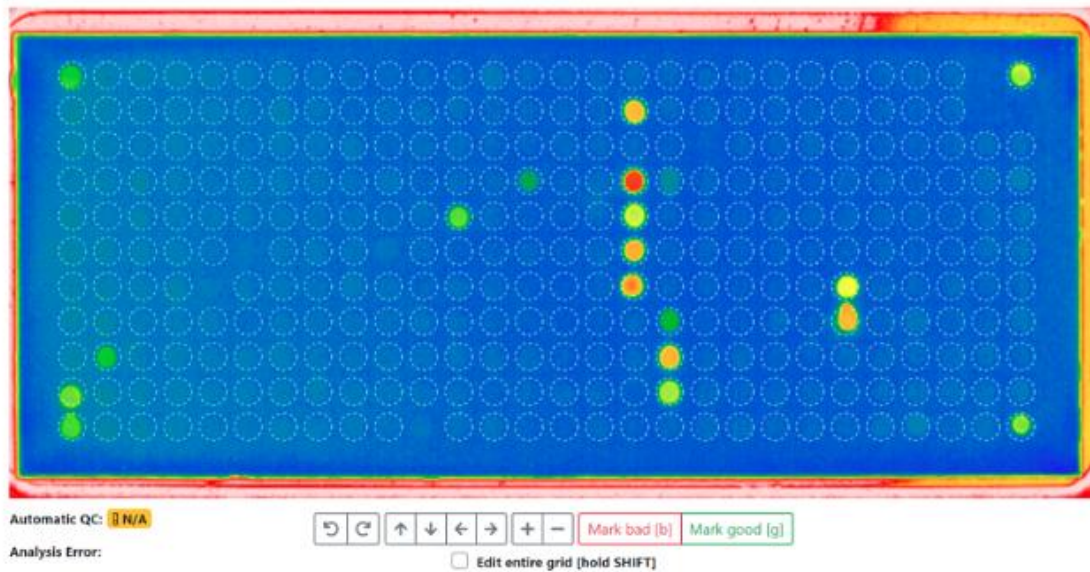
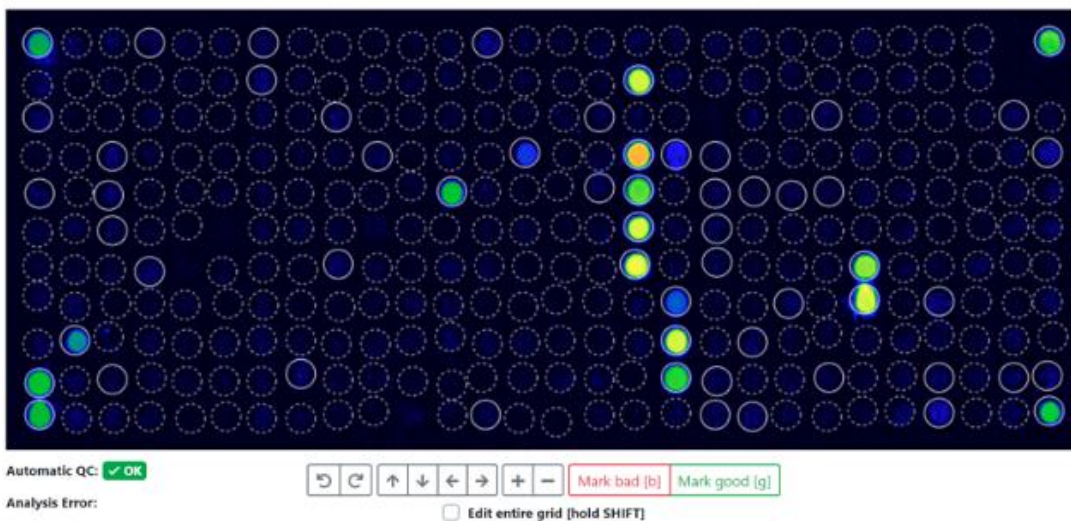
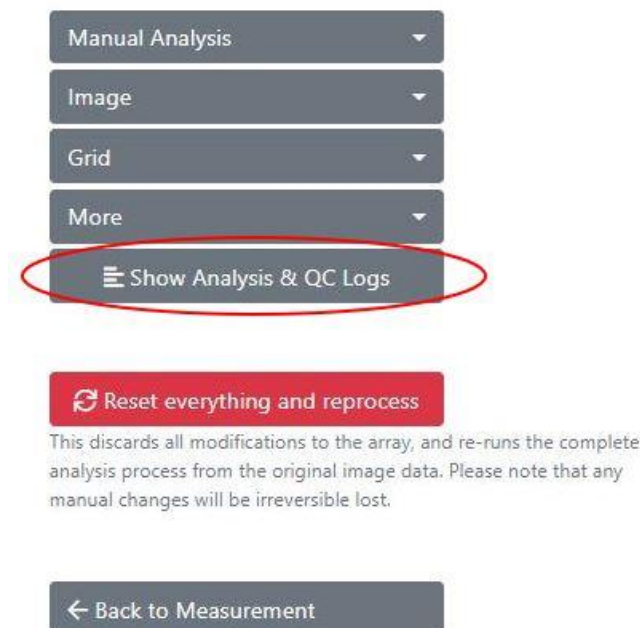
#### Analysis Log

```
'PreProcessedPixels' took 128ms.
Using definition '02_ALEXII_IVD_v1.6.0'.
'ApplyArrayDefinition' took 74ms.
Parsing calibration string ...
'ApplyDefinitions' took 75ms.
'FindGrid' took 1ms.
'AlignFeatures' took 862ms.
'CropImage' took 1ms.
'QuantifyFeatures' took 231ms.
'SubtractBackground' took 681ms.
Parsing calibration string ...
Using calibrators from QR-Code.
Total Igt (tIgt) normalized by calculating 8,330999755859375E-07x^4+0,05697999954223633x+-30,69269921875
(x=8515), which results in 515.
'ReQuantify' took 57ms.
```

#### QC Log

These are the values used for QC (ALEX v2): Curve Correction=0,9997370867894242, tIgt=8515, GD1=28085 (SNR=185.38), GD2=29369 (SNR=185.28), GD3=28569 (SNR=147.53), GD4=28385 (SNR=128.1). Reference values are as follows: Guide Dots must have a value above 5000; each Guide Dot spot must exist and be unique; Guide Dot spot must not be set to bad. Average background must be between 4000 and 16000. QC (ALEX v2) succeeded.

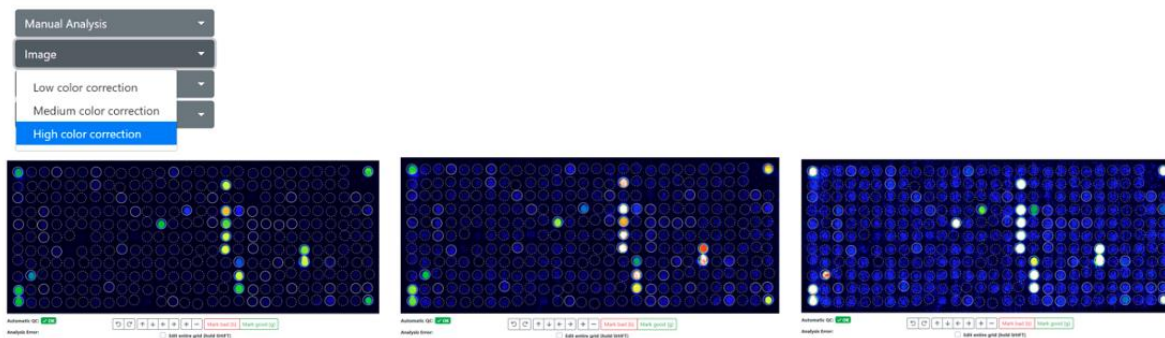
#### Background Log

Calculated background statistics for 164 pixels (from a total of 182, skipped smallest 9, and highest 9 pixels). The average is 9.419, the minimum 7973 and the maximum 11792 (minimum and maximum is calculated including skipped pixels). This results in an average difference of 628, which is an SD of 49,17 and a CV of 0,52%. For comparison, there is an overall total of 282944 pixels. The overall average is 14.871, the minimum 6.012 and the maximum 64.965.

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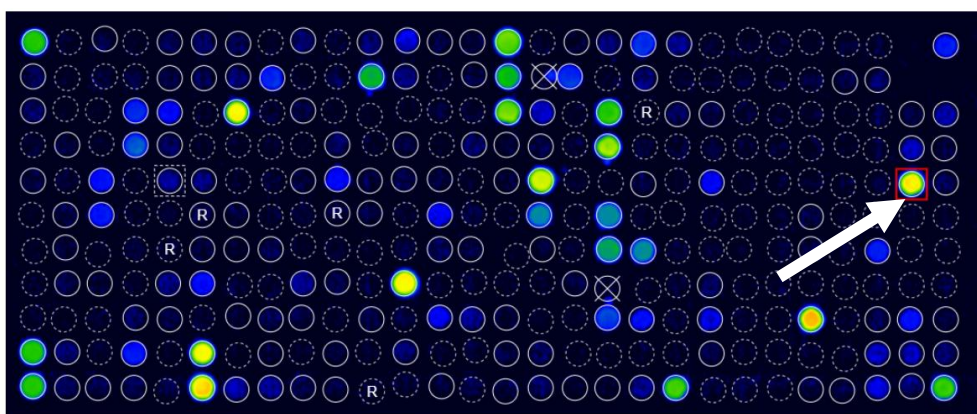
| Tissue & Exposure    |   | Feature | kUA/L  |
|----------------------|---|---------|--------|
| Pollen<br>Inhalation | E | Aca m   | ≤ 0.10 |
|                      | E | Aca s   | 7.11   |

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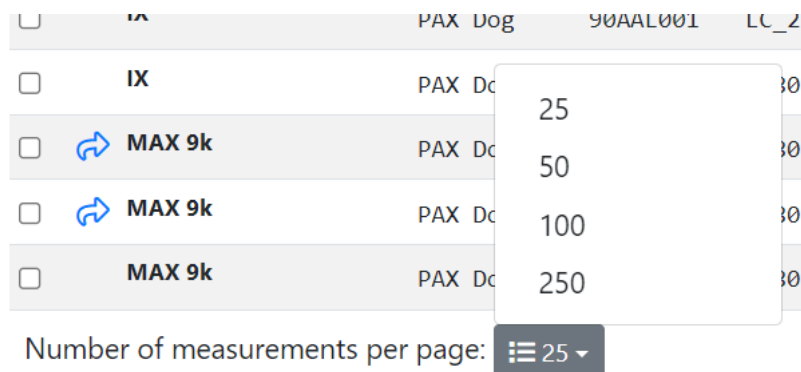
New Measurements

Search by QR-Code, patient, sample, tag, analysis date, approval date ...

|                          | Source | QC   | Tags | Product type | QR Code  | Sample Code | Patient Code | Patient Name | Analysis Date    |
|--------------------------|--------|------|------|--------------|----------|-------------|--------------|--------------|------------------|
| <input type="checkbox"/> | IX     | ✓ OK |      | ALEX2        | 02AGT957 |             |              |              | 29.04.2021 10:51 |
| <input type="checkbox"/> | IX     | ✓ OK |      | ALEX2        | 02AGT956 |             |              |              | 29.04.2021 10:55 |
| <input type="checkbox"/> | IX     | ✓ OK |      | ALEX2        | 02AGT955 |             |              |              | 29.04.2021 10:50 |
| <input type="checkbox"/> | IX     | ✓ OK |      | ALEX2        | 02AGT954 |             |              |              | 29.04.2021 10:51 |
| <input type="checkbox"/> | IX     | ✓ OK |      | ALEX2        | 02AGT953 |             |              |              | 29.04.2021 10:51 |
| <input type="checkbox"/> | IX     | ✓ OK |      | ALEX2        | 02AGT952 |             |              |              | 29.04.2021 10:50 |
| <input type="checkbox"/> | IX     | ✓ OK |      | ALEX2        | 02AGT951 |             |              |              | 29.04.2021 10:53 |
| <input type="checkbox"/> | IX     | ✓ OK |      | ALEX2        | 02AGT950 |             |              |              | 29.04.2021 10:56 |
| <input type="checkbox"/> | IX     | ✓ OK |      | ALEX2        | 02AGT949 |             |              |              | 29.04.2021 10:56 |

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## New Measurements

| Search by QR-Code, patient, sample, tag, analysis date, approval date ... |        |      |      |              |
|---------------------------------------------------------------------------|--------|------|------|--------------|
| <input type="checkbox"/>                                                  | Source | QC   | Tags | Product type |
| <input type="checkbox"/>                                                  | IX     | ✓ OK |      | ALEX2        |
| <input checked="" type="checkbox"/>                                       | IX     | ✓ OK |      | ALEX2        |
| <input checked="" type="checkbox"/>                                       | IX     | ✓ OK |      | ALEX2        |
| <input checked="" type="checkbox"/>                                       | IX     | ✓ OK |      | ALEX2        |

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## New Measurements

| Search by QR-Code, patient |  |
|----------------------------|--|
| Selected measurements ▾    |  |
| Download CSV format        |  |
| Download Excel format      |  |
| Approve                    |  |
| Update interpretation      |  |
| Choose Reporttemplate      |  |
| Modify Tags                |  |
| <b>Re-Process</b>          |  |
| » Move                     |  |
| Delete                     |  |

## Re-process measurements

You are about to re-process the following measurements. undone, the existing analysis result will be irreversibly over

| QR-Code  | Sample Code |
|----------|-------------|
| 02APK3C8 | Dummy8      |
| 02APK258 | Dummy1      |
| 02APK07F | Dummy13     |

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New Measurements

|                          | Source | QC   | Tags | Product type | QR-Code  | Sample Code |
|--------------------------|--------|------|------|--------------|----------|-------------|
| <input type="checkbox"/> | IX     | ✓ OK |      | ALEX2        | 02AGT957 |             |
| <input type="checkbox"/> | IV     | ✓ OK |      |              |          |             |

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## New Measurements

|                          | Source                                         | QC   | Tags       | Product type |
|--------------------------|------------------------------------------------|------|------------|--------------|
| <input type="checkbox"/> | Recaptured                                     | ✓ OK | recaptured | ALEX1        |
| <input type="checkbox"/> | ⚠ Recaptured                                   | ✓ OK | recaptured | ALEX1        |
| <input type="checkbox"/> | QR-Code was scanned more than once. recaptured |      |            | ALEX1        |

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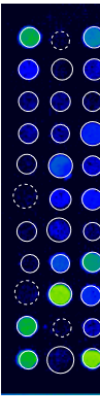
### Approve measurements

You are about to approve the following measurements.

There are measurements with one or more warnings. Please make sure to double-check these measurements before approval.

| QR-Code  | Sample Code | Analysis Date    |
|----------|-------------|------------------|
| 02AKV035 |             | 28.04.2022 11:14 |

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### Exports

Certain types of data exports (for example, the export of many PDF reports at once) take a few minutes to prepare for download. Here download your requested exports. Please note that you can keep working, all exports are done in the background and you can download when they are completed.

| Export requested   | # of measurements included | Export completed   |
|--------------------|----------------------------|--------------------|
| 8/16/2021 11:07 AM | 4                          | 0 / 4              |
| 8/16/2021 11:06 AM | 4                          | 8/16/2021 11:06 AM |
| 8/16/2021 11:04 AM | 3                          | 8/16/2021 11:04 AM |

Administration
Tenant Admin
Exports
Personal Settings
Log out
Support
Imprint
Download
Download

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**Exports**

Certain types of data exports (for example, the export of many PDF reports at a time). Please note that you can keep working, all exports are done in the background.

| Export requested | # of measurements included |
|------------------|----------------------------|
| 06.07.2022 08:52 | 3                          |

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### Export multiple measurements

Please note that **Report** and **ASTM** export is only available for certain products.

**Product Type**

**Analysis Date From**

**Tag**  

use , to separate tags

☐ **Export CSV**

**QR Code**

**Analysis Date**

**Approved or not**

☐ **Export Excel**

## Export multiple measurements

Please note that **Report** and **ASTM** export is only available for certain product types.

### Product Type

ALEX2

### QR Code

### Analysis Date From

TT . MM . JJJJ

### Analysis Date To

TT . MM . JJJJ

### Tag

### Approved by

Approved by

use , to separate tags

☐ Export CSV

☐ Export Report

 Start export ...

← Back

122 Measurements found that match your criteria

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## Edit report template

Your changes have been saved successfully.

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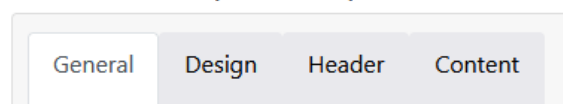
### Configure report template "New report template"

**Name:** New report template  
This is the name of the template which shows up in the dropdown list while generating a report.

**Base Template:** Version 2 (ALEX2 only) ▾  
A base template which specifies basic functionality must be chosen.

**Preferred language:** English (United States) (en-us) ▾  
A default language can be specified which will be used if there isn't yet a language stored to a measurement's report.

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PDF naming

**Setting Type:** Manual

The filename of the report PDF can be customized. Either built automatically based on your selection below or manually adjusted in the order of your needs.

**Name Pattern:** Report\_{PatientId}\_{MeasurementQrCode}

Available Elements: {PatientId} ; {PatientName} ; {SampleId} ; {MeasurementQrCode} ; {DateAndTime}

PDF naming

**Setting Type:** Automatic

The filename of the report PDF can be customized. Either built automatically based on your selection below or manually adjusted in the order of your needs.

**Include Patient ID:** ☒

**Include Patient Name:** ☒

**Include Sample ID:** ☐

**Include Measurement QR-Code:** ☐

**Include Date & Time:** ☒

Footer

**Footer text:**

Will be printed in the footer if provided and rendered as HTML.

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Configure report template "New report template"

The screenshot shows the 'Design' tab of a configuration window. It contains three main settings:

- PDF format:** Two radio buttons, 'A4' (selected) and 'Letter'. A note on the right states: 'Specifies the paper size of the output PDF.'
- Font:** A dropdown menu showing 'Arial'. A note on the right states: 'The font family of the whole report texts can be adjusted.'
- Background image:** A text input field followed by a 'Browse' button. A note on the right states: 'If set, this image is used as background on the first page of the PDF.' Below the input field, it says 'Maximum file size 1 MB'.

- Fargepalett: Grunnfargene i malen og resultatklassene kan tilpasses over hele fargepaletten
- Logoer: I tillegg til logoene på begge sider av forsidenes topptekst, kan det legges til en miniatyrlogo i toppteksten på alle sider. Som standard er dette en miniatyr av logoen på venstre side.
- Signatur: I tillegg kan et bilde av en signatur lastes opp og legges til på slutten av rapporten.
- Egendefinert intensitetsindikator (en egendefinert SVG for visning av allergenintensitet kan legges til)

## Color settings

**Color palette:** Palette 1

**Color 1:**

**Color 2:**

**Color 3:**

**Color 4:**

**First level headline color:**

**Second level headline color:**

**Third level headline color:**

## Logos

**Logo (left):**  Browse

Maximum file size 1 MB

This logo will appear on the **top left corner** of the first page of the report.

**Text (center):**

This text will appear either on the left, right, or center of the top of the first page (depending on your configured logos). Supports the placeholders `{{ PATIENT_PORTAL_QRCODE }}` and `{{ PATIENT_PORTAL_URL }}` to dynamically insert measurement information.

**Logo (right):**  Browse

Maximum file size 1 MB

This logo will appear on the **top right corner** of the first page of the report.

**Small logo:**  Browse

This logo is used on all detail pages. If not set, the default logo is used.

## Signatures

**Left:**  Browse

Maximum file size 1 MB

This signature or image will appear on the last page, left aligned.

**Center:**  Browse

Maximum file size 1 MB

This signature or image will appear on the last page, center aligned.

**Right:**  Browse

Maximum file size 1 MB

This signature or image will appear on the last page, right aligned.

## Indicators

**Custom intensity indicator:**

A custom SVG for displaying allergen intensity can be pasted here. Will be styled according to specified colors.

⚠ The SVG element must contain the attributes **width** and **height** with a minimum dimension of 16 for correct display in PDF.

💡 SVGs paths which should be colored must require the **colorize** class as well as a fill defined in the **defs** segment of the SVG.

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Configure report template "New report template"

| Header content: |  | Left half                  | Right half           |
|-----------------|--|----------------------------|----------------------|
|                 |  | Custom headline            | Custom headline      |
|                 |  | Patient Name               | Doctor Information   |
|                 |  | Patient Name               | None                 |
|                 |  | Custom headline            |                      |
|                 |  | Date of Birth              | None                 |
|                 |  | Sample ID                  | None                 |
|                 |  | QR-Code                    | Additional Info text |
|                 |  | Analysis Date              | None                 |
|                 |  | Amount of tested allergens | None                 |
|                 |  | Test method                | Quality Control text |

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| General                                  | Design                              | Header                                                                                                                                                                    | Content |
|------------------------------------------|-------------------------------------|---------------------------------------------------------------------------------------------------------------------------------------------------------------------------|---------|
| <b>Allergens to include:</b>             | All allergens                       | Specifies which kind of allergens should be included in the report.                                                                                                       |         |
| <b>Allergens sort order:</b>             | Alphabetically by allergen          | Specifies the sort order of the allergens.                                                                                                                                |         |
| <b>Include Overview:</b>                 | <input checked="" type="checkbox"/> | Whether the allergen overview should be printed or not.                                                                                                                   |         |
| <b>Include product type in overview:</b> | <input checked="" type="checkbox"/> | Whether the product type should be printed in the header of allergen overview.                                                                                            |         |
| <b>Include Total IgE:</b>                | On the front page                   | Whether the information of total IgE should be included in the report and where.                                                                                          |         |
| <b>Cross-reactive information:</b>       | Include text                        | This section will be displayed underneath the allergen list itself.                                                                                                       |         |
| <b>Include Interpretation:</b>           | Include manually                    | It can be specified if the Raven report interpretation information should be included in the report or not.<br>This section is printed underneath the allergens overview. |         |

| Additional information (below the list of results) |                                     | Those informations are integrated into the report in an extra section underneath the allergen list. |
|----------------------------------------------------|-------------------------------------|-----------------------------------------------------------------------------------------------------|
| <b>Approval date:</b>                              | <input type="checkbox"/>            |                                                                                                     |
| <b>Assay date:</b>                                 | <input type="checkbox"/>            |                                                                                                     |
| <b>Doctor info:</b>                                | <input type="checkbox"/>            | The information about the doctor which accompanied the measurement/patient.                         |
| <b>Print date:</b>                                 | <input checked="" type="checkbox"/> | When the report was generated.                                                                      |
| <b>Report info:</b>                                | <input type="checkbox"/>            | The „additional info“ field belonging to the measurement.                                           |
| <b>Sample date:</b>                                | <input checked="" type="checkbox"/> |                                                                                                     |

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### Configure report template "New report template - Duplicate"

| General                    | Design                          | Header | Content |
|----------------------------|---------------------------------|--------|---------|
| <b>Name:</b>               | New report template - Duplicate |        |         |
| <b>Support products:</b>   | ALEX <sup>2</sup>               |        |         |
| <b>Preferred language:</b> | Deutsch (de)                    |        |         |

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**General configuration**

**\* Name:** MADx

**Default timezone:** (UTC+01:00) Amsterdam, Berlin, Bern, Rome, Stockholm, Vienna  
Configure a timezone to see date and times in (instead of UTC). Please note that users can choose to use a different timezone.

**Default report template (ALEX 1):** Version 1 (ALEX1 & ALEX2)

**Default report template (ALEX 2):** Version 2 (ALEX2 only)

**Default report language:** English (United Kingdom) (en-gb)

**Default MOD:** < No MOD >

**Require ImageXplorer testarray scan:** Warn after 60 days, enforce after 90 days.

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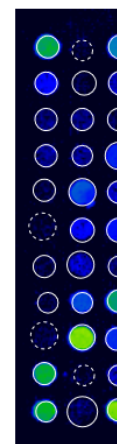
Tags:

MOD:

Report Template: Version 2 (ALEX2 only) - Deutsch (de)

Re  
ImageX  
Analysi

Deretter ser du en forhåndsvisning av rapporten, der du kan velge en rapportmal (standard eller tilpasset) og angi standardspråk. Det er imidlertid ikke mulig å laste ned denne PDF-rapporten. For dette formålet er det nødvendig å godkjenne målingen først.

Raptor

Report

Template: Version 2 (ALEX2 only)

Language: English (en)

Usually this tenant's default template and language will be used when sharing or exporting measurements. You may change the template and language associated with this measurement by saving your current configuration.

You can download this report in PDF format once the measurement has been approved.

You have access to manage the report templates and settings via the tenant administration. Please note that any changes will affect all users.

**MAD X**

PATIENT ID  
PATIENT NAME  
DATE OF BIRTH  
SAMPLE ID  
LAB CODE  
ANALYSIS ON  
TESTED ALLERGENS  
TEST METHOD CODE  
ALEX2

REFERRING PHYSICIAN

NOTE  
The internal QC (Feasibility check for QC) was within acceptance range.

Lab report: Summary on detectable sensitisations

POLLEN  
Grass Pollen

MICROORGANISMS  
Fungal Spores & Yeast

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The screenshot shows the RAPTOR web interface. At the top, there's a navigation bar with 'New measurements' and 'Approved measurements' tabs. Below, the 'Approved Measurements' section has a search bar and a 'Download selected measurements' dropdown menu. The dropdown menu is open, showing options: 'Report (PDF format)' (highlighted with a yellow circle), 'CSV format', 'Excel format', 'ASTM format', 'Update interpretation', 'Choose Reporttemplate', 'Share', 'Move', and 'Unapprove'. Below the menu is a table of measurements.

|                                     | Source | Tags | Product type | QR-Code  | Sample C |
|-------------------------------------|--------|------|--------------|----------|----------|
| <input checked="" type="checkbox"/> | IX     |      | ALEX2        | Ø2AKV39D |          |
| <input checked="" type="checkbox"/> | IX     |      | ALEX2        | Ø2AKV39D |          |
| <input checked="" type="checkbox"/> | IX     |      | ALEX2        | Ø2AKV39D |          |

Hvis du klikker på en måling, kommer du til detaljområdet for denne målingen. Der kan du også vise en forhåndsvisning av rapporten og endre innstillingene for rapportens overskrift og språk. Deretter kan du laste ned rapporten direkte derfra.

### Details

**QR-Code:** Ø2AKV39D

**Sample Code:**

**Requisition Code:**

**Sampling Date:**

**Assay Date:** 27.09.2021

**Patient Code:**

**Patient Name:**

**Date of Birth:**

**Info:**

**Doctor Info:**

**Analysis:** 27.09.2021 15:39

**Approved:** 27.09.2021 15:39

**ImageExplorer:** 23167844

**Tags:**

**MOD:**

**Report Template:** Version 2 (ALEX2 only) - Deutsch (de)

[New measurements](#)
[Approved measurements](#)
[Start measurement](#)
[Automatic](#)

## Report

Template: Default

Language: English

[Save](#)
[Use defaults](#)

Usually this tenants default template and language will be used when sharing or exporting measurements. You may change the template and language associated with this measurement by saving your current configuration.

[Download PDF](#)

[Manage report template and settings](#)

You have access to manage the report templates and settings via the tenant administration. Please note that any changes will affect all users.

|                |                                |
|----------------|--------------------------------|
| Patient ID:    |                                |
| Patient Name:  |                                |
| Date of Birth: |                                |
| Sample ID:     |                                |
| Barcode:       | 02AAE                          |
| Sampled on:    |                                |
| Tested on:     | 12/18/                         |
| Printed on:    | 12/19/                         |
| Note:          | The internal QC (Plausibility) |

Lab report: Summary on de

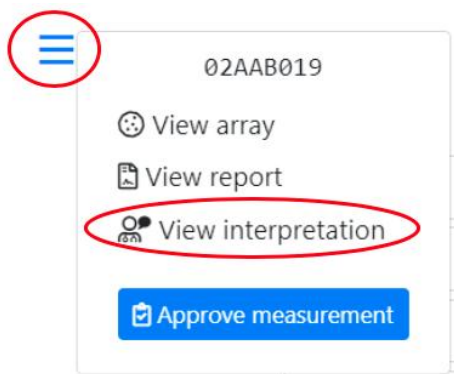
|                  |               |
|------------------|---------------|
| Pollen           | Grass Pollen  |
|                  | Tree Pollen   |
|                  | Weed Pollen   |
| Mites            | House Dust M  |
| Microorganisms   | Fungal Spores |
|                  | Legumes       |
|                  | Grain         |
| Plant-Based Food | Spices        |
|                  | Fruits        |
|                  | Vegetables    |
|                  | Nuts & Seeds  |
|                  | Milk          |

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**Rule set**

RAVEN Ruleset A for ALEX

A rule set defines the RAVEN or RAVEN<sup>2</sup> engine that is used to interpret results.

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AIT

Medication

### MADx data declaration for smartRaven

☒ I accept to enter clinical data for using the full interpretation function of smartRAVEN

### Patient has reported any symptoms

☐ Yes ☐ No

### Additional information about the patient

☐ Atopic Dermatitis

☐ Allergen-specific immunotherapy (AIT) - ongoing, completed or discontinued

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[Request new interpretation](#)

This measurement already has an interpretation stored. Usually, it should not be necessary to request those again, but it could be necessary when the patients information has changed.

#### Interpretation

##### Raven Interpretation Summary

##### Sample Information

The sample was tested on ALEX<sup>2</sup> Barcode 02AAB019, interpretation date 12/19/2019.

Of the tested 295 allergens, 25 were above the detection limit of 0.3 kU<sub>L</sub>/L. A sensitization can be an indicator of an allergy. For selected allergens, comments for interpretation are listed below.

##### Total IgE: 512 kU/L

The measured total IgE was 512 kU/L. Above a total IgE of 100 kU/L, allergy is likely.

##### Cross-Reactive Allergen Sensitization detected

Sensitization against allergen molecules which are markers of (broad) cross-reactivity between different allergen sources.

Detected cross-reactive allergen sensitizations:

- PR-10s: Aln g 1, Bet v 1, Cor a 1.0103, Fag s 1
- Polcalcins: Aln g 4, Phl p 7

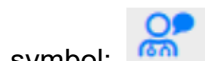
##### PR-10 Proteins (PR10)

PR-10 inhalative: The major birch pollen allergen, Bet v 1, represents the prototype of all PR-10 allergens and is the primary sensitizer in regions with birch pollen exposure. The presence of PR-10 allergens in Fagales tree pollen explains IgE cross-reactivity between pollen from hazel, alder, beech, oak and hornbeam. PR-10 nutritive: PR-10 allergens in raw fruits, nuts, vegetable and legumes can induce oral allergy syndrome and sometimes severe allergic reactions in sensitized individuals. PR-10 allergens are not stable to heat and digestion.

##### Polcalcins (Ca++ binding proteins, PC)


Members of the PC family can cause inhalative symptoms. PC allergens can be found in pollen from grasses, trees and weeds. Inhalative symptoms manifest as allergic rhino-conjunctivitis and/or allergic asthma.

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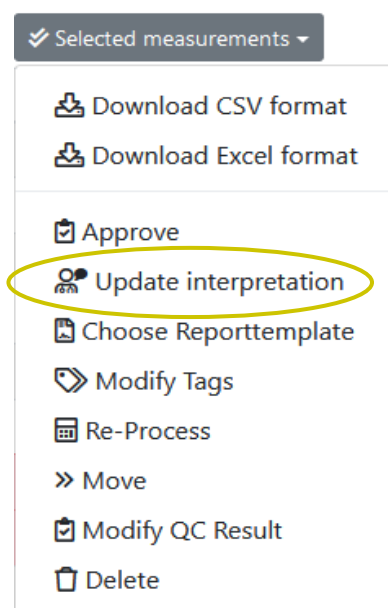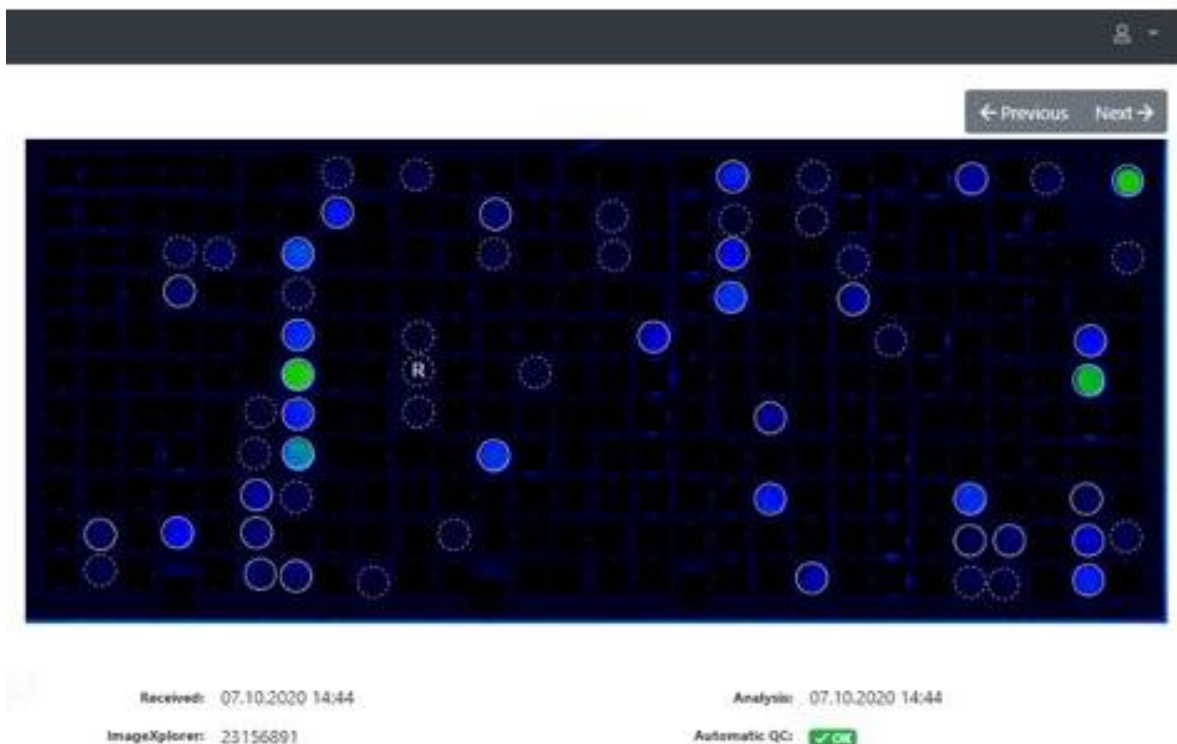
Tags:

MOD:

Report Template: Version 2 (ALEX2 only) - English (United States) (en-us)

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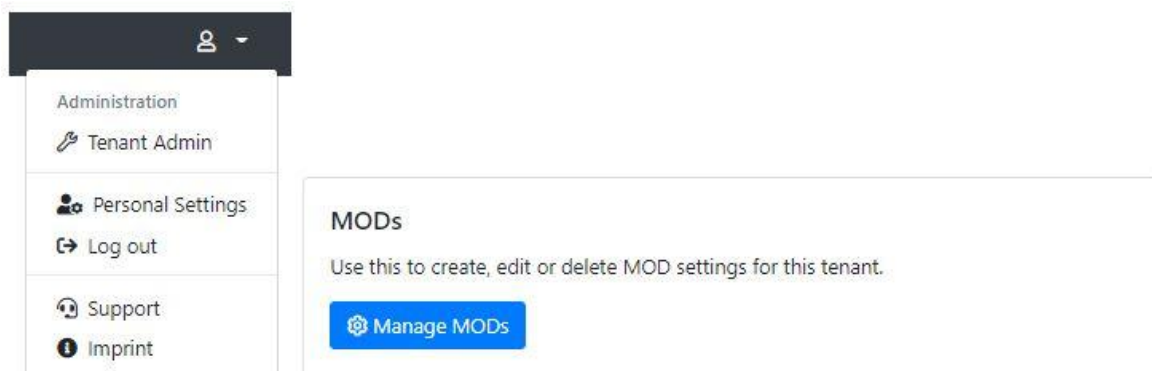
**Type:**

**Deactivated:**

You cannot delete an MOD once it is in use. But you can deactivate it, so that it's not available for new measurements anymore.

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**\* Name:**

**Default timezone:**

Configure a timezone to see date and times in (instead of UTC). Please note that users can choose to use a different timezone.

**Default report template (ALEX 1):**

**Default report template (ALEX 2):**

**Default report template (FOX):**

**Default report template (PAX):**

**Default report language:**

**Default ALEX2 MOD:**

**Default FOX MOD:**

**Default PAX MOD:**

**Force MOD:**

Requires a measurement to always have an MOD assigned. If this setting is enabled, a measurement cannot be approved without an MOD assigned.

**Hide Global MOD:**

If this setting is enabled, only the tenant-specific MOD will be available to select, and the global system-wide MOD will be hidden.

**Display MOD Excluded Spots As Research Use Only:**

If this setting is enabled, spots excluded by an MOD will not be hidden from the measurement image, but displayed as a "RUO spot".

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MOD\_ALEX\_v1.17

Force MOD:

Requires a measurement to always have an MOD assigned. If this setting is enabled, a measurement cannot be approved without an MOD assigned.

Hide Global MOD:

If this setting is enabled, only the tenant-specific MOD will be available to select, and the global system-wide MOD will be hidden.

Display MOD Excluded Spots As Research Use Only:

If this setting is enabled, spots excluded by an MOD will not be hidden from the measurement image, but displayed as a "RUO spot".

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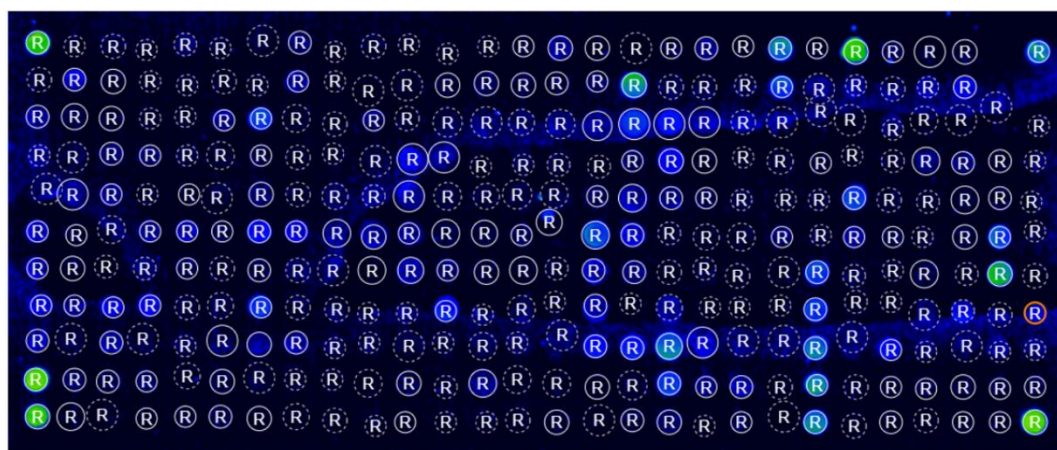
|                                                         |       |                                                                                                                                              |
|---------------------------------------------------------|-------|----------------------------------------------------------------------------------------------------------------------------------------------|
| <b>Force MOD:</b>                                       | False | Requires a measurement to always have an MOD assigned. If this setting is enabled, a measurement cannot be approved without an MOD assigned. |
| <b>Hide Global MOD:</b>                                 | False | If this setting is enabled, only the tenant-specific MOD will be available to select, and the global system-wide MOD will be hidden.         |
| <b>Display MOD Excluded Spots As Research Use Only:</b> | True  | If this setting is enabled, spots excluded by an MOD will not be hidden from the measurement image, but displayed as a "RUO spot".           |

|                            |                                                                                                                                                                                             |
|----------------------------|---------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------|
| <b>Configuration</b>       |                                                                                                                                                                                             |
| <b>Calibration string:</b> | Allows to override the calibration information from the cartridge's QR-Code. Leave empty to use the calibration information from the QR-Code on the cartridge (the usual behaviour).        |
| <b>Research spots:</b>     | Show research spots<br>If enabled, "research use only" spots will be displayed on measurement details and arrays. This does not affect the export behaviour or the reports of measurements. |

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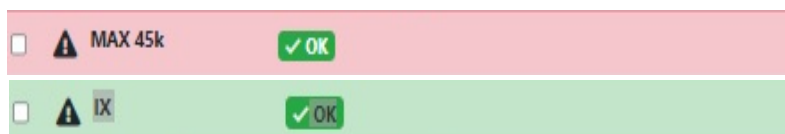
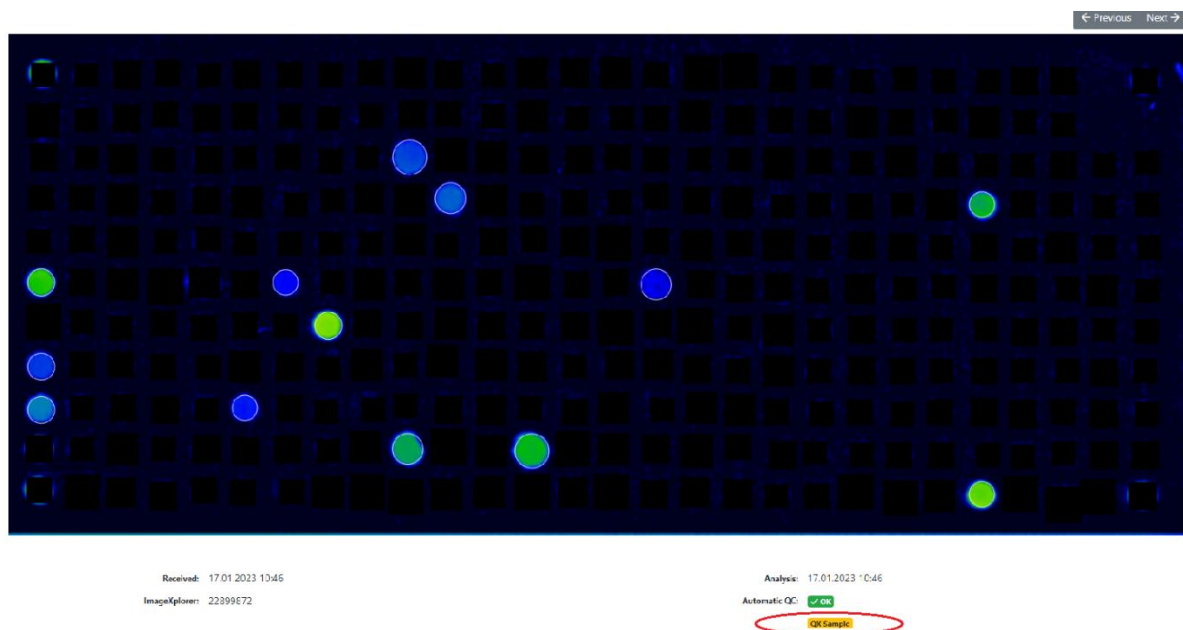
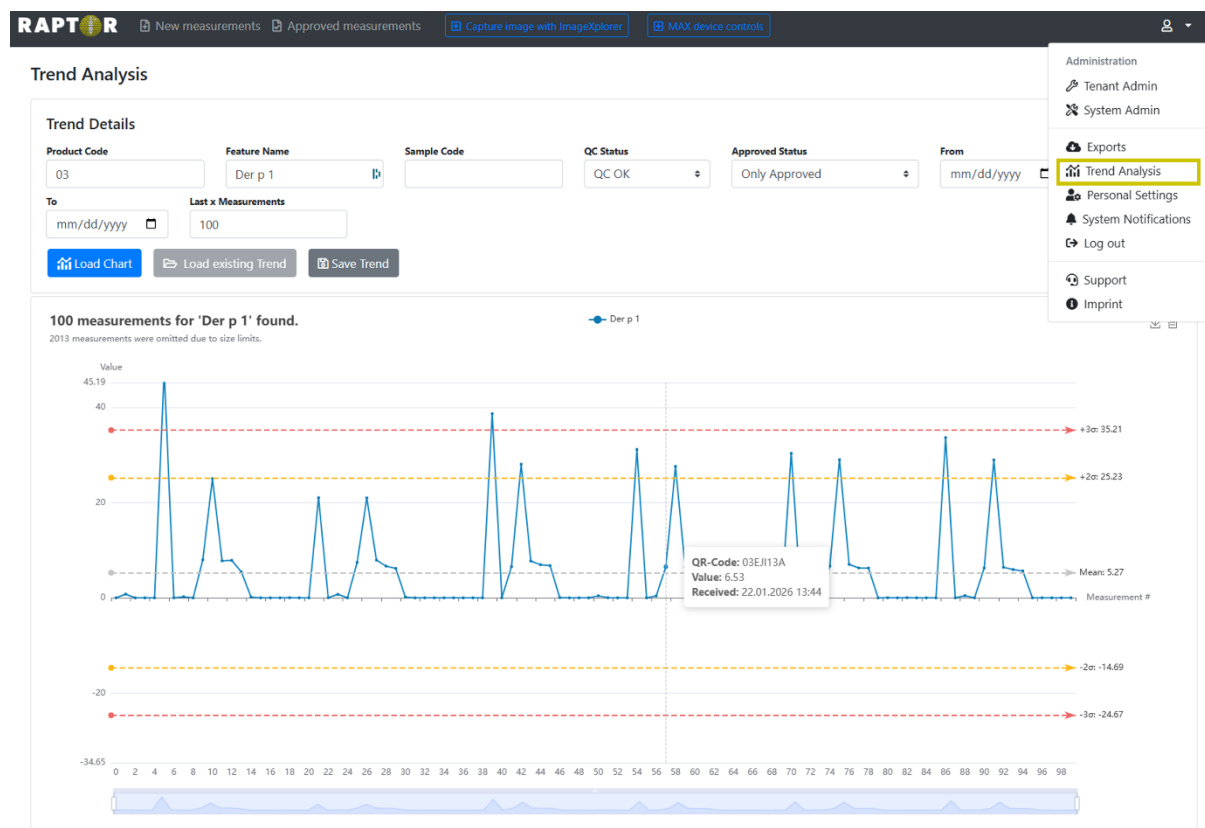
### New Measurements

Search by barcode, patient, sample, tag, analysis date, a

Selected measurements ▾

- Download CSV format
- Download Excel format
- Approve
- Update interpretation
- Re-Process
- » Move
- Delete**

| Tags | Product type |
|------|--------------|
|      | FOX          |
|      | FOX          |
|      | ALEX2        |
|      | FOX          |
|      | ALEX2        |
|      | ALEX1        |

### Delete measurements

You are about to delete the following measurements. Please note that this action is irreversible and cannot be undone.

| Barcode  | Analysis Date    |
|----------|------------------|
| 02AAB160 | 16.12.2019 13:41 |

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|                 |   |         |      |
|-----------------|---|---------|------|
| Plant Food Food | M | Jug r 3 | 0.40 |
| Plant Food Food | M | Jug r 1 | 0.38 |

|            |   |         |      |
|------------|---|---------|------|
| Plant Food | M | Jug r 3 | 0.40 |
| Food       |   |         |      |
| Plant Food | M | Jug r 1 | 0.38 |
| Food       |   |         |      |

Delete request scheduled now. [Cancel delete request](#)

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ign in. Can be

### Background jobs

View details about running background jobs and manually schedule runs.

[View dashboard](#)

### Measurement delete / unapproval log & requests

Show logging and pending requests for measurement delete and unapproval.

[Show log & requests](#)

#### Measurement unapproval / delete requests

| Open requests <a href="#">Log &amp; Finished requests</a> |                           |          |              |                                               |
|-----------------------------------------------------------|---------------------------|----------|--------------|-----------------------------------------------|
| Requested UTC                                             | Requesting User (Email)   | QR-Code  | Request Type | Action                                        |
| 11.10.2023 13:05:28                                       | Test Test (Test@test.com) | 02AYQ145 | Unapprove    | <a href="#">Accept</a> <a href="#">Reject</a> |
| 11.10.2023 12:59:27                                       | Test Test (Test@test.com) | 02AYQ14C | Delete       | <a href="#">Accept</a> <a href="#">Reject</a> |

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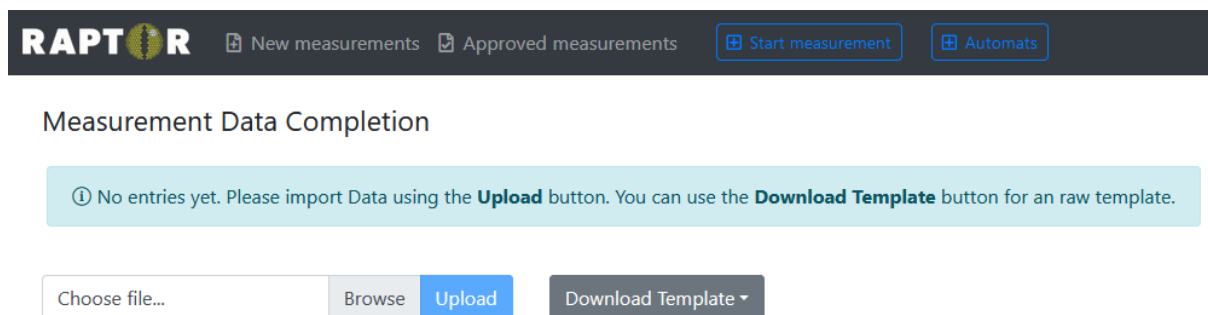
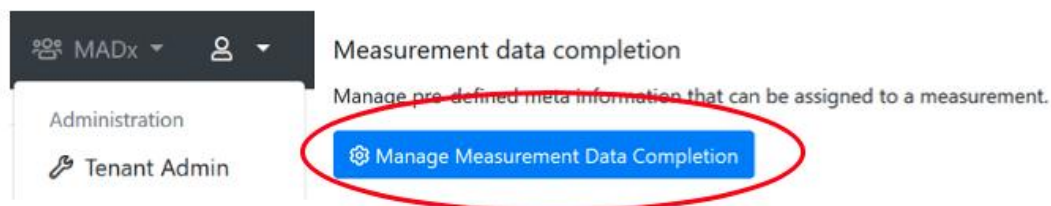
Add new user on Result Sharing Portal

General configuration

First name:

Last name:

E-Mail address:

Status:

Enabled (user can log in)

Username:

Password:

Send an invitation e-mail to user automatically

☒ **Automatically send an email summary in case new measurement(s) have been shared**  
This email is sent once per day as a summary, just in case a new measurement was shared.

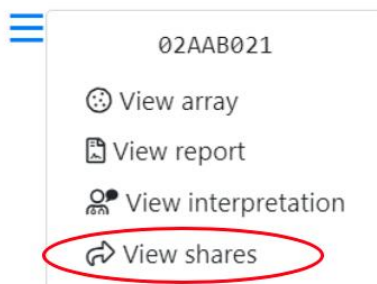
Doctor Info Text:

This information will be printed on a patients report.

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### ≡ Share

You are about to share a measurement with a user of the User Portal. Please note that the doctor will have access to the results of the measurement. Once you share a measurement you'll be provided with a link to access the shared information via the encrypted User F

**Barcode:** 02AAB021

**Share with:** X Dr. (testuser2@macroarraydx.com)

If you can't find a specific user of the User Portal, go to the [tenant administration](#) to configure the list of available users of the User Portal.

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≡ Share

↪ Share this measurement

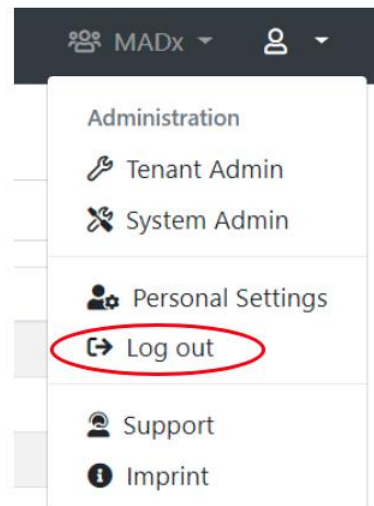
| User   | Link                                                                                                                                                                                              | Shared                             |
|--------|---------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------|------------------------------------|
| RP Bak | <a href="https://resultshare.raptor-server.com/s/07PXEpC7ZNytm4PdRfmct4NR5pUtWq5FJcqOY7Ry6oEx08USj">https://resultshare.raptor-server.com/s/07PXEpC7ZNytm4PdRfmct4NR5pUtWq5FJcqOY7Ry6oEx08USj</a> | 7 months ago <span>↪ Revoke</span> |

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### Measurements

These are the measurements that are available for you. You may view the details of each measurement, or download the report PDF file.

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## Anamneses

You can store anamneses for your patients, to assign them later to a measurement once it is available. An anamnesis then can be used to create a textual interpretation of the measurement.

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## Edit anamnesis

Edit the details of an anamnesis here. This information is especially useful to know which anamneses is for what patient.

|                                                              |                                                                |                                                                 |                                                                           |
|--------------------------------------------------------------|----------------------------------------------------------------|-----------------------------------------------------------------|---------------------------------------------------------------------------|
| <b>QR-Code</b><br><input type="text" value="E.g. 02ABC123"/> | <b>Sample Code</b><br><input type="text" value="E.g. 123456"/> | <b>Patient Code</b><br><input type="text" value="E.g. 987654"/> | <b>Patient Name</b><br><input type="text" value="E.g. Maria Musterfrau"/> |
| <b>Language</b><br><input type="text" value="English (en)"/> | <b>Region</b><br><input type="text" value="Temperate"/>        |                                                                 |                                                                           |

**Notes**

E.g. Questionnaire filled by patient via our tablet.

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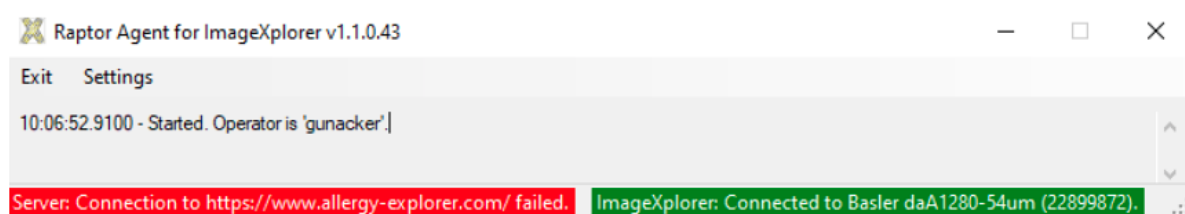
Settings ✕

Please enter the settings for your ImageXplorer. You can find your ImageXplorer key when you log in to Raptor. Please contact support if you're not sure what the proper settings are for your ImageXplorer.

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 Raptor Agent for ImageXplorer v1.1.0.43

Exit Settings Open Raptor

10:06:52.9100 - Started. Operator is 'gunacker'.  
10:11:33.9399 - Disconnecting from server ...  
10:11:35.0321 - Server confirms successful authentication to tenant MADx.]

**Server: Connected to MADx.** **ImageXplorer: Connection failed.**

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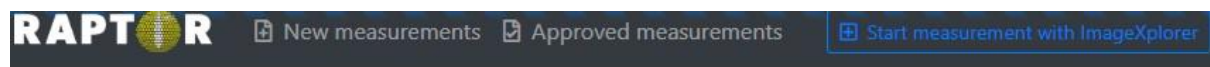
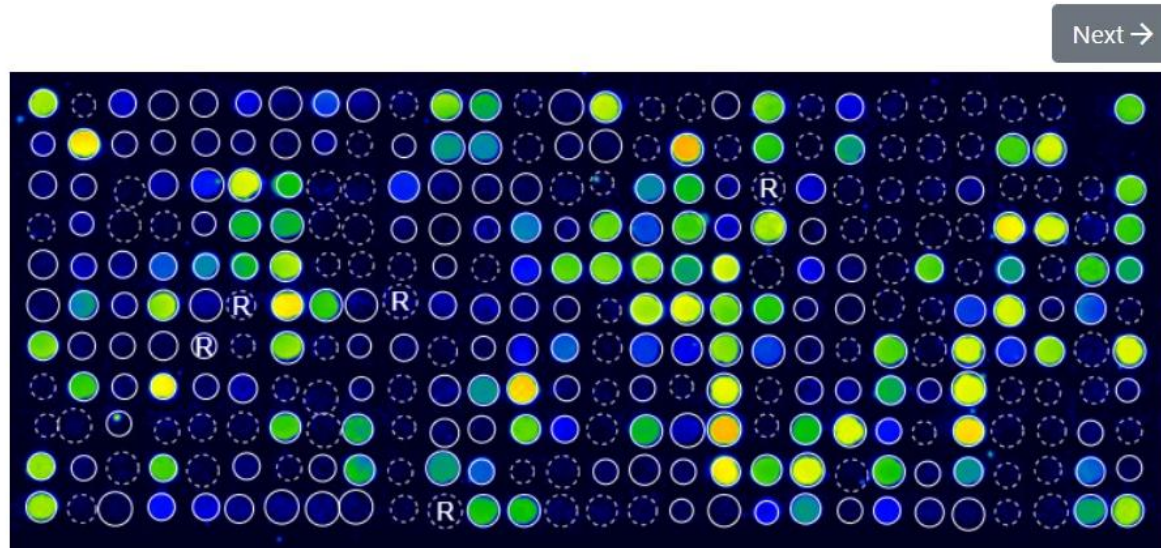
ImageXplorer: 40287249

Automatic QC: **✗ Failed**

Analysis Error: **Insufficient data transfer - check connection.**

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© Copyright by MacroArray Diagnostics

MacroArray Diagnostics (MADx)

Lemböckgasse 59, Top 4

1230 Vienna, Austria

+43 (0)1 865 2573

[www.madx.com](http://www.madx.com)

Versjonsnummer: 22-IFU-01-NO-07 (EN-27)

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