RAPTOR

SERVER ANALYSIS SOFTWARE

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TABLE DES MATIÈRES

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Welcome to RAPTOR SERVER Analysis Software, the analysis software for the [ALEX multiplex test](#).
Please log in below to continue.

Log in with Azure AD / Office 365	Log in with Username & password
If your organization integrates the software into its Microsoft Azure/Office 365 user directory (single-sign-on) click the button below. You will be redirected to the Microsoft login screen.  Log in	Please enter your username and password. Your username Your password   Log in Forgot your password? No problem, you can request a new one.

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The login form is titled "Log in with Username & password". It contains a prompt "Please enter your username and password." followed by two input fields: "Your username" and "password". The password field has an eye icon to its right. Below the fields is a blue "Log in" button with a key icon.

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The dialog is titled "Contract approval required". It contains the text: "Please review the following documents. You are required to confirm that you have read and agree with each document in order to use RAPTOR SERVER:". Below this are three checkboxes with labels: "Bitte lesen (last updated 23.11.2023 18:00)", "RAPTOR SERVER IFU (last updated 04.12.2023 11:05)", and "Terms of Service (last updated 22.11.2023 01:00)". At the bottom left is a blue "Confirm" button. Below the button is the text: "By clicking the button, I confirm that I have reviewed these documents and agree."

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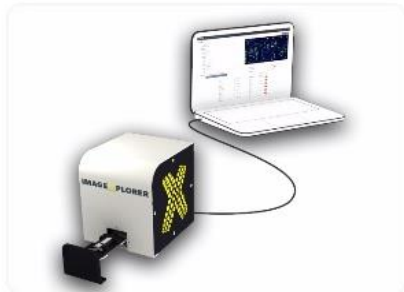
New Measurements					Last 7 days	
QC	Product type / Source	QR-Code	Sample Code	Analysis Date		
Failed	Research MAX 45k	99AFG0F7	NK FOX	30/03/2023 20:50	Details	Report
Failed	Research MAX 45k	99AFG0F8	N.G.	30/03/2023 20:50	Details	Report
Failed	Research MAX 45k	99AFG0F9	E18735	30/03/2023 20:50	Details	Report
Failed	Research MAX 45k	99AFG0FA	20220325w56	30/03/2023 20:50	Details	Report
Failed	Research MAX 45k	99AFG0EE	HM100038	30/03/2023 20:50	Details	Report
Failed	Research MAX 45k	99AFG04D	786118	30/03/2023 20:37	Details	Report
Failed	Research MAX 45k	99AFG016	HM100225	30/03/2023 20:37	Details	Report
Failed	Research MAX 45k	99AFG0CA	E17685_M	30/03/2023 20:37	Details	Report
Failed	Research MAX 45k	99AFG0CC	E17685_M	30/03/2023 20:37	Details	Report
Failed	Research MAX 45k	99AFG015	E17685_F	30/03/2023 20:37	Details	Report
In total there are 270 new measurements in the selected time period.						
View all new measurements						

Approved Measurements						
QR-Code	Product type / Source	Sample Code	Analysis Date			
02AVJ227	ALEX2	Anna LMacey	28/03/2023 21:17	Details	Report	
02ARA379	ALEX2 IX	SM_28032023	28/03/2023 14:29	Details	Report	
02ARA393	ALEX2 IX	RG_28032023	28/03/2023 14:26	Details	Report	
02AVA31F	ALEX2 MAX 45k	PR_Mar2023_11	21/03/2023 15:05	Details	Report	
02AVI3B9	ALEX2 MAX 45k	KF	17/03/2023 14:46	Details	Report	
02AYT337	ALEX2 MAX 45k	2846846+CCD	14/03/2023 19:06	Details	Report	
02ATL349	ALEX2 IX		22/02/2023 14:21	Details	Report	
02AVE383	ALEX2 MAX 45k	AO	03/02/2023 09:41	Details	Report	
02AXI29A	ALEX2 MAX 1k	OW	19/01/2023 19:12	Details	Report	
02AUY3AE	ALEX2 MAX 45k	BSZ_Okt2022_8	18/10/2022 18:12	Details	Report	
View all approved measurements						

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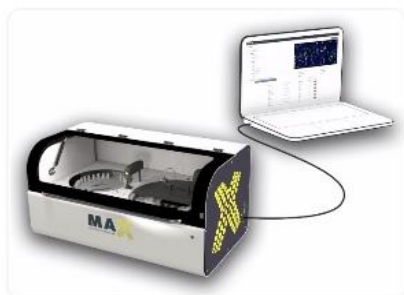
9K-Test-Instrument

Not initialized

[Initialize](#) [Shutdown](#) [Monthly Maintenance](#)

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E-Mail address:

Language:

The language is used to determine the display language.

Culture:

The culture is used to format numbers and dates.

Timezone:

Configure your timezone to see date and times in your local time instead of UTC.

Multi-Factor Authentication

[Manage MFA](#)

Use multi-factor authentication (MFA) to significantly increase security. It is highly recommended to enable MFA; this adds a second layer of security against phishing attacks or weak passwords.


Configuration

Research spots:

If enabled, "research use only" arrays. This does not affect the

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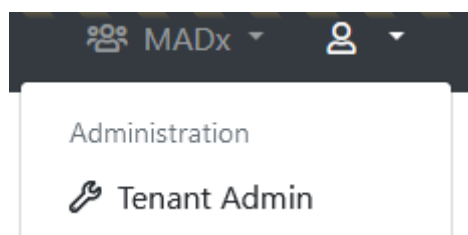Configure "MADx"

General configuration	
* Name:	MADx
Default timezone:	(UTC+01:00) Amsterdam, Berlin, Bern, Ri ⌵ Configure a timezone to see date and times in (instead of UTC). Please note that users can choose to use a different timezone.
Default report template (ALEX2):	ALEX2 ⌵
Default report template (ALEX3, ALEX Air, ALEX Food):	check 1.21 ⌵
Default report template (FOX):	FOX (2025) ⌵
Default report template (PAX):	PAX ⌵
Default report language:	English (en) ⌵
Default ALEX MOD:	< No MOD > ⌵
Default FOX MOD:	< No MOD > ⌵
Default PAX MOD:	< No MOD > ⌵
Force MOD:	False ⌵ Requires a measurement to always have an MOD assigned. If this setting is enabled, a measurement cannot be approved without an MOD assigned.
Hide Global MOD:	False ⌵ If this setting is enabled, only the tenant-specific MOD will be available to select, and the global system-wide MOD will be hidden.
Display MOD Excluded Spots As Research Use Only:	False ⌵ If this setting is enabled, spots excluded by an MOD will not be hidden from the measurement image, but displayed as a "RUO spot".
MFA:	☐ Force MFA If this setting is enabled, users are forced to set up an MFA on sign-in, in case they haven't yet.

MAX device default settings	
* Default Minimum RunId:	10 ⌵ When a new MAX device is created, this value will be used for its Minimum Run Id

Tenant-specific batch releases	
Restrict this tenant to only allow the following batches to be scanned or to restrict certain batches. Enter one per line.	
Use:	☒ Whitelist ☐ Blacklist
Whitelisted Batches:	 Enter one batch code including product identifier per line, for example "02AAA"
Blacklisted Batches:	 Enter one batch code including product identifier per line, for example "02AAA"

Company Details

* Legal Entity Name:	MacroArray Diagnostics
* Street and Number:	Street 4
* Zip Code:	1230
* City:	New York
* Country:	United States of America ⌵
* Phone:	+4318652573
* E-Mail:	office@macroarraydx.com
* Legal Representative:	Dr. Christian Harwanegg
Website:	
Vat Number:	
Company Register Number:	

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Tenant-specific batch releases
Restrict this tenant to only allow the following batches to be scanned or to restrict certain batches. Enter one per line.
Use:
☒ **Whitelist**
☐ **Blacklist**

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You requested a new measurement from ImageXplorer "IX Trainingsraum". Please wait until the measurement is completed ...

Invalid batch detected! The measurement can't be processed because its batch is blocked.

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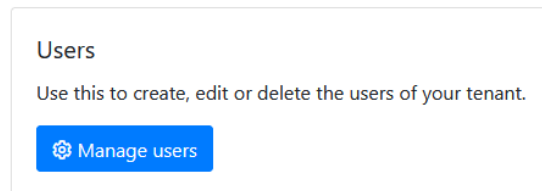
Login Credentials

Type: Username and Password

Please select if the user should be validated against Azure AD (Office 365), or if you want to specify username and password yourself. It is recommended to use Azure AD whenever possible. This allows for increased security by supporting two-factor authentication. Also, it is more convenient for the user, because he or she does not have to remember a new username and password (single-sign-on).

Username:

It is a good idea to use the users e-mail-address as his username, because the username needs to be unique across the entire system.

Password:

Automatically send invitation e-mail

Access Rights

Preconfigured groups

Use these pre-configured groups to automatically assign a set of access rights to this user. You can still add or remove specific access rights.

Reset Lab User Super Lab User Support/Distributor User System Administrator

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Tenants

This user is connected to the following tenants and has the following access rights:

☐ Tenants	Can share measurements	Can edit measurements	Can edit personal data	Can approve measurements	Can manage basic tenant configuration	Can manage API keys	Can manage ImageExplorers	Can manage MAX devices	Can view agent connections	Can manage users	Can manage users of Result Sharing Portal	Can manage report templates	Can manage measurement data completion	Can unapprove specific batch releases	System Administration	Can use a custom calibration string	Can manage Run Configuration	Can skip
☐ Another MADx Tenant	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓
☐ MADx Tenant	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓

Additionally this user can be added to the following tenants by checking the tenant name and hitting save afterwards. User rights w applied from current open tenant user.

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Additionally this user can be added to the following tenants by checking the tenant name and hitting save afterwards. User right applied from current open tenant user.

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Tenants

This user is connected to the following tenants and has the following access rights:

☐	Tenants	Can share measurements	Can edit measurements	Can edit personal data	Can approve measurements	Can manage basic tenant configuration	Can manage API keys	Can manage ImageX
☒	Another MADx Tenant	☒	☒	☒	☒	☒	☒	☒

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ImageXplorers

Use this to configure ImageXplorers, or add new ones. Every ImageXplorer needs to be known to Raptor-Server, otherwise it won't be able to connect.

Manage ImageXplorers

Add new ImageXplorer

General information

Name:

Key:

This key is required when installing an Agent.

ImageXplorer Serialnumber:

Serialnumber of connected ImageXplorer.

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Details

Barcode: 02AE3408
Sample Code: 12
Requisition Code:
Sampling Date: 01.10.2021
Assay Date: 04.10.2021
Patient Code: TE
Patient Name: Test
Date of Birth: 17.05.2010
Info:
Doctor Info: Dr. A
Choose a doctor...
Tags: Tag Info
MOD: < No MOD >
Report Template: Version 2 (ALEX2 only) - Deutsch (de)

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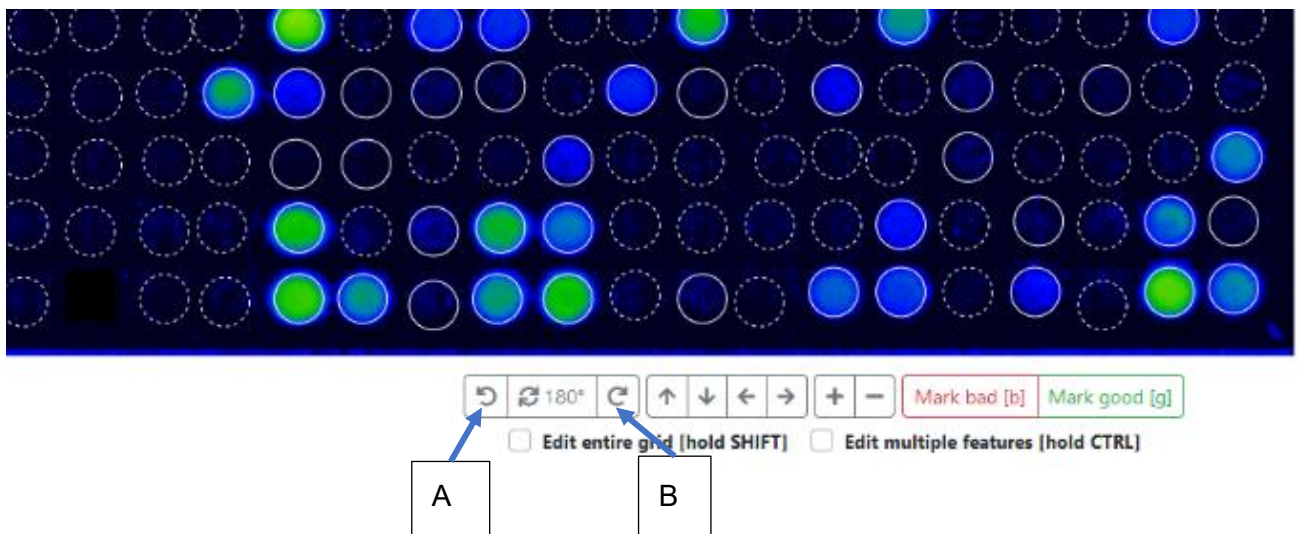
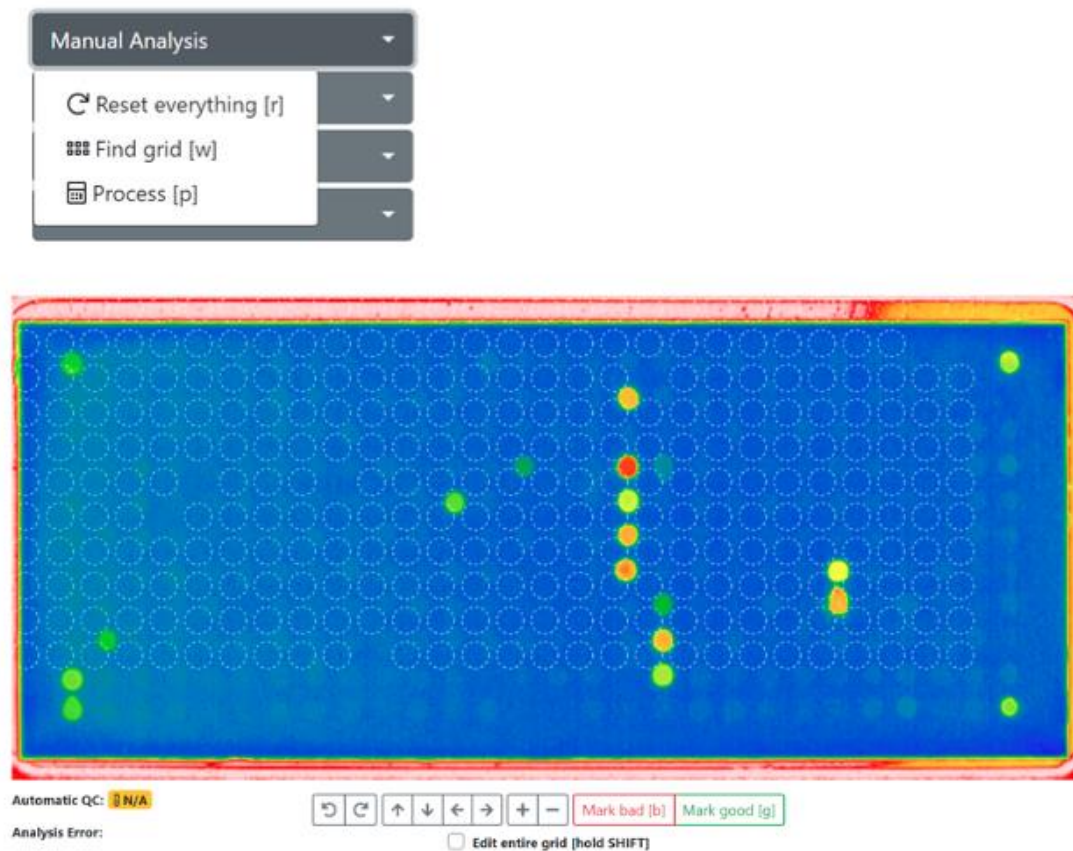
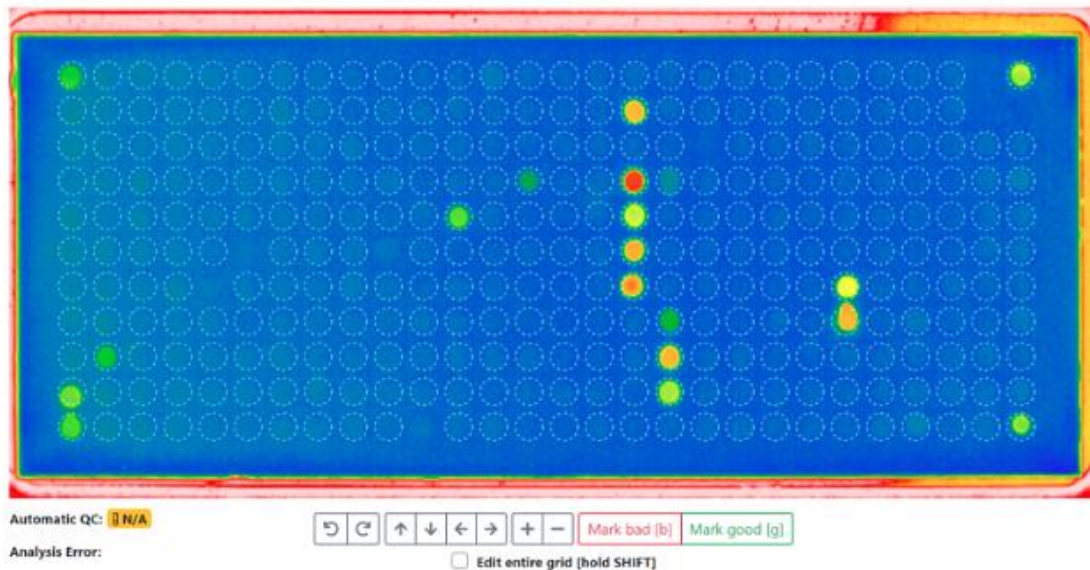
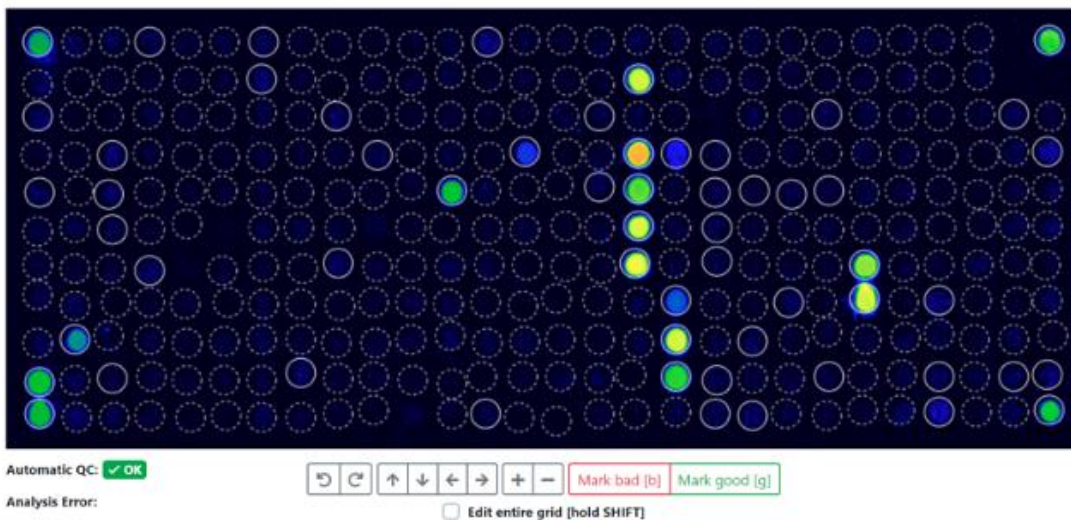
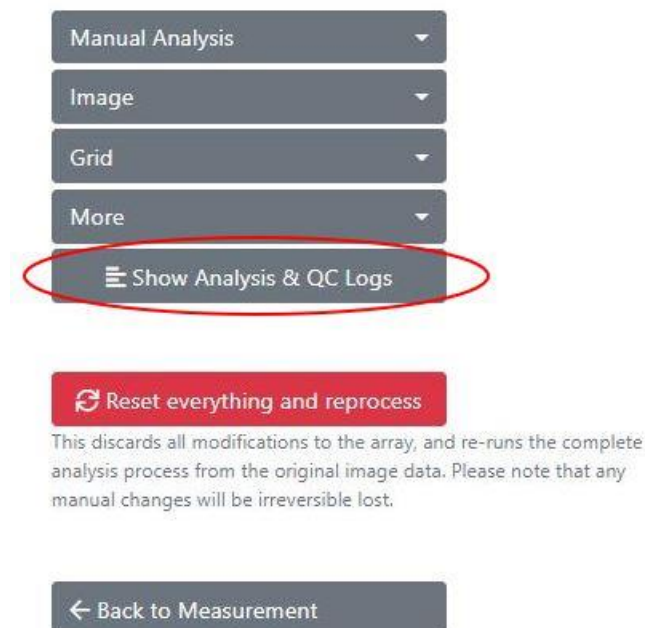
Analysis Log

```
'PreProcessedPixels' took 128ms.
Using definition '02_ALEXII_IVD_v1.6.0'.
'ApplyArrayDefinition' took 74ms.
Parsing calibration string ...
'ApplyDefinitions' took 75ms.
'FindGrid' took 1ms.
'AlignFeatures' took 862ms.
'CropImage' took 1ms.
'QuantifyFeatures' took 231ms.
'SubtractBackground' took 681ms.
Parsing calibration string ...
Using calibrators from QR-Code.
Total IgG (tIgG) normalized by calculating 8,330999755859375E-07x*+0,05697999954223633x--38,69269921875
(x=8515), which results in 515.
'ReQuantify' took 57ms.
```

QC Log

These are the values used for QC (ALEX v2): Curve Correction=0,9997370867894242, tIgG=8515, GD1=28985 (508=185.38), GD2=29369 (508=185.28), GD3=28569 (508=147.53), GD4=28305 (508=128.1). Reference values are as follows: Guide Dots must have a value above 5000; each Guide Dot spot must exist and be unique; Guide Dot spot must not be set to bad. Average background must be between 4000 and 16000. QC (ALEX v2) succeeded.

Background Log

Calculated background statistics for 164 pixels (from a total of 182, skipped smallest 9, and highest 9 pixels). The average is 9.419, the minimum 7973 and the maximum 11792 (minimum and maximum is calculated including skipped pixels). This results in an average difference of 628, which is an SD of 49,17 and a CV of 0,52%. For comparison, there is an overall total of 282944 pixels. The overall average is 14,071, the minimum 6,012 and the maximum 64,965.

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
	All allergens	☑ All allergens
	Only positive allergens	
Tissue & Source	Feature	kUA/L
Pollen Inhalation	E Aca m	≤ 0.10
Mites	E Aca s	7.11

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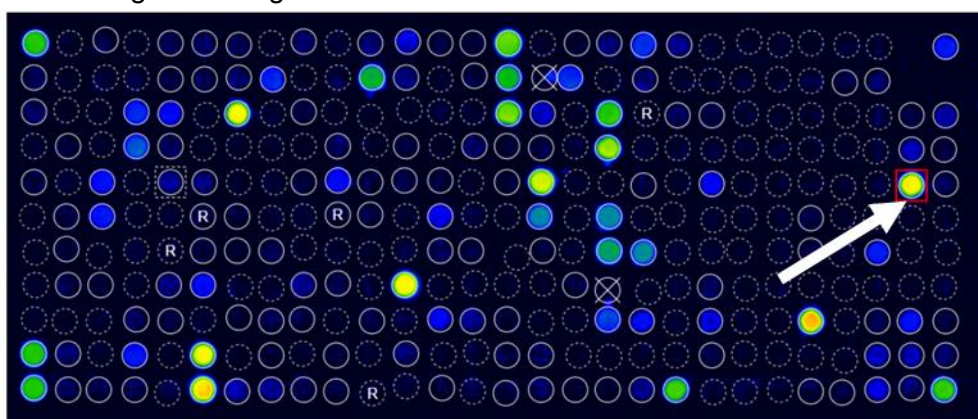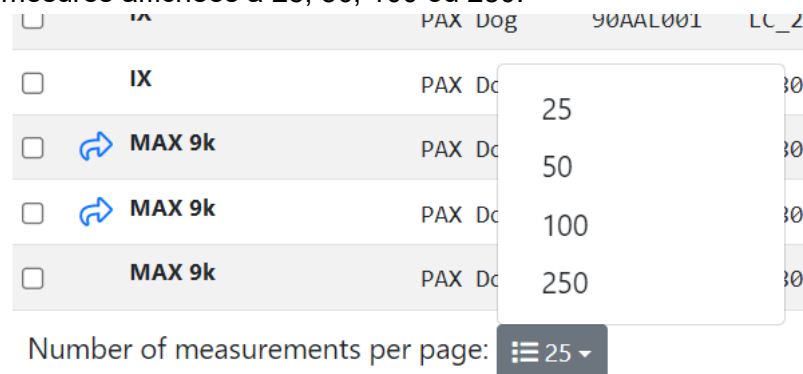
New Measurements

Search by QR-Code, patient, sample, tag, analysis date, approval date ...				
☐	Source	QC	Tags	Product type
☐	IX	✓ OK		ALEX2
☒	IX	✓ OK		ALEX2
☒	IX	✓ OK		ALEX2
☒	IX	✓ OK		ALEX2

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New Measurements

Search by QR-Code, patient	
Selected measurements ▾	
☐	
☒	
☒	
☒	

Re-process measurements

You are about to re-process the following measurements. undone, the existing analysis result will be irreversibly over

QR-Code	Sample Code
02APK3C8	Dummy8
02APK258	Dummy1
02APK07F	Dummy13

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New Measurements

Search by QR-Code, patient, sample, tag, analysis date, approval date ...

☐	Source	QC	Tags	Product type
☐	Recaptured	✓ OK	recaptured	ALEX1
☐	⚠ Recaptured	✓ OK	recaptured	ALEX1
☐	QR-Code was scanned more than once.		recaptured	ALEX1

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RAPTOR New measurements Approved measur

New Measurements

Search by QR-Code, patient, sample, tag, analysis date, approval date ...

Selected measurements

☐	Source	QC	Tags	Product type	QR-Code
☒	IX	✓ OK		ALEX2	02AMC180
☒	IX	✓ OK		ALEX2	02AMC078
☒	IX	✓ OK		ALEX2	02AKV2D9

Selected measurements

- Download CSV format
- Download Excel format
- Approve**
- Update interpretation
- Re-Process
- Move
- Delete

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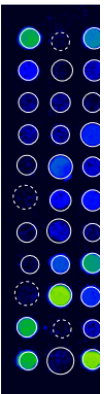
Approve measurements

You are about to approve the following measurements.

There are measurements with one or more warnings. Please make sure to double-check these measurements before approval.

QR-Code	Sample Code	Analysis Date
02AKV035		28.04.2022 11:14

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[New measurements](#)
[Approved measurements](#)
[Start m...](#)

Approved Measurements

Search by QR-Code, patient, sample, tag, analysis date ...

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MADx Testtenant

Exports

Certain types of data exports (for example, the export of many PDF reports at once) take a few minutes to prepare for download. Here download your requested exports. Please note that you can keep working, all exports are done in the background and you can download them when they are completed.

Export requested	# of measurements included	Export completed
8/16/2021 11:07 AM	4	0 / 4
8/16/2021 11:06 AM	4	8/16/2021 11:06 AM
8/16/2021 11:04 AM	3	8/16/2021 11:04 AM

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Exports

Certain types of data exports (for example, the export of many PDF reports at a time) are available. Please note that you can keep working, all exports are done in the background.

Export requested	# of measurements included
06.07.2022 08:52	3

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Export multiple measurements

Please note that **Report PDF** and **ASTM** export is only available for certain data types.

QR-Code/Product/Batch:

Analysis Date (from):

Analysis Date (to):

Tag:

Status:

☐ Export CSV ☐ Export Excel

Export multiple measurements

Please note that **Report PDF** and **ASTM** export is only available for single measurements.

QR-Code/Product/Batch

02aaa

Analysis Date (from)

TT . MM . JJJJ

Analysis Date (to)

TT . MM . JJJJ

Tag

Status

All

☐ Export CSV

☐ Export Excel

 Start export ...

← Back

66 Measurements found that match your criteria

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Edit report template

Your changes have been saved successfully.

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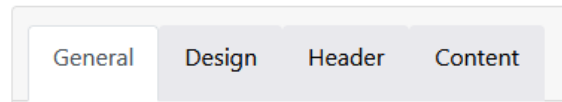
Configure report template "New report template"

Field	Value	Description
Name:	New report template	This is the name of the template which shows up in the dropdown list while generating a report.
Base Template:	Version 2 (ALEX2 only)	A base template which specifies basic functionality must be chosen.
Preferred language:	English (United States) (en-us)	A default language can be specified which will be used if there isn't yet a language stored to a measurement's report.

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PDF naming

Setting Type: Manual

The filename of the report PDF can be customized. Either built automatically based on your selection below or manually adjusted in the order of your needs.

Name Pattern: Report_{PatientId}_{MeasurementQrCode}

Available Elements: {PatientId} ; {PatientName} ; {SampleId} ; {MeasurementQrCode} ; {DateAndTime}

PDF naming

Setting Type: Automatic

The filename of the report PDF can be customized. Either built automatically based on your selection below or manually adjusted in the order of your needs.

Include Patient ID: ☒

Include Patient Name: ☒

Include Sample ID: ☐

Include Measurement QR-Code: ☐

Include Date & Time: ☒

Footer

Footer text:

Will be printed in the footer if provided and rendered as HTML.

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Configure report template "New report template"

The screenshot shows the 'Design' tab of a configuration interface. It includes three main sections: 'PDF format' with radio buttons for 'A4' (selected) and 'Letter'; 'Font' with a dropdown menu showing 'Arial'; and 'Background image' with an empty text box and a 'Browse' button. To the right of each section is a descriptive note. Below the 'Background image' field, it states 'Maximum file size 1 MB'.

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Browse

Maximum file size 1 MB

This logo will appear on the **top left corner** of the first page of the report.

This text will appear either on the left, right, or center of the top of the first page (depending on your configured logos).
Supports the placeholders `{{ PATIENT_PORTAL_QR_CODE }}` and `{{ PATIENT_PORTAL_URL }}` to dynamically insert measurement information.

This logo will appear on the **top right corner** of the first page of the report.

This logo is used on all detail pages. If not set, the default logo is used.

Signatures

Left:

Browse

Maximum file size 1 MB

Center:

Browse

Maximum file size 1 MB

Right:

Browse

Maximum file size 1 MB

This signature or image will appear on the last page, left aligned.

This signature or image will appear on the last page, center aligned.

This signature or image will appear on the last page, right aligned.

Indicators

Custom intensity indicator:

Maximum file size 1 MB

A custom SVG for displaying allergen intensity can be pasted here. Will be styled according to specified colors.

The SVG element must contain the attributes **width** and **height** with a minimum dimension of 16 for correct display in PDF.

SVGs paths which should be colored must require the **colorize** class as well as a fill defined in the **defs** segment of the SVG.

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Configure report template "New report template"

Header content:		Left half	Right half
		Custom headline	Custom headline
Patient Name	☐	Doctor Information	☐
Patient Name	☒	None	☐
Custom headline			
Date of Birth	☐	None	☐
Sample ID	☐	None	☐
QR-Code	☐	Additional Info text	☐
Analysis Date	☐	None	☐
Amount of tested allergens	☐	None	☐
Test method	☐	Quality Control text	☐

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General
Design
Header
Content

Allergens to include:
All allergens

Allergens sort order:
Alphabetically by allergen

Include Overview:
☒

Include product type in overview:
☒

Include Total IgE:
On the front page

Cross-reactive information:
Include text

Include Interpretation:
Include manually

Specifies which kind of allergens should be included in the report.

Specifies the sort order of the allergens.

Whether the allergen overview should be printed or not.

Whether the product type should be printed in the header of allergen overview.

Whether the information of total IgE should be included in the report and where.

This section will be displayed underneath the allergen list itself.

It can be specified if the Raven report interpretation information should be included in the report or not. This section is printed underneath the allergens overview.

Additional information (below the list of results)
Those informations are integrated into the report in an extra section underneath the allergen list.

Approval date:
☐

Assay date:
☐

Doctor info:
☐

Print date:
☒

Report info:
☐

Sample date:
☒

The information about the doctor which accompanied the measurement/patient.

When the report was generated.

The „additional info“ field belonging to the measurement.

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Configure report template "New report template - Duplicate"

General	Design	Header	Content
Name: New report template - Duplicate			
Support products: ALEX<sup>2</sup			
Preferred language: Deutsch (de)			

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General configuration

* Name:	MADx
Default timezone:	(UTC+01:00) Amsterdam, Berlin, Bern, Rome, Stockholm, Vienna Configure a timezone to see date and times in (instead of UTC). Please note that users can choose to use a different timezone.
Default report template (ALEX 1):	Version 1 (ALEX1 & ALEX2)
Default report template (ALEX 2):	Version 2 (ALEX2 only)
Default report language:	English (United Kingdom) (en-gb)
Default MOD:	< No MOD >
Require ImageXplorer testarray scan: Warn after 60 days, enforce after 90 days.	

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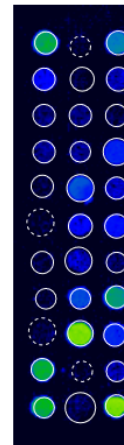
Raptor New measurements Approved measurements Start measurement Automate

Report

Template: Version 2 (ALEX2 only)

Language: English (en)

Usually this tenant's default template and language will be used when sharing or exporting measurements. You may change the template and language associated with this measurement by saving your current configuration.

You can download this report in PDF format once the measurement has been approved.

You have access to manage the report templates and settings via the tenant administration. Please note that any changes will affect all users.

MADX
MACROARRAY DIAGNOSTICS

PATIENT ID
PATIENT NAME
DATE OF BIRTH
SAMPLE ID
SWABCODE
02AD0042
ANALYSED ON
11/04/2022
TESTED ALLERGENS
306
TESTMETHODE
ALEX2

REFERRING PHYSICIAN

NOTE
The internal QC (Pauzability check for Q2) was within acceptance range.

Lab report: Summary on detectable sensitisations

POLLEN
Grass Pollen

MICROORGANISMS
Fungal Spores & Yeast

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The screenshot shows the 'RAPTOR' web application interface. At the top, there are tabs for 'New measurements', 'Approved measurements', and 'Start m'. Below the tabs, the 'Approved Measurements' section is visible, featuring a search bar and a 'Download selected measurements' button. A table lists three measurements, all with 'Source' 'IX', 'Product type' 'ALEX2', and 'QR-Code' '02AKV39D'. A dropdown menu is open, showing options: 'Report (PDF format)' (highlighted with a yellow circle), 'CSV format', 'Excel format', 'ASTM format', 'Update interpretation', 'Choose Reporttemplate', 'Share', 'Move', and 'Unapprove'.

	Source	Tags	Product type	QR-Code	Sample C
☒	IX		ALEX2	02AKV39D	
☒	IX		ALEX2	02AKV39D	
☒	IX		ALEX2	02AKV39D	

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Tags:

MOD:

Report Template: Version 2 (ALEX2 only) - Deutsch (de)

Download ▾

Report

Save

» Move to other tenant

[New measurements](#)
[Approved measurements](#)
[Start measurement](#)
[Automats](#)

Report

Template:

Default

Language:

English

Save

Use defaults

Usually this tenants default template and language will be used when sharing or exporting measurements. You may change the template and language associated with this measurement by saving your current configuration.

Download PDF

Manage report template and settings

You have access to manage the report templates and settings via the tenant administration. Please note that any changes will affect all users.

MACRO ARRAY DIAGNOSTICS

Patient ID:	
Patient Name:	
Date of Birth:	
Sample ID:	
Barcode:	02AAE
Sampled on:	
Tested on:	12/18/
Printed on:	12/19/
Note:	The internal QC (Plausibility)

Lab report: Summary on de


Pollen	Grass Pollen
	Tree Pollen
	Weed Pollen
Mites	House Dust M
Microorganisms	Fungal Spores
	Legumes
	Grain
	Spices
Plant-Based Food	Fruits
	Vegetables
	Nuts & Seeds
	Milk

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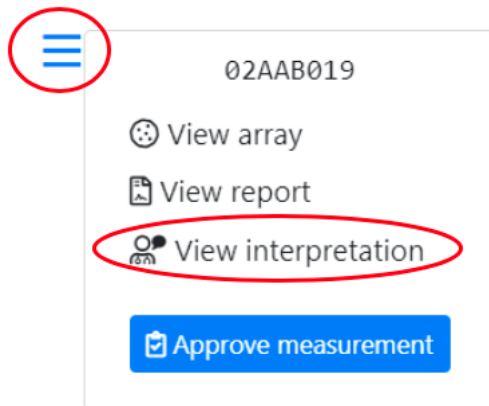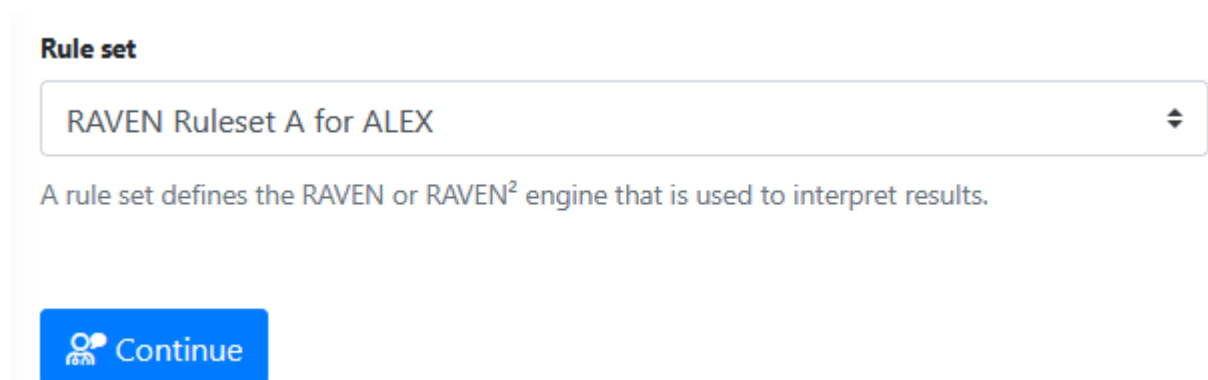
AIT

Medication

MADx data declaration for smartRaven

☒ I accept to enter clinical data for using the full interpretation function of smartRAVEN

Patient has reported any symptoms

☐ Yes ☐ No

Additional information about the patient

☐ Atopic Dermatitis

☐ Allergen-specific immunotherapy (AIT) - ongoing, completed or discontinued

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[Request new interpretation](#)

This measurement already has an interpretation stored. Usually, it should not be necessary to request those again, but it could be necessary when the patients information has changed.

Interpretation

Raven Interpretation Summary

Sample Information

The sample was tested on ALEX² Barcode 02AAB019, interpretation date 12/19/2019.

Of the tested 295 allergens, 25 were above the detection limit of 0.3 kU_L/L. A sensitization can be an indicator of an allergy. For selected allergens, comments for interpretation are listed below.

Total IgE: 512 kU/L

The measured total IgE was 512 kU/L. Above a total IgE of 100 kU/L, allergy is likely.

Cross-Reactive Allergen Sensitization detected

Sensitization against allergen molecules which are markers of (broad) cross-reactivity between different allergen sources.

Detected cross-reactive allergen sensitizations:

- PR-10s: Aln g 1, Bet v 1, Cor a 1.0103, Fag s 1
- Polcalcins: Aln g 4, Phi p 7

PR-10 Proteins (PR10)

PR-10 inhalative: The major birch pollen allergen, Bet v 1, represents the prototype of all PR-10 allergens and is the primary sensitizer in regions with birch pollen exposure. The presence of PR-10 allergens in Fagales tree pollen explains IgE cross-reactivity between pollen from hazel, alder, beech, oak and hornbeam. PR-10 nutritive: PR-10 allergens in raw fruits, nuts, vegetable and legumes can induce oral allergy syndrome and sometimes severe allergic reactions in sensitized individuals. PR-10 allergens are not stable to heat and digestion.

Polcalcins (Ca++ binding proteins, PC)

Members of the PC family can cause inhalative symptoms. PC allergens can be found in pollen from grasses, trees and weeds. Inhalative symptoms manifest as allergic rhino-conjunctivitis and/or allergic asthma.

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The screenshot displays the 'Raptor' software interface. At the top, there are navigation tabs: 'New measurements', 'Approved measurements', 'Start measurement', and 'Automate'. Below this is a 'Details' section with various input fields: 'Barcode' (02461852), 'Sample Code', 'Sampling Date' (tt.mm.aaa), 'Assay Date' (07.10.2020), 'Patient Code', 'Patient Name', 'Date of Birth' (tt.mm.aaa), 'Info', 'Doctor Info', and 'Tags'. A red oval highlights the 'MOD' field, which is set to 'Pollen allergens'. Below the 'MOD' field are buttons for 'Report', 'Save', and 'Save and approve'. Below the form is a large image of a microarray spot grid. The grid shows numerous blue spots, with a few green spots indicating positive results. A red 'R' is visible in the center of the grid. At the bottom of the image, there is a status bar with the following information: 'Received: 07.10.2020 14:44', 'ImageXplorers: 23156891', 'Analysis: 07.10.2020 14:44', and 'Automatic QC: ✓ OK'.

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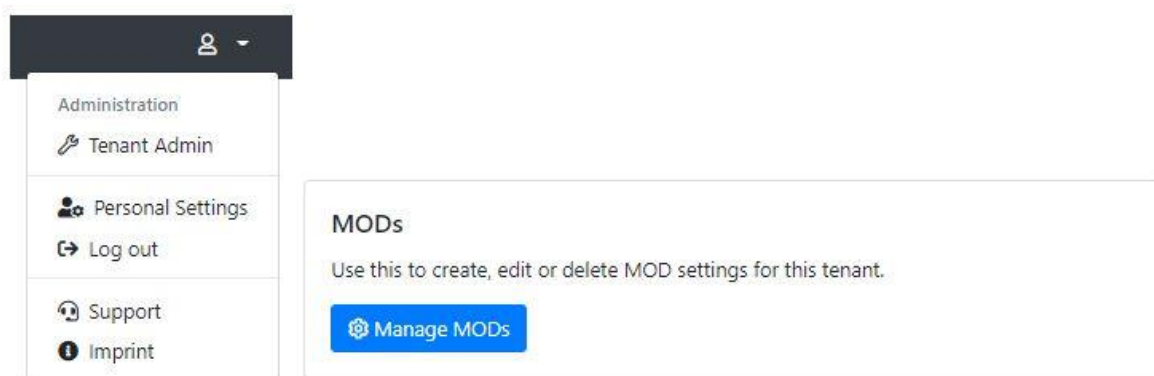
Type:

Deactivated:

You cannot delete an MOD once it is in use. But you can deactivate it, so that it's not available for new measurements anymore.

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*** Name:**

Default timezone:

Configure a timezone to see date and times in (instead of UTC). Please note that users can choose to use a different timezone.

Default report template (ALEX 1):

Default report template (ALEX 2):

Default report template (FOX):

Default report template (PAX):

Default report language:

Default ALEX2 MOD:

Default FOX MOD:

Default PAX MOD:

Force MOD:

Requires a measurement to always have an MOD assigned. If this setting is enabled, a measurement cannot be approved without an MOD assigned.

Hide Global MOD:

If this setting is enabled, only the tenant-specific MOD will be available to select, and the global system-wide MOD will be hidden.

Display MOD Excluded Spots As Research Use Only:

If this setting is enabled, spots excluded by an MOD will not be hidden from the measurement image, but displayed as a "RUO spot".

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MOD:

Report Template:

Force MOD:
Requires a measurement to always have an MOD assigned. If this setting is enabled, a measurement cannot be approved without an MOD assigned.

Hide Global MOD:
If this setting is enabled, only the tenant-specific MOD will be available to select, and the global system-wide MOD will be hidden.

Display MOD Excluded Spots As Research Use Only:
If this setting is enabled, spots excluded by an MOD will not be hidden from the measurement image, but displayed as a "RUO spot".

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Force MOD: False
Requires a measurement to always have an MOD assigned. If this setting is enabled, a measurement cannot be approved without an MOD assigned.

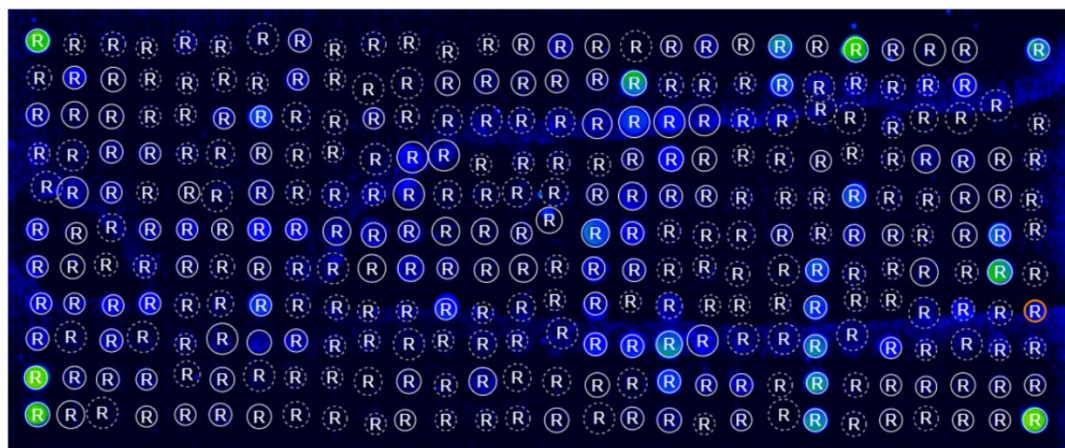
Hide Global MOD: False
If this setting is enabled, only the tenant-specific MOD will be available to select, and the global system-wide MOD will be hidden.

Display MOD Excluded Spots As Research Use Only: True
If this setting is enabled, spots excluded by an MOD will not be hidden from the measurement image, but displayed as a "RUO spot".

Configuration

Calibration string:
Allows to override the calibration information from the cartridge's QR-Code. Leave empty to use the calibration information from the QR-Code on the cartridge (the usual behaviour).

Research spots: Show research spots
If enabled, "research use only" spots will be displayed on measurement details and arrays. This does not affect the export behaviour or the reports of measurements.

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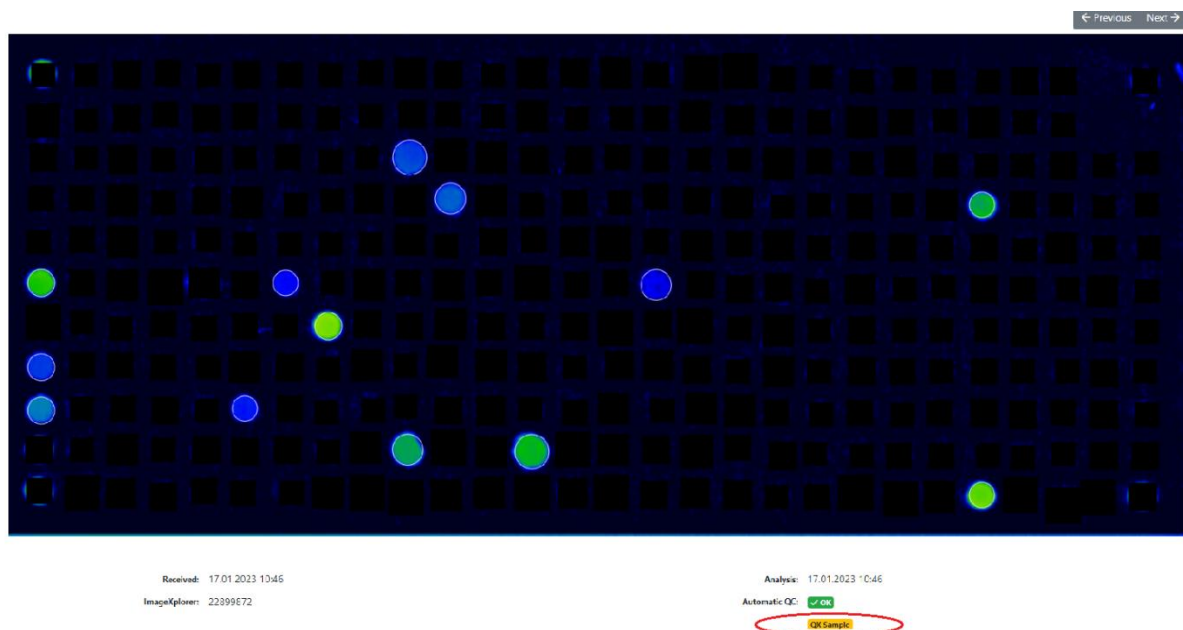
New Measurements

Search by barcode, patient, sample, tag, analysis date, a

Selected measurements ▾

- Download CSV format
- Download Excel format
- Approve
- Update interpretation
- Re-Process
- » Move
- Delete**

Tags	Product type
	FOX
	FOX
	ALEX2
	FOX
	ALEX2
	ALEX1

Delete measurements

You are about to delete the following measurements. Please note that this action is irreversible and cannot be undone.

Barcode	Analysis Date
@2AAB16@	16.12.2019 13:41

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Plant Food Food	M	Jug r 3	0.40
Plant Food Food	M	Jug r 1	0.38

Plant Food Food	M	Jug r 3	0.40
Plant Food Food	M	Jug r 1	0.38

Delete request scheduled now. [Cancel delete request](#)

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ign in. Can be

Background jobs

View details about running background jobs and manually schedule runs.

[View dashboard](#)

Measurement delete / unapproval log & requests

Show logging and pending requests for measurement delete and unapproval.

[Show log & requests](#)

Measurement unapproval / delete requests

Open requests Log & Finished requests				
Requested UTC	Requesting User (Email)	QR-Code	Request Type	Action
11.10.2023 13:05:28	Test Test (Test@test.com)	02AYQ145	Unapprove	Accept Reject
11.10.2023 12:59:27	Test Test (Test@test.com)	02AYQ14C	Delete	Accept Reject

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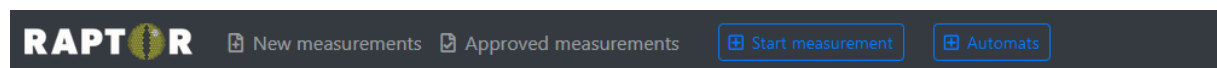
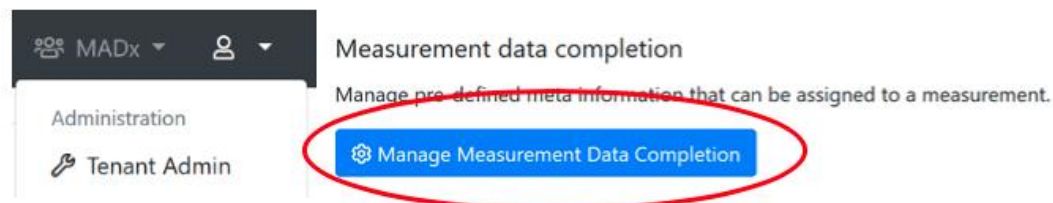
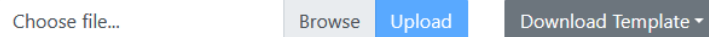
Add new user on Result Sharing Portal

General configuration

First name:

Last name:

E-Mail address:

Status:

Enabled (user can log in)

Username:

Password:

Send an invitation e-mail to user automatically

☒ Automatically send an email summary in case new measurement(s) have been shared
This email is sent once per day as a summary, just in case a new measurement was shared.

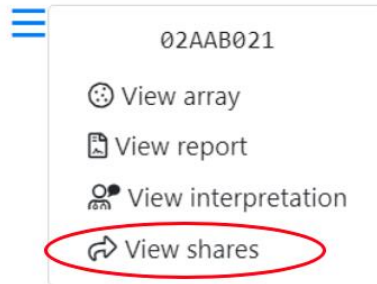
Doctor Info Text:

This information will be printed on a patients report.

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≡ Share

You are about to share a measurement with a user of the User Portal. Please note that the doctor will have access to the results of the measurement. Once you share a measurement you'll be provided with a link to access the shared information via the encrypted User F

Barcode: 02AAB021

Share with: X Dr. (testuser2@macroarraydx.com)

If you can't find a specific user of the User Portal, go to the [tenant administration](#) to configure the list of available users of the User Portal.

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RAPTOR
New measurements
Approved measurements
Start measurement with ImageXplorer
Automats
MADx Testtenant


≡ Share
Share this measurement

User	Link	Shared	
RP Bak	https://resultshare.raptor-server.com/s/07PXepC7ZNytm4PdRfmct4NR5pUtWq5FJcqOY7Ry6oEx08USj	7 months ago	Revoke

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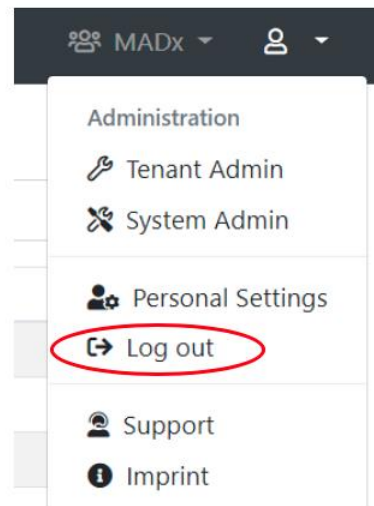The screenshot shows the 'Result Sharing Portal' header with a language dropdown set to 'English (en)'. Below the header, a welcome message states: 'Welcome to the Result Sharing Portal. The Raptor-Server Result Sharing Portal allows users to view shared measurements. You will need a username and password to log in to view all measurements that have been shared with you. Also, a sharing link will allow to view specific shared measurements.' The login form includes a prompt 'Please enter your username and your password:', a 'Your username' input field, a 'Your password' input field with an eye icon, and a blue 'Log in' button. A link for 'request a new one' is provided for users who forgot their password.

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This version of the login form has annotations. The prompt 'Please enter your username and your password:' is highlighted in blue. The 'Your username' input field is highlighted in light blue. The 'password' input field is highlighted in light blue, and the eye icon is highlighted in red. The blue 'Log in' button remains unchanged.

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English (en) ▼
Change password
Log out

Measurements

These are the measurements that are available for you. You may view the details of each measurement, or download the report PDF file.

Measurement	Product	Patient	Sample	Date
01AAK239	ALEX1		11852162	7/12/2019
01AAK281	ALEX1		04824821	12/18/2019
01AAK891	ALEX1		66560260	1/28/2020
02AAA015	ALEX2			2/21/2020

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Measurement 02AYQ143

Product: ALEX2
Patient:
Date of Birth:
Sample:
Info:
Doctor Info:
Sampling Date:
Assay Date: 10/12/2023
Processed Date: 10/12/2023
Approval Date: 10/12/2023

Report Language:

English (United Kingdom) ▼

You can request a new RAVEN interpretation that will be included in the report.

[Request new RAVEN interpretation](#)

You can download a PDF report that includes all information and results of this measurement.

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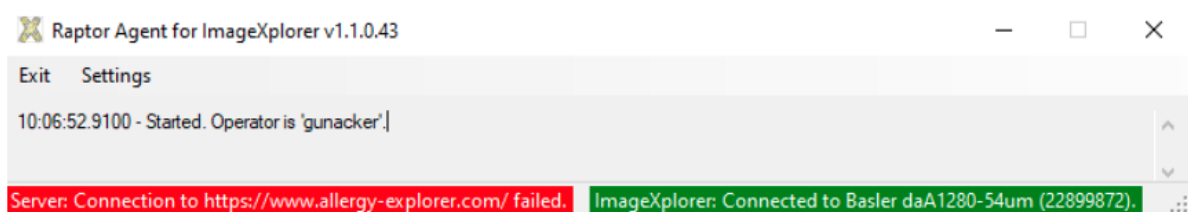
Settings

Please enter the settings for your ImageXplorer. You can find your ImageXplorer key when you log in to Raptor. Please contact support if you're not sure what the proper settings are for your ImageXplorer.

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© Copyright by MacroArray Diagnostics