# RAPTOR

SERVER ANALYSIS SOFTWARE

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### OBSAH

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Welcome to RAPTOR SERVER Analysis Software, the analysis software for the [ALEX multiplex test](#). Please log in below to continue.

| Log in with Azure AD / Office 365                                                                                                                                                                                                                                                                       | Log in with Username & password                                                                                                                                                                                                                                                                                                                                        |
|---------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------|------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------|
| <p>If your organization integrates the software into its Microsoft Azure/Office 365 user directory (single-sign-on) click the button below. You will be redirected to the Microsoft login screen.</p> <p> Log in</p> | <p>Please enter your username and password.</p> <p>Your username</p> <p>Your password </p> <p> Log in</p> <p>Forgot your password? No problem, you can <a href="#">request a new one</a>.</p> |

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Log in with Username & password

Please enter your username and password.

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**Contract approval required**

Please review the following documents. You are required to confirm that you have read and agree with each document in order to use RAPTOR SERVER:

☐ **Bitte lesen** (last updated 23.11.2023 18:00)

☐ **RAPTOR SERVER IFU** (last updated 04.12.2023 11:05)

☐ **Terms of Service** (last updated 22.11.2023 01:00)

Confirm

By clicking the button, I confirm that I have reviewed these documents and agree.

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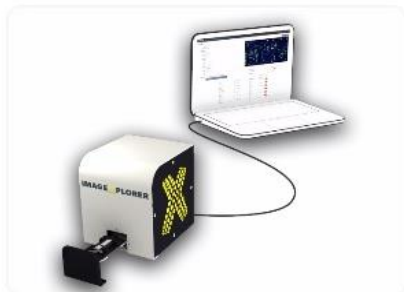
| Approved Measurements                          |                       |               |                  |         |        |  |
|------------------------------------------------|-----------------------|---------------|------------------|---------|--------|--|
| QR-Code                                        | Product type / Source | Sample Code   | Analysis Date    | Details | Report |  |
| 02AVJ227                                       | ALEX2                 | Anna LMacey   | 28/03/2023 21:17 | Details | Report |  |
| 02ARA379                                       | ALEX2 IX              | SM_28032023   | 28/03/2023 14:29 | Details | Report |  |
| 02ARA393                                       | ALEX2 IX              | RG_28032023   | 28/03/2023 14:26 | Details | Report |  |
| 02AVA31F                                       | ALEX2 MAX 45k         | PR_Mar2023_11 | 21/03/2023 15:05 | Details | Report |  |
| 02AVI3B9                                       | ALEX2 MAX 45k         | KF            | 17/03/2023 14:46 | Details | Report |  |
| 02AYT337                                       | ALEX2 MAX 45k         | 2B46B46+CCD   | 14/03/2023 19:06 | Details | Report |  |
| 02ATL349                                       | ALEX2 IX              |               | 22/02/2023 14:21 | Details | Report |  |
| 02AVE383                                       | ALEX2 MAX 45k         | AO            | 03/02/2023 09:41 | Details | Report |  |
| 02AXI29A                                       | ALEX2 MAX 1k          | OW            | 19/01/2023 19:12 | Details | Report |  |
| 02AUY3AE                                       | ALEX2 MAX 45k         | BSZ_Okt2022_8 | 18/10/2022 18:12 | Details | Report |  |
| <a href="#">View all approved measurements</a> |                       |               |                  |         |        |  |

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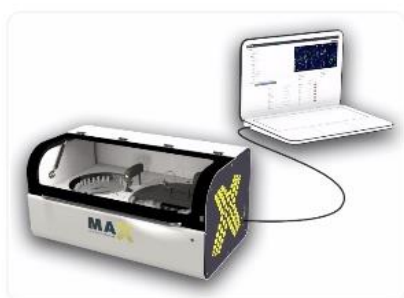
### 9K-Test-Instrument

Not initialized

[Initialize](#) [Shutdown](#) [Monthly Maintenance](#)

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**E-Mail address:**

**Language:** English (en)

The language is used to determine the display language.

**Culture:** English (United States) (en)

The culture is used to format numbers and dates.

**Timezone:** (UTC+01:00) Amsterdam, Berlin, Bern, Rome, Stockholm, Vienna

Configure your timezone to see date and times in your local time instead of UTC.

### Multi-Factor Authentication

[Manage MFA](#)

Use multi-factor authentication (MFA) to significantly reduce the risk of account compromise. It is highly recommended to enable MFA; this adds a second layer of security to your account against phishing attacks or weak passwords.

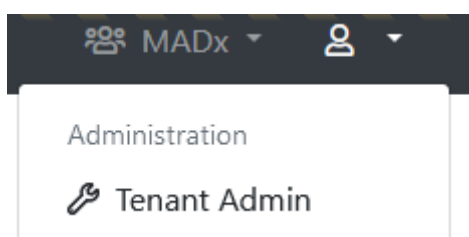
### Configuration

**Research spots:** Show research spots

If enabled, "research use only" arrays. This does not affect the

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## Configure "MADx"

| General configuration                                                                                                                        |                                                                     | MAX device default settings                                                             |                                 |
|----------------------------------------------------------------------------------------------------------------------------------------------|---------------------------------------------------------------------|-----------------------------------------------------------------------------------------|---------------------------------|
| <b>* Name:</b>                                                                                                                               | <input type="text" value="MADx"/>                                   | <b>* Default Minimum RunId:</b>                                                         | <input type="text" value="10"/> |
| <b>Default timezone:</b>                                                                                                                     | <input type="text" value="(UTC+01:00) Amsterdam, Berlin, Bern, R"/> | When a new MAX device is created, this value will be used for its <b>Minimum Run Id</b> |                                 |
| Configure a timezone to see date and times in (instead of UTC). Please note that users can choose to use a different timezone.               |                                                                     |                                                                                         |                                 |
| <b>Default report template (ALEX2):</b>                                                                                                      | <input type="text" value="ALEX2"/>                                  |                                                                                         |                                 |
| <b>Default report template (ALEX3, ALEX Air, ALEX Food):</b>                                                                                 | <input type="text" value="check 1.21"/>                             |                                                                                         |                                 |
| <b>Default report template (FOX):</b>                                                                                                        | <input type="text" value="FOX (2025)"/>                             |                                                                                         |                                 |
| <b>Default report template (PAX):</b>                                                                                                        | <input type="text" value="PAX"/>                                    |                                                                                         |                                 |
| <b>Default report language:</b>                                                                                                              | <input type="text" value="English (en)"/>                           |                                                                                         |                                 |
| <b>Default ALEX MOD:</b>                                                                                                                     | <input type="text" value=" &lt; No MOD &gt;"/>                      |                                                                                         |                                 |
| <b>Default FOX MOD:</b>                                                                                                                      | <input type="text" value=" &lt; No MOD &gt;"/>                      |                                                                                         |                                 |
| <b>Default PAX MOD:</b>                                                                                                                      | <input type="text" value=" &lt; No MOD &gt;"/>                      |                                                                                         |                                 |
| <b>Force MOD:</b>                                                                                                                            | <input type="text" value="False"/>                                  |                                                                                         |                                 |
| Requires a measurement to always have an MOD assigned. If this setting is enabled, a measurement cannot be approved without an MOD assigned. |                                                                     |                                                                                         |                                 |
| <b>Hide Global MOD:</b>                                                                                                                      | <input type="text" value="False"/>                                  |                                                                                         |                                 |
| If this setting is enabled, only the tenant-specific MOD will be available to select, and the global system-wide MOD will be hidden.         |                                                                     |                                                                                         |                                 |
| <b>Display MOD Excluded Spots As Research Use Only:</b>                                                                                      | <input type="text" value="False"/>                                  |                                                                                         |                                 |
| If this setting is enabled, spots excluded by an MOD will not be hidden from the measurement image, but displayed as a "RUO spot".           |                                                                     |                                                                                         |                                 |
| <b>MFA:</b>                                                                                                                                  | <input type="checkbox"/> <b>Force MFA</b>                           |                                                                                         |                                 |
| If this setting is enabled, users are forced to set up an MFA on sign-in, in case they haven't yet.                                          |                                                                     |                                                                                         |                                 |

| Tenant-specific batch releases                                                                                             |                                                                                                                  |
|----------------------------------------------------------------------------------------------------------------------------|------------------------------------------------------------------------------------------------------------------|
| Restrict this tenant to only allow the following batches to be scanned or to restrict certain batches. Enter one per line. |                                                                                                                  |
| Use:                                                                                                                       |                                                                                                                  |
| <input checked="" type="radio"/> <b>Whitelist</b>                                                                          |                                                                                                                  |
| <input type="radio"/> <b>Blacklist</b>                                                                                     |                                                                                                                  |
| <b>Whitelisted Batches:</b>                                                                                                | <div><div></div><div>Enter one batch code including product identifier per line, for example "02AAA"</div></div> |
| <b>Blacklisted Batches:</b>                                                                                                | <div><div></div><div>Enter one batch code including product identifier per line, for example "02AAA"</div></div> |

## Company Details

|                                 |                                                       |
|---------------------------------|-------------------------------------------------------|
| <b>* Legal Entity Name:</b>     | <input type="text" value="MacroArray Diagnostics"/>   |
| <b>* Street and Number:</b>     | <input type="text" value="Street 4"/>                 |
| <b>* Zip Code:</b>              | <input type="text" value="1230"/>                     |
| <b>* City:</b>                  | <input type="text" value="New York"/>                 |
| <b>* Country:</b>               | <input type="text" value="United States of America"/> |
| <b>* Phone:</b>                 | <input type="text" value="+4318652573"/>              |
| <b>* E-Mail:</b>                | <input type="text" value="office@macroarraydx.com"/>  |
| <b>* Legal Representative:</b>  | <input type="text" value="Dr. Christian Harwanegg"/>  |
| <b>Website:</b>                 | <input type="text"/>                                  |
| <b>Vat Number:</b>              | <input type="text"/>                                  |
| <b>Company Register Number:</b> | <input type="text"/>                                  |

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#### Tenant-specific batch releases

Restrict this tenant to only allow the following batches to be scanned or to restrict certain batches. Enter one per line.

Use:

☒ **Whitelist**  
☐ **Blacklist**

**Whitelisted Batches:**

02AAA  
02ABC  
02BAA

Enter one batch code including product identifier per line, for example "02AAA"

**Blacklisted Batches:**

Enter one batch code including product identifier per line, for example "02AAA"

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You requested a new measurement from ImageXplorer "IX Trainingsraum". Please wait until the measurement is completed ...

Invalid batch detected! The measurement can't be processed because its batch is blocked.

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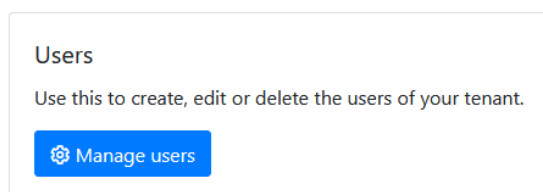
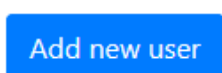
#### Login Credentials

**Type:**

Please select if the user should be validated against Azure AD (Office 365), or if you want to specify username and password yourself. It is recommended to use Azure AD whenever possible. This allows for increased security by supporting two-factor authentication. Also, it is more convenient for the user, because he or she does not have to remember a new username and password (single-sign-on).

**Username:**

It is a good idea to use the users e-mail-address as his username, because the username needs to be unique across the entire system.

**Password:**

#### Access Rights

##### Preconfigured groups

Use these pre-configured groups to automatically assign a set of access rights to this user. You can still add or remove specific access rights.

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#### Tenants

This user is connected to the following tenants and has the following access rights:

| <input type="checkbox"/> Tenants             | Can share measurements | Can edit measurements | Can edit personal data | Can approve measurements | Can manage basic tenant configuration | Can manage API keys | Can manage ImageExplorers | Can view agent connections | Can manage users | Can manage users of Result Sharing Portal | Can manage report templates | Can manage measurement data completion | Can unapprove measurements | Can use a custom calibration string | Can execute MAX r | Can skip r |
|----------------------------------------------|------------------------|-----------------------|------------------------|--------------------------|---------------------------------------|---------------------|---------------------------|----------------------------|------------------|-------------------------------------------|-----------------------------|----------------------------------------|----------------------------|-------------------------------------|-------------------|------------|
| <input type="checkbox"/> Another MADx Tenant | ✓                      | ✓                     | ✓                      | ✓                        | ✓                                     | ✓                   | ✓                         | ✓                          | ✓                | ✓                                         | ✓                           | ✓                                      | ✓                          | ✓                                   | ✓                 | ✓          |
| <input type="checkbox"/> MADx Tenant         | ✓                      | ✓                     | ✓                      | ✓                        | ✓                                     | ✓                   | ✓                         | ✓                          | ✓                | ✓                                         | ✓                           | ✓                                      | ✓                          | ✓                                   | ✓                 | ✓          |

Additionally this user can be added to the following tenants by checking the tenant name and hitting save afterwards. User rights w applied from current open tenant user.

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Additionally this user can be added to the following tenants by checking the tenant name and hitting save afterwards. User right applied from current open tenant user.

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### Tenants

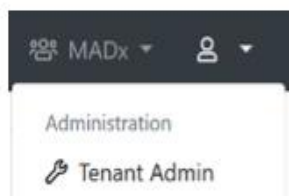
This user is connected to the following tenants and has the following access rights:

| <input type="checkbox"/> Tenants                        | Can share measurements              | Can edit measurements               | Can edit personal data              | Can approve measurements            | Can manage basic tenant configuration | Can manage API keys                 | Can manage ImageX                   |
|---------------------------------------------------------|-------------------------------------|-------------------------------------|-------------------------------------|-------------------------------------|---------------------------------------|-------------------------------------|-------------------------------------|
| <input checked="" type="checkbox"/> Another MADx Tenant | <input checked="" type="checkbox"/> | <input checked="" type="checkbox"/> | <input checked="" type="checkbox"/> | <input checked="" type="checkbox"/> | <input checked="" type="checkbox"/>   | <input checked="" type="checkbox"/> | <input checked="" type="checkbox"/> |

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## ImageXplorers

Use this to configure ImageXplorers, or add new ones. Every ImageXplorer needs to be known to ~~Raptor Server~~, otherwise it won't be able to connect.

[Manage ImageXplorers](#)

Add new ImageXplorer

General information

|                            |                                                               |
|----------------------------|---------------------------------------------------------------|
| Name:                      | <input type="text"/>                                          |
| Key:                       | <input type="text" value="Cxltclg8okSwQ6CqJ4EZLWGnw2PdJ3"/>   |
|                            | <small>This key is required when installing an Agent.</small> |
| ImageXplorer Serialnumber: | <input type="text"/>                                          |
|                            | <small>Serialnumber of connected ImageXplorer.</small>        |
| Notes:                     | <div></div>                                                   |
| Internal Notes:            | <div></div>                                                   |
|                            | <small>For internal use only.</small>                         |
| Disabled:                  | <input type="text" value="Not disabled"/>                     |

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Details

Barcode: 02AE3408  
Sample Code: 12  
Requisition Code:  
Sampling Date: 01.10.2021  
Assay Date: 04.10.2021  
Patient Code: TE  
Patient Name: Test  
Date of Birth: 17.05.2010  
Info:  
Doctor Info: Dr. A  
Choose a doctor...  
Tag: Tag Info  
MOD: < No MOD >  
Report Template: Version 2 (ALEX2 only) - Deutsch (de)

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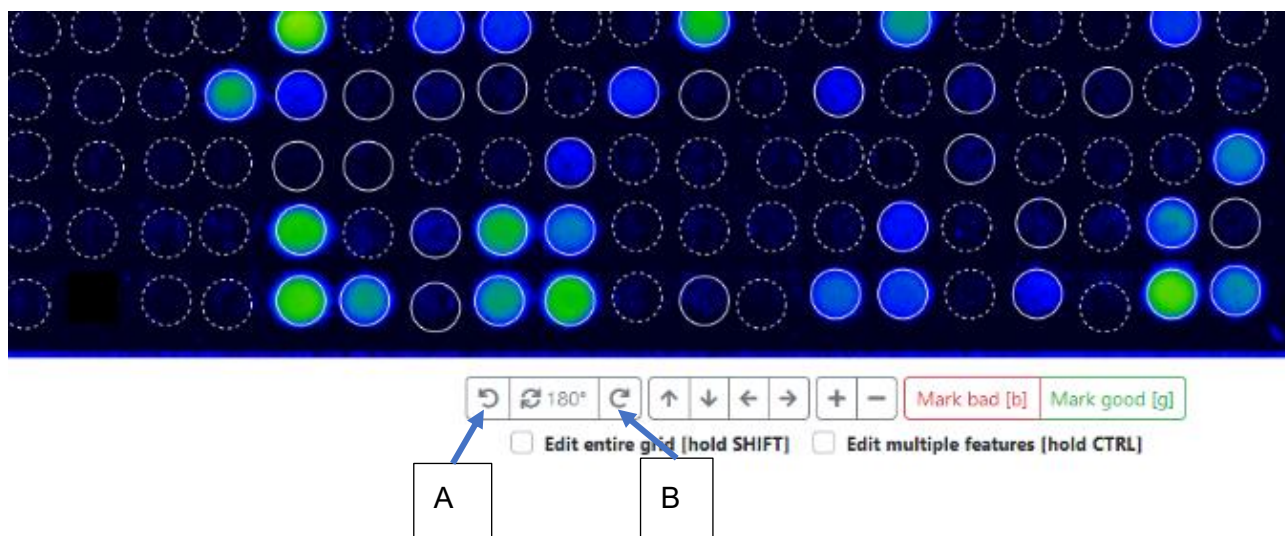
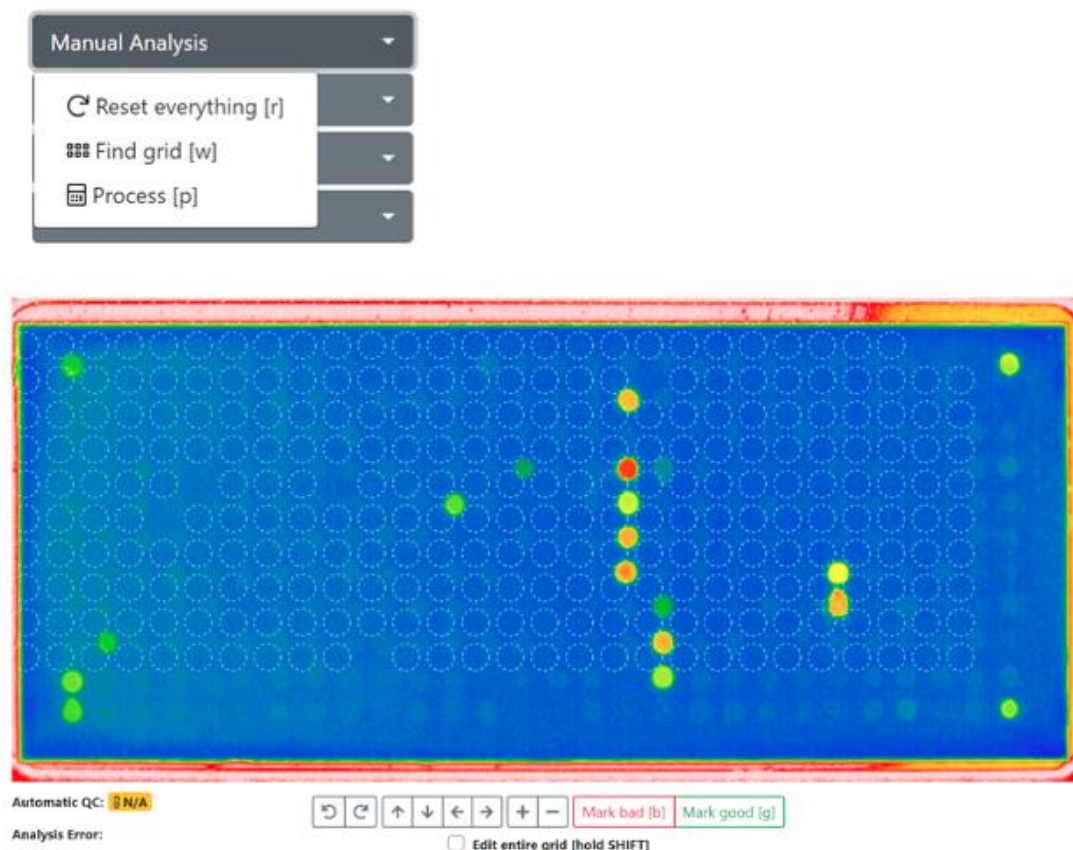
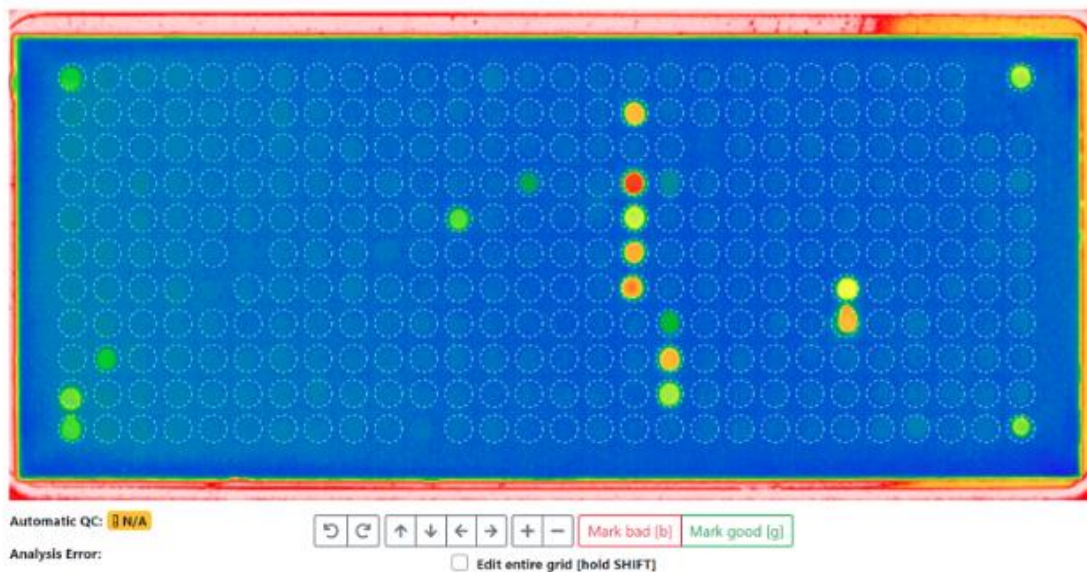
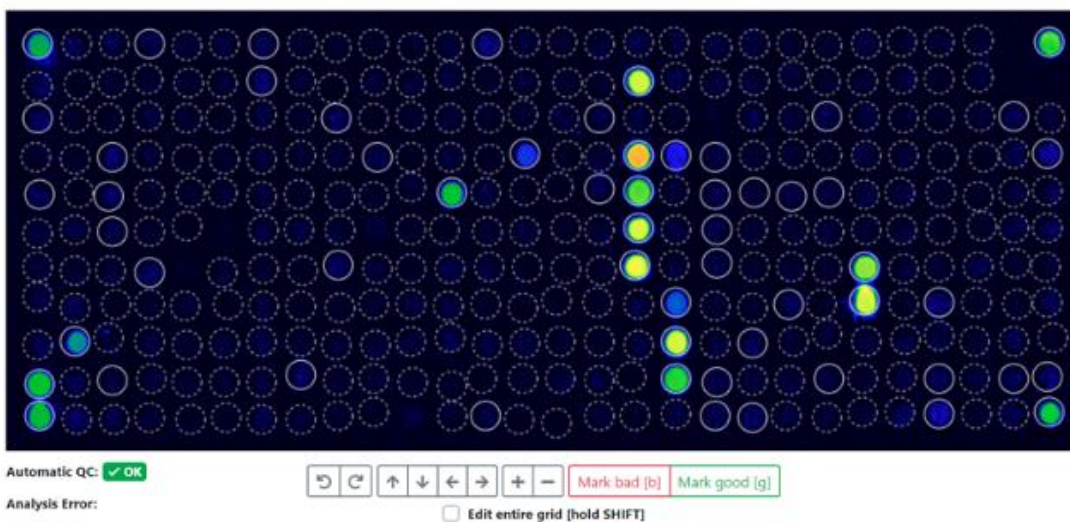
Show Analysis & QC Logs

Reset everything and reprocess

This discards all modifications to the array, and re-runs the complete analysis process from the original image data. Please note that any manual changes will be irreversible lost.

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#### Analysis Log

```
'PreProcessedPixels' took 128ms.
Using definition '02_ALEXII_IVD_v1.6.0'.
'ApplyArrayDefinition' took 74ms.
Parsing calibration string ...
'ApplyDefinitions' took 75ms.
'FindGrid' took 1ms.
'AlignFeatures' took 862ms.
'CropImage' took 1ms.
'QuantifyFeatures' took 231ms.
'SubtractBackground' took 601ms.
Parsing calibration string ...
Using calibrators from QR-Code.
Total iqt (tqg) normalized by calculating 0.330999755859375E-07x²+0.05697999954223633x+-38.69269921875
(x=8515), which results in 515.
'ReQuantify' took 57ms.
```

#### QC Log

These are the values used for QC (ALEX v2): Curve Correction=0.9997370867894242, tlgE=8515, GD1=28095 (SNR=105.38), GD2=29369 (SNR=185.28), GD3=28569 (SNR=147.53), GD4=28385 (SNR=128.1). Reference values are as follows: Guide Dots must have a value above 5000; each Guide Dot spot must exist and be unique; Guide Dot spot must not be set to bad. Average background must be between 4000 and 16000. QC (ALEX v2) succeeded.

#### Background Log

Calculated background statistics for 164 pixels (from a total of 182, skipped smallest 9, and highest 9 pixels). The average is 9,419, the minimum 7973 and the maximum 11792 (minimum and maximum is calculated including skipped pixels). This results in an average difference of 628, which is an SD of 49.17 and a CV of 0.52%. For comparison, there is an overall total of 282944 pixels. The overall average is 14.871, the minimum 6.012 and the maximum 64.965.

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
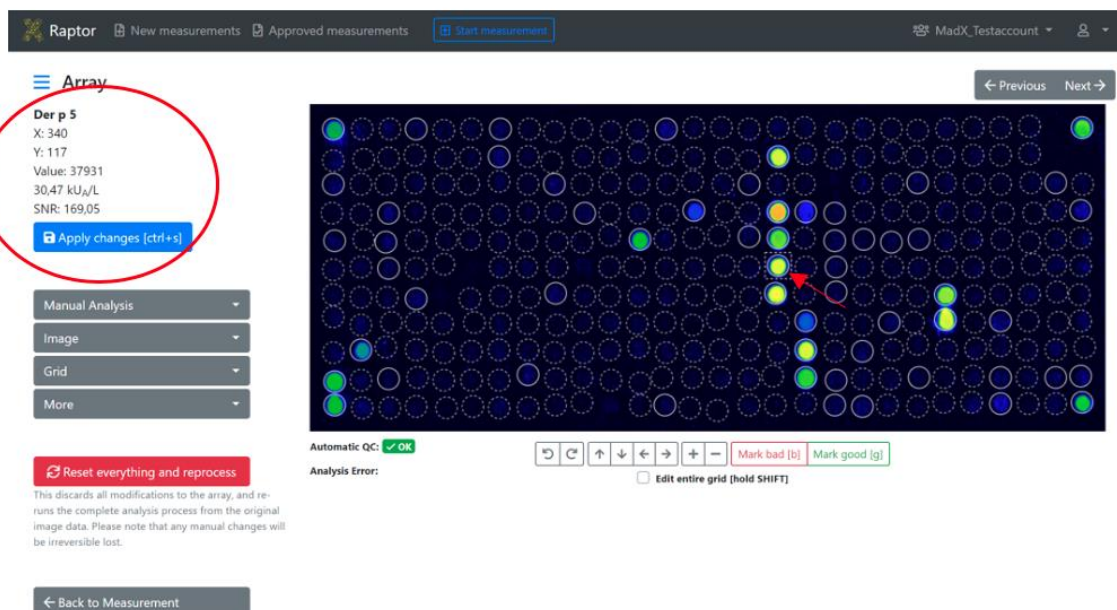
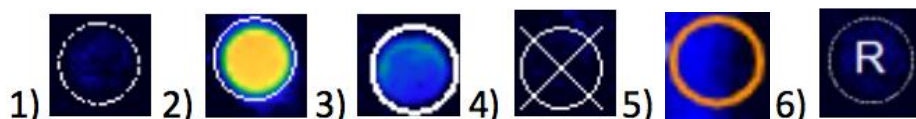
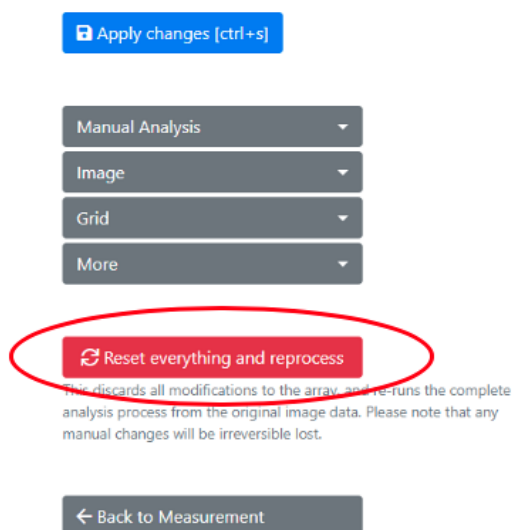
| Milk      |            |   |                 |                                                                                            |
|-----------|------------|---|-----------------|--------------------------------------------------------------------------------------------|
| Cow, milk | Bos d 4    | M | α-Lactalbumin   | 0,38                                                                                       |
| Cow, milk | Bos d 5    | M | β-Lactoglobulin | 0,90  |
| Cow, milk | Bos d 8    | M | Casein          | 6,04                                                                                       |
| Cow, milk | Bos d_milk | E |                 | 3,40                                                                                       |

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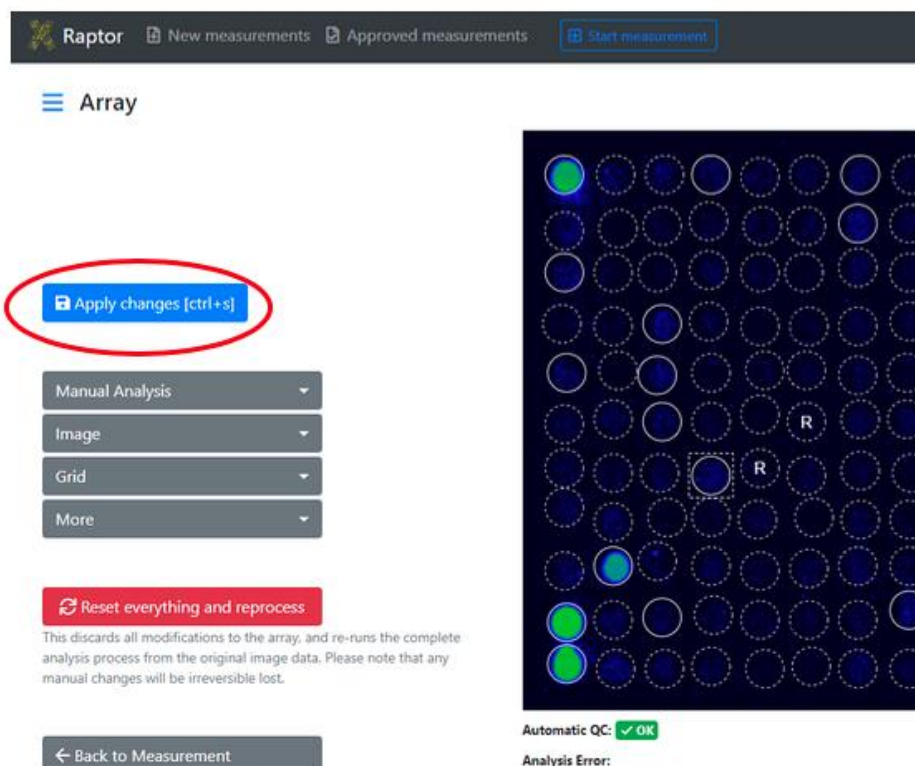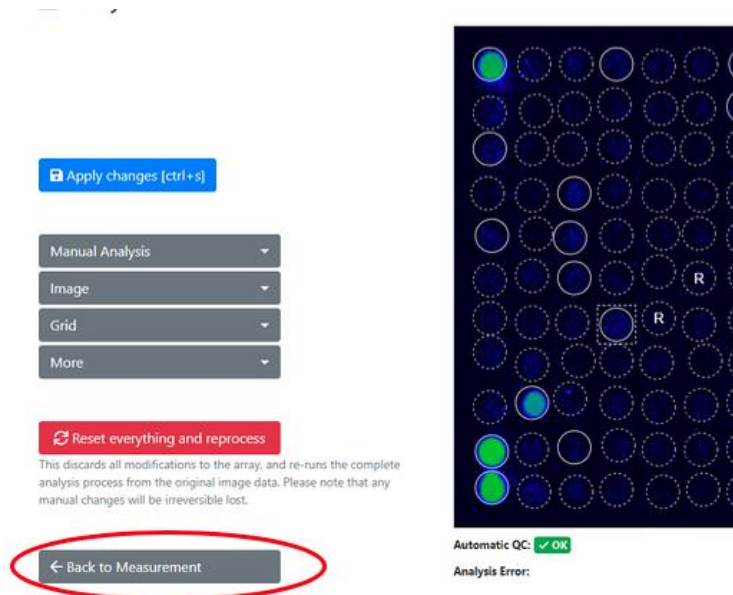
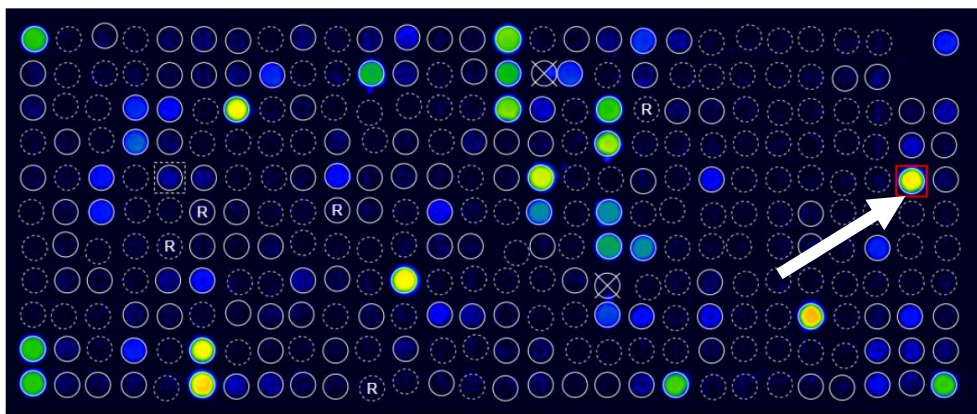
| Tissue & Source |            | All allergens |       | Only positive allergens |  | kUA/L  |
|-----------------|------------|---------------|-------|-------------------------|--|--------|
| Pollen          | Inhalation | E             | Aca m |                         |  | ≤ 0.10 |
| Mites           |            | E             | Aca s |                         |  | 7.11   |

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## New Measurements

Search by QR-Code, patient, sample, tag, analysis date, approval date ...

Selected measurements ▾

| <input type="checkbox"/>            | Source | QC   | Tags | Product type |
|-------------------------------------|--------|------|------|--------------|
| <input type="checkbox"/>            | IX     | ✓ OK |      | ALEX2        |
| <input checked="" type="checkbox"/> | IX     | ✓ OK |      | ALEX2        |
| <input checked="" type="checkbox"/> | IX     | ✓ OK |      | ALEX2        |
| <input checked="" type="checkbox"/> | IX     | ✓ OK |      | ALEX2        |

- Selected measurements ▾
- Download CSV format
  - Download Excel format
  - Approve
  - Update interpretation
  - Choose Reporttemplate
  - Modify Tags**
  - » Move
  - Delete

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## New Measurements

Search by QR-Code, patient, sample, tag, analysis date, approval date ...

Selected measurements ▾

- Download CSV format
- Download Excel format
- Approve
- Update interpretation
- Choose Reporttemplate
- Modify Tags
- Re-Process**
- » Move
- Delete

## Re-process measurements

You are about to re-process the following measurements. If you undo, the existing analysis result will be irreversibly over-

| QR-Code  | Sample Code |
|----------|-------------|
| 02APK3C8 | Dummy8      |
| 02APK258 | Dummy1      |
| 02APK07F | Dummy13     |

Re-process these measurements Cancel

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## New Measurements

| <input type="checkbox"/> | Source                                                                                       | QC   | Tags       | Product type |
|--------------------------|----------------------------------------------------------------------------------------------|------|------------|--------------|
| <input type="checkbox"/> | Recaptured                                                                                   | ✓ OK | recaptured | ALEX1        |
| <input type="checkbox"/> |  Recaptured | ✓ OK | recaptured | ALEX1        |
| <input type="checkbox"/> | QR-Code was scanned more than once.                                                          |      | recaptured | ALEX1        |

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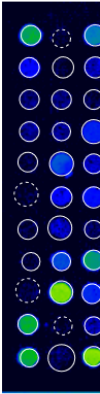
### Approve measurements

You are about to approve the following measurements.

There are measurements with one or more warnings. Please make sure to double-check these measurements before approval.

| QR-Code  | Sample Code | Analysis Date    |
|----------|-------------|------------------|
| 02AKV035 |             | 28.04.2022 11:14 |

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**RAPTOR**
New measurements
Approved measurements
Start m

### Approved Measurements

Search by QR-Code, patient, sample, tag, analysis date ...

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**RAPTOR**
New measurements
Approved measurements
Start measurement
Automats
MADx Testtenant

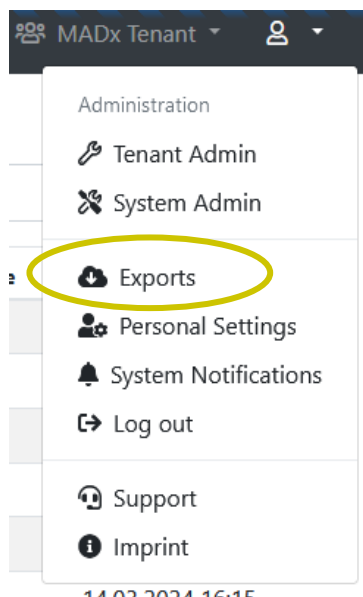
### Exports

Certain types of data exports (for example, the export of many PDF reports at once) take a few minutes to prepare for download. Here download your requested exports. Please note that you can keep working, all exports are done in the background and you can download when they are completed.

| Export requested   | # of measurements included | Export completed   |
|--------------------|----------------------------|--------------------|
| 8/16/2021 11:07 AM | 4                          | 0 / 4              |
| 8/16/2021 11:06 AM | 4                          | 8/16/2021 11:06 AM |
| 8/16/2021 11:04 AM | 3                          | 8/16/2021 11:04 AM |

- Administration
- Tenant Admin
- Exports
- Personal Settings
- Log out
- Support
- Imprint

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## Exports

Certain types of data exports (for example, the export of many PDF reports at once) are done in the background. Please note that you can keep working, all exports are done in the background.

[Export multiple measurements](#)

| Export requested | # of measurements included |
|------------------|----------------------------|
| 06.07.2022 08:52 | 3                          |

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## Export multiple measurements

Please note that **Report PDF** and **ASTM** export is only available for certain data types.

|                                                    |                                         |                                       |
|----------------------------------------------------|-----------------------------------------|---------------------------------------|
| QR-Code/Product/Batch                              |                                         | Sa                                    |
| <input type="text" value="02ABC"/>                 |                                         | <input type="text"/>                  |
| Analysis Date (from)                               | Analysis Date (to)                      |                                       |
| <input type="text" value="TT.MM.JJJJ"/>            | <input type="text" value="TT.MM.JJJJ"/> |                                       |
| Tag                                                | Status                                  |                                       |
| <input type="text"/>                               | <input type="text" value="All"/>        |                                       |
| <input type="checkbox"/> Export CSV                | <input type="checkbox"/> Export Excel   |                                       |
| <input type="button" value="Search Measurements"/> |                                         | <input type="button" value="Cancel"/> |

## Export multiple measurements

Please note that **Report PDF** and **ASTM** export is only available for the current analysis.

QR-Code/Product/Batch

02aaa

Analysis Date (from)

TT . MM . JJJJ

Analysis Date (to)

TT . MM . JJJJ

Tag

Status

All

☐ Export CSV

☐ Export Excel

 Start export ...

← Back

66 Measurements found that match your criteria

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## Edit report template

Your changes have been saved successfully.

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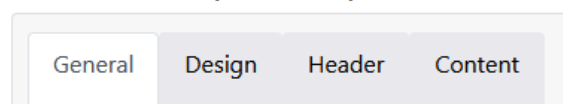
#### Configure report template "New report template"

**Name:** New report template  
This is the name of the template which shows up in the dropdown list while generating a report.

**Base Template:** Version 2 (ALEX2 only) ▾  
A base template which specifies basic functionality must be chosen.

**Preferred language:** English (United States) (en-us) ▾  
A default language can be specified which will be used if there isn't yet a language stored to a measurement's report.

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PDF naming

Setting Type: Automatic

The filename of the report PDF can be customized. Either built automatically based on your selection below or manually adjusted in the order of your needs.

Include Patient ID: ☒

Include Patient Name: ☒

Include Sample ID: ☐

Include Measurement QR-Code: ☐

Include Date & Time: ☒

Footer

Footer text:

Will be printed in the footer if provided and rendered as HTML.

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Configure report template "New report template"

The screenshot shows the 'Design' tab of a configuration window. It contains three main sections: 'PDF format' with radio buttons for 'A4' (selected) and 'Letter'; 'Font' with a dropdown menu showing 'Arial'; and 'Background image' with a text input field and a 'Browse' button. To the right of each section is a descriptive text. Below the 'Background image' field, it says 'Maximum file size 1 MB'.

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**Third level headline color:**

### Logos

**Logo (left):**  Browse  
Maximum file size 1 MB  
This logo will appear on the **top left corner** of the first page of the report.

**Text (center):**   
This text will appear either on the left, right, or center of the top of the first page (depending on your configured logos). Supports the placeholders `{{ PATIENT_PORTAL_QRCODE }}` and `{{ PATIENT_PORTAL_URL }}` to dynamically insert measurement information.

**Logo (right):**  Browse  
Maximum file size 1 MB  
This logo will appear on the **top right corner** of the first page of the report.

**Small logo:**  Browse  
Maximum file size 1 MB  
This logo is used on all detail pages. If not set, the default logo is used.

### Signatures

**Left:**  Browse  
Maximum file size 1 MB  
This signature or image will appear on the last page, left aligned.

**Center:**  Browse  
Maximum file size 1 MB  
This signature or image will appear on the last page, center aligned.

**Right:**  Browse  
Maximum file size 1 MB  
This signature or image will appear on the last page, right aligned.

### Indicators

**Custom intensity indicator:**

A custom SVG for displaying allergen intensity can be pasted here. Will be styled according to specified colors.

**⚠** The SVG element must contain the attributes **width** and **height** with a minimum dimension of 16 for correct display in PDF.

**🌿** SVGs paths which should be colored must require the **colorize** class as well as a fill defined in the **defs** segment of the SVG.

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Configure report template "New report template"

| Header content: |  | Left half                  | Right half           |
|-----------------|--|----------------------------|----------------------|
|                 |  | Custom headline            | Custom headline      |
|                 |  | Patient Name               | Doctor Information   |
|                 |  | Patient Name               | None                 |
|                 |  | Custom headline            |                      |
|                 |  | Date of Birth              | None                 |
|                 |  | Sample ID                  | None                 |
|                 |  | QR-Code                    | Additional Info text |
|                 |  | Analysis Date              | None                 |
|                 |  | Amount of tested allergens | None                 |
|                 |  | Test method                | Quality Control text |

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| General                                  | Design                              | Header                                                                                                                                                                    | Content |
|------------------------------------------|-------------------------------------|---------------------------------------------------------------------------------------------------------------------------------------------------------------------------|---------|
| <b>Allergens to include:</b>             | All allergens                       | Specifies which kind of allergens should be included in the report.                                                                                                       |         |
| <b>Allergens sort order:</b>             | Alphabetically by allergen          | Specifies the sort order of the allergens.                                                                                                                                |         |
| <b>Include Overview:</b>                 | <input checked="" type="checkbox"/> | Whether the allergen overview should be printed or not.                                                                                                                   |         |
| <b>Include product type in overview:</b> | <input checked="" type="checkbox"/> | Whether the product type should be printed in the header of allergen overview.                                                                                            |         |
| <b>Include Total IgE:</b>                | On the front page                   | Whether the information of total IgE should be included in the report and where.                                                                                          |         |
| <b>Cross-reactive information:</b>       | Include text                        | This section will be displayed underneath the allergen list itself.                                                                                                       |         |
| <b>Include Interpretation:</b>           | Include manually                    | It can be specified if the Raven report interpretation information should be included in the report or not.<br>This section is printed underneath the allergens overview. |         |

| Additional information (below the list of results) |                                     | Those informations are integrated into the report in an extra section underneath the allergen list. |
|----------------------------------------------------|-------------------------------------|-----------------------------------------------------------------------------------------------------|
| <b>Approval date:</b>                              | <input type="checkbox"/>            |                                                                                                     |
| <b>Assay date:</b>                                 | <input type="checkbox"/>            |                                                                                                     |
| <b>Doctor info:</b>                                | <input type="checkbox"/>            | The information about the doctor which accompanied the measurement/patient.                         |
| <b>Print date:</b>                                 | <input checked="" type="checkbox"/> | When the report was generated.                                                                      |
| <b>Report info:</b>                                | <input type="checkbox"/>            | The „additional info“ field belonging to the measurement.                                           |
| <b>Sample date:</b>                                | <input checked="" type="checkbox"/> |                                                                                                     |

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### Configure report template "New report template - Duplicate"

| General                    | Design                          | Header | Content |
|----------------------------|---------------------------------|--------|---------|
| <b>Name:</b>               | New report template - Duplicate |        |         |
| <b>Support products:</b>   | ALEX <sup>2</sup>               |        |         |
| <b>Preferred language:</b> | Deutsch (de)                    |        |         |

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**General configuration**

\* Name: MADx

Default timezone: (UTC+01:00) Amsterdam, Berlin, Bern, Rome, Stockholm, Vienna  
Configure a timezone to see date and times in (instead of UTC). Please note that users can choose to use a different timezone.

Default report template (ALEX 1): Version 1 (ALEX1 & ALEX2)

Default report template (ALEX 2): Version 2 (ALEX2 only)

Default report language: English (United Kingdom) (en-gb)

Default MOD: < No MOD >

Require ImageExplorer testarray scan: Warn after 60 days, enforce after 90 days.

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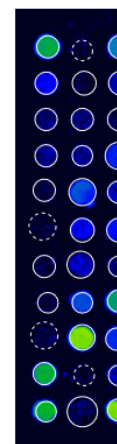
Raptor New measurements Approved measurements Start measurement Automate

### Report

Template: Version 2 (ALEX2 only)

Language: English (en)

Usually this tenant's default template and language will be used when sharing or exporting measurements. You may change the template and language associated with this measurement by saving your current configuration.

You can download this report in PDF format once the measurement has been approved.

[Manage report template and settings](#)  
You have access to manage the report templates and settings via the tenant administration. Please note that any changes will affect all users.

**MADX**  
MACROARRAY DIAGNOSTICS

PATIENT ID  
PATIENT NAME  
DATE OF BIRTH  
SAMPLE ID  
SWABCODE  
02AD0042  
ANALYSED ON  
11/04/2022  
TESTED ALLERGENS  
306  
TESTMETHODE  
ALEX2

REFERRING PHYSICIAN

NOTE  
The internal QC (Pauzability check for QD) was within acceptance range.

Lab report: Summary on detectable sensitisations

POLLEN  
Grass Pollen

MICROORGANISMS  
Fungal Spores & Yeast

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The screenshot shows the 'RAPTOR' web application. At the top, there are tabs for 'New measurements', 'Approved measurements', and 'Start m...'. Below the tabs is a search bar with the text 'Search by QR-Code, patient, sample, tag, analysis date ...'. A dropdown menu 'Download selected measurements' is open, showing options: 'Report (PDF format)' (highlighted with a yellow circle), 'CSV format', 'Excel format', and 'ASTM format'. Below the menu is a table of 'Approved Measurements'.

|                                     | Source | Tags | Product type | QR-Code  | Sample C |
|-------------------------------------|--------|------|--------------|----------|----------|
| <input checked="" type="checkbox"/> | IX     |      | ALEX2        | 02AKV39D |          |
| <input checked="" type="checkbox"/> | IX     |      | ALEX2        | 02AKV39D |          |
| <input checked="" type="checkbox"/> | IX     |      | ALEX2        | 02AKV39D |          |

Below the table, there are several action buttons: 'Update interpretation', 'Choose Reporttemplate', 'Share', 'Move', and 'Unapprove'.

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Tags:

MOD:

Report Template: Version 2 (ALEX2 only) - Deutsch (de)

Download  Save >> Move to other tenant

**Raptor** New measurements Approved measurements Start measurement Automats

### Report

Template: Default  
Language: English

Save Use defaults

Usually this tenants default template and language will be used when sharing or exporting measurements. You may change the template and language associated with this measurement by saving your current configuration.

**Download PDF**

Manage report template and settings

You have access to manage the report templates and settings via the tenant administration. Please note that any changes will affect all users.

|                |                                |
|----------------|--------------------------------|
| Patient ID:    |                                |
| Patient Name:  |                                |
| Date of Birth: |                                |
| Sample ID:     |                                |
| Barcode:       | 02AAE                          |
| Sampled on:    |                                |
| Tested on:     | 12/18/                         |
| Printed on:    | 12/19/                         |
| Note:          | The internal QC (Plausibility) |

Lab report: Summary on de

|                  |               |
|------------------|---------------|
| Pollen           | Grass Pollen  |
|                  | Tree Pollen   |
|                  | Weed Pollen   |
| Mites            | House Dust M  |
| Microorganisms   | Fungal Spores |
|                  | Legumes       |
|                  | Grain         |
| Plant-Based Food | Spices        |
|                  | Fruits        |

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02AAB019

- View array
- View report
- View interpretation**
- Approve measurement

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#### Rule set

RAVEN Ruleset A for ALEX

A rule set defines the RAVEN or RAVEN<sup>2</sup> engine that is used to interpret results.

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AIT

Medication

### MADx data declaration for smartRaven

☒ I accept to enter clinical data for using the full interpretation function of smartRAVEN

### Patient has reported any symptoms

☐ Yes ☐ No

### Additional information about the patient

☐ Atopic Dermatitis

☐ Allergen-specific immunotherapy (AIT) - ongoing, completed or discontinued

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[Request new interpretation](#)

This measurement already has an interpretation stored. Usually, it should not be necessary to request those again, but it could be necessary when the patients information has changed.

#### Interpretation

##### Raven Interpretation Summary

##### Sample Information

The sample was tested on ALEX<sup>2</sup> Barcode 02AAB019, interpretation date 12/19/2019.

Of the tested 295 allergens, 25 were above the detection limit of 0.3 kU<sub>A</sub>/L. A sensitization can be an indicator of an allergy. For selected allergens, comments for interpretation are listed below.

##### Total IgE: 512 kU/L

The measured total IgE was 512 kU/L. Above a total IgE of 100 kU/L, allergy is likely.

##### Cross-Reactive Allergen Sensitization detected

Sensitization against allergen molecules which are markers of (broad) cross-reactivity between different allergen sources.

Detected cross-reactive allergen sensitizations:

- PR-10s: Aln g 1, Bet v 1, Cor a 1.0103, Fag s 1
- Polcalcins: Aln g 4, Pnl p 7

##### PR-10 Proteins (PR10)

PR-10 inhalative: The major birch pollen allergen, Bet v 1, represents the prototype of all PR-10 allergens and is the primary sensitizer in regions with birch pollen exposure. The presence of PR-10 allergens in Fagales tree pollen explains IgE cross-reactivity between pollen from hazel, alder, beech, oak and hornbeam. PR-10 nutritive: PR-10 allergens in raw fruits, nuts, vegetable and legumes can induce oral allergy syndrome and sometimes severe allergic reactions in sensitized individuals. PR-10 allergens are not stable to heat and digestion.

##### Polcalcins (Ca++ binding proteins, PC)

Members of the PC family can cause inhalative symptoms. PC allergens can be found in pollen from grasses, trees and weeds. Inhalative symptoms manifest as allergic rhino-conjunctivitis and/or allergic asthma.

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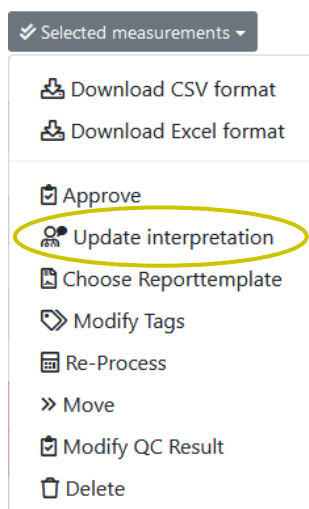
The screenshot displays the Raptor software interface. At the top, there is a navigation bar with 'Raptor', 'New measurements', 'Approved measurements', 'Start measurement', and 'Automate'. Below this is a 'Details' section with various input fields: Barcode (024E1852), Sample Code, Sampling Date (tt.mm.rrrr), Assay Date (07.10.2020), Patient Code, Patient Name, Date of Birth (tt.mm.rrrr), Info, Doctor Info, and Tags. A red oval highlights the 'MOD: Pollen allergens' field. Below the form are buttons for 'Report', 'Save', and 'Save and approve'. The bottom section shows a large image of a multiplex assay with numerous blue and green spots on a dark background. Navigation arrows 'Previous' and 'Next' are visible at the top right of the image. At the bottom, there is a status bar with 'Received: 07.10.2020 14:44', 'ImageXplorer: 23156891', 'Analysis: 07.10.2020 14:44', and 'Automatic QC: ✓ OK'.

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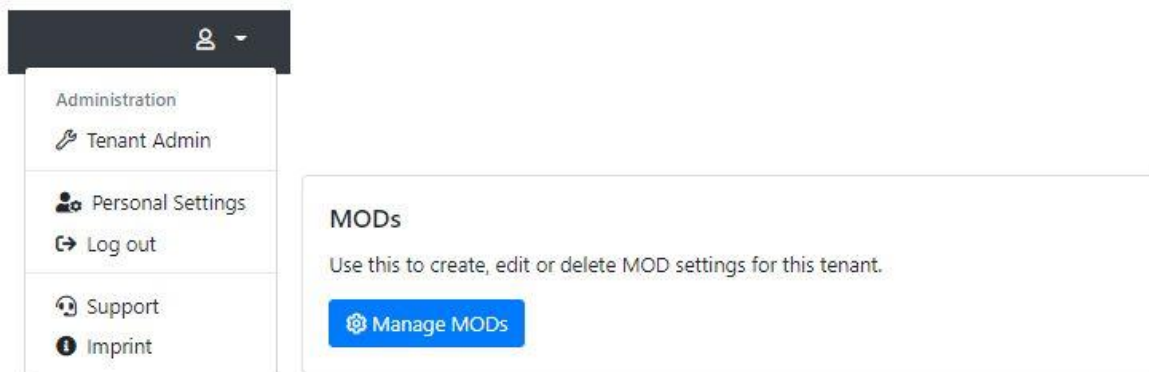
**Type:**

**Deactivated:**

You cannot delete an MOD once it is in use. But you can deactivate it, so that it's not available for new measurements anymore.

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**\* Name:**

**Default timezone:**

Configure a timezone to see date and times in (instead of UTC). Please note that users can choose to use a different timezone.

**Default report template (ALEX 1):**

**Default report template (ALEX 2):**

**Default report template (FOX):**

**Default report template (PAX):**

**Default report language:**

**Default ALEX2 MOD:**

**Default FOX MOD:**

**Default PAX MOD:**

**Force MOD:**

Requires a measurement to always have an MOD assigned. If this setting is enabled, a measurement cannot be approved without an MOD assigned.

**Hide Global MOD:**

If this setting is enabled, only the tenant-specific MOD will be available to select, and the global system-wide MOD will be hidden.

**Display MOD Excluded Spots As Research Use Only:**

If this setting is enabled, spots excluded by an MOD will not be hidden from the measurement image, but displayed as a "RUO spot".

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Report Template:

Force MOD:

Requires a measurement to always have an MOD assigned. If this setting is enabled, a measurement cannot be approved without an MOD assigned.

Hide Global MOD:

If this setting is enabled, only the tenant-specific MOD will be available to select, and the global system-wide MOD will be hidden.

Display MOD Excluded Spots As Research Use Only:

If this setting is enabled, spots excluded by an MOD will not be hidden from the measurement image, but displayed as a "RUO spot".

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**Force MOD:** False  
Requires a measurement to always have an MOD assigned. If this setting is enabled, a measurement cannot be approved without an MOD assigned.

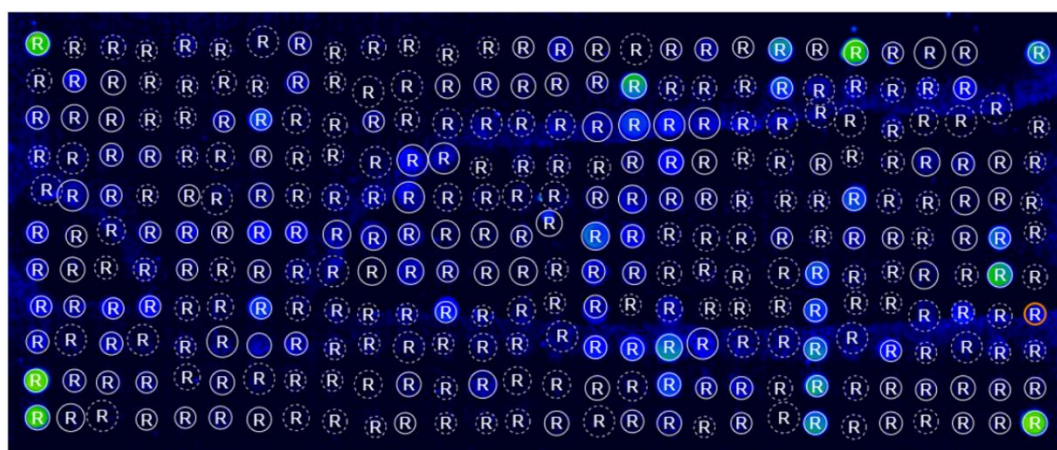
**Hide Global MOD:** False  
If this setting is enabled, only the tenant-specific MOD will be available to select, and the global system-wide MOD will be hidden.

**Display MOD Excluded Spots As Research Use Only:** True  
If this setting is enabled, spots excluded by an MOD will not be hidden from the measurement image, but displayed as a "RUO spot".

**Configuration**

**Calibration string:**  
Allows to override the calibration information from the cartridge's QR-Code. Leave empty to use the calibration information from the QR-Code on the cartridge (the usual behaviour).

**Research spots:** Show research spots  
If enabled, "research use only" spots will be displayed on measurement details and arrays. This does not affect the export behaviour or the reports of measurements.

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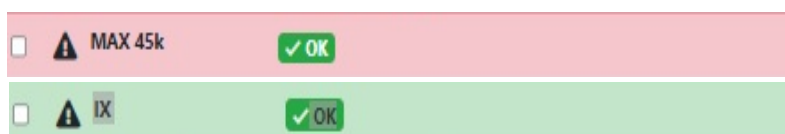
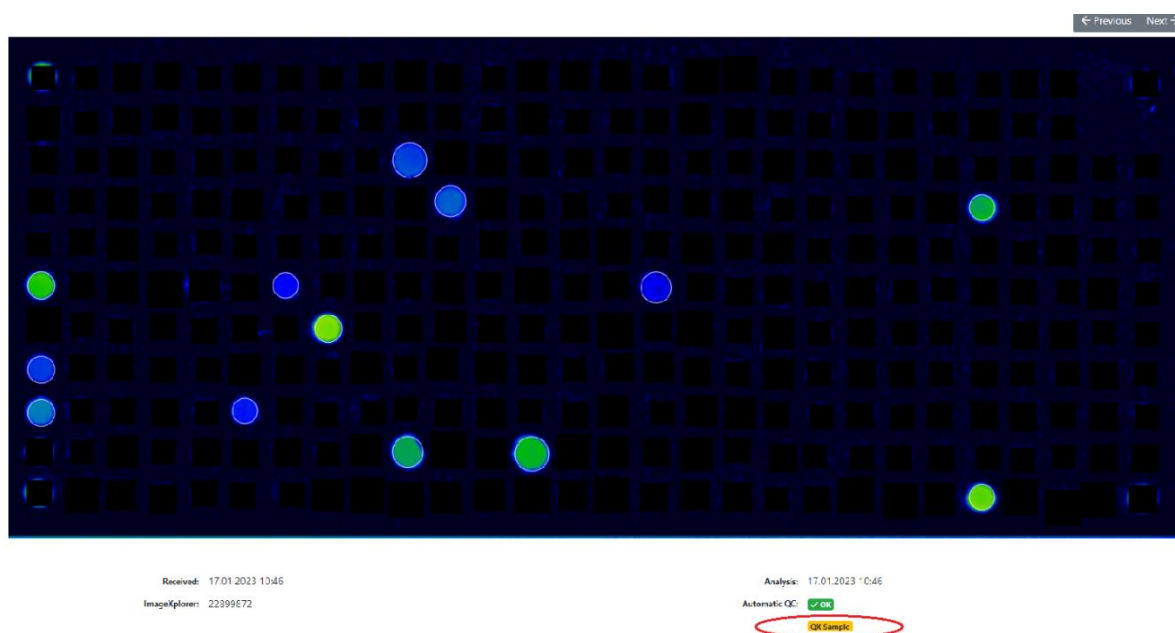
**New Measurements**

Search by barcode, patient, sample, tag, analysis date, a

Selected measurements ▾

- Download CSV format
- Download Excel format
- Approve
- Update interpretation
- Re-Process
- » Move
- Delete**

| Tags | Product type |
|------|--------------|
|      | FOX          |
|      | FOX          |
|      | ALEX2        |
|      | FOX          |
|      | ALEX2        |
|      | ALEX1        |

### Delete measurements

You are about to delete the following measurements. Please note that this action is irreversible and cannot be undone.

| Barcode  | Analysis Date    |
|----------|------------------|
| 02AAB160 | 16.12.2019 13:41 |

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|                 |   |         |      |
|-----------------|---|---------|------|
| Plant Food Food | M | Jug r 3 | 0.40 |
| Plant Food Food | M | Jug r 1 | 0.38 |

|            |   |         |      |
|------------|---|---------|------|
| Plant Food | M | Jug r 3 | 0.40 |
| Food       |   |         |      |
| Plant Food | M | Jug r 1 | 0.38 |
| Food       |   |         |      |

Delete request scheduled now. [Cancel delete request](#)

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ign in. Can be

### Background jobs

View details about running background jobs and manually schedule runs.

[View dashboard](#)

### Measurement delete / unapproval log & requests

Show logging and pending requests for measurement delete and unapproval.

[Show log & requests](#)

#### Measurement unapproval / delete requests

Open requests [Log & Finished requests](#)

| Requested UTC       | Requesting User (Email)   | QR-Code  | Request Type | Action                                        |
|---------------------|---------------------------|----------|--------------|-----------------------------------------------|
| 11.10.2023 13:05:28 | Test Test (Test@test.com) | 02AYQ145 | Unapprove    | <div><div>Accept</div><div>Reject</div></div> |
| 11.10.2023 12:59:27 | Test Test (Test@test.com) | 02AYQ14C | Delete       | <div><div>Accept</div><div>Reject</div></div> |

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Measurement data completion

Manage pre-defined meta information that can be assigned to a measurement.

Administration

Tenant Admin

Manage Measurement Data Completion

**RAPTOR** New measurements Approved measurements Start measurement Automats

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## Add new user on Result Sharing Portal

General configuration

First name:

Last name:

E-Mail address:

Username:

It is a good idea to use the users e-mail-address as username as well.

Password:

Send an invitation e-mail to user automatically

Status:

Locked (user cannot log in)

Enabled (user can log in)

Locked (user cannot log in)

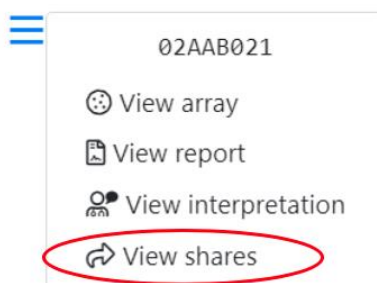
Doctor Info Text:

This information will be printed on a patients report.

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## ≡ Share

You are about to share a measurement with a user of the User Portal. Please note that the doctor will have access to the results of the measurement. Once you share a measurement you'll be provided with a link to access the shared information via the encrypted User F

Barcode: 02AAB021

Share with: X Dr. (testuser2@macroarraydx.com)

If you can't find a specific user of the User Portal, go to the [tenant administration](#) to configure the list of available users of the User Portal.

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| User   | Link                                                                                                                                                                                              | Shared       |        |
|--------|---------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------|--------------|--------|
| RP Bak | <a href="https://resultshare.raptor-server.com/s/07PXEpC7ZNYthM4PdRfmc4NR5pUtWq5FJcqOY7Ry6oEx08U5j">https://resultshare.raptor-server.com/s/07PXEpC7ZNYthM4PdRfmc4NR5pUtWq5FJcqOY7Ry6oEx08U5j</a> | 7 months ago | Revoke |

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- Administration
  - Tenant Admin
  - System Admin
- Personal Settings
- Log out**
- Support
- Imprint

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<https://resultshare.raptor-server.com>.

The screenshot shows the 'Result Sharing Portal' header with a language dropdown set to 'English (en)'. Below the header, a welcome message states that users need a username and password to log in. The login form includes two input fields: 'Your username' and 'Your password' (with a toggle icon). A blue 'Log in' button is positioned below the fields. A link for 'request a new one' is provided for users who forgot their password.

Result Sharing Portal English (en) ▼

### Welcome to the Result Sharing Portal

The Raptor-Server Result Sharing Portal allows users to view shared measurements. You will need a username and password to log in to view all measurements that have been shared with you. Also, a sharing link will allow to view specific shared measurements.

Please enter your username and your password:

Your username

Your password

Log in

Did you forget your password? No problem, you can [request a new one](#).

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This block shows a detailed view of the login form. It includes the instruction 'Please enter your username and your password:', followed by two input fields. The first field is labeled 'Your username'. The second field is labeled 'password' and has a toggle icon. A blue 'Log in' button is at the bottom.

Please enter your username and your password:

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## Measurements

These are the measurements that are available for you. You may view the details of each measurement, or download the report PDF file.

Search by barcode, patient or sample.

| Measurement              | Product | Patient | Sample   | Date       |
|--------------------------|---------|---------|----------|------------|
| <a href="#">01AAK239</a> | ALEX1   |         | 11852162 | 7/12/2019  |
| <a href="#">01AAK281</a> | ALEX1   |         | 04824821 | 12/18/2019 |
| <a href="#">01AAK891</a> | ALEX1   |         | 66560260 | 1/28/2020  |
| <a href="#">02AAA015</a> | ALEX2   |         |          | 2/21/2020  |

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### Measurement 02AYQ143

**Product:** ALEX2  
**Patient:**  
**Date of Birth:**  
**Sample:**  
**Info:**  
**Doctor Info:**  
**Sampling Date:**  
**Assay Date:** 10/12/2023  
**Processed Date:** 10/12/2023  
**Approval Date:** 10/12/2023

#### Report Language:

English (United Kingdom) ▼

You can request a new RAVEN interpretation that will be included in the report.

[Request new RAVEN interpretation](#)

You can download a PDF report that includes all information and results of this measurement.

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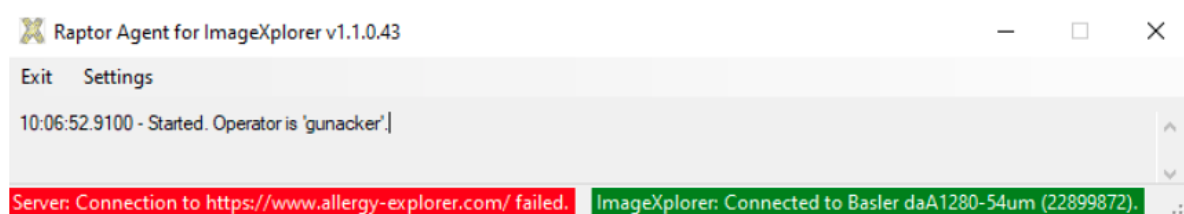
Settings

Please enter the settings for your ImageXplorer. You can find your ImageXplorer key when you log in to Raptor. Please contact support if you're not sure what the proper settings are for your ImageXplorer.

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Raptor Agent for ImageXplorer v1.1.0.43

Exit Settings Open Raptor

10:06:52.9100 - Started. Operator is 'gunacker'.  
10:11:33.9399 - Disconnecting from server ...  
10:11:35.0321 - Server confirms successful authentication to tenant MADx.]

Server: Connected to MADx. ImageXplorer: Connection failed.

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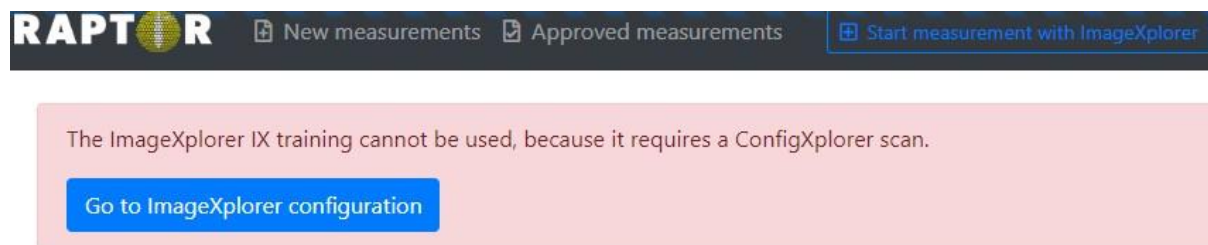
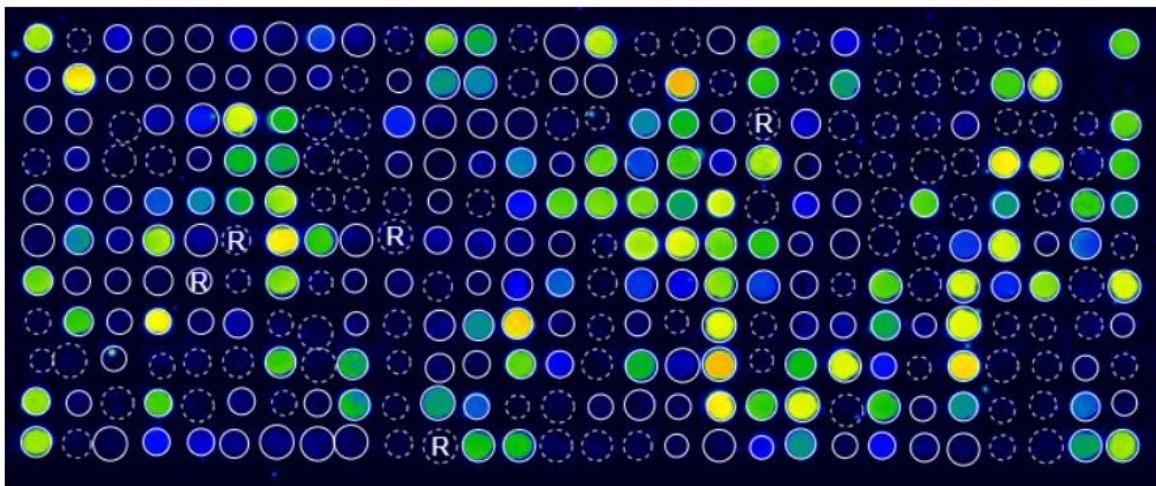
Analysis: 06.04.2023 13:37

ImageXplorer: 40287249

Automatic QC: ✗ FailedAnalysis Error: Insufficient data transfer - check connection.

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© Copyright by MacroArray Diagnostics

MacroArray Diagnostics (MADx)

Lemböckgasse 59, Top 4

1230 Vienna Austria

+43 (0)1 865 2573

[www.madx.com](http://www.madx.com)

Číslo verze: 22-IFU-01-CZ-08 (EN-25)

Datum vydání: 2025-11