RAPTOR

SERVER ANALYSIS SOFTWARE

UPUTA ZA KORIŠTENJE

SADRŽAJ

I. Kratice.....	4
II. Predviđena namjena	4
III. Sažetak i objašnjenje	5
III.1 Operacija	5
III.1.1 Operacija On-premise	6
IV. Princip postupka.....	7
V. Rječnik simbola	8
VI. Sigurnosne upute	9
VI.1 Kvalifikacija operatera.....	9
VI.2 kopija podataka.....	9
VI.3 Upozorenja i mjere opreza	9
VII. upute.....	10
VII.1 Postavljanje softvera RAPTOR SERVER.....	10
VII.1.1 Prijaviti se	10
VII.1.2 Osobne postavke	13
VII.1.3 Administrator stanara	13
VII.1.4 Dodajte korisnika stanarima	16
VII.1.5 Preuzimanje softvera agenta.....	20
VII.1.6 Ažuriranje softvera agenta	23
VII.1.7 Snimanje slike, čitanje QR koda i pronalaženje mreže	24
VII.1.8 Način obrade slike.....	28
VII.1.9 Pregled mjerenja	37
VII.1.10 Odobravanje i izvoz mjerenja	40
VII.2 Izvješće.....	45

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Welcome to RAPTOR SERVER Analysis Software, the analysis software for the [ALEX multiplex test](#). Please log in below to continue.

Log in with Azure AD / Office 365	Log in with Username & password
If your organization integrates the software into its Microsoft Azure/Office 365 user directory (single-sign-on) click the button below. You will be redirected to the Microsoft login screen.  Log in	Please enter your username and password. Your username Your password   Log in Forgot your password? No problem, you can request a new one.

Klikom na ikonu oka unesenu lozinku možete učiniti vidljivom.

Log in with Username & password

Please enter your username and password.

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Contract approval required

Please review the following documents. You are required to confirm that you have read and agree with each document in order to use RAPTOR SERVER:

☐ [Bitte lesen](#) (last updated 23.11.2023 18:00)

☐ [RAPTOR SERVER IFU](#) (last updated 04.12.2023 11:05)

☐ [Terms of Service](#) (last updated 22.11.2023 01:00)

Confirm

By clicking the button, I confirm that I have reviewed these documents and agree.

istim novim skočnim prozorom odmah nakon prijave u RAPTOR SERVER Analysis Software i mora ih potvrditi.

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New Measurements					Last 7 days
QC	Product type / Source	QR-Code	Sample Code	Analysis Date	
❌ Failed	Research MAX 45k	99AFG0F7	NK FOX	30/03/2023 20:50	Details Report
❌ Failed	Research MAX 45k	99AFG0F8	N.G.	30/03/2023 20:50	Details Report
❌ Failed	Research MAX 45k	99AFG0F9	E18735	30/03/2023 20:50	Details Report
❌ Failed	Research MAX 45k	99AFG0FA	20220325w56	30/03/2023 20:50	Details Report
❌ Failed	Research MAX 45k	99AFG0EE	HM100038	30/03/2023 20:50	Details Report
❌ Failed	Research MAX 45k	99AFG04D	786118	30/03/2023 20:37	Details Report
❌ Failed	Research MAX 45k	99AFG016	HM100225	30/03/2023 20:37	Details Report
❌ Failed	Research MAX 45k	99AFG0CA	E17685_M	30/03/2023 20:37	Details Report
❌ Failed	Research MAX 45k	99AFG0CC	E17685_M	30/03/2023 20:37	Details Report
❌ Failed	Research MAX 45k	99AFG015	E17685_F	30/03/2023 20:37	Details Report

In total there are 270 new measurements in the selected time period.

[View all new measurements](#)

Approved Measurements					
QR-Code	Product type / Source	Sample Code	Analysis Date		
02AVI227	ALEX2	Anna LMacey	28/03/2023 21:17	Details	Report
02ARA379	ALEX2 IX	SM_28032023	28/03/2023 14:29	Details	Report
02ARA393	ALEX2 IX	RG_28032023	28/03/2023 14:26	Details	Report
02AVA31F	ALEX2 MAX 45k	PR_Mar2023_11	21/03/2023 15:05	Details	Report
02AVI3B9	ALEX2 MAX 45k	KF	17/03/2023 14:46	Details	Report
02AYT337	ALEX2 MAX 45k	2B46B46+CCD	14/03/2023 19:06	Details	Report
02ATL349	ALEX2 IX		22/02/2023 14:21	Details	Report
02AVE383	ALEX2 MAX 45k	AO	03/02/2023 09:41	Details	Report
02AXI29A	ALEX2 MAX 9k	OW	19/01/2023 19:12	Details	Report
02AUY3AE	ALEX2 MAX 45k	BSZ_Okt2022_8	18/10/2022 18:12	Details	Report

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9K-Test-Instrument

Not initialized

[Initialize](#) [Shutdown](#) [Monthly Maintenance](#)

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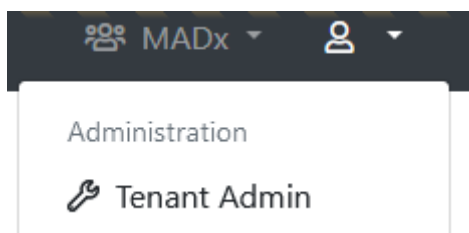
Configure "MADx"

General configuration		MAX device default settings	
* Name:	MADx	* Default Minimum RunId:	10
Default timezone:	(UTC+01:00) Amsterdam, Berlin, Bern, R	When a new MAX device is created, this value will be used for its Minimum Run Id	
Configure a timezone to see date and times in (instead of UTC). Please note that users can choose to use a different timezone.			
Default report template (ALEX2):	ALEX2		
Default report template (ALEX3, ALEX Air, ALEX Food):	check 1.21		
Default report template (FOX):	FOX (2025)		
Default report template (PAX):	PAX		
Default report language:	English (en)		
Default ALEX MOD:	< No MOD >		
Default FOX MOD:	< No MOD >		
Default PAX MOD:	< No MOD >		
Force MOD:	False		
Requires a measurement to always have an MOD assigned. If this setting is enabled, a measurement cannot be approved without an MOD assigned.			
Hide Global MOD:	False		
If this setting is enabled, only the tenant-specific MOD will be available to select, and the global system-wide MOD will be hidden.			
Display MOD Excluded Spots As Research Use Only:	False		
If this setting is enabled, spots excluded by an MOD will not be hidden from the measurement image, but displayed as a "RUO spot".			
MFA:	☐ Force MFA		
If this setting is enabled, users are forced to set up an MFA on sign-in, in case they haven't yet.			

Tenant-specific batch releases	
Restrict this tenant to only allow the following batches to be scanned or to restrict certain batches. Enter one per line.	
Use:	
☒ Whitelist	
☐ Blacklist	
Whitelisted Batches:	Enter one batch code including product identifier per line, for example "02AAA"
Blacklisted Batches:	Enter one batch code including product identifier per line, for example "02AAA"

Company Details

* Legal Entity Name:	MacroArray Diagnostics
* Street and Number:	Street 4
* Zip Code:	1230
* City:	New York
* Country:	United States of America
* Phone:	+4318652573
* E-Mail:	office@macroarraydx.com
* Legal Representative:	Dr. Christian Harwanegg
Website:	
Vat Number:	
Company Register Number:	

VII.1.3.1 GRUPE STAVLJANJA NA CRNU I BIJELU LISTU

Pojedinačne serije može staviti na crnu ili bijelu listu po zakupcu korisnik koji ima pravo pristupa zakupca "Can manage basic tenant configuration". Za crnu ili bijelu listu samo prebacite na "Bjeli popis" ("Whitelist") ili "Crni popis" ("Blacklist") i unesite šifre serije koje želite staviti na crnu ili bijelu listu.

Tenant-specific batch releases
Restrict this tenant to only allow the following batches to be scanned or to restrict certain batches. Enter one per line.
Use:
☒ **Whitelist**
☐ **Blacklist**

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You requested a new measurement from ImageExplorer "IX Trainingsraum". Please wait until the measurement is completed ...

Invalid batch detected! The measurement can't be processed because its batch is blocked.

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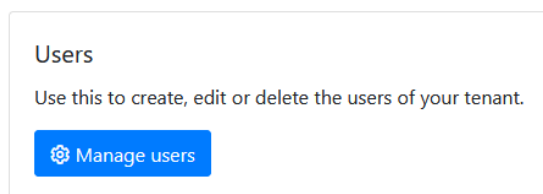
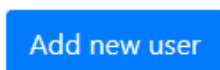
Login Credentials

Type: Username and Password ↕

Please select if the user should be validated against Azure AD (Office 365), or if you want to specify username and password yourself. It is recommended to use Azure AD whenever possible. This allows for increased security by supporting two-factor authentication. Also, it is more convenient for the user, because he or she does not have to remember a new username and password (single-sign-on).

Username:

It is a good idea to use the users e-mail-address as his username, because the username needs to be unique across the entire system.

Password:

Automatically send invitation e-mail ↕

Access Rights

Preconfigured groups

Use these pre-configured groups to automatically assign a set of access rights to this user. You can still add or remove specific access rights.

Reset Lab User Super Lab User Support/Distributor User System Administrator

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Tenants

This user is connected to the following tenants and has the following access rights:

	Tenants	Can share measurements	Can edit measurements	Can edit personal data	Can approve measurements	Can manage basic tenant configuration	Can manage API keys	Can manage ImageXplorers	Can view agent connections	Can manage users	Can manage users of Result Sharing Portal	Can manage report templates	Can manage measurement data completion	Can unapprove tenant specific batch releases	System Administration	Can use a custom calibration string	Can manage Run Configuration	Can skip
☐	Another MADx Tenant	☒	☒	☒	☒	☒	☒	☒	☒	☒	☒	☒	☒	☒	☒	☒	☒	☒
☐	MADx Tenant	☒	☒	☒	☒	☒	☒	☒	☒	☒	☒	☒	☒	☒	☒	☒	☒	☒

Additionally this user can be added to the following tenants by checking the tenant name and hitting save afterwards. User rights w applied from current open tenant user.

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Additionally this user can be added to the following tenants by checking the tenant name and hitting save afterwards. User right applied from current open tenant user.

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Tenants

This user is connected to the following tenants and has the following access rights:

Edit selected tenants rights

☐	Tenants	Can share measurements	Can edit measurements	Can edit personal data	Can approve measurements	Can manage basic tenant configuration	Can manage API keys	Can manage ImageY
☒	Another MADx Tenant	☒	☒	☒	☒	☒	☒	☒

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Add new ImageXplorer

General information

Name:	
Key:	Cxltcg8okSwQ6CqJ4EZLWGnw2PdJ3
	This key is required when installing an Agent.
ImageXplorer Serialnumber:	
	Serialnumber of connected ImageXplorer.
Notes:	
Internal Notes:	
	For internal use only.
Disabled:	Not disabled

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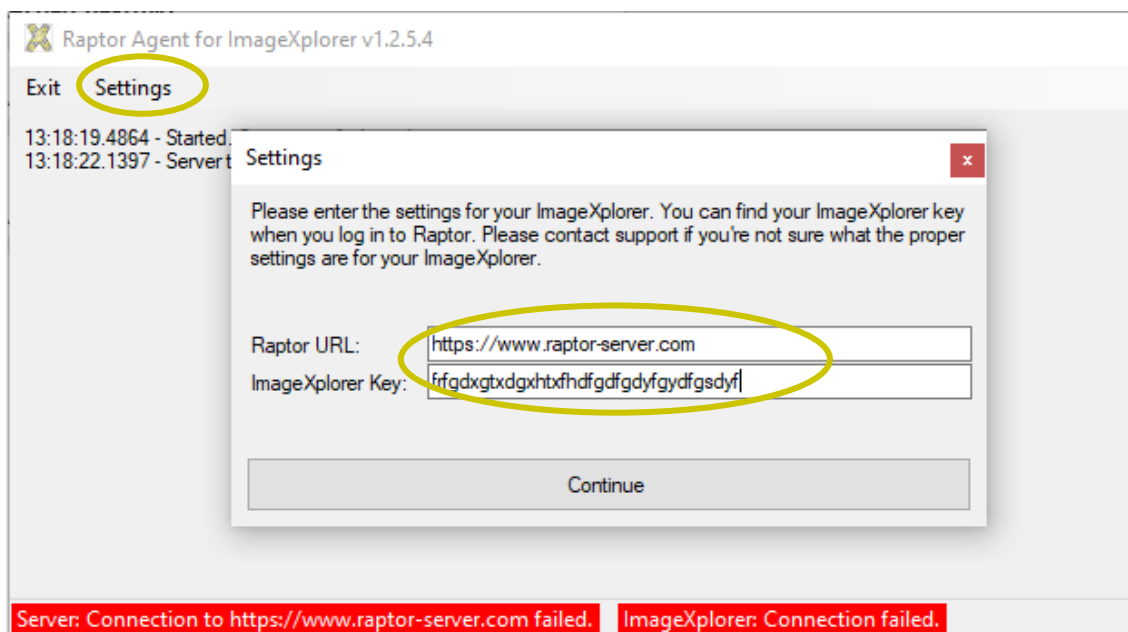
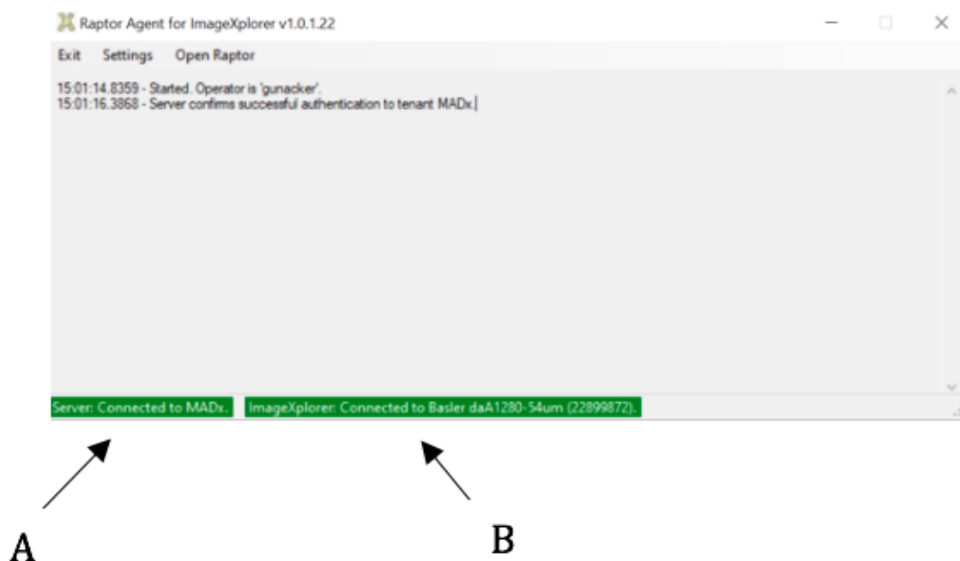
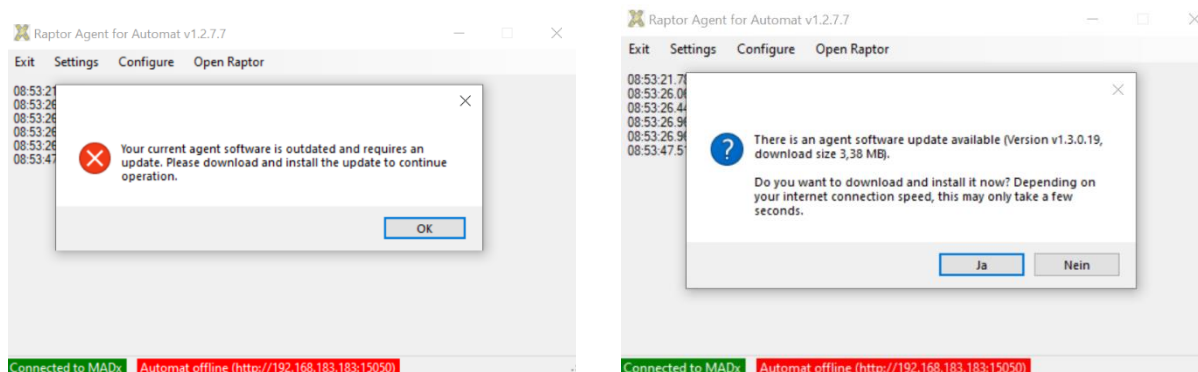
Details

Barcode: 02AE348B
Sample Code: T2
Requisition Code:
Sampling Date: 01.10.2021
Assay Date: 04.10.2021
Patient Code: TE
Patient Name: Test
Date of Birth: 17.05.2010
Info:
Doctor Info: Dr. A
Choose a doctor... Insert doctor info
Tags: Tag Info
MOD: < No MOD >
Report Template: Version 2 (ALEX2 only) - Deutsch (de)

Report Save Save and approve

Previous Next

Received: 04.10.2021 16:23
ImageXplorer: Z2124137
Analysis: 04.10.2021 16:23
Automatic QC: OK

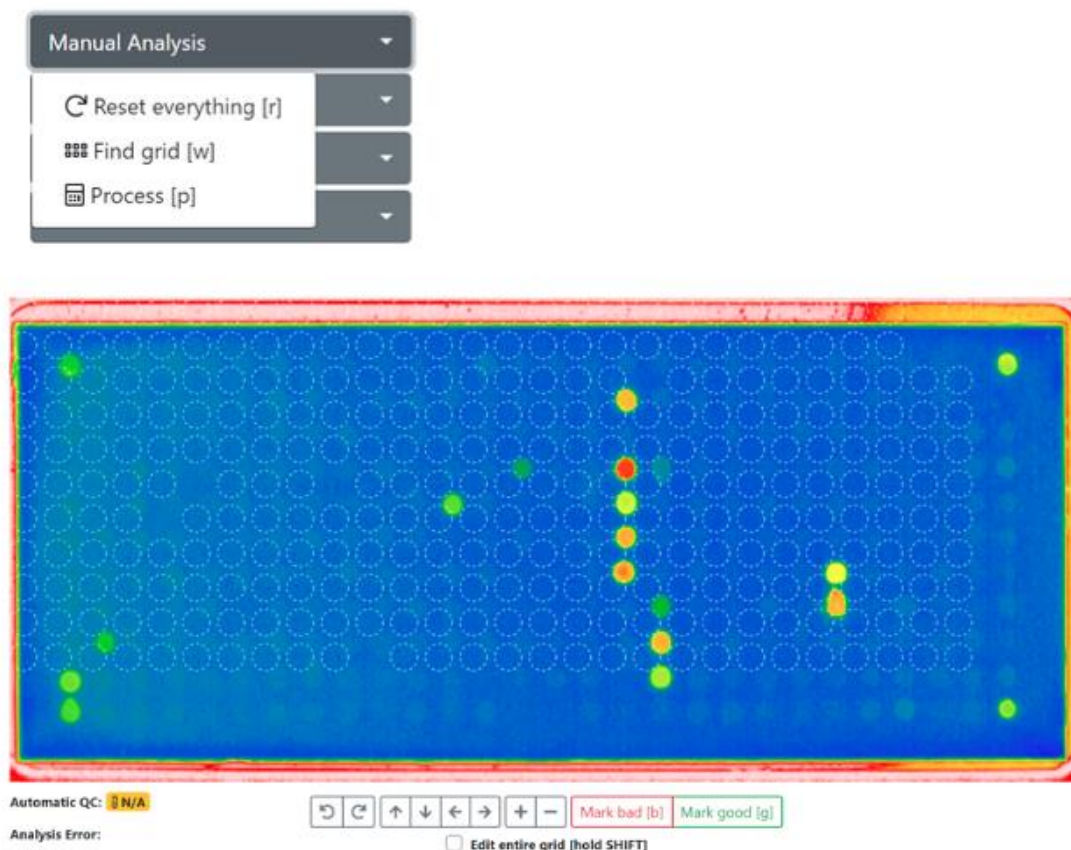
180°

+ -

Mark bad [b] Mark good [g]

☐ Edit entire grid [hold SHIFT] ☐ Edit multiple features [hold CTRL]


Ako automatsko pronalaženje položaja ne uspije, korisnik će biti upozoren, a kontrola kvalitete mjerenja (QC) bit će postavljena na FAIL. U tom slučaju, korisnik može pokušati ručno pozicionirati niz resetiranjem mreže pritiskom na „R” ili otići na „Manual Analysis“ → „Reset Grid”. Nakon resetiranja, cijelu rešetku možete pomicati mišem dodatnim pritiskom na „Shift”, ili postavljanjem kvačice na „edit entire grid”. Imajte na umu da rešetku možete pomicati samo unutar određenih granica.

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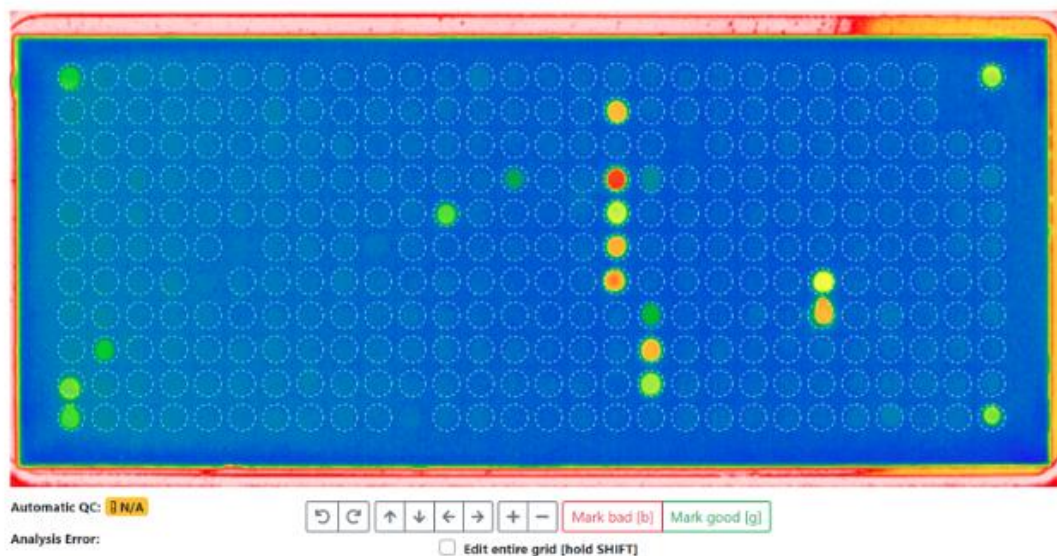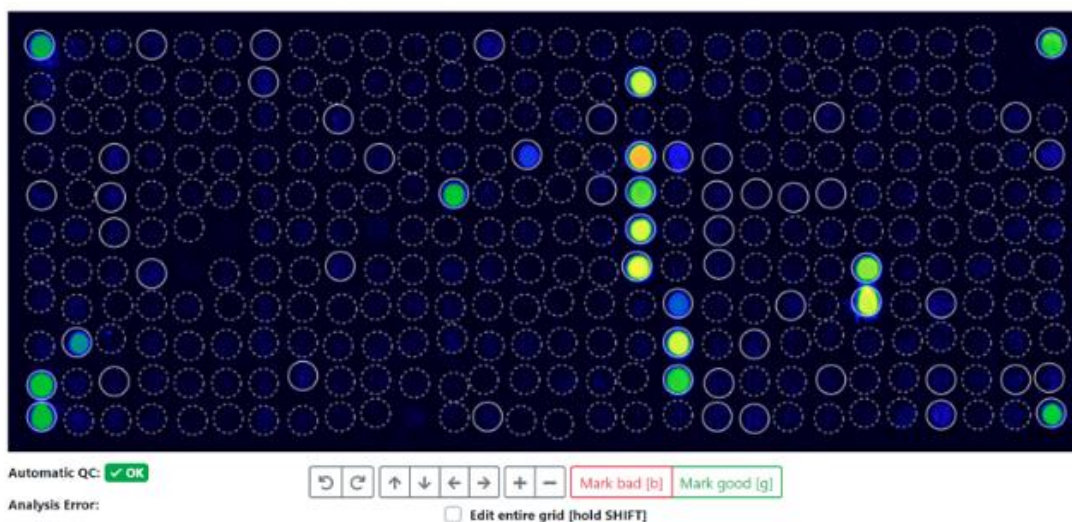
Show Analysis & QC Logs

Reset everything and reprocess

This discards all modifications to the array, and re-runs the complete analysis process from the original image data. Please note that any manual changes will be irreversible lost.

Back to Measurement

Kliknite na ovaj gumb da biste dobili informacije o pozadini, vrijednostima QC točaka i primijenjenim referentnim vrijednostima. Ovo je posebno korisno ako trebate saznati zašto mjerenje nije uspjelo u QC-u. Da bi QC bio u redu, pozadina mora biti unutar određenog raspona i sve točkice moraju doseći određenu vrijednost praga. Ako ovi kriteriji kontrole kvalitete još uvijek nisu ispunjeni nakon ručnog podešavanja točaka na slici, obratite se svom lokalnom distributeru ili MADx podršci.

Analysis Log

```
'PreProcessedPixels' took 128ms.
Using definition '02_ALEXII_IVD_v1.6.0'.
'ApplyArrayDefinition' took 74ms.
Parsing calibration string ...
'ApplyDefinitions' took 75ms.
'FindGrid' took 1ms.
'AlignFeatures' took 862ms.
'CropImage' took 1ms.
'QuantifyFeatures' took 231ms.
'SubtractBackground' took 681ms.
Parsing calibration string ...
Using calibrators from QR-Code.
Total Igt (tIgt) normalized by calculating 0.330999755859375E-07x²+0.05697999954223633x+-38.69269921875
(x=8515), which results in 515.
'ReQuantify' took 57ms.
```

QC Log

These are the values used for QC (ALEX v2): Curve Correction=0.9997370867894242, tIgt=8515, GD1=28085 (SNR=105.38), GD2=29369 (SNR=185.28), GD3=28569 (SNR=147.53), GD4=28385 (SNR=128.1). Reference values are as follows: Guide Dots must have a value above 5000; each Guide Dot spot must exist and be unique; Guide Dot spot must not be set to bad. Average background must be between 4000 and 16000. QC (ALEX v2) succeeded.


Background Log

Calculated background statistics for 164 pixels (from a total of 182, skipped smallest 9, and highest 9 pixels). The average is 9.439, the minimum 7973 and the maximum 11792 (minimum and maximum is calculated including skipped pixels). This results in an average difference of 628, which is an SD of 49.17 and a CV of 0.52%. For comparison, there is an overall total of 282944 pixels. The overall average is 14.871, the minimum 6.012 and the maximum 64.965.

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
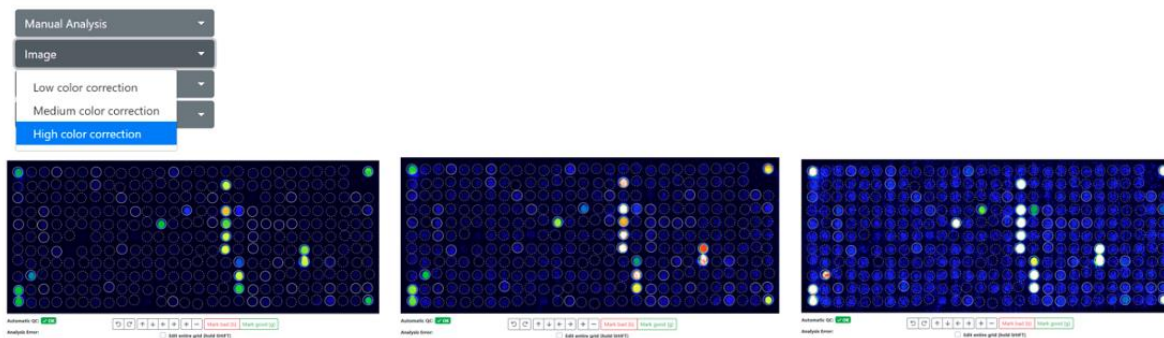
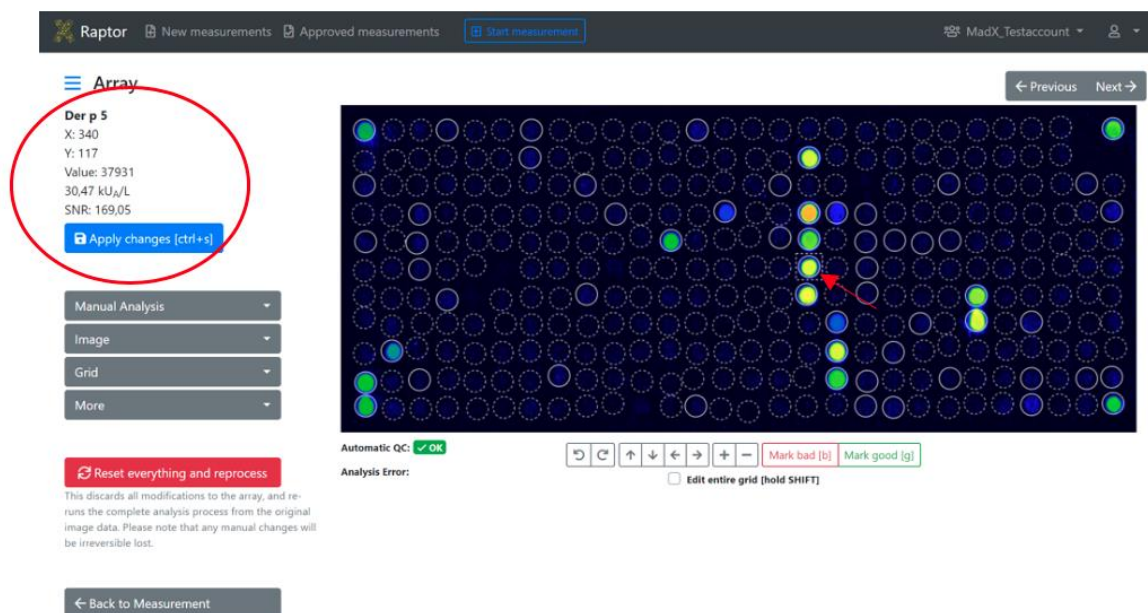
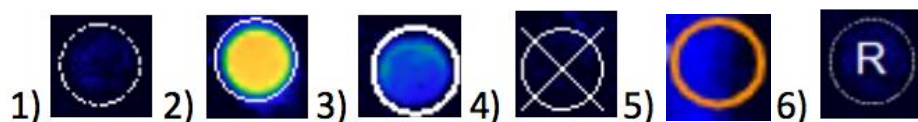
Milk				
Cow, milk	Bos d 4	M	α-Lactalbumin	0,38
Cow, milk	Bos d 5	M	β-Lactoglobulin	0,90 
Cow, milk	Bos d 8	M	Casein	6,04
Cow, milk	Bos d_milk	E		3,40

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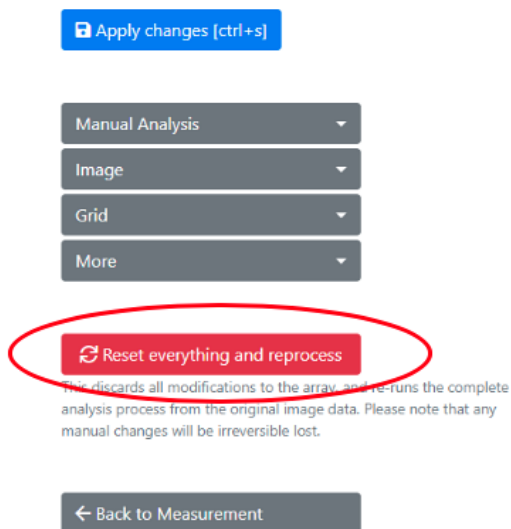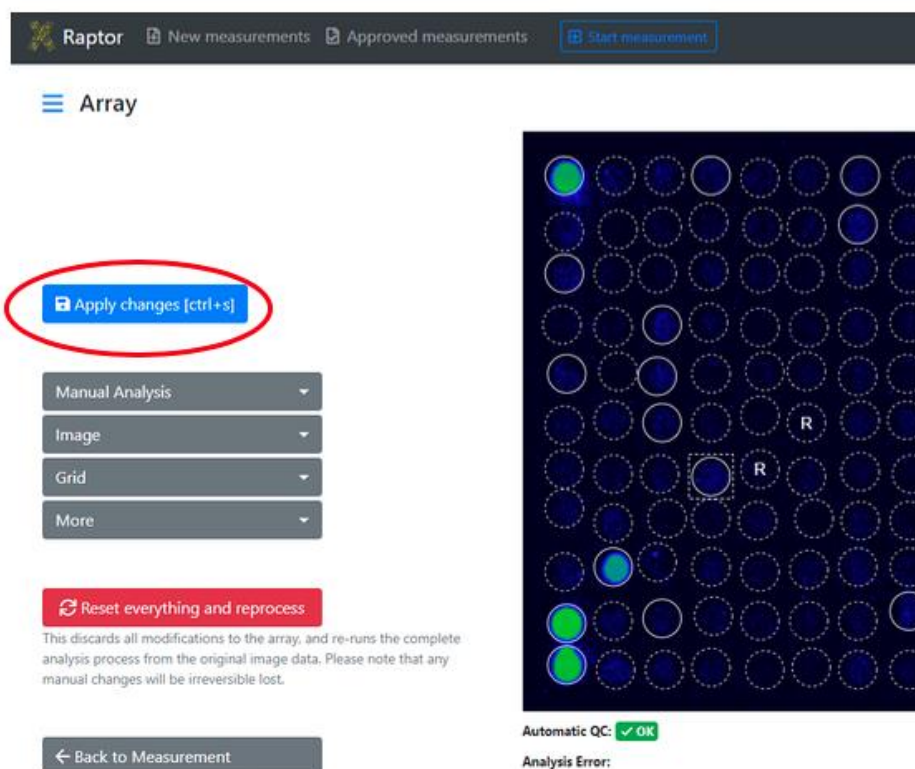
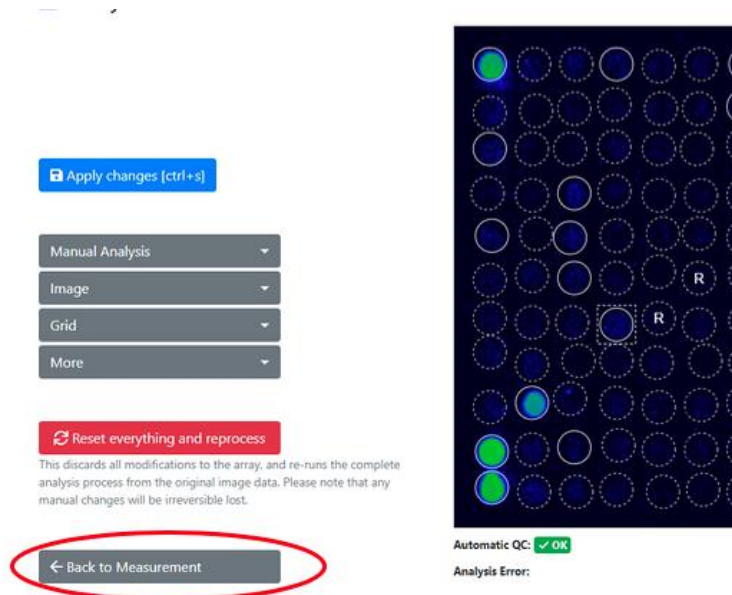
	All allergens	All allergens
	Only positive allergens	
Tissue &	ire	kUA/L
Pollen Inhalation	E Aca m	≤ 0.10
Mites	E Aca s	7.11

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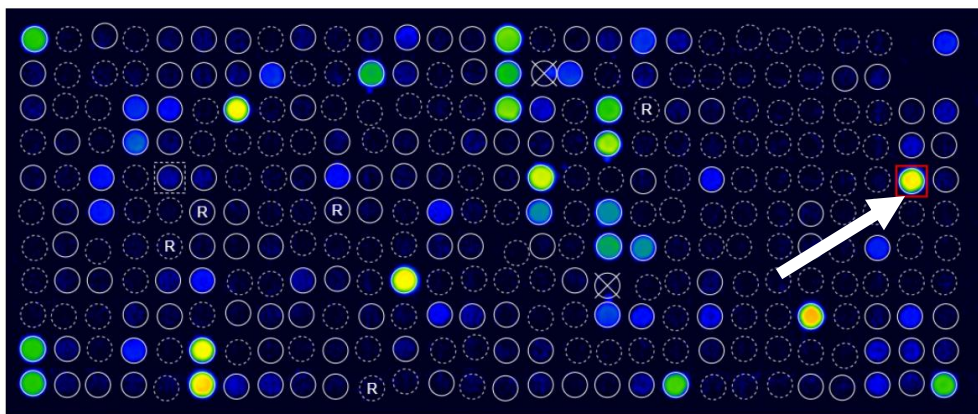New Measurements

Search by QR-Code, patient, sample, tag, analysis date, approval date ...

☐	Source	QC	Tags	Product type
☐	IX	✓ OK		ALEX2
☒	IX	✓ OK		ALEX2
☒	IX	✓ OK		ALEX2
☒	IX	✓ OK		ALEX2

-
- Download CSV format
 - Download Excel format
 - Approve
 - Update interpretation
 - Choose Reporttemplate
 - Modify Tags**
 - >> Move
 - Delete

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Re-process measurements

You are about to re-process the following measurements. undone, the existing analysis result will be irreversibly over

QR-Code	Sample Code
02APK3C8	Dummy8
02APK258	Dummy1
02APK07F	Dummy13

U polju za pretraživanje mjerenja se mogu pretraživati prema njihovoj oznaci, vrsti proizvoda, QR-kodu, šifri uzorka, šifri pacijenta i imenu pacijenta kao i datumu analize.

New Measurements

Search by QR-Code, patient, sample, tag, analysis date, approval date ...

☐	Source	QC	Tags	Product type	QR-Code	Sample Code
☐	IX	✓ OK		ALEX2	02AGT957	

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New Measurements

☐	Source	QC	Tags	Product type
☐	Recaptured	✓ OK	recaptured	ALEX1
☐	Recaptured	✓ OK	recaptured	ALEX1
☐	QR-Code was scanned more than once.		recaptured	ALEX1

VII.1.10 ODOBRAVANJE I IZVOZ MJERENJA

Nakon analize i pažljive revizije rezultata, mjerenja se mogu odobriti na dva načina. Međutim, pobrinite se da riješite neizvjesna mjesta prije odobrenja mjerenja, vidi također VII.1.8.1:

1. Jedno ili više mjerenja može se odabrati označavanjem i zatim odobriti u padajućem izborniku u skupu:

New measurements
Approved measurements

New Measurements

Search by QR-Code, patient, sample, tag, analysis date, approval date ...

☐	Source	QC	Tags	Product type	QR-Code
☒	IX	✓ OK		ALEX2	02AMC180
☒	IX	✓ OK		ALEX2	02AMC078
☒	IX	✓ OK		ALEX2	02AKV2D9

- Download CSV format
- Download Excel format
- Approve**
- Update interpretation
- Re-Process
- Move
- Delete

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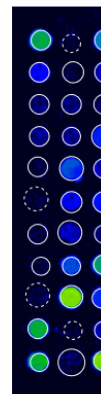
Approve measurements

You are about to approve the following measurements.

There are measurements with one or more warnings. Please make sure to double-check these measurements before approval.

QR-Code	Sample Code	Analysis Date
02AKV035		28.04.2022 11:14

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RAPTOR
New measurements
Approved measurements
Start measurement

Download selected measurements

Approved Measurements

Search by QR-Code, patient, sample, tag, analysis date ...

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RAPTOR
New measurements
Approved measurements
Start measurement
Automats
MADx Testtenant


Exports

Certain types of data exports (for example, the export of many PDF reports at once) take a few minutes to prepare for download. Here download your requested exports. Please note that you can keep working, all exports are done in the background and you can download when they are completed.

Export requested	# of measurements included	Export completed
8/16/2021 11:07 AM	4	0 / 4
8/16/2021 11:06 AM	4	8/16/2021 11:06 AM
8/16/2021 11:04 AM	3	8/16/2021 11:04 AM

Administration
Tenant Admin
Exports
Personal Settings
Log out
Support
Imprint
Download
Download

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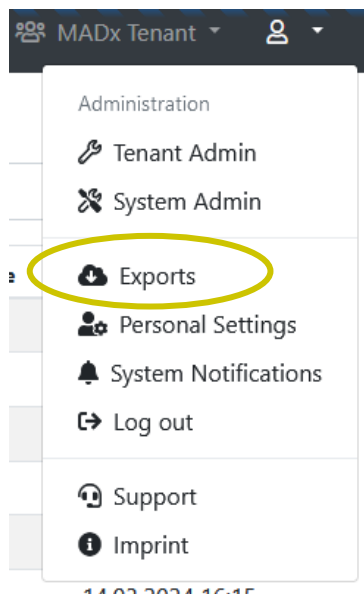Exports

Certain types of data exports (for example, the export of many PDF reports at once) are processed in the background. Please note that you can keep working, all exports are done in the background.

[Export multiple measurements](#)

Export requested	# of measurements included
06.07.2022 08:52	3

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Export multiple measurements

Please note that **Report PDF** and **ASTM** export is only available for certain data types.

QR-Code/Product/Batch		Sa
02ABC		
Analysis Date (from)	Analysis Date (to)	
TT.MM.JJJJ	TT.MM.JJJJ	
Tag	Status	
	All	
☐ Export CSV	☐ Export Excel	
Search Measurements		Cancel

Please note that **Report PDF** and **ASTM** export is only

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Edit report template

Your changes have been saved successfully.

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PDF naming

Setting Type: Manual

The filename of the report PDF can be customized. Either built automatically based on your selection below or manually adjusted in the order of your needs.

Name Pattern: Report_{PatientId}_{MeasurementQrCode}

Available Elements: {PatientId} ; {PatientName} ; {SampleId} ; {MeasurementQrCode} ; {DateAndTime}

PDF naming

Setting Type: Automatic

The filename of the report PDF can be customized. Either built automatically based on your selection below or manually adjusted in the order of your needs.

Include Patient ID: ☒

Include Patient Name: ☒

Include Sample ID: ☐

Include Measurement QR-Code: ☐

Include Date & Time: ☒

Footer

Footer text:

Will be printed in the footer if provided and rendered as HTML.

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Configure report template "New report template"

The screenshot shows a configuration window titled "Configure report template 'New report template'". It has four tabs: "General", "Design", "Header", and "Content". The "Design" tab is active. It contains three main settings:

- PDF format:** Two radio buttons, "A4" (selected) and "Letter". A note to the right says: "Specifies the paper size of the output PDF."
- Font:** A dropdown menu showing "Arial". A note to the right says: "The font family of the whole report texts can be adjusted."
- Background image:** A text input field followed by a "Browse" button. A note to the right says: "If set, this image is used as background on the first page of the PDF." Below this, it says "Maximum file size 1 MB".

- boja: Osnovne boje predloška i klase rezultata mogu biti prilagođeno u cijeloj boji paleta
- Logotipi: Dodatak logotipima na obje strane zaglavlja naslovne stranice, minijatura logotip se može dodati u zaglavlje svih stranica. Prema zadanim postavkama, ovo je minijaturalogotip zaglavlja s lijeve strane.
- Potpis: Osim toga, slika potpisa se može učitati i dodati nakraj izvještaja.
- Prilagođeni indikator intenziteta (prilagođeni SVG za prikaz intenziteta alergena može biti dodano)

Color settings

Color palette: Palette 1

Color 1:

Color 2:

Color 3:

Color 4:

First level headline color:

Second level headline color:

Third level headline color:

Logos

Logo (left): Browse
Maximum file size 1 MB
This logo will appear on the **top left corner** of the first page of the report.

Text (center):
This text will appear either on the left, right, or center of the top of the first page (depending on your configured logos). Supports the placeholders `{{ PATIENT_PORTAL_QRCODE }}` and `{{ PATIENT_PORTAL_URL }}` to dynamically insert measurement information.

Logo (right): Browse
Maximum file size 1 MB
This logo will appear on the **top right corner** of the first page of the report.

Small logo: Browse
This logo is used on all detail pages. If not set, the default logo is used.

Signatures

Left: Browse
Maximum file size 1 MB
This signature or image will appear on the last page, left aligned.

Center: Browse
Maximum file size 1 MB
This signature or image will appear on the last page, center aligned.

Right: Browse
Maximum file size 1 MB
This signature or image will appear on the last page, right aligned.

Indicators

Custom intensity indicator:

A custom SVG for displaying allergen intensity can be pasted here. Will be styled according to specified colors.

⚠ The SVG element must contain the attributes **width** and **height** with a minimum dimension of 16 for correct display in PDF.

🌿 SVGs paths which should be colored must require the **colorize** class as well as a fill defined in the **defs** segment of the SVG.

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Configure report template "New report template"

Header content:		Left half	Right half
		Custom headline	Custom headline
		Patient Name	Doctor Information
		Patient Name	None
		Custom headline	
		Date of Birth	None
		Sample ID	None
		QR-Code	Additional Info text
		Analysis Date	None
		Amount of tested allergens	None
		Test method	Quality Control text

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General	Design	Header	Content
Allergens to include:	All allergens	Specifies which kind of allergens should be included in the report.	
Allergens sort order:	Alphabetically by allergen	Specifies the sort order of the allergens.	
Include Overview:	☒	Whether the allergen overview should be printed or not.	
Include product type in overview:	☒	Whether the product type should be printed in the header of allergen overview.	
Include Total IgE:	On the front page	Whether the information of total IgE should be included in the report and where.	
Cross-reactive information:	Include text	This section will be displayed underneath the allergen list itself.	
Include Interpretation:	Include manually	It can be specified if the Raven report interpretation information should be included in the report or not. This section is printed underneath the allergens overview.	

Additional information (below the list of results)		Those informations are integrated into the report in an extra section underneath the allergen list.
Approval date:	☐	
Assay date:	☐	
Doctor info:	☐	The information about the doctor which accompanied the measurement/patient.
Print date:	☒	When the report was generated.
Report info:	☐	The „additional info“ field belonging to the measurement.
Sample date:	☒	

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
Configure report template "New report template - Duplicate"

General	Design	Header	Content
Name:	New report template - Duplicate		
Support products:	ALEX ²		
Preferred language:	Deutsch (de)		

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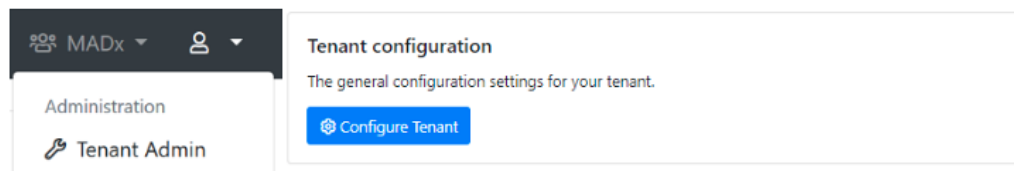* Name: MADx

Default timezone: (UTC+01:00) Amsterdam, Berlin, Bern, Rome, Stockholm, Vienna

Configure a timezone to see date and times in (instead of UTC). Please note that users can choose to use a different timezone.

Default report template (ALEX 1): Version 1 (ALEX1 & ALEX2)

Default report template (ALEX 2): Version 2 (ALEX2 only)

Default report language: English (United Kingdom) (en-gb)

Default MOD: < No MOD >

Require ImageXplorer testarray scan: Warn after 60 days, enforce after 90 days.

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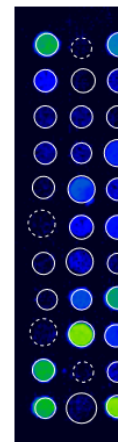
Raptor New measurements Approved measurements Start measurement Automate

Report

Template: Version 2 (ALEX2 only)

Language: English (en)

Usually this tenant's default template and language will be used when sharing or exporting measurements. You may change the template and language associated with this measurement by saving your current configuration.

You can download this report in PDF format once the measurement has been approved.

Management report template and settings
You have access to manage the report templates and settings via the tenant administration. Please note that any changes will affect all users.

MADX
MACROARRAY DIAGNOSTICS

PATIENT ID
PATIENT NAME
DATE OF BIRTH
SAMPLE ID
ASSAY CODE
02APH008
ANALYZED ON
11/04/2022
TESTED ALLERGENS
255
TEST METHOD CODE
ALEX2

REFERRING PHYSICIAN

NOTE
The internal QC (Fluorescence check for QC) was within acceptance range.

Lab report: Summary on detectable sensitisations

POLLEN
Grass Pollen

MICROORGANISMS
Fungal Spores & Yeast

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The screenshot shows the RAPTOR web application interface. At the top, there's a navigation bar with 'RAPTOR' logo and links for 'New measurements', 'Approved measurements', and 'Start m...'. Below this, the 'Approved Measurements' section features a search bar with the placeholder 'Search by QR-Code, patient, sample, tag, analysis date ...' and a 'Download selected measurements' button. A table displays three approved measurements, each with a checkbox, a source icon, the source 'IX', the product type 'ALEX2', and the QR-code '02AKV39D'. To the right, a dropdown menu is open, showing options: 'Report (PDF format)' (highlighted with a yellow circle), 'CSV format', 'Excel format', 'ASTM format', 'Update interpretation', 'Choose Reporttemplate', 'Share', 'Move', and 'Unapprove'.

	Source	Tags	Product type	QR-Code	Sample C
☒		IX	ALEX2	02AKV39D	
☒		IX	ALEX2	02AKV39D	
☒		IX	ALEX2	02AKV39D	

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Tags:

MOD:

Report Template: Version 2 (ALEX2 only) - Deutsch (de)

Download ▾

Report

Save

» Move to other tenant

[New measurements](#)
[Approved measurements](#)
[Start measurement](#)
[Automats](#)

Report

Template: Default

Language: English

[Save](#)
[Use defaults](#)

Usually this tenants default template and language will be used when sharing or exporting measurements. You may change the template and language associated with this measurement by saving your current configuration.

[Download PDF](#)

[Manage report template and settings](#)

You have access to manage the report templates and settings via the tenant administration. Please note that any changes will affect all users.

Patient ID:	
Patient Name:	
Date of Birth:	
Sample ID:	
Barcode:	02AAE
Sampled on:	
Tested on:	12/18/
Printed on:	12/19/
Note:	The internal QC (Plausibility)

Lab report: Summary on de

Pollen	Grass Pollen
	Tree Pollen
	Weed Pollen
Mites	House Dust M
Microorganisms	Fungal Spores
	Legumes
	Grain
	Spices
Plant-Based Food	Fruits
	Vegetables
	Nuts & Seeds
	Milk

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02AAB019

- View array
- View report
- View interpretation
- Approve measurement

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Rule set

RAVEN Ruleset A for ALEX

A rule set defines the RAVEN or RAVEN² engine that is used to interpret results.

Continue

Za "RAVEN Ruleset A and B for ALEX" otvara se stranica na kojoj se mogu unijeti postavke, kao što su jezik, specifični podaci o pacijentu (dob, spol, okolina) i simptomi pacijenta.

Patient symptoms

Asthma	None	⌵
Rhinitis	None	⌵
Conjunctivitis	None	⌵
Urticaria	None	⌵
Angioedema	None	⌵
Oral Allergy Syndrome	None	⌵
Eczema	None	⌵
Anaphylaxis	None	⌵
GastroentericSymptoms	None	⌵

≡ Interpretation

Settings

Language: English ⌵

Patient information

Gender: Female ⌵

Age: 30 years

Environment: Continental ⌵

 Request interpretation

Za “RAVEN² for ALEX” jezik i regija mogu se odabrati prije nastavka na anamnezu, koja se nudi kao upitnik.

Interpretation settings

Language:

English (en)

Region:

No specific region

 Continue with Anamnesis

RAVEN[®]

INTERPRETATION SOFTWARE

Overview

Spring

Summer

Perennial

Food

Insects

AIT

Medication

MADx data declaration for smartRaven

☒ I accept to enter clinical data for using the full interpretation function of smartRAVEN

Patient has reported any symptoms

☐ Yes ☐ No

Additional information about the patient

☐ Atopic Dermatitis

☐ Allergen-specific immunotherapy (AIT) - ongoing, completed or discontinued

☐ Medication

Submit

By clicking “Request interpretation” in case of “RAVEN Rulesets A and B for ALEX” or “Submit” in case of “RAVEN² for ALEX” the interpretation will be generated

Interpretation

Patient information

Gender: Female
 Age: 30 years
 Environment: Continental
 Asthma: None
 Rhinitis: None
 Conjunctivitis: None
 Urticaria: None
 Angioedema: None
 Oral Allergy Syndrome: None
 Eczema: None
 Anaphylaxis: None
 GastroentericSymptoms: None

[Request new interpretation](#)

This measurement already has an interpretation stored. Usually, it should not be necessary to request those again, but it could be necessary when the patients information has changed.

Interpretation

Raven Interpretation Summary

Sample Information

The sample was tested on ALEX² Barcode 02AAB019, interpretation date 12/19/2019.

Of the tested 295 allergens, 25 were above the detection limit of 0.3 kU_L/L. A sensitization can be an indicator of an allergy. For selected allergens, comments for interpretation are listed below.

Total IgE: 512 kU/L

The measured total IgE was 512 kU/L. Above a total IgE of 100 kU/L, allergy is likely.

Cross-Reactive Allergen Sensitization detected

Sensitization against allergen molecules which are markers of (broad) cross-reactivity between different allergen sources.

Detected cross-reactive allergen sensitizations:

- PR-10s: Aln g 1, Bet v 1, Cor a 1.0103, Fag s 1
- Polcalcins: Aln g 4, Phi p 7

PR-10 Proteins (PR10)

PR-10 inhalative: The major birch pollen allergen, Bet v 1, represents the prototype of all PR-10 allergens and is the primary sensitizer in regions with birch pollen exposure. The presence of PR-10 allergens in Fagales tree pollen explains IgE cross-reactivity between pollen from hazel, alder, beech, oak and hornbeam. PR-10 nutritive: PR-10 allergens in raw fruits, nuts, vegetable and legumes can induce oral allergy syndrome and sometimes severe allergic reactions in sensitized individuals. PR-10 allergens are not stable to heat and digestion.

Polcalcins (Ca++ binding proteins, PC)

Members of the PC family can cause inhalative symptoms. PC allergens can be found in pollen from grasses, trees and weeds. Inhalative symptoms manifest as allergic rhino-conjunctivitis and/or allergic asthma.

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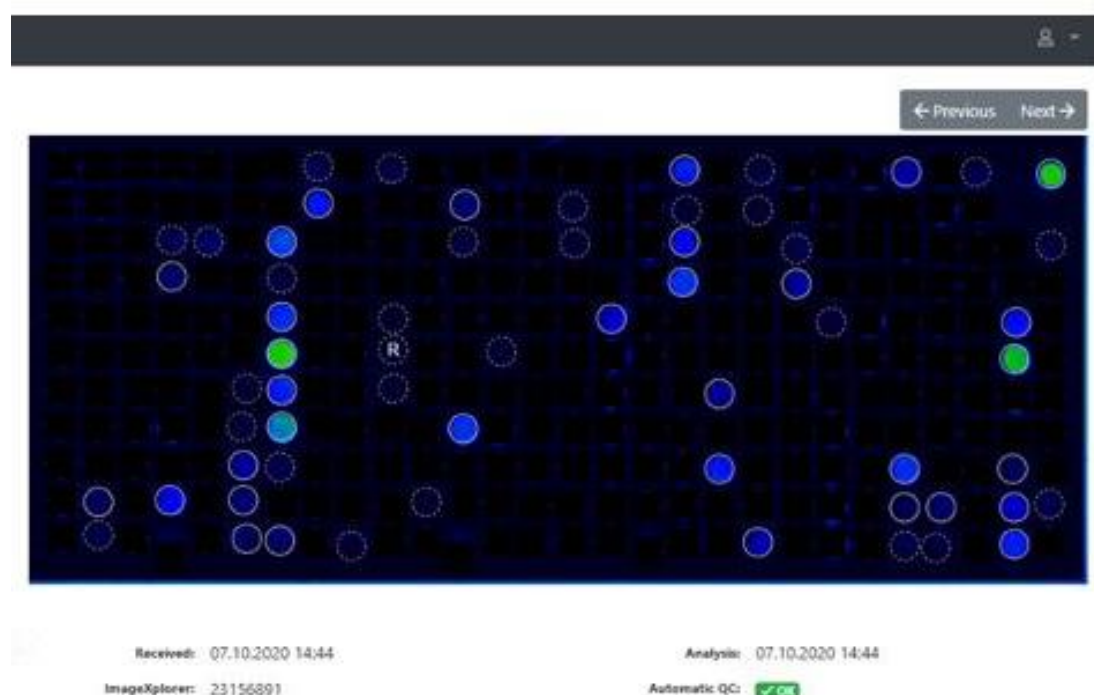
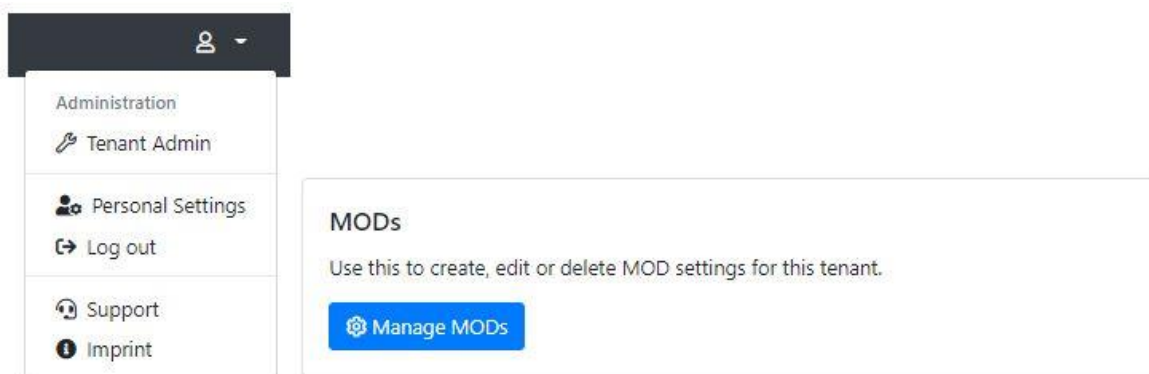
Type:

Deactivated:

You cannot delete an MOD once it is in use. But you can deactivate it, so that it's not available for new measurements anymore.

Allergens:

Enter one allergen
2"

[Save](#) [Cancel](#)

Nadalje, u konfiguraciji stanara (Tenant Admin → Configure Tenant) dostupne su tri MOD postavke.

General configuration

*** Name:**

Default timezone:

Configure a timezone to see date and times in (instead of UTC). Please note that users can choose to use a different timezone.

Default report template (ALEX 1):

Default report template (ALEX 2):

Default report template (FOX):

Default report template (PAX):

Default report language:

Default ALEX2 MOD:

Default FOX MOD:

Default PAX MOD:

Force MOD:

Requires a measurement to always have an MOD assigned. If this setting is enabled, a measurement cannot be approved without an MOD assigned.

Hide Global MOD:

If this setting is enabled, only the tenant-specific MOD will be available to select, and the global system-wide MOD will be hidden.

Display MOD Excluded Spots As Research Use Only:

If this setting is enabled, spots excluded by an MOD will not be hidden from the measurement image, but displayed as a "RUO spot".

Require ImageXplorer ConfigXplorer scan:

- Force MOD: Ako je ova postavka istinita, mora se odabrati MOD prilikom spremanja ili odobravanja mjerenja.

Doctor Info:

Choose a doctor...

Tags:

MOD:

MOD is required!

Report Template: ALEX2_v1.17 - Español (es)

- Hide Global MOD: Kada je ova postavka istinita, globalni MOD-ovi se ne prikazuju na padajućem popisu MOD-a prilikom uređivanja mjerenja.

MOD:

Report Template:

Force MOD:

Requires a measurement to always have an MOD assigned. If this setting is enabled, a measurement cannot be approved without an MOD assigned.

Hide Global MOD:

If this setting is enabled, only the tenant-specific MOD will be available to select, and the global system-wide MOD will be hidden.

Display MOD Excluded Spots As Research Use Only:

If this setting is enabled, spots excluded by an MOD will not be hidden from the measurement image, but displayed as a "RUO spot".

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Force MOD: False

Requires a measurement to always have an MOD assigned. If this setting is enabled, a measurement cannot be approved without an MOD assigned.

Hide Global MOD: False

If this setting is enabled, only the tenant-specific MOD will be available to select, and the global system-wide MOD will be hidden.

Display MOD Excluded Spots As Research Use Only: True

If this setting is enabled, spots excluded by an MOD will not be hidden from the measurement image, but displayed as a "RUO spot".

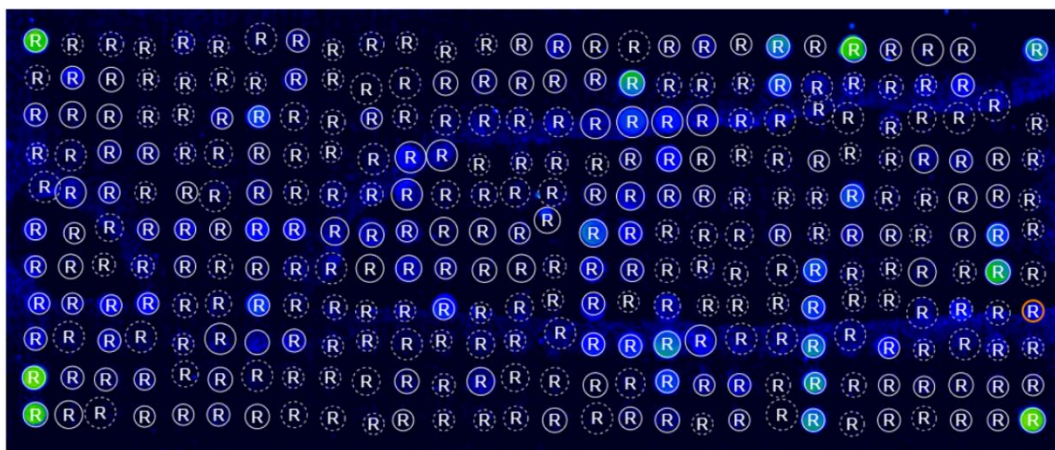
Configuration

Calibration string:

Allows to override the calibration information from the cartridge's QR-Code. Leave empty to use the calibration information from the QR-Code on the cartridge (the usual behaviour).

Research spots: Show research spots

If enabled, "research use only" spots will be displayed on measurement details and arrays. This does not affect the export behaviour or the reports of measurements.

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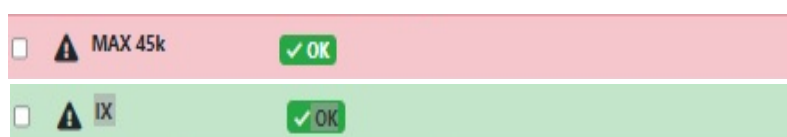
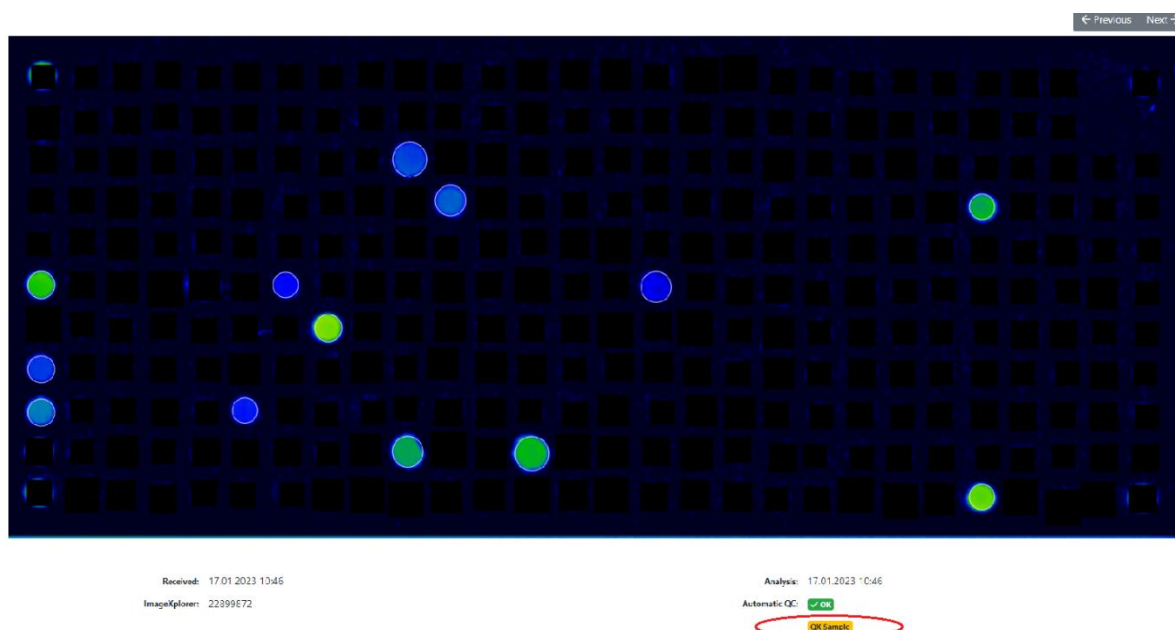
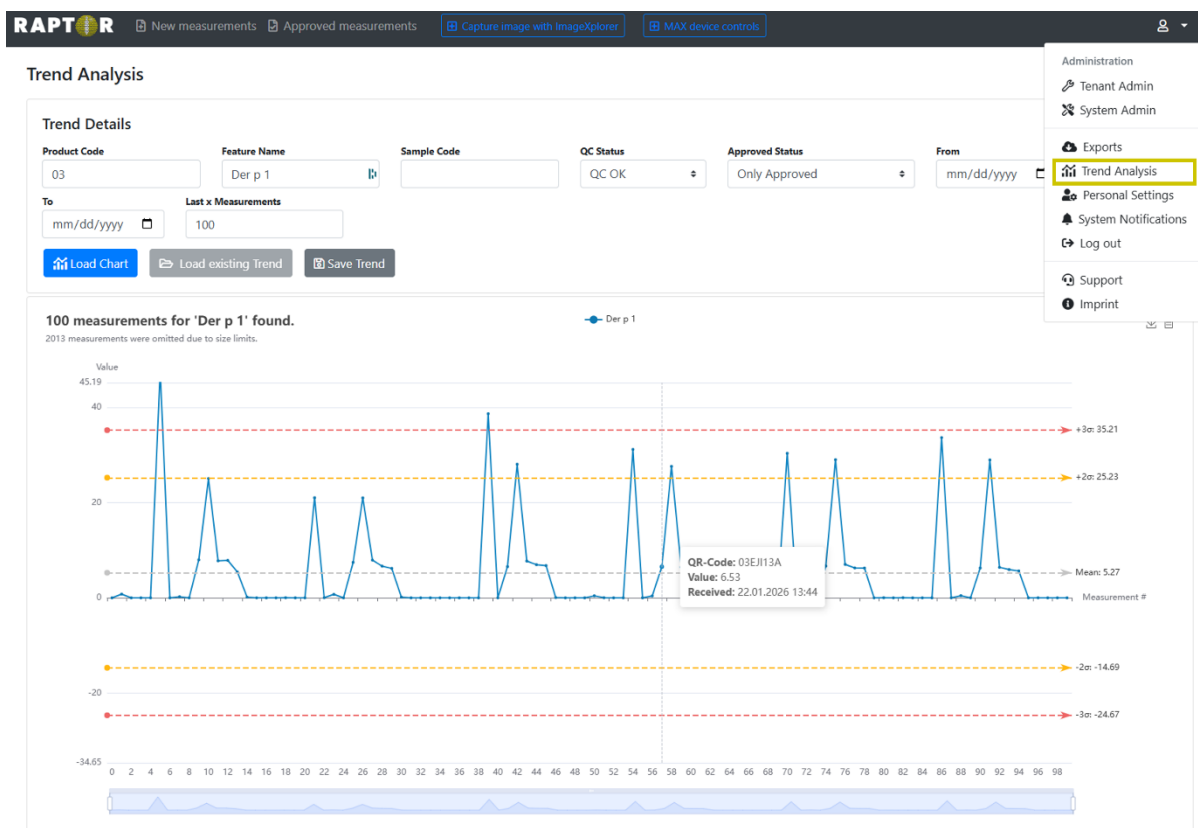
New Measurements

Search by barcode, patient, sample, tag, analysis date, a

Selected measurements ▾

- Download CSV format
- Download Excel format
- Approve
- Update interpretation
- Re-Process
- >> Move
- Delete**

Tags	Product type
	FOX
	FOX
	ALEX2
	FOX
	ALEX2
	ALEX1

Delete measurements

You are about to delete the following measurements. Please note that this action is irreversible and cannot be undone.

Barcode	Analysis Date
@2AAB16@	16.12.2019 13:41

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Plant Food Food	M	Jug r 3	0.40
Plant Food Food	M	Jug r 1	0.38

Plant Food Food	M	Jug r 3	0.40
Plant Food Food	M	Jug r 1	0.38

Delete request scheduled now.

Korisnici administratora sustava primit će ovaj zahtjev (posebna značajka u pravima administratora sustava: „Measurement delete/unapproval log & requests”) i mogu ga prihvatiti ili odbiti.

ign in. Can be

Background jobs
View details about running background jobs and manually schedule runs.
[View dashboard](#)

Measurement delete / unapproval log & requests
Show logging and pending requests for measurement delete and unapproval.
[Show log & requests](#)

Measurement unapproval / delete requests

Open requests

Log & Finished requests

Requested UTC	Requesting User (Email)	QR-Code	Request Type	Action
11.10.2023 13:05:28	Test Test (Test@test.com)	02AYQ145	Unapprove	Accept Reject
11.10.2023 12:59:27	Test Test (Test@test.com)	02AYQ14C	Delete	Accept Reject

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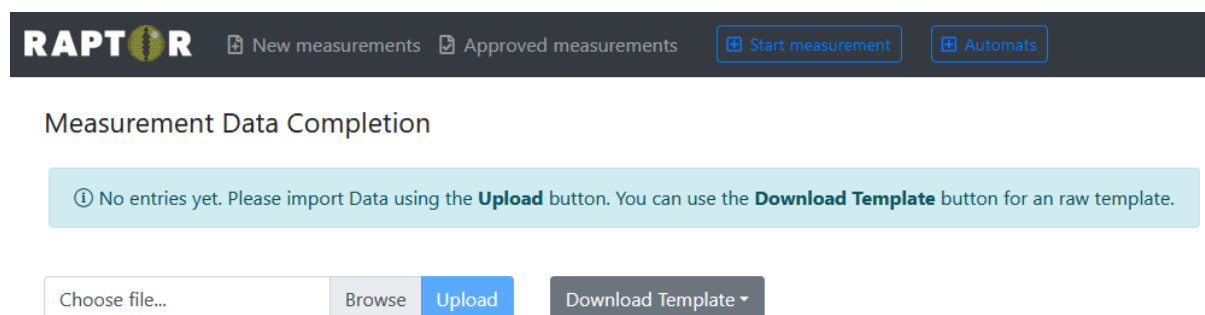
Add new user on Result Sharing Portal

General configuration

First name:

Last name:

E-Mail address:

Status:

Enabled (user can log in) ▾

Username:

Password:

Send an invitation e-mail to user automatically ▾

☒ Automatically send an email summary in case new measurement(s) have been shared

This email is sent once per day as a summary, just in case a new measurement was shared.

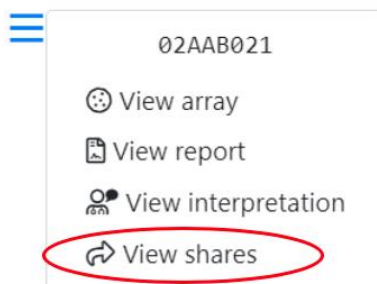
Doctor Info Text:

This information will be printed on a patients report.

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Share

You are about to share a measurement with a user of the User Portal. Please note that the doctor will have access to the results of the measurement. Once you share a measurement you'll be provided with a link to access the shared information via the encrypted User F

Barcode: 02AAB021

Share with: X Dr. (testuser2@macroarraydx.com)

If you can't find a specific user of the User Portal, go to the [tenant administration](#) to configure the list of available users of the User Portal.

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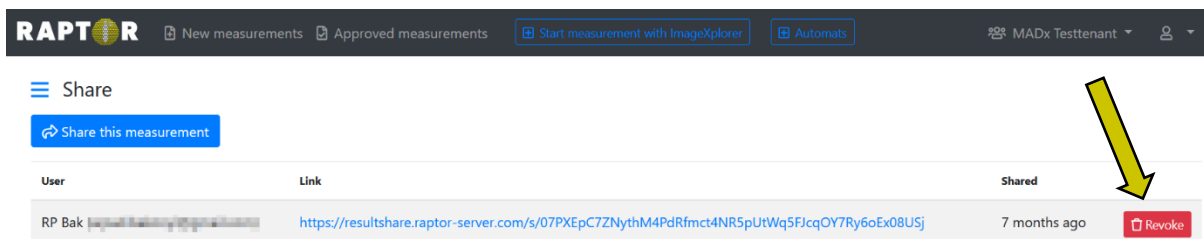
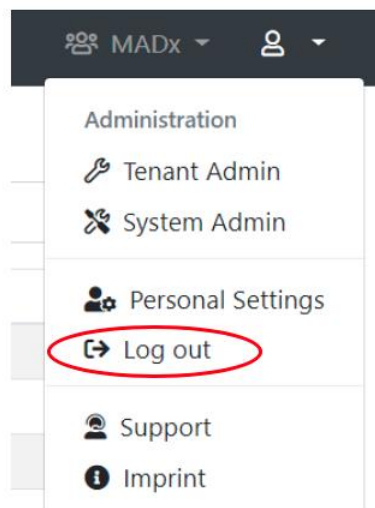
 Result Sharing Portal English (en) ▾

Welcome to the Result Sharing Portal

The Raptor-Server Result Sharing Portal allows users to view shared measurements. You will need a username and password to log in to view all measurements that have been shared with you. Also, a sharing link will allow to view specific shared measurements.

Please enter your username and your password:

Did you forget your password? No problem, you can [request a new one](#).

Jezik se može promijeniti d na vrhu ekrana. Klikom na ikonu oka unesenu lozinku možete učiniti vidljivom. Novu lozinku možete zatražiti klikom na „Request a new one“.

Please enter your username and your password:

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Result Sharing Portal
English (en)
Change password
Log out

Measurements

These are the measurements that are available for you. You may view the details of each measurement, or download the report PDF file.

Measurement	Product	Patient	Sample	Date
01AAK239	ALEX1		11852162	7/12/2019
01AAK281	ALEX1		04824821	12/18/2019
01AAK891	ALEX1		66560260	1/28/2020
02AAA015	ALEX2			2/21/2020

Popis zajedničkih mjerenja može se poredati prema QR-kodu, vrsti proizvoda, nazivu proizvoda, imenu pacijenta, ID- u uzorka ili datumu.

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Measurement 02AYQ143

Product: ALEX2
Patient:
Date of Birth:
Sample:
Info:
Doctor Info:
Sampling Date:
Assay Date: 10/12/2023
Processed Date: 10/12/2023
Approval Date: 10/12/2023

Report Language:

English (United Kingdom) ▼

You can request a new RAVEN interpretation that will be included in the report.

[Request new RAVEN interpretation](#)

You can download a PDF report that includes all information and results of this measurement.

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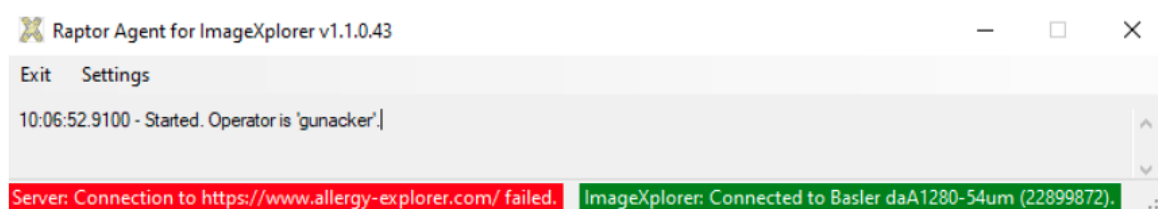
Settings

Please enter the settings for your ImageXplorer. You can find your ImageXplorer key when you log in to Raptor. Please contact support if you're not sure what the proper settings are for your ImageXplorer.

Raptor URL:

ImageXplorer Key:

Continue

Početak mjerenja nije moguć:

RAPTOR  [New measurements](#) [Approved measurements](#) [Start measurement with ImageXplorer](#)

The ImageXplorer IX training cannot be used, because it requires a ConfigXplorer scan.

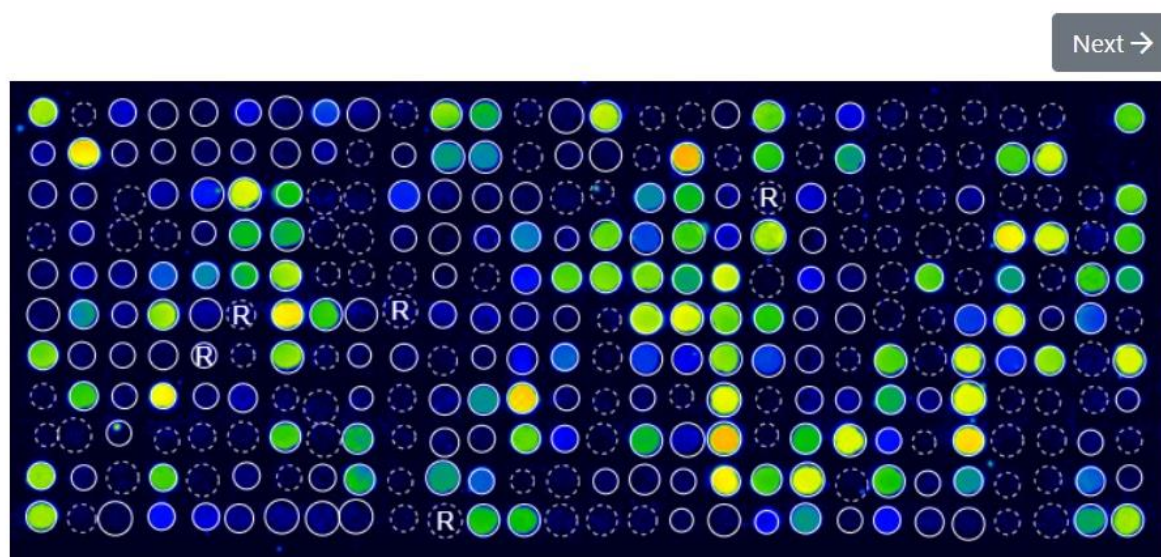
[Go to ImageXplorer configuration](#)

Moguća rješenja:

- Provjerite je li ImageXplorer povezan
- Provjerite je li izvršeno posljednje potrebno ConfigXplorer skeniranje

Neuspješni prijenos mjerenja na RAPTOR-SERVER:

Svako mjerenje sastoji se od 257 pojedinačnih slika (256 slika ROI-a i 1 slika QR koda). Ako je broj prenesenih slika manji (zbog nestabilnog prijenosa), poruka o pogrešci („Analysis Error: Insufficient data transfer – check connection” pojavljuje se uz sliku u RAPTOR-SERVER. U tom slučaju, mjerenje je potrebno ponoviti).

Received: 06.04.2023 13:37

Analysis: 06.04.2023 13:37

ImageXplorer: 40287249

Automatic QC: **✗ Failed**

Analysis Error: **Insufficient data transfer - check connection.**

Moguća rješenja:

- Molimo koristite originalni USB kabel koji ste dobili uz uređaj.
- Koristite drugi USB priključak na računalu na koje je povezan ImageXplorer.
- ImageXplorer mora biti spojen putem USB 3.0/3.1 priključka. Prijenosna ili prijenosna računala imaju takozvani „always-on“ USB priključak (obično priključak pored priključka za punjenje). Ako je moguće, koristite ovaj priključak.
- Ponovno povežite ImageXplorer s računalom (izvucite i ponovno uključite USB kabel).

Ako i dalje imate problema ili imate bilo kakvih drugih pitanja o proizvodima i uslugama MacroArray Diagnostics, obratite se svom lokalnom distributeru.

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Verzija	Opis	Zamjenjuje
9.0	Dodane su opcije rasporeda zaglavlja VII.2.1.2, opcije sadržaja VII.2.1.3, uklonjene su opcije jezika VII.2.2.4 budući da nisu obavezni dio IFU sadržaja, dodana je Analiza trendova VII.4, dodana je opcija za dnevni sažetak putem e-pošte VII.7, ažuriran je snimak zaslona.	8.0

RAPTOR SERVER Analysis Software u kombinaciji s test kitom ALEX², ALEX Air, ALEX Food i ALEX³:

RAPTOR SERVER Analysis Software u kombinaciji s drugim MADx test kitovima:

© Autorsko pravo MacroArray Diagnostics

MacroArray Diagnostics (MADx)

Lemböckgasse 59, Top 4

1230 Vienna, Austria

+43 (0)1 865 2573

www.madx.com

Broj verzije: 22-IFU-01-KR-09 (EN-26)

Datum izdavanja: 2026-02