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**LABOR SAMPLING SET (STOOL)
WITH ANALYSIS**

**INSTRUCTIONS
FOR USE**

Table of content

Intended use of the sampling set	13
A: (Warning) notes	14
B: Just as important	15
C: Components of the sampling kit	16
1. Carrying out the sampling.....	17
1.1 Take a stool sample.....	18
Take a stool sample (Images).....	19
2. Dispatch of the sample.....	20
3. Receive the results.....	21

Intended use of the sampling set:

This sampling kit, along with laboratory analysis, is used to collect a stool sample for the detection of occult blood (non-visible blood) using an immunological stool blood test (iFOBT test).

BEFORE YOU START:

- Please read and follow each step of the instructions before using the sampling kit to ensure that your stool sample can be analysed by the laboratory. The following instructions are based on the usage guidelines of the included components for the independent collection of a stool sample.
 - Allow approx. **10 to 15 minutes** for the sampling.

Links to the manufacturer's instructions

SENTIFIT® pierceTube

The manufacturer does not provide instructions for use for the examination glove. The instructions for the stool collection aid can be found on the product.

A: (Warning) notes

1. SENTiFIT® pierceTube

- For in-vitro diagnostic purposes
- Do not use components after the Expiry date.
- Keep the container out of the reach of children.
- Never insert the container or any part of it into the anus.
- Do not attempt to open the colourless, pierceable cap.
- Use the sampling device only once.
- Safety data sheets are available at www.sentinel diagnostics.com or contact your local representative.
- As haemoglobin in some samples may undergo rapid denaturation or degradation, samples should be stored at 2–8°C and analysed as soon as possible.

Further information on the product can be found in the manufacturer's instructions (QR code on the page before).

2. Examination glove

- Sterile, made of ethylene–vinyl acetate copolymer, sealed in punched paper.
- For external use.
- Contents sterile, as long as the pack is intact.

B: Just as important:

- Store the sampling set with the components at temperatures between 5° and 30 °C and do not use it after the expiry date.
- Please inform us (DoctorBox) or the manufacturers of the components if any serious incidents occur during sampling.
- The liquid contained in the container must not be swallowed and must not come into contact with the eyes or mucous membranes. The container is designed to prevent leakage. In the event of accidental contact, please consult a doctor immediately.
- Collect the stool sample immediately after excretion.
- Stool samples must not be taken during menstruation if there is bleeding from the anus (e.g. due to haemorrhoids or other reasons).
- If the stool is contaminated with urine, the sample is invalid.

C: What components does this sampling kit contain?

Extraction fluid

Green cap with collection stick for the stool sample

SENTiFIT® pierceTube

- A. 1 x SENTiFIT® pierceTube
- B. 1 x DoctorBox instruction manual
- C. 1 x Barcode card with test ID

In the cardboard

- D. 1 x Examination Glove
- E. 1 x Stool catcher
- F. 1 x Blue safety bag
- G. 1 x Mailing bag (prepaid)

Purpose of the components in the sampling set:

Stool sample tube: The SENTiFIT® pierceTube is used only for the collection and storage of human stool samples.

Examination glove: Disposable gloves made of copolymer, transparent, sterile, and powder-free, can be worn on the right and left. Applicable in the food sector, industry, and medical fields.

Stool collection aid: The stool collection aid is an accessory for sample containers (in-vitro diagnostics) and facilitates the hygienic collection of stool samples.

Carrying out the sampling

Before you start sampling:

- Ensure that you can ship the sample on the same day as collection. A fast shipment ensures that the sample is stable.
- If shipping on the same day is not possible, store the sample in a cool place (2–8°C) and send it the following day.
- Place all utensils on a clean surface. If any components are missing or damaged, do not use the set and contact us (DoctorBox).

You can reach us Monday to Friday from 09:00 to 17:00 at the following telephone number and e-mail address:

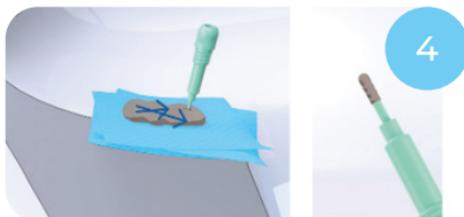
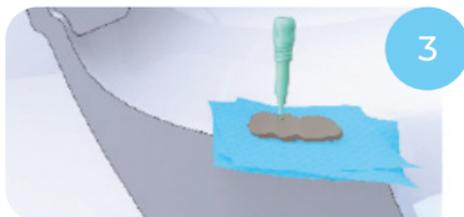
Tel.: +49 30 544 538 991

Mail: info@doctorbox.de

Take a stool sample

1. Use the enclosed stool collection aid to collect the sample.
2. Unscrew the green cap with the removal stick.
3. Do not prick too deeply into the stool sample at four different points with the green sampling stick.
4. Swipe the collection stick crosswise over the surface of the stool sample. Make sure that all indentations are filled with stool. This is exactly the right amount.
5. Screw the green cap firmly back onto the tube and carefully invert it several times. Now it must **NOT** be opened again.

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2

Dispatch of the sample

1. Place the tube in the protective bag with suction insert (blue).
2. Fold the protective bag with the tube and place it in the prepaid mailing bag.
3. Close the prepaid mailing bag and send the sample (e.g., via mailbox) on the same **day of collection**.

Receive the result

Depending on the option selected during registration, you will receive your result either by mail or digitally – depending on what your insurance provider offers:

Mail Delivery: If your insurance supports mail delivery and you selected this option during registration, your result will be sent to the address you provided.

Please note that not all insurance providers support mail delivery. If this is the case, you will only be able to access your result digitally.

Digital Access: As soon as your test result is available, you will receive an email with a link to access it.

This requires that you selected the digital option and provided a valid email address.

Important: To access your result digitally, you will need the password you set during registration.

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DoctorBox Service GmbH
MaiMed® GmbH