0

>80%

1

43% - 80%

2

18% - 43%

3

8% - 18%

4

3% - 8%

99082_V2 2025-11--13

A:

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EN MANUFACTURER'S DECLARATION OF CONFORMITY

All filter spectacles from Multilens AB are CE-marked and meet the requirements of regulation (EU) 2016/425, (EU) 2017/745 and the EN ISO 12312-1:2022 standard.

GENERAL

See the attached label for the name of the filter in your filter spectacles. Table A gives a description of the filter categories, their transmission values and recommended usage.

USAGE LIMITATIONS

Our filter spectacles provide 100% protection from natural UV radiation, but they are not suitable for looking directly at the sun, for protection from artificial light or for use in solariums.

Some filters cause colour distortions and should not be worn in traffic, day or night. This applies to all combinations of the ML511, ML527, ML550 and ML585 models, as well as ND23, ML Emerald Medium and ML Emerald Dark.

Categories 1 to 4 of filter spectacles are not suitable for driving in the dark or in poor lighting conditions.

Photochromatic filters with less than 75% light transmission in bright light must not be worn when driving in twilight or darkness.

Filter spectacles are not suitable eye protection against mechanical risks such as impacts.

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CARE

Clean the glasses with a soft lens cleaning cloth. Remove heavy dirt with water and a mild washing-up liquid. Do not use solvents such as alcohol or acetone.

STORAGE

Spectacles should preferably be stored in a case. Do not use scratched or damaged spectacles.

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✖ : Not suitable for driving

VEDLIKEHOLD

Clean the glasses with a soft lens cleaning cloth. Remove heavy dirt with water and a mild washing-up liquid. Do not use solvents such as alcohol or acetone.

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