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INSTRUCTIONS FOR USE, RIGID TRIAL CONTACT LENSES
These instructions for use apply to rigid trial contact lenses manufactured by Multilens AB.

Lens type	Brand name	Material
Corneal rigid gas permeable (RGP) contact lens	A Design	Boston ES
	Envision	
	Rose K2 KC	Optimum Extra
	Rose K2 NC	
	Rose K2 IC	
Scleral and corneo scleral (RGP) lens	Rose K2 PG	
	Rose K2 XL	Optimum Extra

INTENDED PURPOSE
Reusable rigid gas permeable trial lens for multi-patient use, intended for fitting of contact lenses.

PRODUCT DESCRIPTION

Information regarding trial lens design, parameters, and material is listed on the product label. It is delivered disinfected, dry, and individually packed in a lens case with a plastic screw cap. Do not use the product if the package is damaged.

INTENDED USER

The rigid trial lens is intended for use by eye care professionals during the fitting of rigid gas permeable contact lenses on the end user.

RECOMMENDED TIME OF USE

The trial lenses are recommended for use only during the fitting session (typically 10-60 minutes).

RECOMMENDED EXCHANGE FREQUENCY

The rigid trial lens shall be discarded after the expiry date on the label or if the lens is damaged and/or its physical appearance has changed. A lens used in an infected eye shall always be discarded.

DIRECTION FOR USE

- Prepare for use**
 - Ensure the work surface is clean and disinfected.
 - With clean and dry hands, select the appropriate disinfected trial contact lens.
 - Rub and rinse the lens with a multi-purpose solution (MPS) approved for rigid lenses.
 - Visually inspect the lens for damage or changes in physical appearance. Discard if any issues are identified.
- Use the lens**
 - Examine the surface of the eye to ensure it is healthy for contact lens fitting, and that no eye infection is present. In case of infection, contact lens shall not been used.
 - Insert the appropriate trial lens.
 - The lens may sometimes feel uncomfortable a few minutes after application. This may occur due to dust particles. In that case, remove the lens, rinse it, check that it is clean and reapply it.
 - Perform an optical assessment of the lens on the eye.
 - Remove the lens, rub and rinse it with an approved disinfection solution.
 - Document the use of the trial contact lens in a multi-patient trial contact lens management log.
- Inspect lens**
 - Visually inspect the trial contact lens for damage and changes in physical appearance.
 - If the lens is damaged and/or has changed in physical appearance, discard the lens.
 - Wash and dry hands after contact with patients and used lens.
- Disinfection trial lens and lens case**
 - Ensure the work surface is clean and disinfected.
 - Disinfect the trial contact lens and lens case using 3% ophthalmic hydrogen peroxide with a three (3) hour soak time.
 - After 3 hours neutralize, rinse, and make sure that lens and case contain no hydrogen peroxide.
 - An alternative disinfection method, approved for trial lenses, can be used.
- Store the lens**
 - Ensure that the lens and lens case are completely dry before storage, using a clean, lint-free paper tissue, not exceeding 25°C.
- Documentation**
 - Document the use of the trial contact lens on a multi-patient diagnostic lens management log (including patient's reference, date of use, date of hygienic management, contact lens details, person performing the disinfection).

WARNINGS AND PRECAUTIONS

- Always wash and dry your hands before handling contact lenses.
- Do not use contact lenses and lens care products that have passed their expiry date.

Serious adverse events when using this product shall be reported to Multilens AB (the manufacturer) as well as the competent authority.

CONTRAINDICATIONS

(When should trial lenses not be used)

- Contact lenses should not be used under certain medical and environmental conditions that can affect safe contact lens wear, such as:
- Allergy, inflammation, infection, or irritation in or around the eye or eyelid.
 - Diseases such as severe cold or flu.
 - When using certain pharmaceuticals including eye medication.

ADVERSE REACTION (problems that may occur)

When wearing contact lenses, some problems may occur that can be identified by one or more of the following symptoms:

- Foreign body sensation/ chafing/ poor comfort.
- Red eyes.
- Light sensitivity (photophobia).
- Heat sensitivity.
- Burning sensation.
- Increased tear flow or unusual eye secretion.
- Reduced visual acuity.
- Colored rings or light dispersal (halos) around light sources.
- Dry eyes caused by your contact lenses.
- Sudden changes in eye appearance e.g. swollen eyelids or conjunctiva.

If these symptoms are ignored, they can lead to serious complications/ infections that could result in mild, severe, or even permanent tissue damage, for example scar tissue.

IF YOU ARE EXPERIENCING PROBLEMS

If you are experiencing any of the problems listed above: REMOVE THE CONTACT LENS.

If the problem ceases when you remove the contact lens from the eye, check the contact lens thoroughly. If there are visible particles on the lens and it appears undamaged, clean the lens thoroughly and reapply it. If the lens is damaged in any way, do not reapply it. If the symptoms persist after removing the lens or when applying it again, remove the lens and inspect the eye for any reactions and/or damage. Certain contact lens complications can be serious and lead to permanent sight impairment if neglected.

DISPOSAL

- Dispose of as follows:
- Contact lens as plastic.
 - Lens case as plastic.

SYMBOLS & ABBREVIATIONS GLOSSARY

Symbol	Definition
	Manufacturer
	Date of manufacture
	Use-by date
	Batch code
	Catalogue number
	Do not use if package is damaged
	Maximum storage temperature
	Consult instructions for use
	Medical device
	CE mark (Conformité Européen)
BC	Base Curve
Dia	Diameter
Sph	Sphere

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