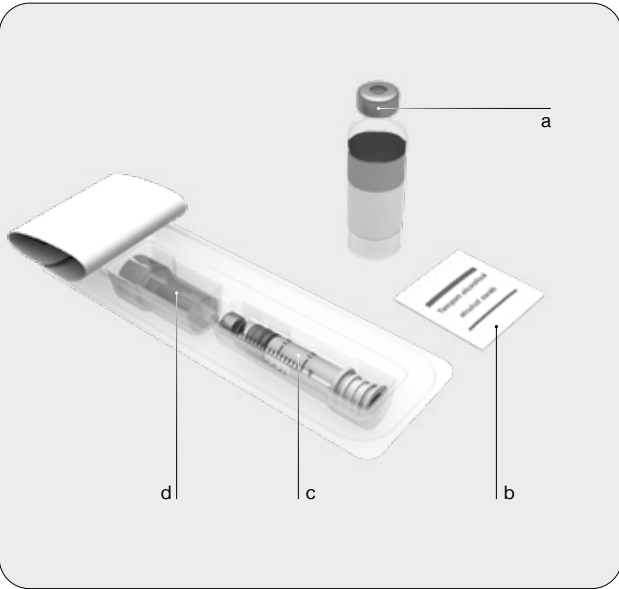
en

Sterile device

The myPump+ Reservoir is sterile provided that the packaging has neither been opened nor damaged. If the packaging shows signs of damage, then use a different reservoir.

Intended use

The myPump+ Reservoir is intended to be filled with a rapid-acting insulin at a concentration of 100 U/ml and used with the YpsoPump insulin pump.

Warnings

Prevent risk of incorrect medication:

- Ensure the use of the correct drug liquid and check the label of the vial before filling the reservoir.
- Remove air bubbles from the reservoir prior to its insertion into the YpsoPump.

Prevent risk of infection:

- Never pull the stopper beyond the 1.6-ml mark.
- Do not re-use the transfer adapter or the reservoir.
- A damaged septum may lead to leakage.
- Store the unopened reservoir in a dry place.

Safety measures

Check the reservoir for cracks and damages and do not use the reservoir after the expiration date on the packaging. Ensure sterility by checking if the sterile paper and tamper-proof seal are undamaged. Follow the user manuals for YpsoPump and myOrbit infusion sets and the explanations of the symbols on the packaging. The reservoir is solely intended for single use.

You will need the following

- a) Insulin vial
- b) Alcohol swabs
- c) Reservoir with plunger rod
- d) Transfer adapter

- 1 Wash your hands thoroughly with soap and water.
- 2 Clean the septum of the insulin vial with an alcohol swab.
- 3 Remove the reservoir and the transfer adapter from the packaging (c and d) without touching the septum.
- 4 Attach the transfer adapter to the reservoir and advance the stopper all the way forward. Do not penetrate the septum several times.
- 5 Then pull the plunger rod backwards until the entire reservoir is filled with air.
- 6 Attach the other end of the transfer adapter to the insulin vial.
- 7 Inject all of the air into the insulin vial. This will make it easier to then withdraw insulin from the vial.

- 8 Turn the reservoir, including the insulin vial, upside down. Very gently draw the insulin into the reservoir.
- 9 Check the reservoir for air bubbles. If air bubbles are in the reservoir, carefully tap the side of the reservoir. Then press the plunger rod to push the air bubbles out of the reservoir. Repeat if necessary.
- 10 Remove the transfer adapter from the vial. Dispose of the transfer adapter in accordance with local laws.
- 11 Twist off the plunger rod and dispose of it.
- 12 Follow the instructions in the user guide for the YpsoPump in order to insert the reservoir into the YpsoPump.

Storage

You can fill the reservoir with a rapid acting insulin analogue as described by your doctor/physician and store it in the fridge (2–8 °C) for up to 30 days. The filled reservoir can be used in the YpsoPump for up to 7 days. For more information regarding your insulin, please consult the package leaflet issued by the insulin manufacturer. The storage time and storage conditions may differ. Please consult the YpsoPump IFU for compatible insulins.

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mylife Diabetes Care AG