



ACRAGUN WITH ACRACLIPS

Scalp Clip System



Designed for Efficient Hemostasis

The ACRA-CUT clip gun system is designed to provide efficient and reliable scalp clip application. Engineered for consistent clip delivery and secure clip retention, it helps simplify scalp clip placement during surgical procedures.

Simple. Reliable. Economical.

Key Features

- 

Designed to apply clips efficiently across a wide range of operating angles, including inverted positions, while maintaining control during application.
- 

Combines the durability of a reusable gun with the convenience of sterile, preloaded clip cartridges.
- 

Cartridges are designed to securely snap and lock into the gun.
- 

ACRAGUN features a simple design with only four moving parts to support reliable operation.
- 


ACRACLIPS are available with serrated or straight jaws and a choice of three clamping forces.
- 

Integral cartridge lock is designed to help prevent unintended clip ejection.
- 

Ergonomic design promotes comfortable handling and control.

ACRACLIPS


Color-coded ACRACLIPS are available with two jaw configurations and three clamping force options to accommodate a range of surgical preferences and applications. Medium-force clips are designed for general scalp clip applications. All ACRACLIPS provide a consistent maximum clip opening to support efficient placement.

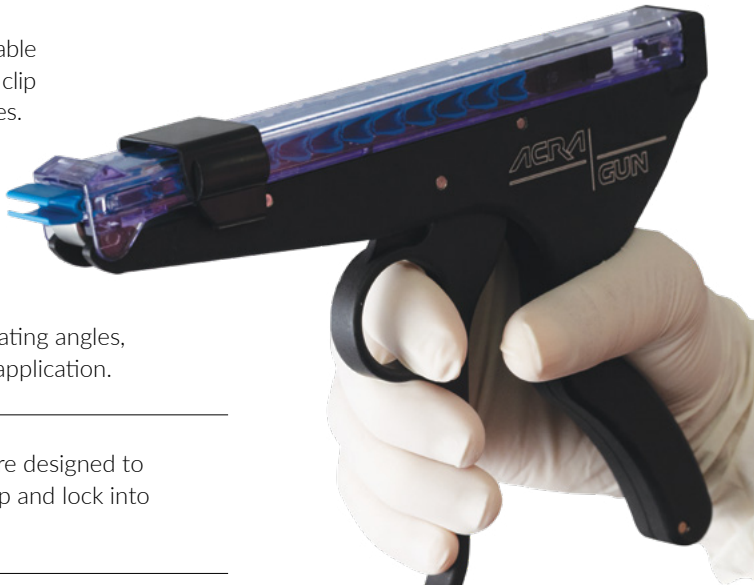


Clip Loading and Application

Easy to load

- 1

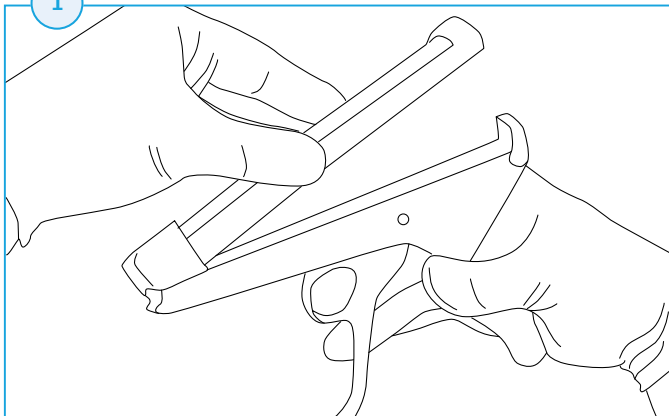


Insert the preloaded sterile cartridge into the ACRAGUN until it securely snaps into place.
- 2

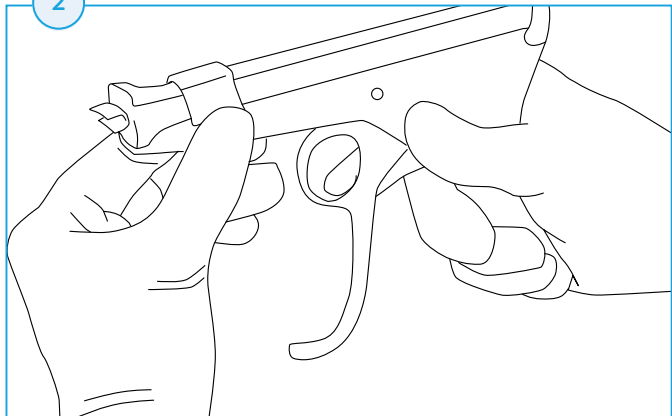


Lock the cartridge in place. The first clip automatically advances to the ready position.

Easy to use


- 1

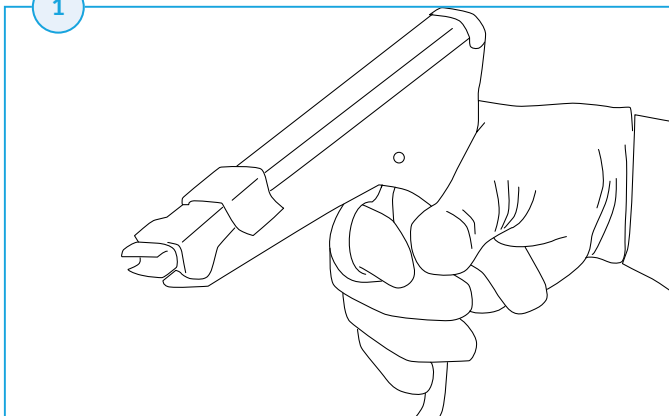


Squeeze the trigger to open the clip.
- 2

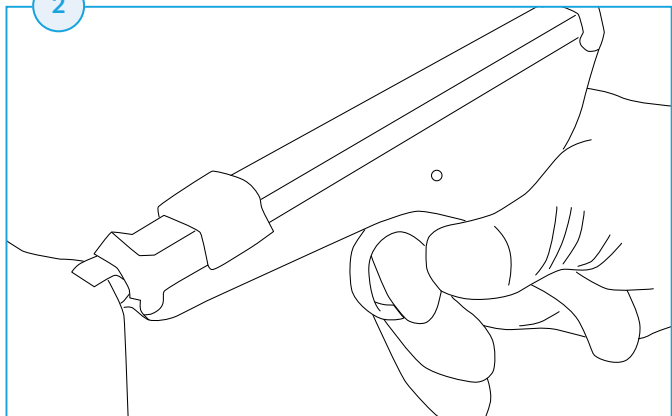

Apply the clip to the scalp.
- 3

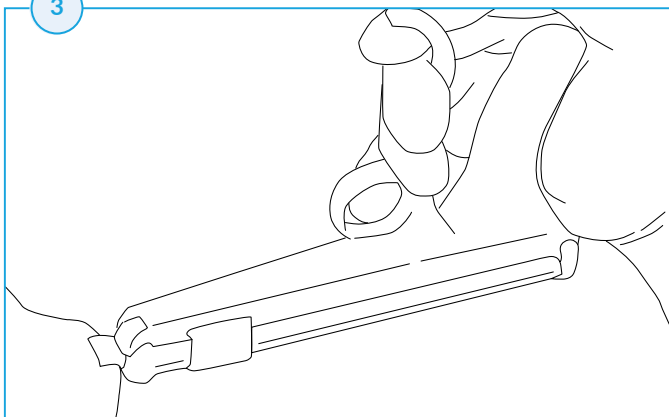

ACRAGUN can be used to apply clips from multiple angles, including inverted positions.
- 4

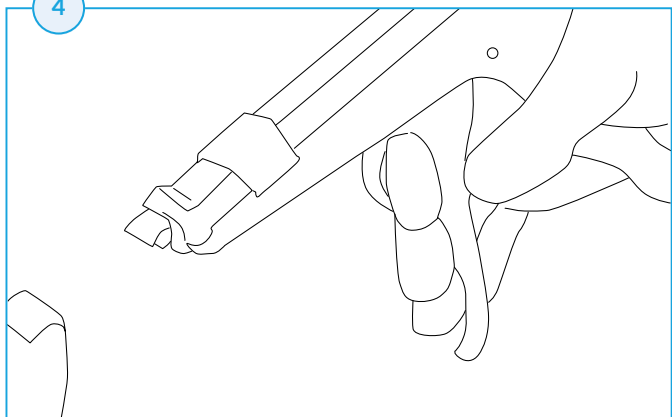

Release the trigger to close the clip. The next clip automatically advances into the ready position.

Product Overview

ACRAGUN

ACRAGUN comes with a protective storage case.

Catalog No.	Description
500-100	Reusable ACRAGUN with Storage Case
500-135	Reusable ACRA-CUT Clip Applier/Remover



ACRACLIP Cartridges



ACRACLIPS are supplied sterile in single or double-cartridge blister packs. Each cartridge contains 15 clips.

Two Cartridges Pack, 15 clips per cartridge

Catalog No.	No. of Cartridges	Jaw Description	Clamping Force	Color	Application
500-103	Two (2)	Straight Edge	Heavy	Green	Adult Scalp
500-104	Two (2)	Serrated Edge	Light	Yellow	Pediatric Scalp
500-105	Two (2)	Serrated Edge	Medium	Blue	Adult Scalp
500-106	Two (2)	Serrated Edge	Heavy	Green	Adult Scalp

Single Cartridge Pack, 15 clips per cartridge

Catalog No.	No. of Cartridges	Jaw Description	Clamping Force	Color	Application
500-108	One (1)	Straight Edge	Medium	Blue	Adult Scalp
500-109	One (1)	Straight Edge	Heavy	Green	Adult Scalp
500-110	One (1)	Serrated Edge	Light	Yellow	Pediatric Scalp
500-111	One (1)	Serrated Edge	Medium	Blue	Adult Scalp
500-112	One (1)	Serrated Edge	Heavy	Green	Adult Scalp

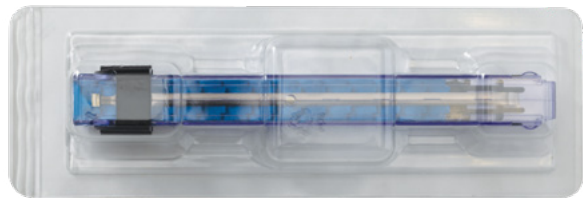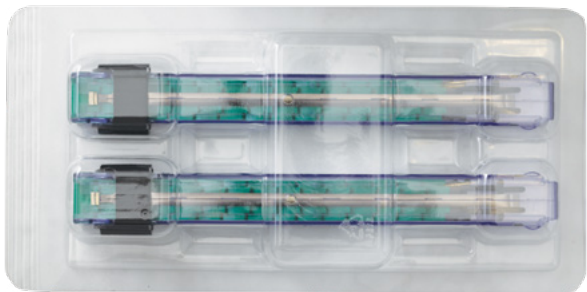

Specifications are subject to change without prior notice. The images, descriptions, quotations, dimensions, and technical data in this document should be considered for illustrative purposes only.