Patient Information

Long Term Catheters

Patient Information

de - Wichtige Patienteninformation	4
en - Important patient information	8
ar - معلومات هامة للمرضى	12
bg - Важна пациентска информация	16
cs - Důležité informace pro pacienty	20
da - Vigtig patientinformation	24
el - Σημαντικές πληροφορίες για τον ασθενή	28
es - Información importante para el paciente	32
et - Oluline teave patsientidele	36
fi - Tärkeitä tietoja potilaalle	40
fr - Importantes informations aux patients	44
hr - Važne informacije za bolesnika	48
hu - Fontos betegtájékoztató	52
id - Informasi penting untuk pasien	56
it - Informazioni importanti per il paziente	60
lt - Svarbi informacija pacientui	64
lv - Svarīga pacienta uzvedība	68
nl - Belangrijke informatie voor de patiënt	72
no - Viktig pasientinformasjon	76
pl - Ważna informacja dla pacjentów	80
pt - Importantes informações aos pacientes	84
ro - Informații importante pentru pacienți	88
ru - Важная информация для пациента	92
sk - Dôležité informácie pre pacientov	96
sl - Pomembne informacije za pacienta	100
sv - Viktig patientinformation	104

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Your doctor has ordered long-term access to your central blood circulation system for your medical treatment. This is performed with a vascular catheter (referred to as a catheter in the text below). This leaflet contains information about your catheter and the implant card (International Implant Card). This information is intended to supplement the instructions from your doctor or nurse. If you have any questions or concerns about your catheter, always discuss these with your doctor or nurse.

Implant card

Your implant card documents all the information needed to identify the catheter that is implanted in you. This includes the device name, the catalogue number (REF) and the lot number (LOT), the Unique Device Identification (UDI) for identifying the device and the name, address and website of the manufacturer (Joline). On the website you will always find the currently valid version of this patient information leaflet.

The above information is next to the following symbols on the implant card:

	Patient name or ID		Batch code
	Name and address of the implanting health facility		Unique device identification
	Date of implantation		MR compatible
	Website with patient information		Medical device
	Catalogue number		Name and address of the manufacturer

Make sure that the implant card has been fully completed by the medical facility and handed over to you.

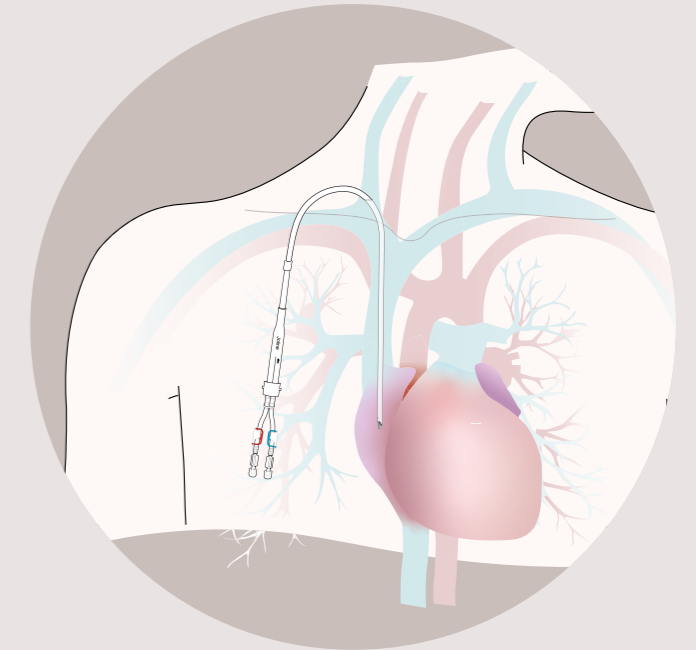
Keep the implant card in a safe place. Always carry it with you.

It is recommended to laminate it after receiving it to protect it from wear and tear and to make it easier to read.

When receiving treatment at medical facilities, always show your implant card to staff to inform them about your implant during medical examinations or treatment. This applies, for example, to magnetic resonance imaging (MRI). The catheter has been classified as suitable for magnetic resonance, i.e. safe for MRI examinations. This is marked with a corresponding MR symbol on your implant card.

General information about the catheter

The catheter is essentially a long plastic tube that is impermeable to X-rays (for configuration, see below). Its tip is located in the right atrium of your heart. Implanting the catheter provides long-term access to your central blood circulatory system. Within a few weeks, the catheter grows into the tissue under the skin via a small plastic felt. The catheter is attached to the skin with sutures until the felt has grown in completely and securely.

During dialysis the ends of the catheter that are located outside the body are connected to the tubes of the dialysis machine.

This allows the blood to be pumped out of your body, cleaned in the dialysis machine and pumped back again.

After each dialysis session the catheter is flushed with saline solution. Then it is filled with a special fluid, the "lock solution", to prevent blood clots clogging the tube. Nevertheless, blood may still run back into the catheter and clot there. Such blocked catheters usually have to be surgically removed and replaced.

Safety clamps and small caps at the ends of the catheter prevent blood loss between the dialyses.

The exit site of the catheter and the parts of the catheter outside the body are covered with protective dressing while the catheter is not being used for dialysis. The main purpose of this protective dressing is to prevent infections by protecting the exit site from contamination and pathogens.

The catheter is intended for long-term implantation of more than 30 days. Permanent implantation is not intended. The indwelling period of the catheter is determined by your treating doctor or the dialysis centre. If necessary, it will be replaced by a shunt (also called a fistula) or a port system according to the treating doctor's instructions. The catheter can also be surgically removed after a kidney transplant.

The table below shows information on the type and quantity of the most important materials that you as a patient may come into contact with when a **silicone catheter** is implanted:

Catheter component	Contact	Material	%
Luer-lock connector	Blood	PEEK	100
Connection tube*	Blood	Silicone	100
Catheter bifurcation*	Blood, skin	Silicone Barium sulphate	87 13
Fixation wing	Skin	Silicone	100
Felt (cuff)	Subcutaneous tissue	PET	100
Catheter shaft	Central blood circulation system	Silicone Barium sulphate	80-87 13-20

* only with double lumen variants

The table below shows information on the type and quantity of the most important materials that you as a patient may come into contact with when a **polyurethane catheter** is implanted:

Catheter component	Contact	Material	%
Luer-lock connector	Blood	PEEK	100
Connection tube	Blood	Silicone (Evolution Flow)	100
		PU (Smooth Flow)	100
Catheter bifurcation	Blood, skin	PU Barium sulphate	80 20
Felt (cuff)	Subcutaneous tissue	PET	100
Catheter shaft	Central blood circulation system	PU Barium sulphate	80 20

Behaviour in the event of complications

Please always follow the doctor's orders.

You must be aware of the following signs and symptoms that may indicate a possible problem. In the cases below, please contact your dialysis centre or doctor immediately:

- Bleeding, swelling, pain, redness of the skin or a local feeling of heat at the surgical site or at the exit site of the catheter
- Chest pain or other heart complaints after surgery
- If you have a fever (over 38 °C) or chills, inform the treating doctor that you have a catheter
- Swelling of the arms or legs
- A feeling of cold or numbness in the affected leg if the catheter was inserted from the groin
- Blood or other fluids leaking from your dressing
- Detachment or soaking of the protective dressing

Warnings and precautions

All catheters must be carefully cared for to ensure that they remain functional. Therefore, please observe the following warnings and precautions for everyday use of your catheter:

- Do not manipulate the catheter. For example, you must not pull on it or hold it under tension. **Opening the clamps and unscrewing the closure caps is strictly forbidden to avoid blood loss! Otherwise there is a risk to life!**
- **Never use scissors or other sharp and pointed objects (e.g. needles or razor blades) near the catheter! If the catheter is punctured or cut, there is a risk to life!**
- **The protective dressing may only be removed and renewed by trained specialists!** The exit site is carefully cleaned and sterilely re-banded by trained specialists at each dialysis.
- Showering is recommended instead of bathing or swimming to avoid contamination. Before showering, cover the protective dressing with an additional waterproof dressing. Remove the waterproof dressing after the shower.
- Using a sauna is not recommended to avoid contamination. The extremely high temperature and high humidity in the sauna can cause the protective dressing over the catheter to detach.
- For dialysis treatment, the catheter must be easily accessible and visible in order to keep the situation under control at all times and to respond quickly if complications arise. Please keep this in mind when selecting your clothing for dialysis treatment. We recommend blouses, shirts or cardigans.

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Joline GmbH & Co. KG