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Safety and warranty manual DRY® Cooling Vest Dog.
Product: (Medical) Recovery wear for dogs.
Read all safety and product information carefully before using the DRY® Cooling Vest on your pet. Failure to follow these instructions may cause injury. Suitital® products are not toys or fashion items. Adult supervision is required during the use of this product. Improper use may lead to hotspots, chafing, skin irritation, inflammation, or infection. In case of emergency or doubt, consult your veterinarian immediately.
Important safety instructions: 1. Usage. The DRY® Cooling Vest should only be used on the advice/consultation of your veterinarian and only after a medical procedure, for medical conditions, and skin problems. In case of emergency or doubt, consult your veterinarian immediately! Do not use the vest if it is damaged or if it contains loose parts. 2. Supervision. Your pet must always be under the supervision of a responsible adult while wearing the DRY® Cooling Vest. Regular checks for cleanliness, friction, constriction, chafing, tripping hazard, or choking hazard are mandatory. 3. Adjustment period. Get your pet used to the vest by wearing it for a shorter period the first few times. Start with one hour, two hours and slowly build up to a full day. If in doubt, always consult your veterinarian for advice. If your pet tends to chew of gnaw on materials, this product is not recommended. Therefore, always keep your pet under supervision. 4. Unusual behavior. In case of unusual behavior from your pet, which may include but is not limited to walking backward, sneezing, coughing, or incontinence, remove the DRY® Cooling Vest immediately and consult your veterinarian. 5. Visible issues. Check for proper fit and any skin irritation daily. Report skin irritation or any other problems to your veterinarian immediately for proper treatment and care. 6. Fit. Ensure the right fit to avoid safety risks. A vest that is too large can cause tripping or choking hazards; a vest that is too small can cause friction or skin irritation. Always measure your pet as described in the manual, which is available through the link at the bottom of the page. The vest is designed to have a snug fit. It is made of lightweight material and does not retain litres of water, making it very comfortable to wear. 7. Wear time. The Vest cools for up to 3 days. We do advise not to wear the vest for 3 consecutive days. Take the vest off when it cools down in the evening or when the vest no longer cools significantly. See the washing instructions, available in this link.
Disclaimer: While every effort has been made to ensure that the information in this manual is accurate and complete, Suitital is not liable for errors in or incorrect use of the manual and the product. We reserve the right to modify the manual. Warranty: Our products are guaranteed against manufacturing defects. In case of a product defect, please contact your veterinarian or the place of purchase.

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