

## Cooling comfort: the summer essential for dogs!

The DRY® Cooling Vest offers safe and effective temperature control for heat-sensitive and brachycephalic patients during recovery, hospitalisation, or transport. Designed for veterinary use, the vest is comfortable for the patient, easy to (re)fill, and activates simply by adding water. The cooling effect begins within minutes and can last for up to 3 days. Thanks to the patented DRY® Cooling technology, the vest remains dry and lightweight throughout use. Cooling is achieved through evaporation and is influenced by humidity and airflow around the vest.


- ✓ Safe cooling system for dogs.
- ✓ Ready to use in just 1 minute.
- ✓ Not limited to a single location.
- ✓ 100% Dry and lightweight.
- ✓ Refillable up to 800 times.
- ✓ Stays cool up to 3 days.



Sizing guide DRY® Cooling Vest

Size	Length (A)	Chest (B)	Neck (C)	Water
XS	25 - 35 cm 10 - 14 in	40 - 60 cm 16 - 24 in	< 35 cm < 14 in	100 ml 3.40 oz.
S	35 - 50 cm 14 - 20 in	55 - 70 cm 22 - 28 in	< 45 cm < 18 in	200 ml 6.75 oz.
M	50 - 60 cm 20 - 24 in	60 - 80 cm 24 - 32 in	< 50 cm < 20 in	290 ml 10.20 oz.
L	60 - 75 cm 24 - 30 in	65 - 85 cm 26 - 34 in	< 55 cm < 22 in	430 ml 14.50 oz.
XL	70 - 90 cm 28 - 36 in	75 - 95 cm 30 - 38 in	< 65 cm < 26 in	525 ml 17.70 oz.

**Size Chart**  
Measure the **length (A)** from the neck (collar) to the base of the tail, the chest **circumference (B)** and the neck **circumference (C)**. Add the amount of water as indicated in the chart. If a dog falls between two sizes, we recommend choosing the larger size.

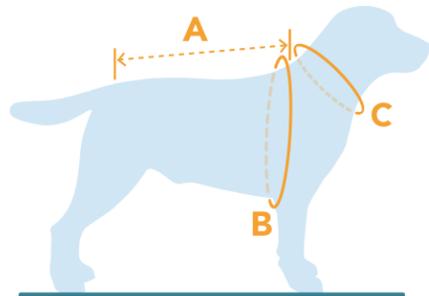

### How to put the DRY® Cooling Vest on the dog?

Put on	Secure the straps	Safety check	Putting on a collar/ harness
Put the dog's head through the opening of the DRY® Cooling Vest.	Take the black strap(s) on the belly side and secure them along the torso by fastening the buckles. Adjust the straps by gently pulling until the vest fits snugly but comfortably.	It is important to check that the straps are not too tight around the body and neck, as this could lead to breathing and circulation issues.	The dog's collar or harness can be worn over the DRY® Cooling Vest. Always adjust the harness to ensure it fits properly over the DRY® Cooling Vest.

## Features of the DRY® Cooling Vest.

- 1. Large filling cap**  
Easily fill the vest with a small amount of water to activate the DRY® cooling technology. No gels or chemicals required, and it is not recommended to further cool the vest in the refrigerator or with ice-cold water.
- 2. Optimal cooling of the torso**  
Cooling is most effective on the chest and abdomen because the fur is usually shortest in these areas. The DRY Cooling Vest® provides safe cooling on the back, neck, chest, and abdomen.
- 3. Easy to use**  
The elastic straps are adjustable, making the cooling vest suitable for almost every breed. The buckles allow the vest to be put on and taken off quickly, making it ideal for brief use before a walk. There are guiding loops on the sides to keep the vest securely in place.
- 4. Comfortable fit**  
The stretch material ensures optimal comfort, providing complete freedom of movement while wearing the vest—wherever you are.
- 5. Adjustable at the neck**  
To prevent breathing and circulation issues, special attention has been given to the design around the neck. The elastic cords allow you to comfortably adjust the vest to fit your dog's specific shape.

### How to activate the DRY® Cooling Vest?

Fill with water	Distribute	Squeeze out	Important!
Open the cap. Carefully pour the indicated amount of water into the vest according to the size chart on the previous page. Close the fill cap.  Do not overfill the vest with water!	Gently distribute the water throughout the vest by sliding and pressing your hands over the vest. Ensure the water is evenly spread on all sides.	Next, fold the test pad/cooling vest, open the fill cap, and gently squeeze out any excess water.  Close the filling cap, and the cooling process will begin within a few minutes.	Wash the cooling vest at a maximum of 40°C and hang it out to dry. Close the fill cap before washing.