

## Optimize recovery for head and ear issues with dogs in your clinic.

Designed with comfort and protection in mind, the MPS-Head Cover is the ideal solution for dogs that need protection on and around the head and/or ears. It can be used during infections, aural hematoma's, skin issues, after medical procedures and with wounds. With the cover, vets and owners can check the sensitive area easier and more regularly, which limits the need to replace and waste bandages and limits stress for the dogs. The MPS-Head Cover is available in 5 sizes and have adjustable fastening straps for the skull length, neck and head, so there's a perfect fit for every breed.


### Why the MPS-Protective Head Cover sets the standard in pet recovery

- ✓ Prevents flapping of the ears.
- ✓ Easily adjustable for a perfect fit.
- ✓ Patented, breathable design.
- ✓ Space for extra protective pads.
- ✓ Easy and fast check-ups.
- ✓ Limits stress.



### Sizing guide of the MPS-Head Cover

Size	Head (A)	Neck (B)	Length (C)	Pads
MPS-XS	30 - 36 cm 11.8 - 14.1 in	32 - 46 cm 12.6 - 18.1 in	17 - 24 cm 6.7 - 9.4 in	SMALL
MPS-S	32 - 40 cm 12.6 - 15.7 in	38 - 50 cm 15.0 - 19.7 in	19 - 28 cm 7.5 - 11.0 in	SMALL
MPS-M	38 - 46 cm 14.9 - 18.1 in	38 - 54 cm 15.0 - 19.7 in	21 - 32 cm 8.3 - 12.5 in	SMALL
MPS-L	38 - 50 cm 14.9 - 19.7 in	42 - 62 cm 16.5 - 24.4 in	21 - 33 cm 8.3 - 12.9 in	MEDIUM
MPS-XL	42 - 54 cm 16.5 - 21.3 in	46 - 66 cm 18.1 - 25.9 in	21 - 34 cm 8.3 - 13.3 in	MEDIUM

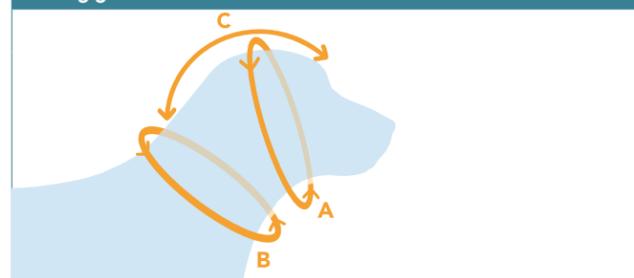

**Sizing guide**  
The **head circumference (A)** is measured around the widest part of the head. The **neck circumference (B)** is measured around the collar. The **length (C)** is measured from the top of the head to the collar.

### How to put the Protective Head Cover on the dog


Optional: Add the pad(s)	Cover the head and ears	Secure at the neck	Optional: Neck length
For extra protection you can place the pads in the MPS-Head Cover. Place the Velcro® of the pad directly on the Velcro® on the inside of the head cover.	Place the head cover on the head of the dog and start with closing the band under the chin using the press studs and velcro. Be sure to use at least two rows of press studs for a secure fit.	Close the band around the neck using velcro and press studs. Be sure to use at least two rows of press studs for a secure fit.	You can also adjust the neck length for a better fit by adjusting the Velcro at the back of the neck. Pull the velcro toward the back of the neck and attach it to the velcro on the inside of the neck band. Make sure to fasten the head strap first, followed by the neck strap.

## Features of the MPS-Protective Head Cover Dog.

- 1. Extra space for pads**  
The Head Cover comes with two specially designed inner pockets. These pockets hold the Ear Pads (sold separately) for extra soft protection. The improved pads are breathable and contain Kooltex fabric.
- 2. Flexible supporting ribs**  
These flexible plastic ribs prevent the Head Cover from hanging loosely and help support the protection of wounds and issues.
- 3. Elastic cord**  
The head cover includes an elastic cord around the face of the dog which helps keep the head cover in place as much as possible.
- 4. Adjustable fastening straps\***  
The head cover can be fully closed with adjustable fastening straps, press studs and velcro. The press studs help ensure a secure fit.
- 5. Velcro® strip**  
There is a velcro strip on the backside of the neck band. This way you can pull back the head part and shorten the skull length for an even better fitting.

### Improved KoolTex® technology

The improved and breathable KoolTex® material in the lightblue inner layer and pads create a better air circulation, perfect for ear infections and healing wounds.

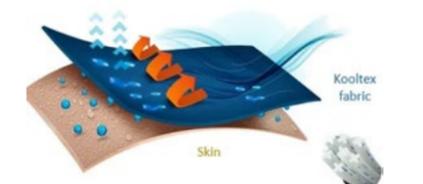

### MPS-Head Cover washing instructions

Remove the ribs from the Head Cover before washing.

MPS-HC - pads

Washing, maintaining and handling of delivered items always remains the responsibility of the customer.

The MPS-Head Cover Dog is available in 1 color:

MPS-Denim Blue