

Optimize recovery for dogs in your practice.

The MPS-Top Shirt is an ideal solution for wound protection after surgery whilst allowing freedom of movement and providing comfort. This versatile Top Shirt can be used for dogs after a medical procedure, when having skin conditions, to reduce stress, to hold a portable infusion system or Holter ECG device holder. The Top Shirt has an adjustable closure around the chest which makes the fitting effortless.

Why the MPS-Protective Top Shirt sets the standard in pet recovery

- ✓ Comes with a special device holder.
- ✓ Protects after medical procedures, wounds and stitches.
- ✓ Breathable fabric to support fast healing.
- ✓ Faster, safer and more comfortable recovery compared to a cone.
- ✓ Can help reduce stress and anxiety.
- ✓ Allows fixation of the MPS-Hind Leg Sleeves (sold separately).



Sizing guide of the MPS-Top Shirt Dog		
	Size	Chest circumference (A)
	MPS-2XS	28 - 43 cm 11.0 - 17.0 in
	MPS-XS	38 - 52 cm 15.0 - 22.0 in
	MPS-S	48 - 68 cm 19.0 - 27.0 in
	MPS-M	60 - 80 cm 24.0 - 31.0 in
	MPS-L	72 - 92 cm 28.0 - 36.0 in
	MPS-XL	78 - 100 cm 31.0 - 39.0 in
Always measure the dog for the right fit.		

Size guide

The size of the TOP shirt is best measured around the widest part of the chest (A), usually right behind the front legs.

How to put the Protective Top Shirt on the dog

Insert the head	Close the shirt	Attach the sleeves
Put the MPS-Protective Top Shirt on by inserting the head first followed by each front leg separately into the openings provided.	Close the MPS-Protective Top Shirt around the chest with the adjustable Velcro®.	If you want a more calming effect slightly increase the pressure by adjusting the Velcro®. Take a good look around the legs and neck to see if there is enough room to move and breathe!

Features of the MPS-Protective Top Shirt Dog.

- 1. Space between the shoulders**
 This space is intended for the insertion of a portable infusion system or Holter ECG, for example.
- 2. Device holder**
 The shirt has a sealable pouch on the back between the shoulders for holding a portable infusion system or Holter ECG. The dimensions of the sealable pouch are 12 x 9 x 3 cm.
- 3. Red loops for good fixation**
 The MPS-Protective Top Shirt comes with red loops on the inside for securing the MPS-Protective Hind Leg Double Sleeve if necessary.
- 4. Adjustable with Velcro® strap**
 The MPS-Protective Top Shirt fully covers the dog's chest and chest area. The band with Velcro® strap makes the shirt adjustable.
- 5. Elastic binder**
 The elastic binder near the abdomen allows you to increase pressure easily to create a calming effect, to apply a pressure bandage, or to keep a portable infusion system/ Holter ECG in place.

Washing instructions



- Always wash inside out
- Fasten all press studs and Velcro® before washing
- Use a laundry bag

The washing, maintenance and handling of delivered items always remains the responsibility of the customer.

TIP! To ensure the dog remains protected, Two Sleeves are often purchased to ensure proper hygiene and protection in between washings. Especially when a sleeve is needed for more than one week, a back-up is recommended.

The MPS-Protective Hind Leg Double Sleeve Dog is available in 2 colors/prints:

MPS-Blue

MPS-Zebra

