


## OPTIMIZE RECOVERY FOR CATS IN YOUR PRACTICE


The Protective Pet Shirt offers an innovative and animal-friendly alternative to the traditional E-collar and bandages. Specifically designed for medical applications, protecting wounds, and covering skin issues, the shirt is tailored to the anatomy of cats, ensuring a perfect fit and comfort. Made from breathable cotton and 4-way-stretch Invista Lycra®, the shirt provides full freedom of movement. The convenient press studs on the back allow for easy and quick access to sensitive areas.

### Why the MPS-Protective Pet Shirt sets the standard in pet recovery

- ✓ Covers wounds, hotspots, stitches, skin issues and ointments.
- ✓ Protects after spaying and other medical procedures.
- ✓ Breathable material supports faster healing.
- ✓ A quick, safe and comfortable alternative to the cone.
- ✓ Provides warmth post-surgery and helps reduce stress.
- ✓ Can also protect IV-lines.

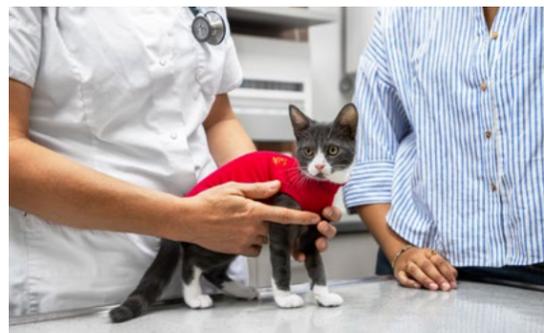

### Sizing guide for the Protective Pet Shirt Cat

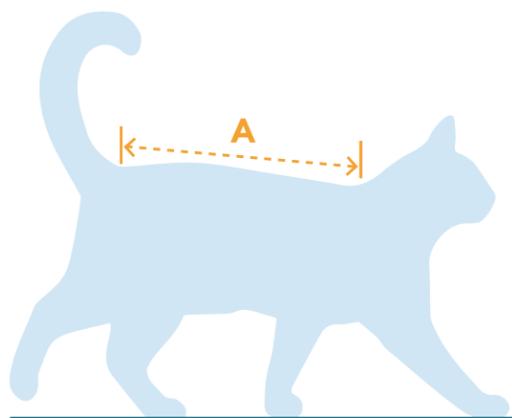


Size	Length (A)	
MPS-7XS	14 - 18 cm	5.5 - 7.1 inch
MPS-6XS	18 - 24 cm	7.1 - 9.4 inch
MPS-5XS	22 - 28 cm	8.7 - 11.0 inch
MPS-4XS	25 - 32 cm	9.9 - 12.6 inch
MPS-3XS	29 - 37 cm	11.4 - 14.6 inch
MPS-2XS	33 - 42 cm	13.0 - 16.5 inch
MPS-XS	40 - 45 cm	15.7 - 17.7 inch
MPS-S	43 - 51 cm	16.9 - 20.1 inch


**Sizing guide**  
Measure the **length (A)** from the neck (collar) to the base of the tail. If a cat falls between two sizes we recommend choosing the smaller size.

### How to put the Protective Pet Shirt on the cat?

Start with the head	Pull the legs through	Cover the body	Secure it
Roll up the shirt with the middle head opening in your hand. Make sure that the light blue inner layer is facing down. Insert your cat's head through the opening.	Followed by inserting the legs into the provided smaller leg openings.	Now pull the MPS-Protective Pet Shirt over your cat's body all the way to the back.	The lower belly flaps must be pulled between the hind legs towards the upper part near the base of the tail. Secure the two flaps with the press studs. We advise to close all press studs to keep the shirt perfectly in place.

## FEATURES OF THE MPS-PROTECTIVE PET SHIRT

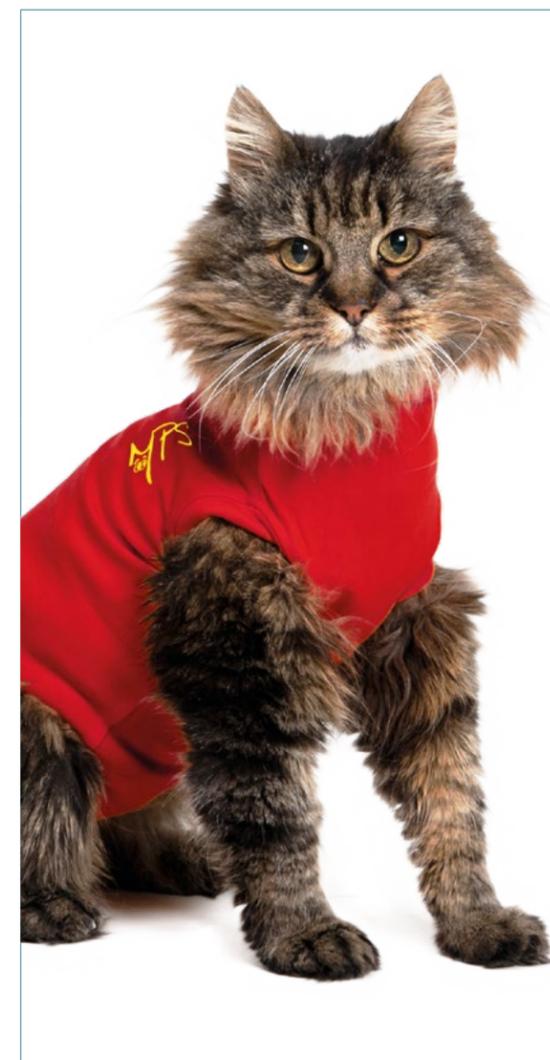

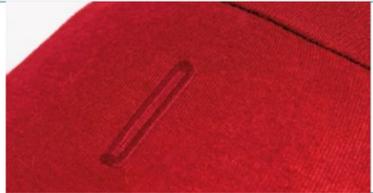

**1. Blind buttonhole on the back**  
The blind buttonhole between the shoulders can be used to apply an IV-line without the cat being able to reach it. The extra stitching around the buttonhole helps to prevent tearing.



**2. Collar fastening**  
There are loops on the inside of the collar. The collar of the cat can be pulled through these loops and attached. Which makes it more difficult for the cat to squeeze out of the shirt.



**3. Blue inner layer**  
The interior of the MPS-Protective Pet Shirt is light blue so fluid loss can be checked by the veterinarian or owner. In the event of fluid loss, a sanitary towel can be applied on the inside of the shirt or between the double layer.



**4. Special opening for litter box**  
The MPS-Protective Pet Shirt can stay fully closed on most cats when using the litter box because of the specially designed opening at the rear. How easy is that!



**5. Press studs for a good fit**  
It is difficult for the cat to open the MPS-Protective Pet Shirt itself because the closure is on the back. However, it is quick, safe and easy for the owner or veterinarian to put on and off the shirt.

### Washing Instructions



The MPS-Protective Pet Shirt is also sterilizable in an autoclave. At 121°C / 250°F on sensitive tissue.

The washing, maintenance and handling of delivered items always remains the responsibility of the customer.

**TIP!** To ensure the cat remains protected, two shirts are often purchased. While one can be washed, the other can be worn. Especially when a shirt is needed for more than one week, a back-up shirt is recommended.

The MPS-Protective Pet Shirt Cat is available in 4 colors/prints:

