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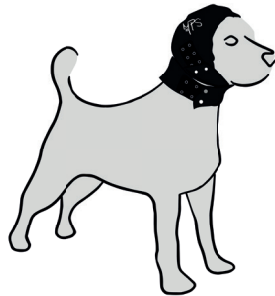
Medical Pet Shirts
The best recovery wear a pet can get.

MANUAL

MPS-Protective Head Cover Dog
+ 1 earpad | Dog • Hond • Chien • Hund • Perro

EN	Safety and warranty manual for dog owner.
NL	Veiligheids- en garantiehandleiding voor hondeneigenaar.
FR	Manuel de sécurité et de garantie pour propriétaire de chien.
DE	Sicherheits- und Garantiehinweise für Hundebesitzer.
ES	Manual de seguridad y garantía para propietarios de perros.

Incl. one (1)
EarPad
For extra soft
padding.

English	
Safety and warranty manual MPS- Protective Head Cover Dog.	
Product type: (Medical) head and ear cover for dogs.	
Read all safety and product information carefully before using the MPS-Protective Head Cover on your pet. Failure to follow these instructions may cause injury. MPS products are not toys or fashion items. Adult supervision is required during the use of this product. Improper use may lead to hotspots, chafing, skin irritation, inflammation, or infection. In case of emergency or doubt, consult your veterinarian immediately. Important safety instructions: 1. Usage. The MPS-Head Cover should only be used on the advice/consultation of your veterinarian and only after a medical procedure, for medical conditions, and skin problems. In case of emergency or doubt, consult your veterinarian immediately! Do not use the MPS-Head Cover if it is damaged or if it contains loose parts. 2. Supervision. Your pet must always be under the supervision of a responsible adult while wearing the MPS-Head Cover. Regular checks for cleanliness, friction, constriction, chafing, tripping hazard, or choking hazard are mandatory. 3. Adjustment period. Always keep your pet under supervision and allow him to get used to the Head Cover. If in doubt, always consult your veterinarian for advice. 4. Unusual behavior. In case of unusual behavior from your pet, which may include but is not limited to walking backward, sneezing, coughing, or excessive head shaking, remove the MPS-Head Cover immediately and consult your veterinarian. 5. Visible issues. Check for proper fit and any skin irritation daily. Report skin irritation or any other problems to your veterinarian immediately for proper treatment and care. 6. Fit. Ensure the right fit to avoid safety risks. A Head Cover that is too large can cause entanglement injuries; a Head Cover that is too small can cause friction or skin irritation. Always measure your pet as described in the manual, which is available through the link at the bottom of the page. There are three adjustable fastenings to create a snug fit: the adjustable strap under the chin, the adjustable strap around the neck and the Velcro® strap on the back of the neck. To secure the Head Cover properly, it is important to use at least two rows of snaps with the two fastening straps. Do not secure the Head Cover too tightly as this can result in breathing and blood circulation problems. Check at least once a day for any irritation around the head, neck, chin, and ears. 7. Wear time. Never use the Head Cover for more than eight (8) consecutive hours unless prescribed by your veterinarian. Check the problem area or surgical zone every day. Remove the Head Cover every few days, wash it, and let it air dry. For constant protection, we recommend using a second MPS-Head Cover. See the washing instructions in the user manual, available in the link below.	
Disclaimer: While every effort has been made to ensure that the information in this manual is accurate and complete, Medical Pet Shirts is not liable for errors in or incorrect use of the manual and the product. We reserve the right to modify the manual. Warranty: Our products are guaranteed against manufacturing defects. In case of a product defect, please contact your veterinarian or the place of purchase.	Want to know more about this product? Visit: links.medicalpetshirts.com/pdp-headcover-dog Scan the QR code for detailed information about usage, sizes and more. 

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