

OPTIMIZE RECOVERY FOR RABBITS IN YOUR PRACTICE

The Protective Pet Shirt offers an innovative and animal-friendly alternative to the traditional E-collar and bandages. Specifically designed for medical applications, protecting wounds, and covering skin issues, the shirt is tailored to the anatomy of rabbits, ensuring a perfect fit and comfort. Made from breathable cotton and 4-way-stretch Invista Lycra®, the shirt provides full freedom of movement. The convenient press studs on the back allow for easy and quick access to sensitive areas.

Why the MPS-Protective Pet Shirt sets the standard in pet recovery

- ✓ Covers wounds, hotspots, stitches, skin issues, and ointments.
- ✓ Protects after spaying and other medical procedures.
- ✓ Breathable material supports faster healing.
- ✓ A quick, safe and comfortable alternative to the cone.
- ✓ Provides warmth post-surgery and helps reduce stress.
- ✓ Can also protect IV-lines.



Sizing guide for the Protective Pet Shirt Rabbit																						
	<table border="1"> <thead> <tr> <th>Size</th> <th colspan="2">Weight (A)</th> </tr> </thead> <tbody> <tr> <td>MPS-3XS</td> <td>0.6 - 0.8 kg</td> <td>1.3 - 1.8 lbs</td> </tr> <tr> <td>MPS-2XS</td> <td>0.8 - 1.5 kg</td> <td>1.8 - 3.3 lbs</td> </tr> <tr> <td>MPS-XS</td> <td>1.5 - 2.0 kg</td> <td>3.3 - 4.4 lbs</td> </tr> <tr> <td>MPS-S</td> <td>2.0 - 3.5 kg</td> <td>4.4 - 7.7 lbs</td> </tr> <tr> <td>MPS-M</td> <td>3.5 - 4.5 kg</td> <td>7.7 - 9.9 lbs</td> </tr> <tr> <td>MPS-L</td> <td>4.5 - 5.5 kg</td> <td>9.9 - 12.1 lbs</td> </tr> </tbody> </table>	Size	Weight (A)		MPS-3XS	0.6 - 0.8 kg	1.3 - 1.8 lbs	MPS-2XS	0.8 - 1.5 kg	1.8 - 3.3 lbs	MPS-XS	1.5 - 2.0 kg	3.3 - 4.4 lbs	MPS-S	2.0 - 3.5 kg	4.4 - 7.7 lbs	MPS-M	3.5 - 4.5 kg	7.7 - 9.9 lbs	MPS-L	4.5 - 5.5 kg	9.9 - 12.1 lbs
	Size	Weight (A)																				
	MPS-3XS	0.6 - 0.8 kg	1.3 - 1.8 lbs																			
	MPS-2XS	0.8 - 1.5 kg	1.8 - 3.3 lbs																			
	MPS-XS	1.5 - 2.0 kg	3.3 - 4.4 lbs																			
	MPS-S	2.0 - 3.5 kg	4.4 - 7.7 lbs																			
MPS-M	3.5 - 4.5 kg	7.7 - 9.9 lbs																				
MPS-L	4.5 - 5.5 kg	9.9 - 12.1 lbs																				
<p>Sizing guide The correct size is determined by the weight of the rabbit (A). If the rabbit is somewhere between two sizes, we recommend choosing the smaller size.</p>																						

How to put the Protective Pet Shirt on the rabbit?			
Start with the head	Pull the legs through	Cover the body	Secure it
Roll up the shirt with the middle head opening in your hand. Make sure that the light blue inner layer is facing down. Insert your rabbit's head through the opening.	Followed by inserting the legs into the provided smaller leg openings.	Now pull the MPS-Protective Pet Shirt over your rabbit's body all the way to the back.	The lower belly flaps must be pulled between the hind legs towards the upper part near the base of the tail. Secure the two flaps with the press studs. We advise to close all press studs to keep the shirt perfectly in place.

FEATURES OF THE MPS-PROTECTIVE PET SHIRT

1. Longer neck stretch band
The neck area needs to be protected as well sometimes. That is why the shirt is equipped with a longer, soft, elastic neck protective collar.

2. Blue inner layer
The interior of the shirt is light blue so fluid loss can be checked by the veterinarian or owner. In the event of fluid loss, a sanitary towel can be applied on the inside of the shirt or between the double layer.

3. On the rabbit toilet
The shirt can stay fully closed on most rabbits when using the rabbit toilet because of the specially designed opening at the rear. How easy is that!

4. Press studs for a good fit
It is difficult for the rabbit to open the shirt itself because the closure is on the back. However, it is quick, safe and easy for the owner or veterinarian to put on and off the shirt.

5. Soft stretch bands
A soft, elastic protective collar around the front and hind legs and around the tail ensures that the shirt does not rub and cut into the legs and tail of the rabbit, but that the rabbit can move and heal in complete freedom.

Washing Instructions	
	<p>The MPS-Protective Pet Shirt is also sterilizable in an autoclave. At 121°C / 250°F on sensitive tissue.</p>
The washing, maintenance and handling of delivered items always remains the responsibility of the customer.	
<p>TIP! Little accidents can happen while wearing the shirt. To ensure the cat remains protected, two shirts are often purchased. While one can be washed, the other can be worn. Especially when a shirt is needed for more than one week, a back-up shirt is recommended.</p>	
<p>The MPS-Protective Pet Shirt Rabbit is available in 1 color:</p>	<p>MPS-Green</p>