

Optimize recovery for dogs in your practice.


The MPS-Protective Front Leg Double Sleeves can be used for protection on the front legs during hotspots, wounds, bandages, sensitive elbows and skin problems. The sleeves can even be useful in covering ointments to prevent the dog from licking the medicine. Expedite the healing process of each dog with the Double Sleeves, made of high-quality cotton and stretchable Invista Lycra® for the best comfort and care.

Why the MPS-Front Leg Double Sleeves sets the standard in pet recovery

- ✓ Can protect IV-lines.
- ✓ Extra protection on the elbows.
- ✓ Easily adjustable for the perfect fit.
- ✓ Easy and quick check-ups.
- ✓ Limits stress and can have a calming effect.
- ✓ Available in 8 sizes.



Sizing guide of the MPS-Protective Single Sleeve Dog

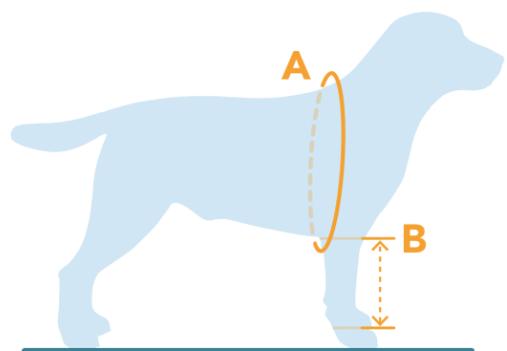

Size	Chest (A)		LengthLeg (B)	
	cm	in	cm	in
3XS	27 - 33	10.6 - 13.0	10	3.9
2XS	30 - 43	11.8 - 16.9	12	4.7
XS	40 - 52	15.7 - 20.5	17	6.7
S	45 - 61	17.7 - 24.0	23	9.1
M	54 - 68	21.3 - 26.8	26	10.2
L	64 - 84	25.2 - 33.1	35	13.8
XL	80 - 104	31.5 - 40.9	41	16.1
2XL	96 - 124	37.8 - 48.8	48	18.9

Size guide


Measure the chest circumference around the widest part of the **chest (A)**, usually right behind the front legs. Measure the **Length (B)** on the inside of the leg from the armpit to ankle.

How to put the Protective Front Leg Double Sleeves

Putting the sleeves on:


Insert one leg through the sleeve. Insert the other leg through the sleeve on the opposite side. Make sure that the sleeve and flap with MPS logo on it are on the left leg when inserting.

*When closed, the MPS Logo should be on the right shoulder.

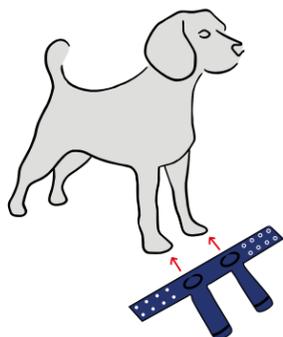

Securing the sleeve:

Secure the flaps around the chest by using the press studs and Velcro®. It is important to use at least two rows press studs. Take a good look around the legs and chest area to see if there is enough room to move and breathe!

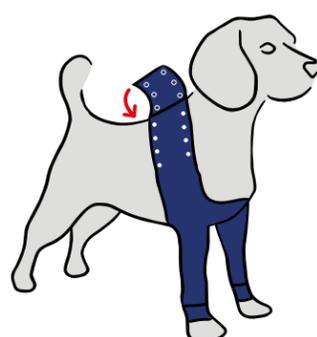

Features of the MPS-Protective Front Leg Double sleeve Dog.



1. Press studs for a good fixation

It is nearly impossible for the dog to open the Double Front Leg Sleeve due to the closure being located on or near the shoulders. This however allows for easy access for the owner and the veterinarian.




2. Calming effect

The Front Leg Double Sleeve has an additional advantage: the gentle pressure around the chest thanks to the adjustable closure and soft stretch band offers a calming effect on most dogs.



3. Soft stretch band


A soft stretch band at the bottom of the sleeve ensures that the sleeve fit more snugly around the leg. Sleeves that are too long can be rolled up. A soft stretch band is also around the chest area.

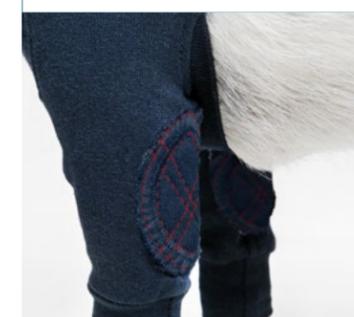

4. Elbow pads

An additional soft protective layer has been added around the elbows. This is designed for dogs that need a bit more comfort around the elbow. This section includes an extra piece of fabric with a thin layer of polyester in between.

Washing instructions

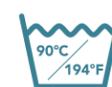

- Always wash the sleeve inside out
- Fasten all press studs and Velcro® before washing
- Use a laundry bag

The washing, maintenance and handling of delivered items always remains the responsibility of the customer.

TIP! To ensure the dog remains protected, Two Sleeves are often purchased to ensure proper hygiene and protection in between washings. Especially when a sleeve is needed for more than one week, a back-up is recommended.

The MPS-Protective Double Sleeve Dog is available in 1 color:

MPS-Blue