

## Optimize recovery for dogs in your practice.

Two hind leg sleeves for wound and bandage protection whilst allowing freedom of movement. Give our furry friends the comfort and freedom they deserve for a faster recovery! The MPS-Hind Leg Sleeves comes with two sleeves. The two sleeves must always be worn as otherwise they will twist. Moreover, always secure Hind Leg Sleeves to the MPS-Protective Top Shirt as not doing so will cause the Hind Leg Sleeves to slip down while being worn.

### Why the MPS-Protective Hind Leg Sleeve sets the standard in pet recovery

- ✓ Replaces a full hind leg bandage.
- ✓ Protects after medical procedures, wounds and stitches.
- ✓ Breathable fabric to support fast healing.
- ✓ Faster, safer and more comfortable recovery compared to a cone.
- ✓ Double layered protection.
- ✓ Available in 6 sizes.



Sizing guide of the MPS-Protective Hind Leg Double Sleeve Dog		
Size	Length Hind Leg (A)	
MPS-2XS	10 cm	3.9 in
MPS-XS	14 cm	5.5 in
MPS-S	20 cm	7.8 in
MPS-M	26 cm	10.2 in
MPS-L	32 cm	12.5 in
MPS-XL	37 cm	14.5 in

The MPS-HLS must be worn with the MPS-TOP Shirt, purchased separately. Always measure the dog for the right fit.

### Size guide

Measure the length of the back leg (A) on the inside, from the groin (near the belly) down to the hock.

### How to put the Protective Hind Leg Double Sleeves on the dog

Put on the MPS-Top Shirt	Put the legs into the sleeves	Attach the sleeves	Attach the straps	Link the sleeves
Put the MPS-Top Shirt on by inserting the head first, followed by each front leg separately into the openings provided. Close the MPS-Top Shirt with the adjustable Velcro®.	Separate the MPS-Hind Leg Sleeves. Put a hind leg in the first sleeve. Then put the other leg in the second sleeve.	Attach both sleeves to each other with the Velcro® closure on the back part near the base of the tail. Make sure that the sleeve with the MPS logo on it is on the right hind leg.	Fasten the straps on the back crosswise to the red loops of the MPS-Top Shirt. On the sides, attach the straps in a straight line to the shirt.	Link the sleeves together with the straps under the tail. Secure firmly to prevent movement of the sleeves.

## Features of the MPS-Protective Hind Leg Double Sleeve Dog.

- Adjustable Velcro® fastening**  
The MPS-Hind Leg Sleeves come with two sleeves with a Velcro® fastening at the top. The two sleeves must always be worn as otherwise they will twist.
- Double layer & soft protection**  
The sleeves are made of a double layer material with an extra soft thin padding at the front for additional protection of the legs.
- Securing the MPS-Top Shirt**  
Fasten the straps on the back of the Hind Leg Sleeves crosswise to the red loops of the Top shirt. On the sides, attach the straps in a straight line to the shirt.
- Elastic straps**  
The adjustable strap with small hook must be secured under the tail to prevent the sleeves from twisting.
- Always wear with the MPS-Top Shirt**  
Always secure the MPS-Hind Leg Sleeves to the MPS-Top Shirt as not doing so will cause the sleeves to slip down while being worn.

Washing instructions	
	<ul style="list-style-type: none"> <li>• Always wash the sleeve inside out</li> <li>• Fasten all press studs and Velcro® before washing</li> <li>• Use a laundry bag</li> </ul>

The washing, maintenance and handling of delivered items always remains the responsibility of the customer.

**TIP! Little accidents can happen while wearing the sleeves. To ensure the dog remains protected, Two Sleeves are often purchased to ensure proper hygiene and protection in between washings. Especially when a sleeve is needed for more than one week, a back-up is recommended.**

The MPS-Protective Hind Leg Double Sleeve Dog is available in 1 color:

MPS-Blue