

## Optimize recovery for dogs in your practice.

The innovative Protective Pet Shirt offers an animal-friendly alternative to the traditional E-collar and bandages. Specifically designed for medical applications, protecting wounds and covering skin issues, the shirt is tailored to the anatomy of dogs, ensuring a perfect fit and comfort. Made from breathable cotton and 4-way-stretch Invista Lycra®, the shirt provides full freedom of movement and sufficient oxygen flow for healing wounds. The convenient press studs on the back allow for easy and quick access to sensitive areas.

### Why the MPS-Protective Pet Shirt sets the standard in pet recovery

- ✓ Covers wounds, hotspots, stitches, skin issues and ointments.
- ✓ Protects after spaying/neutering and other medical procedures.
- ✓ Breathable material supports faster healing.
- ✓ A quick, safe and comfortable alternative to the cone.
- ✓ Provides warmth post-surgery and helps reduce stress.
- ✓ Can also protect IV-lines.



| Sizing guide of the MPS-Protective Pet Shirts dog |            |                  |
|---|------------|------------------|
|   | Size       | Length (A)       |
|   | PPS-4XS    | 19 - 30 cm       |
| PPS-3XS   | 22 - 35 cm | 8.7 - 13.8 inch  |
| PPS-2XS   | 33 - 42 cm | 13.0 - 16.5 inch |
| PPS-XS  | 40 - 45 cm | 15.7 - 17.7 inch |
| PPS-S   | 43 - 51 cm | 16.9 - 20.1 inch |
| PPS-S+  | 49 - 57 cm | 19.3 - 22.4 inch |
| PPS-M   | 55 - 69 cm | 21.7 - 27.2 inch |
| PPS-M+  | 61 - 72 cm | 24.0 - 28.3 inch |
| PPS-L   | 67 - 76 cm | 26.4 - 29.9 inch |
| PPS-XL  | 74 - 82 cm | 29.1 - 32.3 inch |
| PPS-2XL   | 80 - 92 cm | 31.5 - 36.2 inch |

### How to put the Protective Pet Shirt on the dog?

| Start with the head   | Pull the legs through  | Cover the body  | Secure it  |
|---|--|---|--|
| Roll up the shirt with the middle head opening in your hand. Make sure that the light blue inner layer is facing up toward the belly. Insert your dog's head through the opening. | Followed by inserting the legs into the provided smaller leg openings. | Now pull the MPS-Protective Pet Shirt over your dog's body all the way to the back. | The lower belly flaps must be pulled between the hind legs towards the upper part near the base of the tail. Secure the two flaps with the press studs. We advise to close all press studs to keep the shirt perfectly in place. |
|   |  |   |  |

## Features of the MPS-Protective Pet Shirt Dog.

- 1. Blind buttonhole on the back**  
 You can use the blind buttonhole between the shoulders if you want to insert an IV-line. Open the buttonhole by cutting it in the middle of the seam. The extra stitching around the buttonhole helps to prevent tearing.
- 2. Collar fastening**  
 There are loops on the inside of the shirt. The dog's collar can be pulled through these loops. Which makes it more difficult for the dog to squeeze out of the MPS-Protective Pet Shirt.
- 3. Easy hold-up system**  
 In order to be able to walk the dog or check the surgical area, Medical Pet Shirts has developed a hold-up system. The shirt can be rolled up inside and secured to the chest with a press stud.
- 4. Blue inner layer**  
 The interior of the MPS-Protective Pet Shirt is light blue so fluid loss can be checked by the veterinarian or owner. In the event of fluid loss, a sanitary towel can be applied on the inside of the shirt or between the double layer.
- 5. Press studs for a good fit**  
 It is difficult for the dog to open the Protective Pet Shirt itself because of the placement of the closure on the back. However, it is quick, safe and easy for the owner or veterinarian to put the shirt on and off.

**Washing instructions**

**121°** The MPS-Protective Pet Shirt is also sterilizable in an autoclave. At 121°C / 250°F on sensitive tissue.

The washing, maintenance and handling of delivered items always remains the responsibility of the customer.

**TIP! Little (peeing) accidents can happen while wearing the shirt. To ensure the dog remains protected, Two shirts are often purchased to ensure proper hygiene and protection in between washings. Especially when a shirt is needed for more than one week, a back-up shirt is recommended.**

The MPS-Protective Pet Shirt Dog is available in 3 colors/prints: