Sicherheits- und garantiehinweise MPS-Protective Front Leg Single Sleeve Hund.

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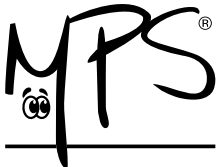
Medical Pet Shirts
The best recovery wear a pet can get.

MANUAL

MPS- Protective Front Leg Single Sleeve
Dog • Hond • Chien • Hund • Perro

EN	Safety and warranty manual for dog owner.
NL	Veiligheids- en garantiehandleiding voor hondeneigenaar.
FR	Manuel de sécurité et de garantie pour propriétaire de chien.
DE	Sicherheits- und garantiehinweise für Hundebesitzer.
ES	Manual de seguridad y garantía para propietarios de perros.

Safety and warranty manual MPS- Protective Front Leg Single Sleeve Dog.	
Product type: (Medical) Protective front leg sleeve for dogs.	
Read all safety and product information carefully before using the MPS- Protective Front Leg Single Sleeve on your pet. Failure to follow these instructions may cause injury. MPS products are not toys or fashion items. Adult supervision is required during the use of this product. Improper use may lead to hotspots, chafing, skin irritation, inflammation, or infection. In case of emergency or doubt, consult your veterinarian immediately. Important safety instructions: 1. Usage. The MPS- Protective Front Leg Single Sleeve should only be used on the advice/consultation of your veterinarian and only after a medical procedure, for medical conditions, and skin problems. In case of emergency or doubt, consult your veterinarian immediately! Do not use the sleeve if it is damaged or if it contains loose parts. 2. Supervision. Your pet must always be under the supervision of a responsible adult while wearing the MPS- Protective Front Leg Single Sleeve. Regular checks for cleanliness, friction, constriction, chafing, tripping hazard, or choking hazard are mandatory. 3. Adjustment period. Allow your pet to get used to the sleeve before a surgery and have your veterinarian put the sleeve on your pet during anesthesia. If in doubt, always consult your veterinarian for advice. If your pet tends to chew on materials, this product is not recommended. Therefore, always keep your pet under supervision. 4. Unusual behavior. In case of unusual behavior from your pet, which may include but is not limited to walking backward or excessive licking, remove the MPS- Protective Front Leg Single Sleeve immediately and consult your veterinarian. 5. Visible issues. Check for proper fit and any skin irritation daily. Report skin irritation or any other problems to your veterinarian immediately for proper treatment and care. 6. Fit. Ensure the right fit to avoid safety risks. A sleeve that is too large can cause tripping or choking hazards; a sleeve that is too small can cause friction or skin irritation. Always measure your pet as described in the manual, which is available through the link at the bottom of the page. The sleeve is designed to have a snug fit. It is made of lightweight, breathable, and stretchable material. Close at least two rows of snaps/poppers to keep the sleeve in place. Make sure there is always room for 1 to 2 fingers around the legs, chest, and armpit. Check at least once a day for any irritation around the legs, chest, or armpit. 7. Wear time. Never use the sleeve for more than eight (8) consecutive hours unless prescribed by your veterinarian. Check the problem area or surgical zone every day. Remove the sleeve every few days, wash it, and let it air dry. For constant protection, we recommend using a second MPS- Protective Front Leg Single Sleeve. See the washing instructions in the user manual, available in the link below.	
Disclaimer: While every effort has been made to ensure that the information in this manual is accurate and complete, Medical Pet Shirts is not liable for errors in or incorrect use of the manual and the product. We reserve the right to modify the manual. Warranty: Our products are guaranteed against manufacturing defects. In case of a product defect, please contact your veterinarian or the place of purchase.	Want to know more about this product? Visit: links.medicalpetshirts.com/pdp-front-leg-single-sleeve-dog Scan the QR code for detailed information about usage, sizes and more. 

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