

Optimize recovery for dogs in your practice.

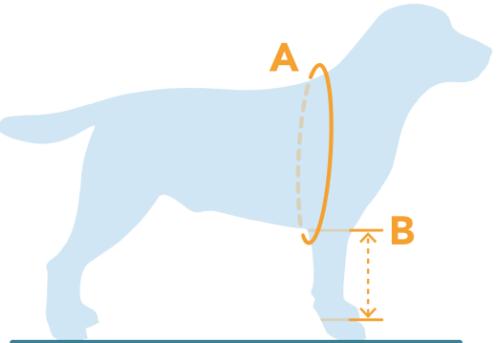
The MPS-Protective Front Leg Single Sleeve can be used for protection on the front leg during hotspots, wounds, bandages, sensitive elbows and skin problems. The sleeve can even be useful in covering ointments to prevent the dog from licking the medicine. Expedite the healing process of each dog with the Single Sleeve, made of high-quality cotton and stretchable Invista Lycra® for the best comfort and care.

Why the MPS-Protective Single Sleeve sets the standard in pet recovery

- ✓ Can protect IV-lines.
- ✓ To be used on the left or right front leg - Interchangeable.
- ✓ Easily adjustable for the perfect fit.
- ✓ Easy and quick check-ups.
- ✓ Limits stress and can have a calming effect.
- ✓ Available in 8 sizes.

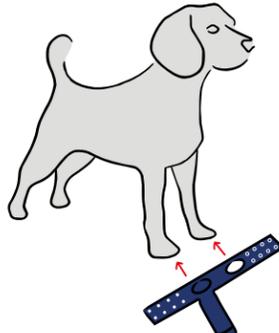
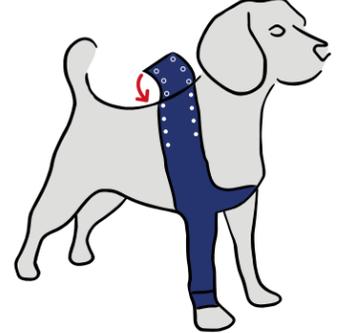


Sizing guide of the MPS-Protective Single Sleeve Dog					
Size	Chest (A)		Length Leg (B)		
	3XS	27 - 33 cm	10.6 - 13.0 in	10cm	3.9 in
2XS	30 - 43 cm	11.8 - 16.9 in	12 cm	4.7 in	
XS	40 - 52 cm	15.7 - 20.5 in	17 cm	6.7 in	
S	45 - 61 cm	17.7 - 24.0 in	23 cm	9.1 in	
M	54 - 68 cm	21.3 - 26.8 in	26 cm	10.2 in	
L	64 - 84 cm	25.2 - 33.1 in	35 cm	13.8 in	
XL	80 - 104 cm	31.5 - 40.9 in	41 cm	16.1 in	
2XL	96 - 124 cm	37.8 - 48.8 in	48 cm	18.9 in	



Size guide

Measure the chest circumference around the widest part of the chest (A), usually right behind the front legs. Measure Length Leg (B) on the inside of the leg from the armpit to ankle.

How to put the Protective Front Leg Single Sleeve	
<p>Putting the sleeve on:</p> <p>Roll up the sleeve for easier access. Pull the recovering leg through the sleeve. Pull the other leg through the opening on the other side of the sleeve.</p>	<p>Securing the sleeve:</p> <p>Secure the flaps around the chest by using the press studs and Velcro®. It is important to use at least two rows of press studs. Take a good look around the legs and chest area to see if there is enough room to move and breathe!</p>
	

Features of the MPS-Protective Front Leg Sleeve Dog.



1. Press studs for a good fixation
It is nearly impossible for the dog to open the Single Front Leg Sleeve due to the closure being located on or near the shoulders. This however allows for easy access for the owner and the veterinarian.

2. Calming effect
The Front Leg Single Sleeve has an additional advantage: the gentle pressure around the chest thanks to the adjustable closure and soft stretch band offers a calming effect on most dogs.

3. Soft stretch band
A soft stretch band at the bottom of the sleeve ensures that the sleeve fit more snugly around the leg. Sleeves that are too long can be rolled up. A soft stretch band is also around the chest area.

4. Interchangeable
The MPS-Protective Front Leg Single Sleeve has a symmetrical design which makes it applicable on the left and right leg. One leg goes into the sleeve and the other goes through the opening on the other side.

Washing instructions	
	<ul style="list-style-type: none"> • Always wash the sleeve inside out • Fasten all press studs and Velcro® before washing • Use a laundry bag
<p>The washing, maintenance and handling of delivered items always remains the responsibility of the customer.</p>	
<p>TIP! To ensure the dog remains protected, Two Sleeves are often purchased to ensure proper hygiene and protection in between washings. Especially when a sleeve is needed for more than one week, a back-up is recommended.</p>	
<p>The MPS-Protective Single Sleeve Dog is available in 1 color:</p>	<p>MPS-Blue</p>