English

Instructions for use

DESCRIPTION

Daily test for the efficacy evaluation of the air removal in pre-vacuum sterilizers. The device is intended for professional use by trained people only.

CONFORMITY

AAMI/ISO 11140-5: 2007
AAMI/ISO 11140-1, Type 2: 2014

SYMBOLS

- Keep away from sunlight and from heat
- Temperature limits
- Do not re-use
- Keep dry

REF Catalogue number

LOT Lot number

AIR REMOVAL For air removal process only

STEAM For steam sterilizers only

Consult the instructions for use

Expiration date

Manufacturing date

Manufacturer

DIRECTIONS FOR USE:

- Preheat the sterilizer.
- Place one test pack on the lowest tray over the drain. The sterilizer must be empty except for the test pack, the tray holder and the lowest tray.
- From the sterilizer menu, run a B&D TEST cycle at 273°F (134°C). If the sterilizer does not feature a Bowie-Dick test cycle, run a 273°F (134°C), 3.5-minute sterilization cycle.
- At the end of the cycle, retrieve the test pack from the sterilizer.
Caution: the test pack may be hot.
- Remove the test sheet from the pack and examine the indicator test pattern.

INTERPRETATION OF THE INDICATORS

Fig. 1: a **uniform blue pattern** represents an unexposed test sheet.

Fig. 2: a **uniform pink pattern** represents an acceptable cycle.

Fig. 3: a **pink pattern with a dark blue area** near the center of the test sheet represents an unacceptable air removal condition.

Fig. 4: a **uniform dark blue pattern** represents a low temperature condition.

In case the test fails, unacceptable test conditions must be remedied, and the test repeated until an acceptable test pattern is observed.

Note: the device is for single use.

Warning: Bowie - Dick test pass result is not an indication that the sterilizer has reached the parameters for sterilization.

STORAGE CONDITIONS AND WARNINGS

Avoid direct exposure to the sun or heat sources and storage in highly polluted environments. Store the packaging avoiding direct contact with the floor or walls, since the material is sensitive to humidity and heat. Store in protected and dry places, away from heat and humidity sources.

The packaging's characteristics and properties remain unaltered if the conditions of good preservation, use and storage are maintained.

Do not use if the packaging is damaged or not properly stored.

EXPIRY AND TRACEABILITY

Don't use B&D TEST after the expiration date shown on the package.

DEVICE CODE

The product may be reordered with code T800003X.

DISPOSAL AND OTHERS

The device must be disposed of in accordance with current / applicable laws and rules in the Country of use.

Lead and heavy metals have not been used in the fabrication of the product.

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